

crocodile™

Exploded views / Part lists

Rev-date (05-2018)

On the following pages you will find the parts used to assemble this product. Please note that some spare parts cannot be purchased individually but are part of a complete kit. Please also note that for some spare parts extra delivery time may occur.




Contents

Crocodile size 1+2	3
--------------------------	---

Accessories

Back support - for sizes 1-3	5
Anti tip device fixed - for sizes 1-2	5
Anti tip swing away - for sizes 1-2	5
Hip support - for sizes 1-3	5
Seat - size 1-2	5
Armrest size 1+2	7
Helpers handle size 1+2	8
Back support - for sizes 1-3	10
Handbrake kit - for size 1-2	11


Crocodile size 3	12
Armrest - size 3	15
Cross bar - for size 3	16
Back support - size 3	17
Seat - size 3	18
Anti tip device fixed - for sizes 3	19
Anti tip swing away - for sizes 3	20
Handbrake kit - size 3	21

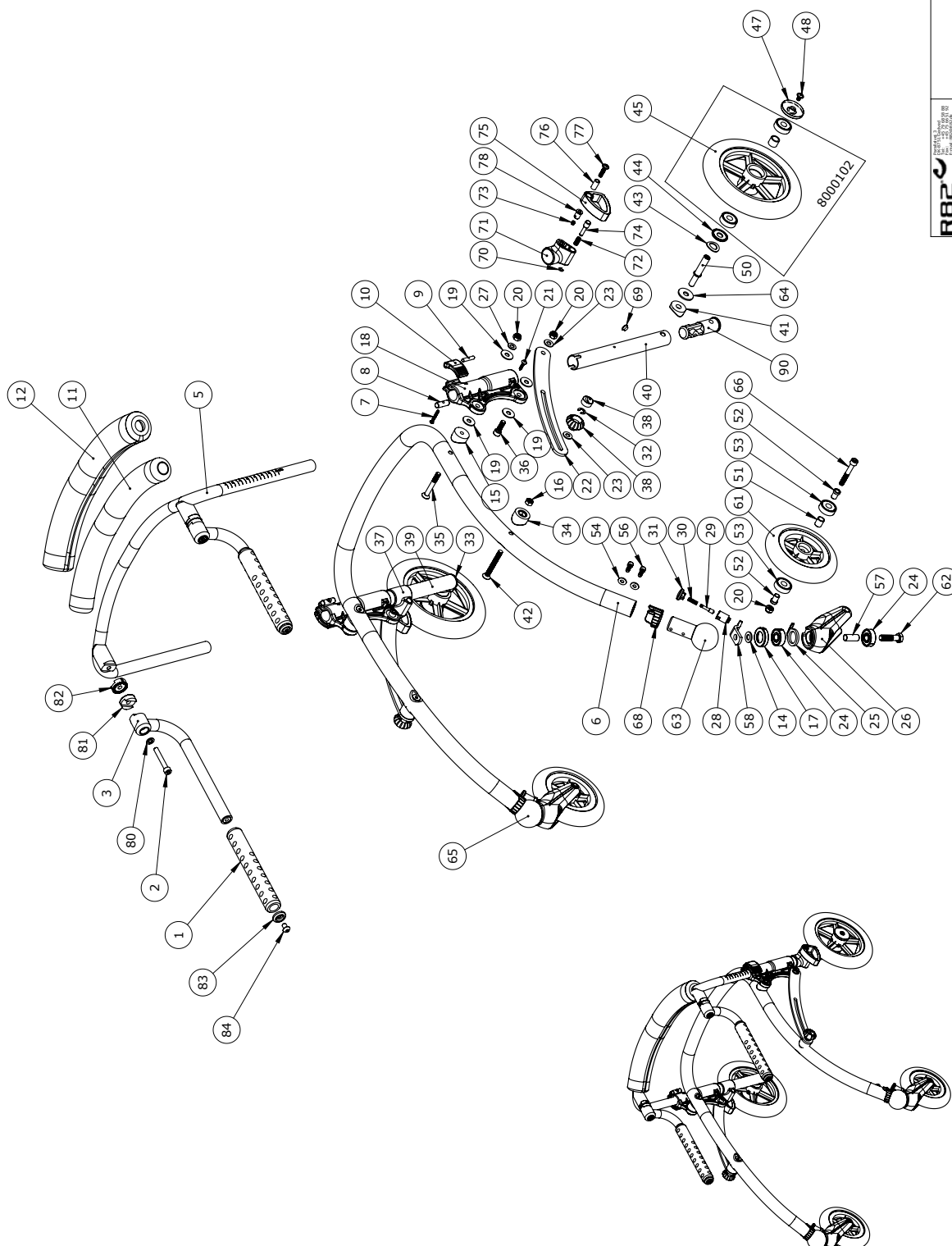

R82 <small>Nothing compares to a smile</small> Crocodile	Created by jpn	Revision E	Revision by PER
	Created date 23-02-04	Revision date 30-11-2015	Revision no. 107812


Position	No.	Description	Unit of Measure	Quantity	Ean code
1	2500530	Tpe-cover	PCS	2	5707292381781
2	0603600	Insex screw 8*55	PCS	2	5707292138262
3	100782	Handle for Crocodile	PCS	2	5707292569448
11	4395000	Pipe isolating Ø18mm. 2m long	MTR	0,41	5707292149596
15	73428	Saddle washer Ø30-r16, alu	PCS	2	5707292447852
16	1603500	Steel nut 8mm, large pack	PCS	2	5707292141286
19	1108000	Washer 8mm	PCS	4	5707292140388
20	1607004	Lock nut M8, hight 6,5 mm	PCS	2	5707292444516
22	73417	Banana	PCS	1	5707292351760
23	1102000	Planing washer 8mm	PCS	2	5707292140227
27	1103200	Fazet washer 8mm	PCS	2	5707292140296
32	1100202	Lockring for 6mm axle	PCS	2	5707292445070
34	73442	Saddle washer	PCS	2	5707292354778
35	0625061	Countersunk screw M8*60,stainl	PCS	2	5707292355850
38	3470393	R82 handle M8*14mm nut w.cover	PCS	2	5707292427458
42	73404	Countersunk screw M8*60,stainl	PCS	2	5707292447845
54	1103000	Fazet washer 6mm	PCS	4	5707292140289
56	0602300	Insex screw 6*20	PCS	4	5707292138101
68	73427	Case for length adj.	PCS	2	5707292361349
80	1101000	Center spring washer 8mm	PCS	2	5707292140173
83	109106	Stop disc	PCS	2	5707292575821
84	0661100	Button head screw 8*12	PCS	2	5707292139511
81+82	74061	Tooth washer Ø30	PCS	4	5707292373816
90	73483	Alu rear wheel retainer	PCS	2	5707292447876

SPAREPART KITS					
11 12	86813	Crocodile padding size 1	PCS	1	5707292356536
11 12	86814	Crocodile padding size 2	PCS	1	5707292356550
1 2 3 5 11 12 80 81 82 83 84	100783-1	Front brace assembly 1	PCS	1	5707292563583
1 2 3 5 11 12 80 81 82 83 84	100783-2	Front brace assembly 2	PCS	1	5707292563606
7 8 9 10	86831	Handle incl. lock and screw	PCS	1	5707292433930
28 29 30 31 58	86826	Directional stabilizer	PCS	1	5707292416308
75 74 76 37 70 72 78 73 77 71	86828	On/Off anti-reverse stop	PCS	1	5707292365309
33 37 39 41 43 44 45 47 48 50 64 69 70 72 73 74 75 76 77 78	86835-1HM	Right backwheel size 1	PCS	1	5707292406422
40 41 43 44 45 47 48 50 64 69 70 71 72 73 74 75 76 77 78 90	86835-1VM	Left backwheel size 1	PCS	1	5707292406538
33 37 39 41 43 44 45 47 48 50 64 69 70 72 73 74 75 76 77 78	86835-2HM	Right backwheel size 2	PCS	1	5707292406545
40 41 43 44 45 47 48 50 64 69 70 71 72 73 74 75 76 77 78 90	86835-2VM	Left backwheel size 2	PCS	1	5707292406552
61	86838	Front wheel	PCS	1	5707292373663
14 17 24 25 26 28 29 30 31 51 57 58 62 63 66 20	86837-V	Front fork left	PCS	1	5707292373199
14 17 24 25 26 28 29 30 31 51 57 58 62 65 66 20	86837-H	Front fork right	PCS	1	5707292373205

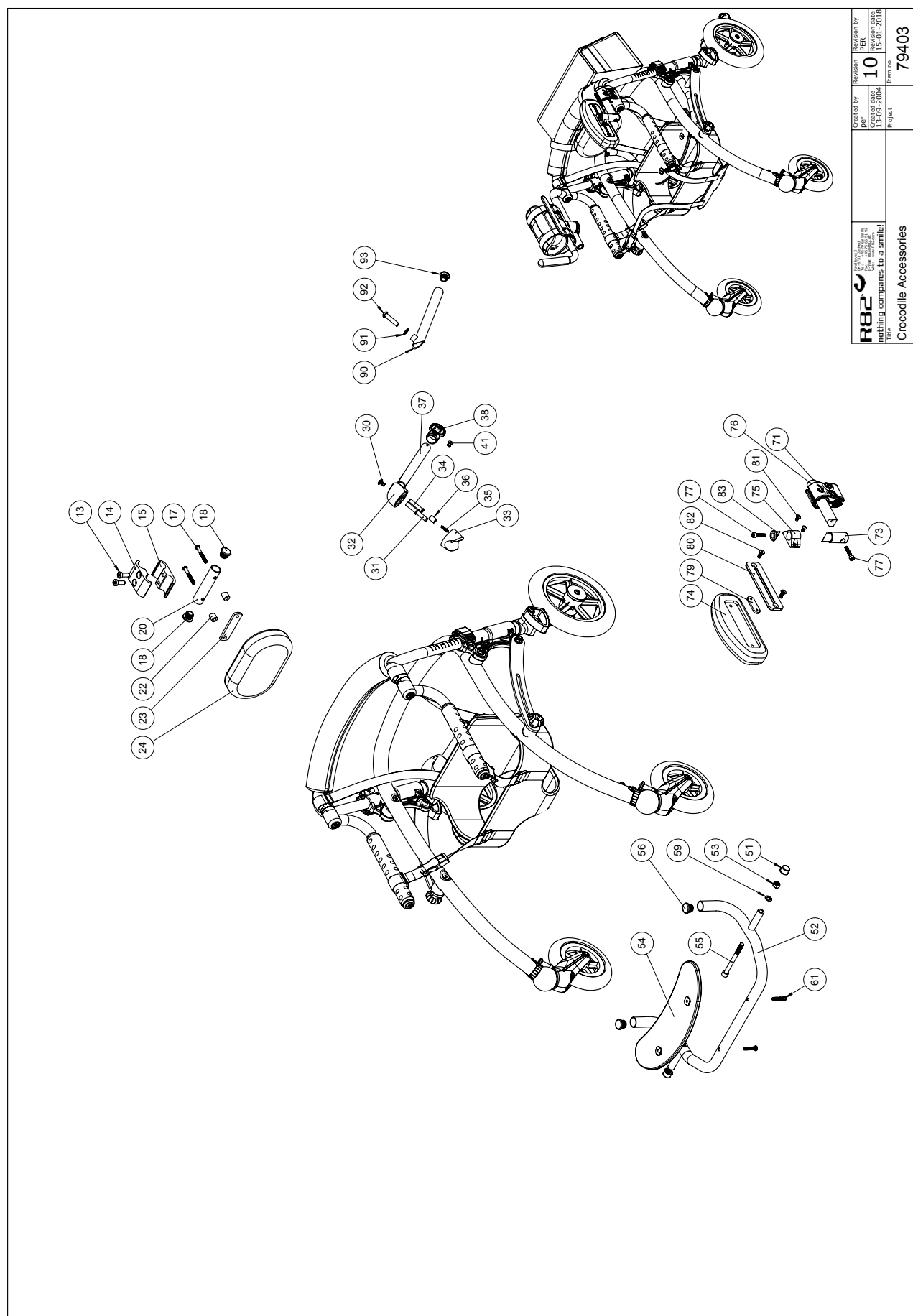


R82 <small>nothing compares to a smile!</small> Title Crocodile Accessories	Created by	Revision	Item no
	Per 13-09-2004	10 15-01-2018	79403

Position	No.	Description	Unit of Measure	Quantity	Ean code
30	0659010	Button head with flange M6*10	PCS	2	5707292356819
31	73459	Staybolt for kip security	PCS	2	5707292355287
32	73439	Swivel joint removable	PCS	2	5707292355089
33	73438	Joint fixed, Crocodile	PCS	2	5707292354747
34	73458	Release button, Crocodile	PCS	2	5707292355263
35	0309175	Comp. Spring 0,8*5,8*17,5mm*10	PCS	2	5707292135490
36	82555	lower pin	PCS	2	5707292124722
90	101174	Anti-tip simple for Crocodile	PCS	2	5707292555267
91	1100500	Spring washer 8mm, stainless	PCS	2	5707292140142
92	0663035	Button head screw 8*35 stainl.	PCS	2	5707292139627
93	2502510	Plug 20*2,0-2,5 inside black	PCS	2	5707292557759
71	112656	Clamping unit for crocodile	PCS	2	5707292603968
73	73465	Tightening bush for hip supp	PCS	2	5707292355300
74	6175171	Cushion size 1	PCS	2	5707292189301
75	82542	House tool	PCS	2	5707292124593
76	73468	Bar for side support	PCS	2	5707292357328
77	0602400	Insex screw 6*25	PCS	4	5707292138118
79	82974	Threaded plate for hip support	PCS	2	5707292126184
80	82953	Plate for hip support	PCS	2	5707292126047
81	0658010	Button head M5 x 10, stainless	PCS	4	5707292139375
82	0609728	Insex, stainless 6*16 uh	PCS	4	5707292138491
83	82939	Tightening bushing f. Swingarm	PCS	2	5707292125941
51	2501407	Nut protection, black, f/M8	PCS	2	5707292447227
52	73393	Seat brace 1, yellow, new model	PCS	1	5707292440655
52	73394	Seat brace 2, orange, new model	PCS	1	5707292440679
52	73394-60	Seat brace 2, black, new model	PCS	1	5707292440686
53	1607000	Lock nut 8mm, large pack	PCS	2	5707292141378
54	5020900	Upholstered Seat size 1+2, croc	PCS	1	5707292358240
55	0603900	Insex screw 8*90	PCS	2	5707292138316
56	2503000	Ferrule d22*1,0-2,0 int. black	PCS	2	5707292144041
59	1102000	Planing washer 8mm	PCS	2	5707292140227
61	0659580	Button head screw 6*30 mm	PCS	2	5707292139481
15	112656	Clamping unit for crocodile	PCS	1	5707292603968
17	0659700	Button head M6*50 stainless	PCS	2	5707292566041
18	2503010	Ferrule d22*1,8-3,5, black	PCS	2	5707292357632
20	73486	Pipe for back support	PCS	1	5707292355171
22	73466	Saddle bush	PCS	2	5707292355317
23	80543-01	Mounting 20*5*80	PCS	1	5707292361431
24	6175221	Abd. Cushion size 1+2	PCS	1	5707292189400

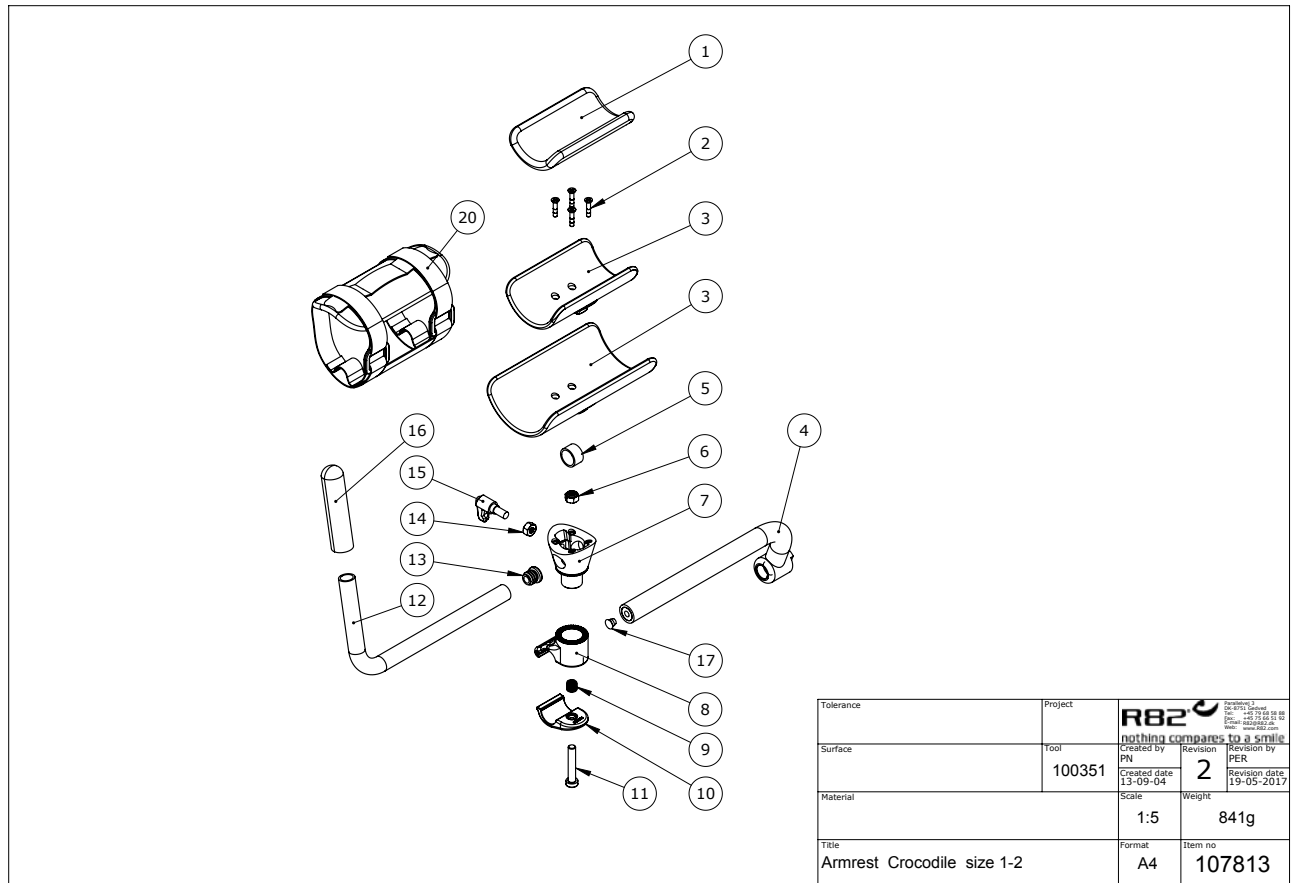


Position	No.	Description	Unit of Measure	Quantity	Ean code
1	5020910	Arm rest cover size 1	S/ET	1	5707292361158
1	5020920	Cover for forearm support sz 2	S/ET	1	5707292361165
2	1940016	Plate screw 4,2*19 FZB	PCS	8	5707292356796
3	73448	Armholder 1,crocodile	PCS	2	5707292355126
3	73449	Arm rest plate sz 2,crocodile	PCS	2	5707292355133
4	100782	Handle for Crocodile	PCS	2	5707292569448
5	128485	Steel bushing for armrest	PCS	2	5707292795304
6	1607000	Lock nut 8mm, large pack	PCS	2	5707292141378
7	73446	Retainer forearm support	PCS	2	5707292355102
8	102798	Armrest adjustment (top)	PCS	2	5707292564559
9	0310129	pressure spring 0,9*11*26mm*7v	PCS	2	5707292438782
10	102799	Armrest adjustment (bottom)	PCS	2	5707292564566
11	0663041	Button head M8*40, rf	PCS	2	5707292575388
12	73467	Tube for handle Crocodile	PCS	2	5707292356314
13	2502002	Ferrule d18*0,8-2,5 black	PCS	2	5707292433947
14	1750800	Lock nut, stainless M8	PCS	2	5707292141484
15	3502516	Tightening handle 45mm M8*16	PCS	2	5707292146915
16	2500516	Grip 87.00 mm length	PCS	2	5707292356727
17	1310528	Covering stud, black 13mm	PCS	2	5707292470348

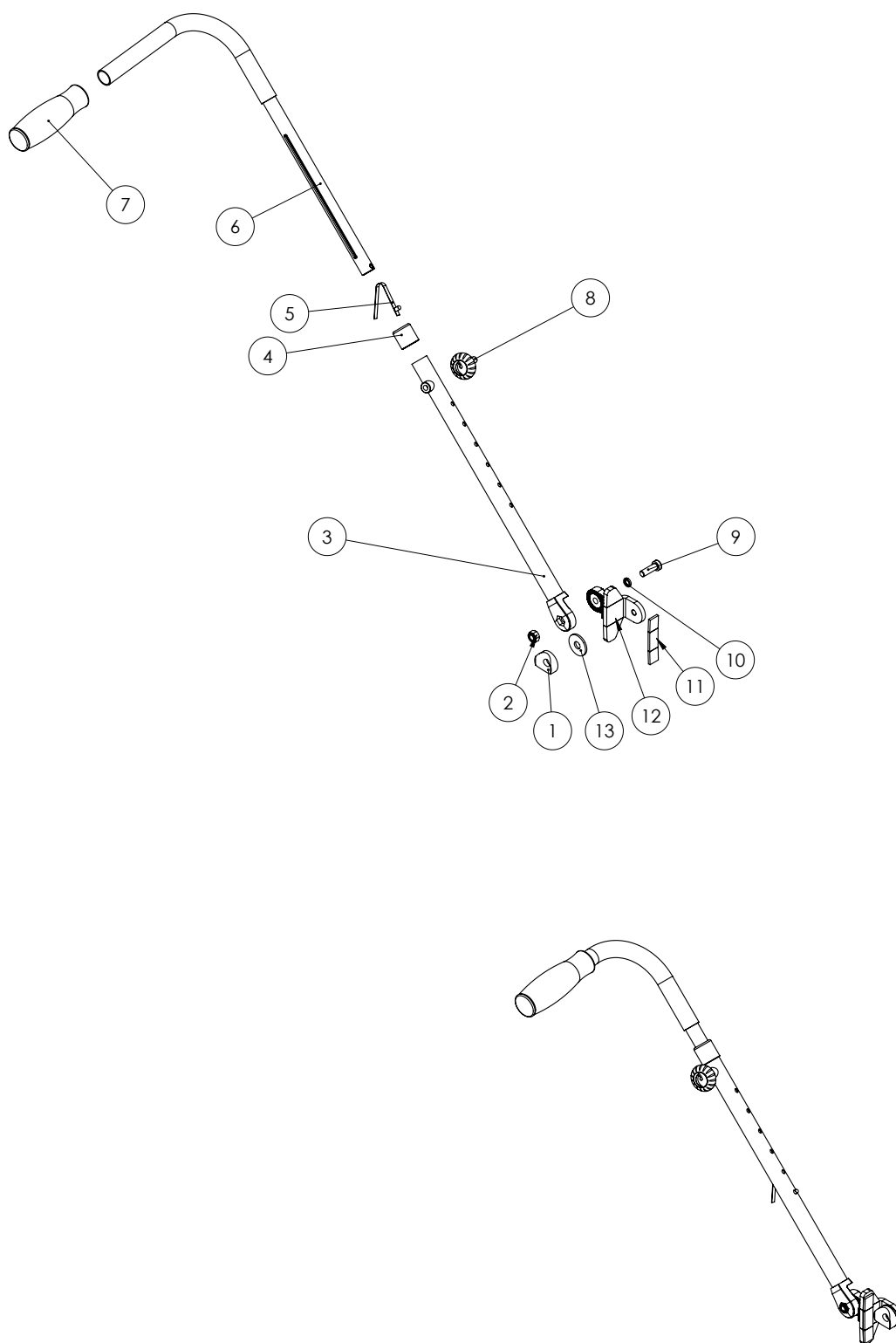


R82 nothing compares to a smile Title Push bar complete	Created by BOT	Revision 3	Revision by PER
	Created date 17-08-2016	Revision date 19-06-2017	Item no 122022

Position	No.	Description	Unit of Measure	Quantity	Ean code
1	82894	Saddle washer	PCS	1	5707292387578
2	1607000	Lock nut 8mm, large pack	PCS	1	5707292141378
3	122014	Ø22 tube with toothdisc	PCS	1	5707292762870
4	72215	Ferrule for Panther	PCS	1	5707292106667
5	6153500	Knob spring 0,5*6,5 mm.	PCS	1	5707292188571
6	109837	Hand brace, left	PCS	1	5707292583178
7	108371	rubber grip for handle	PCS	1	5707292583857
8	3503001	R82 handle M8x15mm screw	PCS	1	5707292413925
9	0603535	Insex low head M8*25	PCS	1	5707292575319
10	1100495	Spring washer M8	PCS	1	5707292458674
11	128718	Support rubber	PCS	1	5707292796769
12	128427	Bracket for push bar welded	PCS	1	5707292796752
13	1108050	Washer M10, din9021	PCS	1	5707292140401

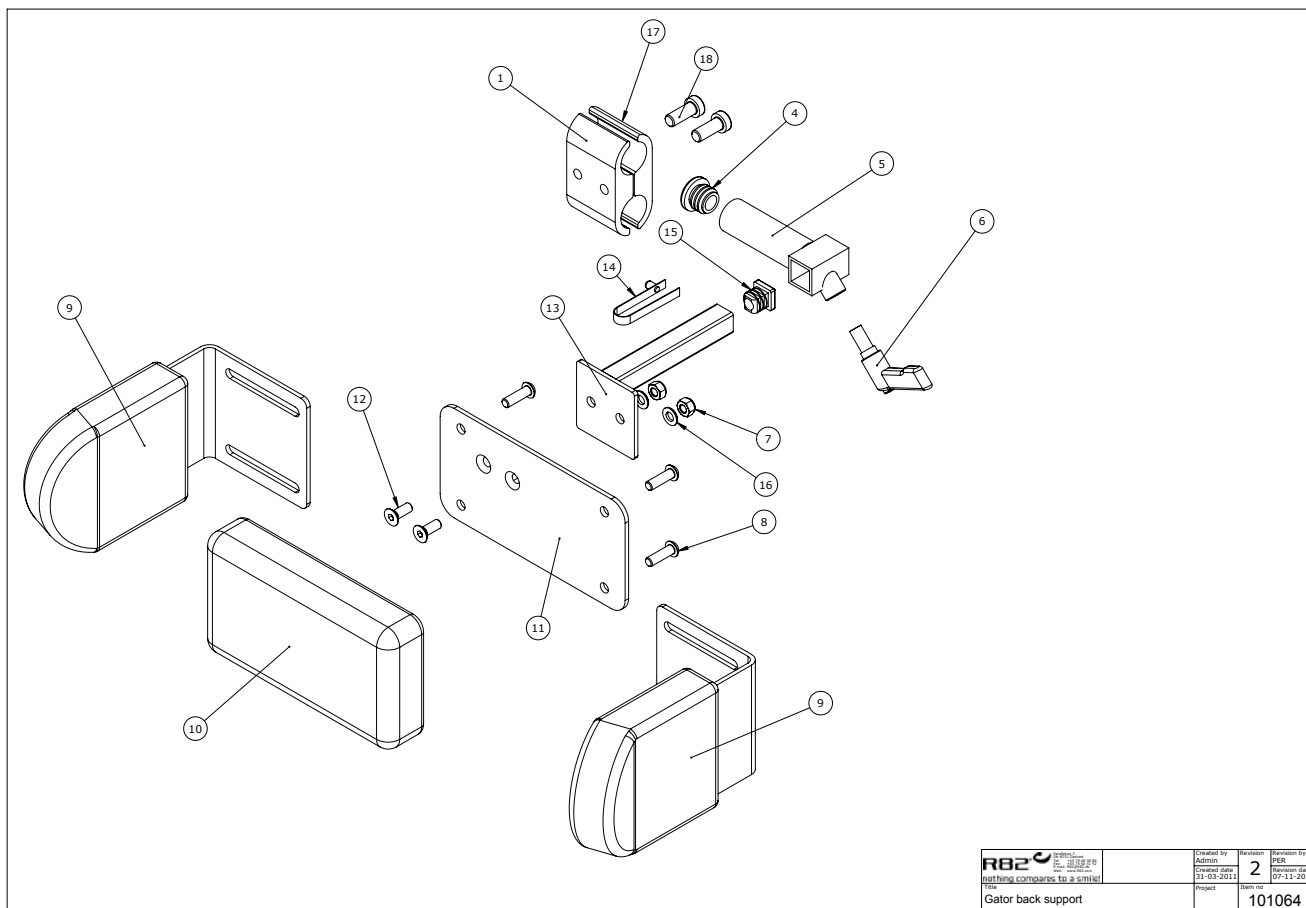


R82 <small>RESEARCH & DESIGN</small> <small>getting companies to a smile!</small>	<small>Created by</small> Admin <small>Created date</small> 31-05-2011 <small>Revision</small> 2 <small>Revised by</small> PER <small>Revision date</small> 07-11-2017	<small>Project</small> Gator back support <small>Item</small> 101064

Position	No.	Description	Unit of Measure	Quantity	Ean code
1	73487	Pipe jaw upper part	PCS	1	5707292355218
2	73488	Pipe jaw lower part	PCS	1	5707292355225
3	0602800	Insex screw 8*20	PCS	2	5707292138194
4	2503010	Ferrule d22*1,8-3,5, black	PCS	1	5707292357632
5	82653	Bracket for neck support, croc	PCS	1	5707292390431
6	3502516	Tightening handle 45mm M8*16	PCS	1	5707292146915
7	1606500	Lock nut 6 mm	PCS	2	5707292141361
8	0641620	Button head screw 6*20	PCS	4	5707292139276
9	SN4015	Lateral pads, large, set	SET	1	5707292219619
10	PU-R82-03	Small flat pelvic/trunk pad	PCS	1	5707292489517
11	111088	Mounting plate small	PCS	1	5707292586087
12	0621600	Insex screw counters. 6*16	PCS	2	5707292138965
13	111085	Mounting f. side support weld	PCS	1	5707292588944
14	6153500	Knob spring 0,5*6,5 mm.	PCS	1	5707292188571
15	2504000	Ferrule e15*1,0-2,0 int.black	PCS	1	5707292144102
16	1103000	Fazet washer 6mm	PCS	2	5707292140289
10	PU-R82-04	Large flat pelvic/trunk pad	PCS	1	5707292491398
11	111089	Mounting plate large	PCS	1	5707292586100

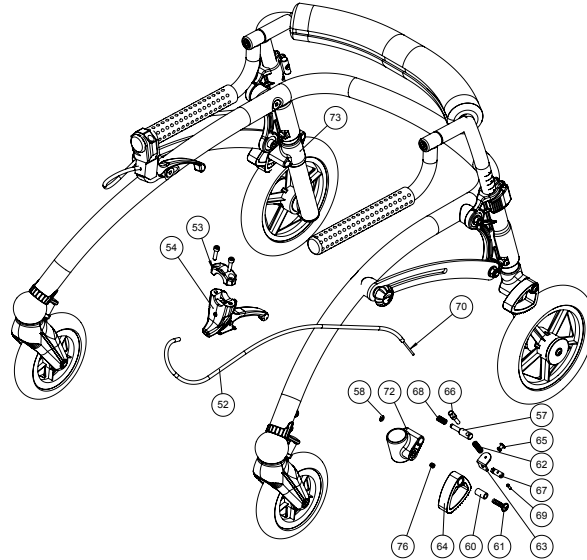

R82 <small>nothing compares to a smile!</small>	Created by per	Revision 3	Revision by jeso
	Created date 05-07-2004	Revision date 19-09-2014	
Project Crocodile with handbrake	Project	Item no. 79415	

Position	No.	Description	Unit of Measure	Quantity	Ean code
52	5002951	Wire f/tiger,panther Ø200+10cm	PCS	2	5707292154644
53	3503536	Clamp w/screws	PCS	2	5707292427762
54	3503535	Brake parking lever wo/clamp	PCS	2	5707292413536
57	73476	Release pin	PCS	2	5707292398055
58	1145040	Lock washer 4mm, stainless	PCS	2	5707292140562
60	73452	Bush o10/o6,2*18,1,crocodile	PCS	2	5707292354952
61	0659571	Button head w/flange M6*25 FZB	PCS	2	5707292355843
62	0310157	Stainl.spr.1,1x7insidex30x10v	PCS	2	5707292397782
63	73415	Mountings for cable	PCS	2	5707292398024
64	73474	Anti reverse item - machined	PCS	2	5707292398093
65	0506513	Stainless blind rivet Ø4, 8*12	PCS	2	5707292137326
66	0380003	Brake adjuster M6*24mm	PCS	2	5707292135964
67	0388006	Screwnipple 5*7,5mm	PCS	2	5707292136114
68	0310156	Stainless spring 0,5*8,2*25mm	PCS	2	5707292355911
70	0371853	Strained wires 1,5mm 190 cm.	PCS	2	5707292135889
71	2503010	Ferrule d22*1,8-3,5, black	PCS	2	5707292357632
72	73437	Retainer part left	PCS	1	5707292354730
73	73436	Retainer part right	PCS	1	5707292354723
75	1600862	Hsct-b-M6, screw-sert bush	PCS	2	5707292355928
76	0700050	Stopscrew M6*6 rf	PCS	2	5707292355898

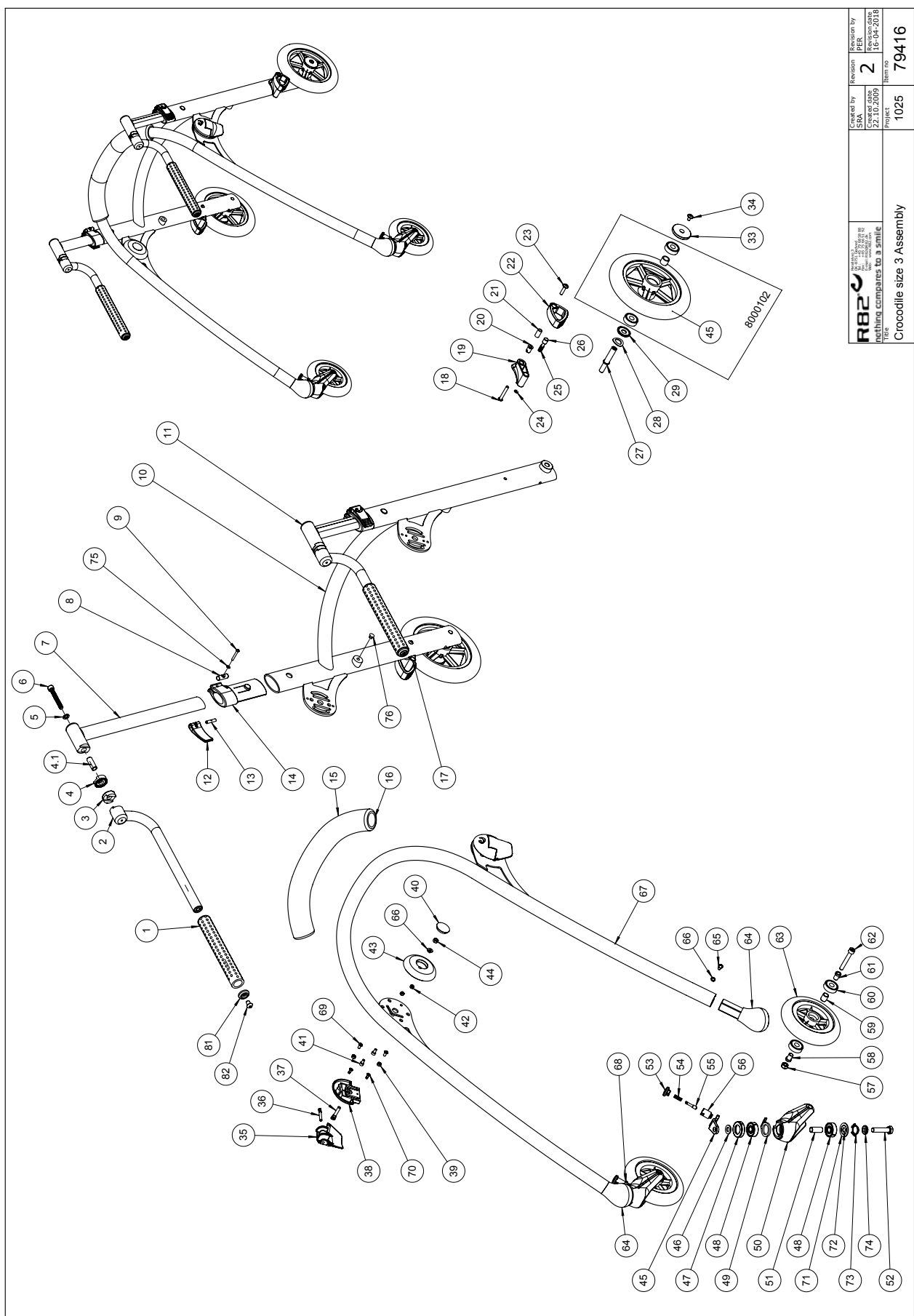


R82 <small>Nothing compares to a smile</small> Title Crocodile size 3 Assembly	Created by SJA	Revision 2	Revision by PER
	22.10.2009		16-04-2018
	Project 1025	Item no. 79416	


Position	No.	Description	Unit of Measure	Quantity	Ean code
1	2500520	TPE-Cover ø18 l=300	PCS	2	5707292585332
2	75478	Handle assembly 1	PCS	2	5707292549884
3	74061	Tooth washer Ø30	PCS	2	5707292373816
4	74061	Tooth washer Ø30	PCS	2	5707292373816
5	1101000	Center spring washer 8mm	PCS	2	5707292140173
6	0603950	Insex screw 8*100	PCS	2	5707292138323
7	75479	Handbrake assembly, right	PCS	1	5707292549907
8	73461	Transversebush ø10-M4,croco	PCS	2	5707292354976
9	0631950	Button head M4*30,stainless	PCS	2	5707292355836
10	75461	Rear leg assembly	PCS	1	5707292543790
11	75480	Handbrake assembly, left	PCS	1	5707292549914
12	75458	Flip handle	PCS	2	5707292549723
13	73460	Transversebush ø6 M4	PCS	2	5707292354921
14	75456	Mounting for handlebar	PCS	2	5707292549709
17	75471	Rear brake mount, right	PCS	1	5707292549822
18	0659590	Button head M6*40,stainless	PCS	2	5707292139504
19	75472	Rear brake mount, left	PCS	1	5707292549839
20	1600862	Hsct-b-M6, screw-sert bush	PCS	2	5707292355928
21	73452	Bush ø10/ø6,2*18,1,crocodile	PCS	2	5707292354952
22	73435	Anti-reverse item	PCS	2	5707292354716
23	0659571	Button head w/flange M6*25 FZB	PCS	2	5707292355843
24	1145040	Lock washer 4mm, stainless	PCS	2	5707292140562
25	0310156	Stainless spring 0,5*8,2*25mm	PCS	2	5707292355911
26	73451	Release pin, Crocodile	PCS	2	5707292354969
27	73470	Wheel axle, Crocodile	PCS	2	5707292392404
28	1101620	Spring washer M14 FZB	PCS	2	5707292355867
29	73455	Friction washer, Crocodile	PCS	2	5707292354938
30	8000100	Ballbearing Ø30mm, Crocodile	PCS	2	5707292397027
31	8000102	Rear wheel for Crocodile	PCS	2	5707292397041
32	8000100	Ballbearing Ø30mm, Crocodile	PCS	2	5707292397027
33	73456	Frictions nut, Crocodile	PCS	2	5707292354907
34	0659010	Button head with flange M6*10	PCS	2	5707292356819
35	75492	Flip handle	PCS	2	5707292549969
36	75497	Ø5x39mm Pin	PCS	2	5707292550040
37	75496	M6 eye-pin	PCS	2	5707292550033
38	75493	Flip handle, housing	PCS	2	5707292549976
39	1606300	Lock nut 5 mm	PCS	4	5707292141354
40	2985000	Pressureplug black soft rubber	PCS	2	5707292145314
41	0602007	Stainless insex M5*10	PCS	4	5707292542779
42	1606300	Lock nut 5 mm	PCS	4	5707292141354
43	104403	Tension disc for locking joint	PCS	2	5707292569516
44	1606500	Lock nut 6 mm	PCS	2	5707292141361
45	100899	Bracket f. direction lock,left	PCS	2	5707292552754
46	8000012	Washer Ø20x2mm	PCS	2	5707292397072
47	8000010	Cover for ballbearing	PCS	2	5707292396907
48	8000100	Ballbearing Ø30mm, Crocodile	PCS	2	5707292397027
49	8000002	Direction bracket	PCS	2	5707292396921
50	8000001	Front fork for Crocodile	PCS	2	5707292396938

51	8000009	Distance bush	PCS	2	5707292396990
52	1908000	Steel bolt M10*45 stainless	PCS	2	5707292355881
53	8000004	Handle for direction lock	PCS	2	5707292396945
54	8000007	Spring for direction lock	PCS	2	5707292396952
55	8000005	Pin for direction lock	PCS	2	5707292396969
56	8000003	House for direction lock	PCS	2	5707292396976
57	1607000	Lock nut 8mm, large pack	PCS	2	5707292141378
58	8000016	Bush for frontwheel	PCS	2	5707292397959
59	8000015	Innerbush for frontwheel	PCS	2	5707292397966
60	2985215	Ball bearing 6000-2rs	PCS	4	5707292420855
61	8000016	Bush for frontwheel	PCS	2	5707292397959
62	0603600	Insex screw 8*55	PCS	2	5707292138262
63	8000101	Front wheel for Crocodile	PCS	2	5707292397034
64	75450	Front wheel hub, right	PCS	2	5707292549686
65	0641600	Button head screw 6*16	PCS	4	5707292139269
66	1100800	Center spring washer 6mm	PCS	2	5707292140166
67	75463	Frame assembly	PCS	1	5707292549747
68	100898	Bracket f. direction lock, right	PCS	1	5707292552761
69	104462	M5x10 Special Screw	PCS	4	5707292569523
70	1940014	Plastite 4*14mm	PCS	4	5707292552785
71	74066	End washer 20*31,7*1,7	PCS	2	5707292378323
72	1103400	Fazet washer 10 mm	PCS	2	5707292140319
73	6108150	Spring screw 29,57*17,48	PCS	2	5707292378200
74	74069	Stop washer 10,2*19,9*3,7	PCS	2	5707292377418
75	1102800	Fazet washer 4mm	PCS	4	5707292140265
76	1310532	Cover plug, black o9,5 mm	PCS	4	5707292375797
81	109106	Stop disc	PCS	2	5707292575821
82	0661100	Button head screw 8*12	PCS	2	5707292139511
15+16	86883	Crocodile, padding, size 3	PCS	1	5707292550064


SPAREPART KITS					
15 16	86883	Crocodile, Upholstery and cover, size 3	PCS	1	5707292550064
1 2 3 4 5 6 81 82	86862	Handle for Crocodile size 3	PCS	1	5707292553812
8 9 12 13 75	86831-3U	Lock part for handles and cross bar	PCS	1	5707292560230
17 18 19 20 21 22 23 24 25 26	86828-3U	Crocodile 3 anti reverse	PCS	1	5707292616623
14 17 24 25 26 51 57 58 62 65 66 20	86837-H	Front fork for Crocodile right	PCS	1	5707292373205
14 17 24 25 26 51 57 58 62 63 66 20	86837-V	Front fork for Crocodile left	PCS	1	5707292373199
61	86838	Front wheel, Crocodile	PCS	1	5707292373663

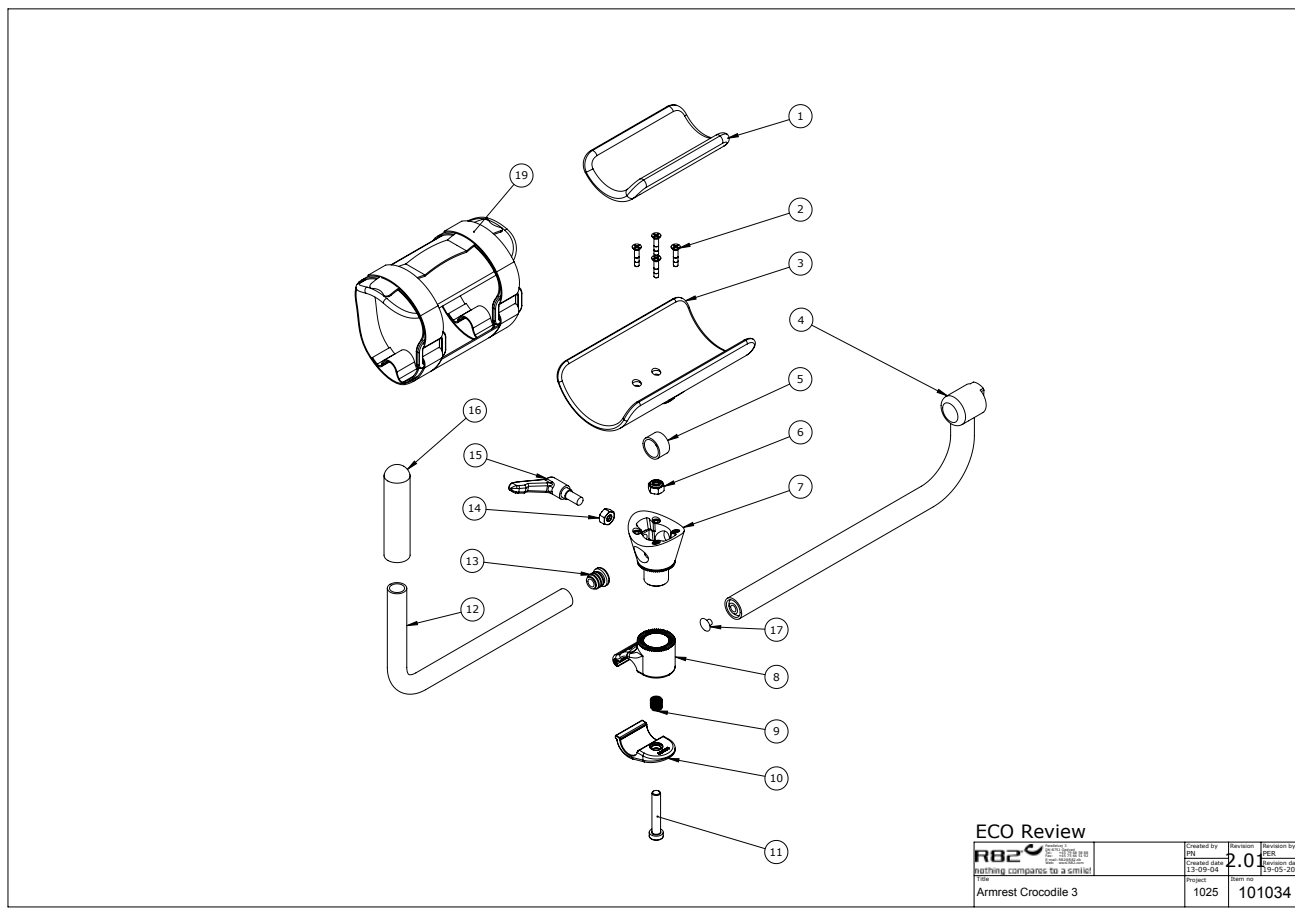


Position	No.	Description	Unit of Measure	Quantity	Ean code
1	5020920	Cover for forearm support sz 2	SÆT	1	5707292361165
2	1940016	Plate screw 4,2*19 FZB	PCS	8	5707292356796
3	73449	Arm rest plate sz 2,crocodile	PCS	2	5707292355133
4	75478	Handle assembly 1	PCS	2	5707292549884
5	128485	Steel bushing for armrest	PCS	2	5707292795304
6	1607000	Lock nut 8mm, large pack	PCS	2	5707292141378
7	73446	Retainer forearm support	PCS	2	5707292355102
8	102798	Armrest adjustment (top)	PCS	2	5707292564559
9	0310129	pressure spring 0,9*11*26mm*7v	PCS	2	5707292438782
10	102799	Armrest adjustment (bottom)	PCS	2	5707292564566
11	0663041	Button head M8*40, rf	PCS	2	5707292575388
12	73467	Tube for handle Crocodile	PCS	2	5707292356314
13	2502002	Ferrule d18*0,8-2,5 black	PCS	2	5707292433947
14	1750800	Lock nut, stainless M8	PCS	2	5707292141484
15	3502516	Tightening handle 45mm M8*16	PCS	2	5707292146915
16	2500516	Grip 87.00 mm length	PCS	2	5707292356727
17	1310528	Covering stud, black 13mm	PCS	2	5707292470348

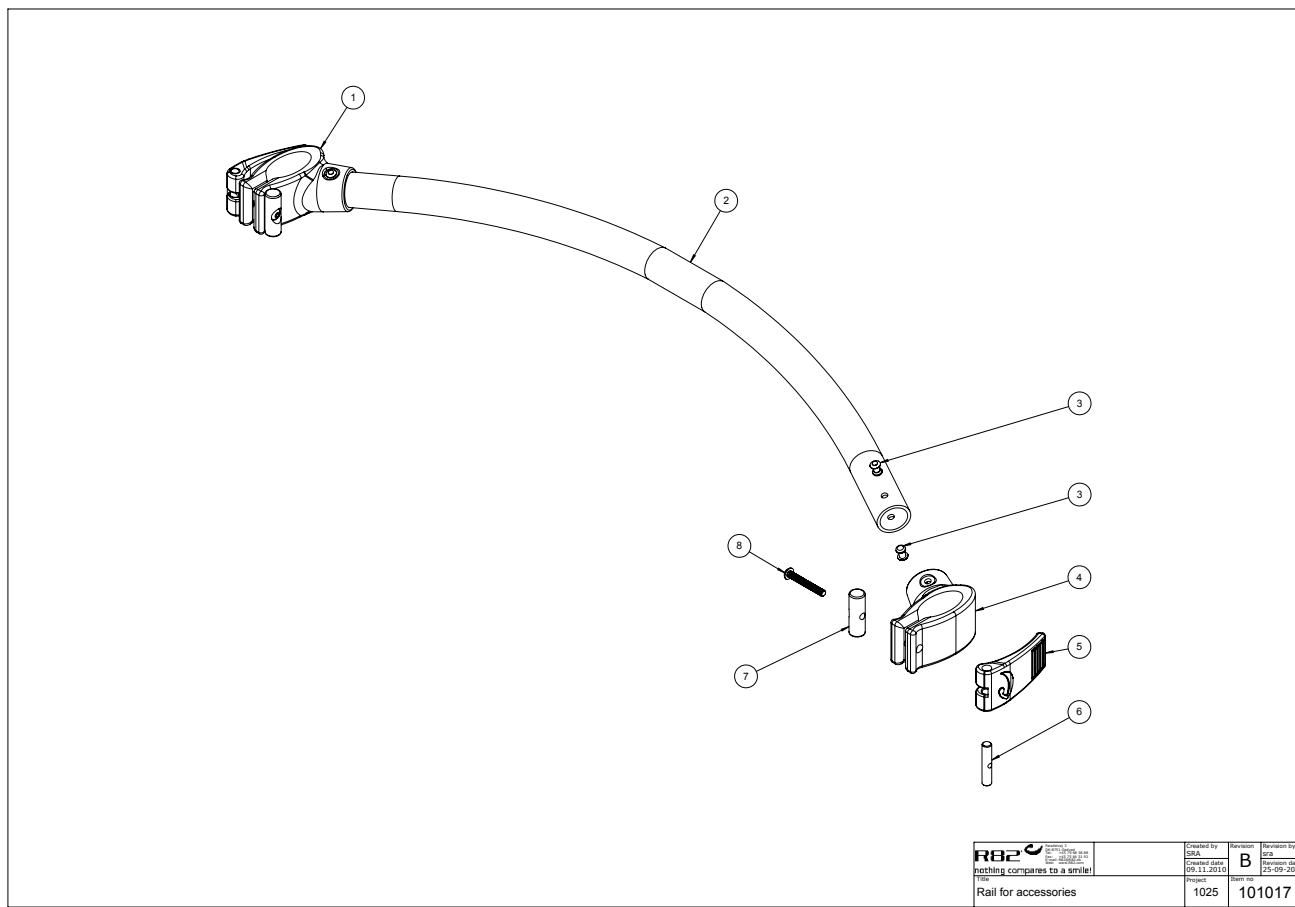


Position	No.	Description	Unit of Measure	Quantity	Ean code
1	75481	Mounting for brace, right	PCS	1	5707292549921
2	75483	Brace for Accessories	PCS	1	5707292543257
3	0506509	Rivet 4*10, stainless	PCS	2	5707292472441
4	75482	Mounting for brace, left	PCS	1	5707292549938
5	75458	Flip handle	PCS	2	5707292549723
6	73460	Transverse bush o6 M4	PCS	2	5707292354921
7	73461	Transverse bush o10-M4, croco	PCS	2	5707292354976
8	0631950	Button head M4*30, stainless	PCS	2	5707292355836

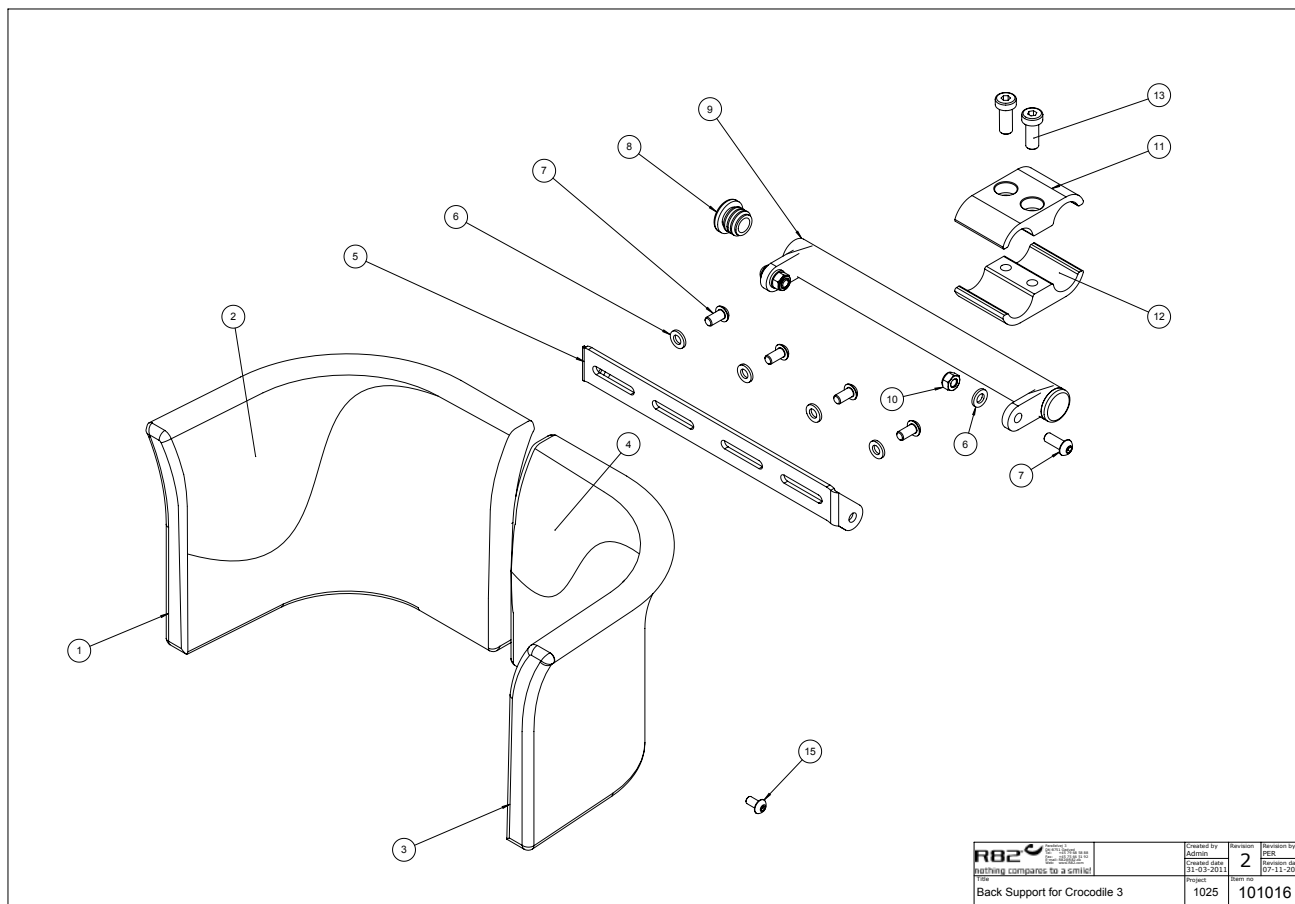


C:\SPIN\BOM\BOM\BOM\Crocodile size 3\201004

 Nothing compares to a smile!	Revision: 2 Rev A11 - 31-03-2011 Created by: 1025 (31-03-2011) Created date: 31-03-2011	Created by Admin	Revision 2	Revision by PER
		Created date 31-03-2011		Revision date 07-11-2011
	Title Back Support for Crocodile 3	Project 1025	Item no 101016	


Position	No.	Description	Unit of Measure	Quantity	Ean code
1	5700593	Upholst back support,right 3+4	PCS	1	5707292489173
3	5700592	Upholst back support,left 3+4	PCS	1	5707292489166
5	77544	Bracket f. Back adjust	PCS	1	5707292487674
6	1103000	Fazet washer 6mm	PCS	6	5707292140289
7	0641600	Button head screw 6*16	PCS	6	5707292139269
8	2502000	Ferrule d18*1,0-2,0 int.black	PCS	2	5707292144003
9	75510	Support bracket	PCS	1	5707292550057
10	1606500	Lock nut 6 mm	PCS	2	5707292141361
14	75306	Snap lock R82 25	PCS	2	5707292437624
15	0640911	Button head M5x10 FZB tuflok	PCS	2	5707292428479
2+4	85639-260	Back cover, nandu size 2, Blac	SET	1	5707292491220

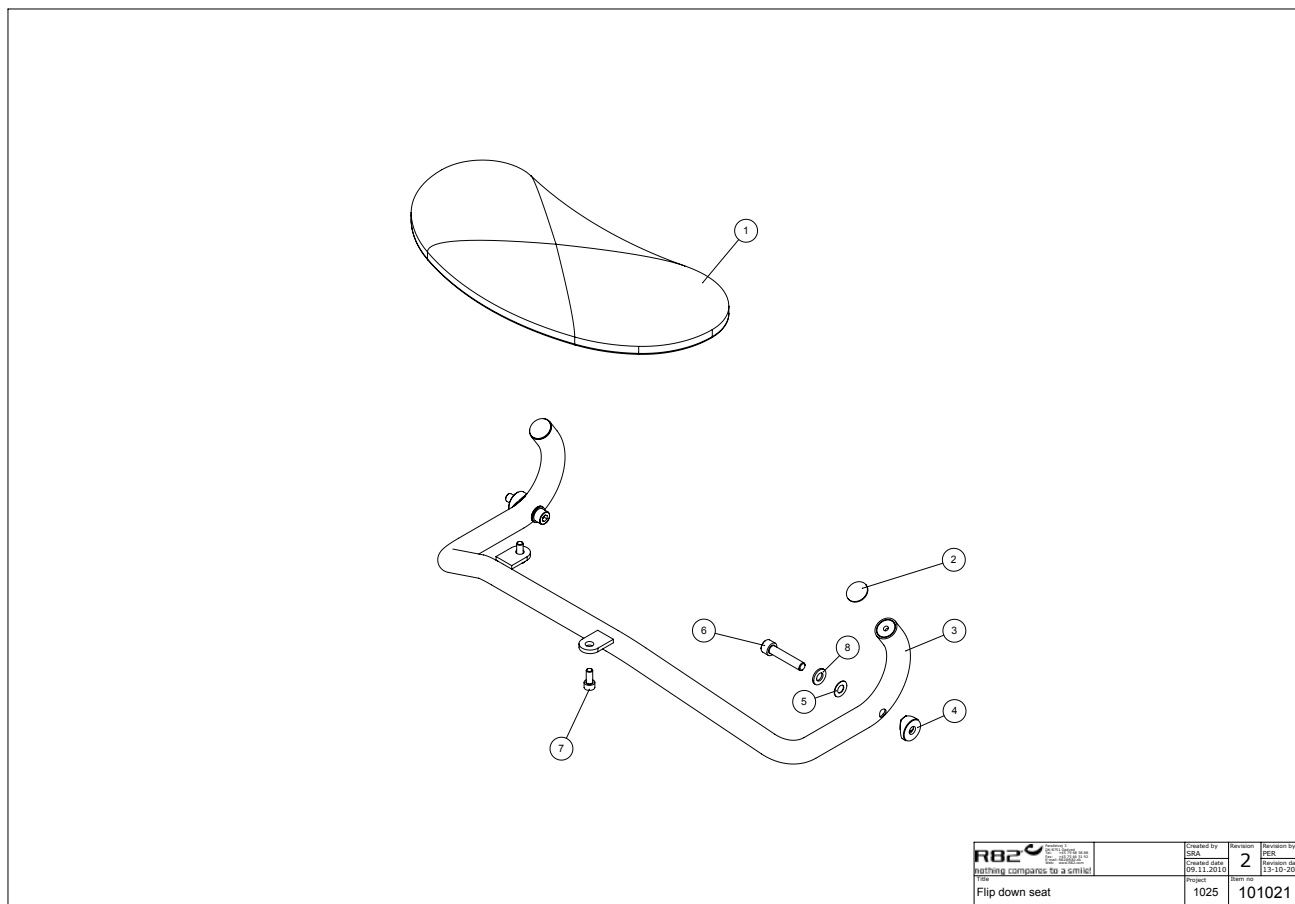


Position	No.	Description	Unit of Measure	Quantity	Ean code
1	5020901	Polst.sæde str. 3,crocodile	PCS	1	5707292553980
2	102428	3M round Sticker	PCS	2	5707292561053
3	75486	Seat frame	PCS	1	5707292549945
4	71110-60	Black saddle washer M8	PCS	2	5707292574268
5	1310601	Nylon washer din 125, m8	PCS	2	5707292557261
6	0663040	Insex M8*40, rf, dryloc	PCS	2	5707292573377
7	0659200	Button head screw 6*12 stainl.	PCS	2	5707292139436
8	1105080	Stainless washer 8mm	PCS	2	5707292140364

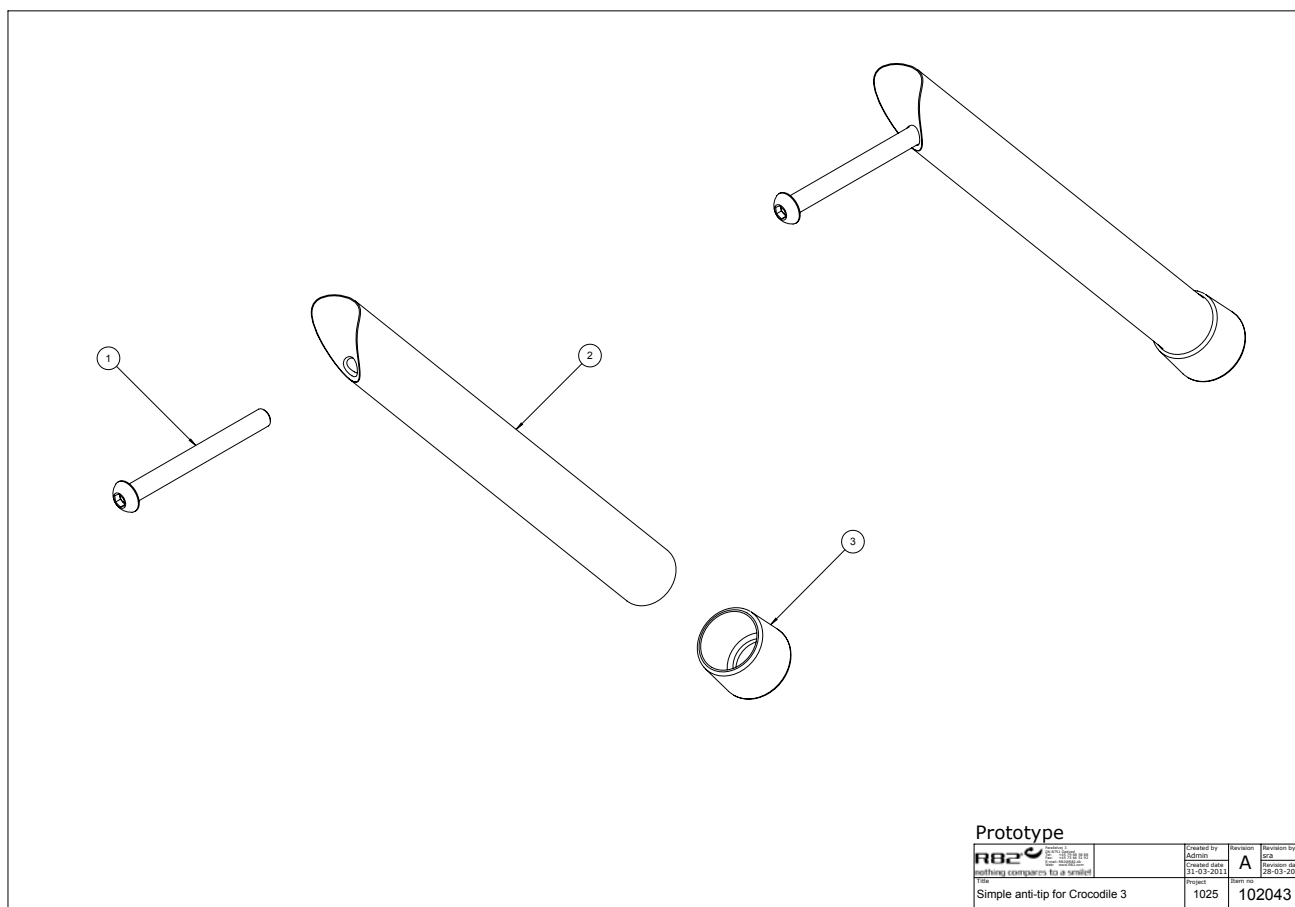

Position	No.	Description	Unit of Measure	Quantity	Ean code
1	0602465	Insex M6*65, stainless	PCS	2	5707292564665
2	102044	Anti-tip arm size 3	PCS	2	5707292564061
3	2508250	Wear 20 outside	PCS	2	5707292564405

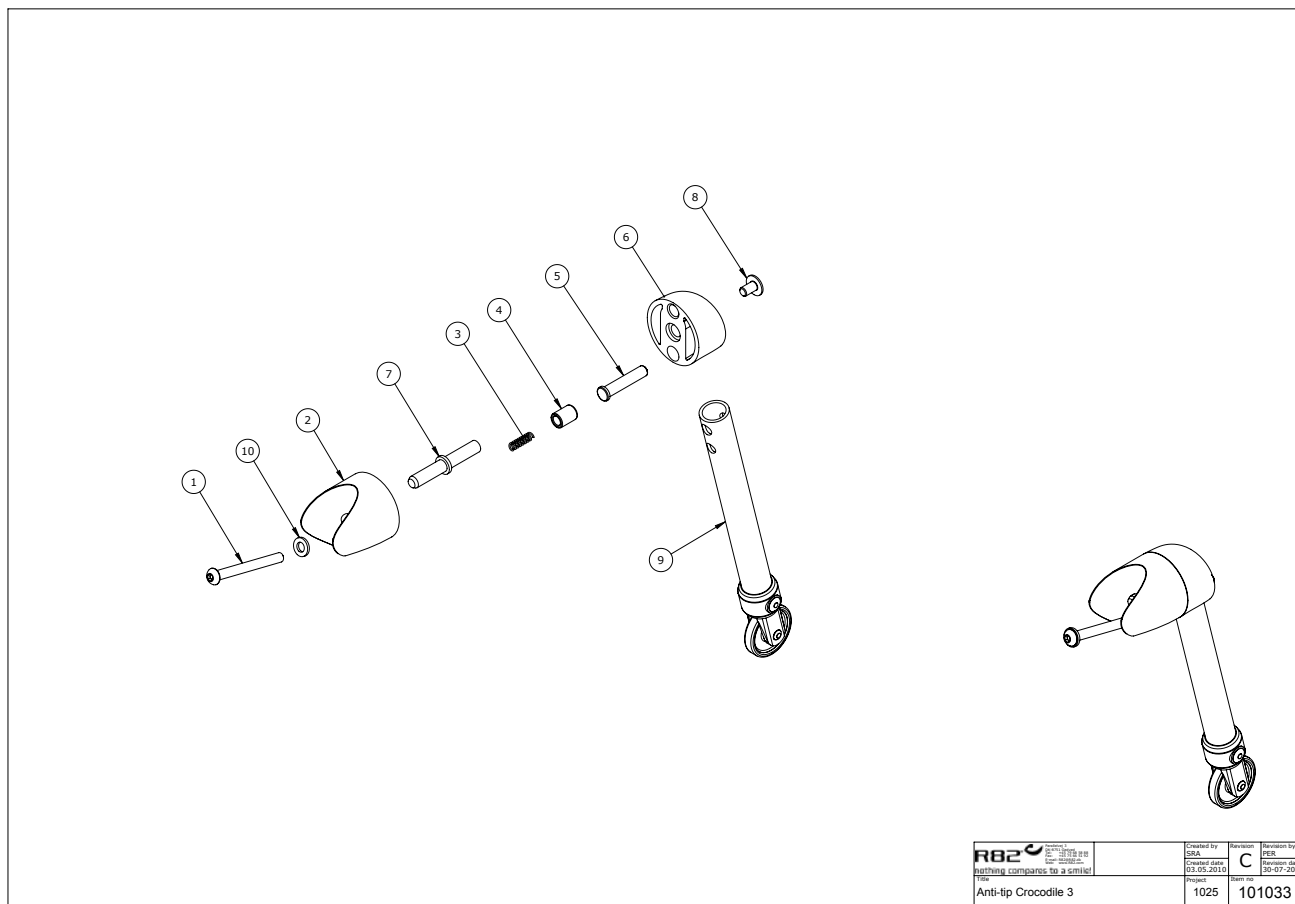

Position	No.	Description	Unit of Measure	Quantity	Ean code
1	0602465	Insex M6*65, stainless	PCS	2	5707292564665
2	75457	Antitip adapter piece	PCS	2	5707292549716
3	0309175	Comp. Spring 0,8*5,8*17,5mm*10	PCS	2	5707292135490
4	82555	lower pin	PCS	2	5707292124722
5	73458	Release button, Crocodile	PCS	2	5707292355263
6	73439	Swivel joint removable	PCS	2	5707292355089
7	73459	Staybolt for kip security	PCS	2	5707292355287
8	0659010	Button head with flange M6*10	PCS	2	5707292356819
9	5086800	Extens tube w wheel, antitipper	PCS	2	5707292416179
10	1105060	Stainless washer 6mm	PCS	2	5707292140357

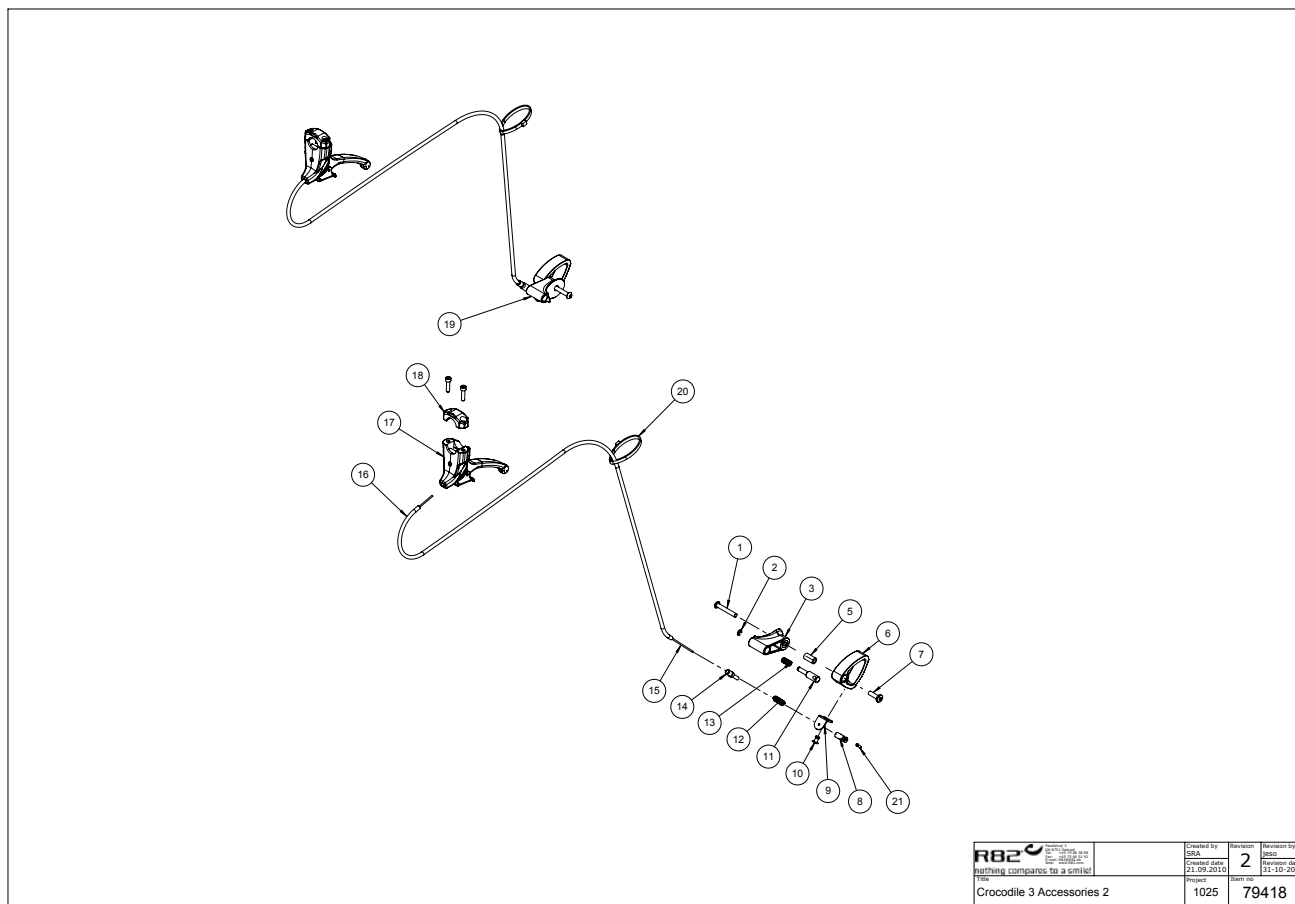

R82 nothing compares to a smile!	Revision: 2 Created by: SRA Created date: 21.09.2010 Revision date: 31-10-2016	Revision: 2 Revision date: 31-10-2016
	Title: Crocodile 3 Accessories 2	Item no: 79418

Position	No.	Description	Unit of Measure	Quantity	Ean code
1	0659590	Button head M6*40, stainless	PCS	2	5707292139504
2	1145040	Lock washer 4mm, stainless	PCS	2	5707292140562
3	102817-L	Brake mount assembly left	PCS	1	5707292563392
5	73452	Bush o10/o6,2*18,1, crocodile	PCS	2	5707292354952
6	73474	Anti reverse item - machined	PCS	2	5707292398093
7	0659571	Button head w/flange M6*25 FZB	PCS	2	5707292355843
8	0388006	Screwnipple 5*7,5mm	PCS	2	5707292136114
9	73415	Mountings for cable	PCS	2	5707292398024
10	0506513	Stainless blind rivet Ø4, 8*12	PCS	2	5707292137326
11	73476	Release pin	PCS	2	5707292398055
12	0308713	Stainless spring 1,0*6*40mm*17	PCS	2	5707292502032
13	0310156	Stainless spring 0,5*8,2*25mm	PCS	2	5707292355911
14	0380003	Brake adjuster M6*24mm	PCS	2	5707292135964
15	0371853	Strained wires 1,5mm 190 cm.	PCS	2	5707292135889
16	5002951	Wire f/tiger, panther Ø200+10cm	PCS	2	5707292154644
17	3503535	Brake parking lever wo/clamp	PCS	2	5707292413536
18	3503536	Clamp w/screws	PCS	2	5707292427762
19	102817-R	Brake mount assembly right	PCS	1	5707292563408
20	0235075	Cable tape, black	PCS	2	5707292135308
21	0387000	End nipple for outer casing	PCS	2	5707292136091