

Flamingo high-low

Exploded views / Part lists


Rev. date (09.2018)

On the following pages you will find the parts used to assemble this product. Please note that some spare parts cannot be purchased individually but are part of a complete kit. Please also note that for some spare parts extra delivery time may occur.




Contents

Seat	3
Frame	5
Arm support, back sides, pan with holder, suction cups.....	8
Separate armrests.....	11
Foot support.....	13

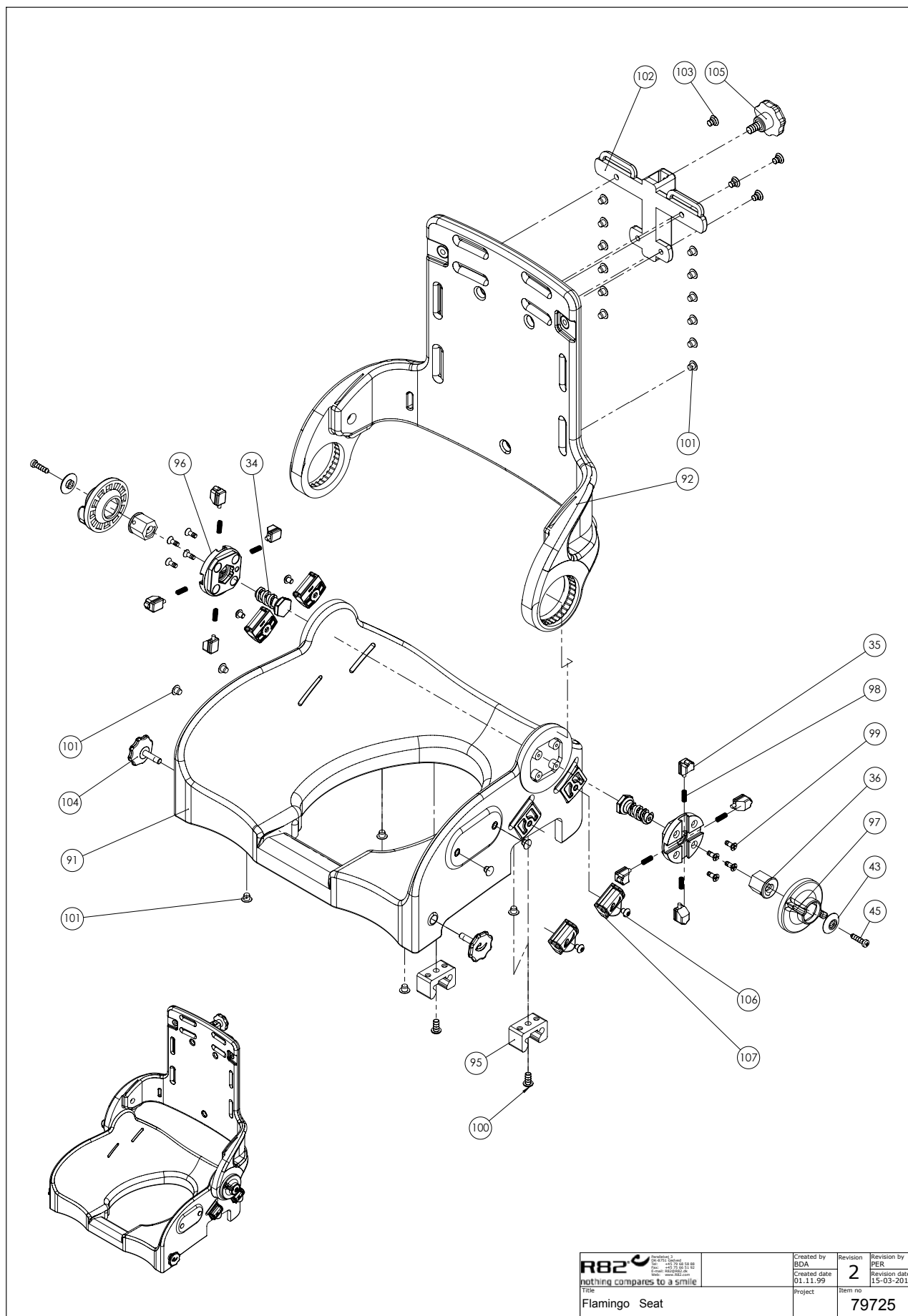

R82 <small>nothing compares to a smile</small> R82 A/S Parallelvej 3 8751 Gedved Denmark Tel: +45 79 68 58 88 Fax: +45 75 66 51 92 E-mail: R82@R82.dk Web: R82.org	Created by BDA Created date 01.11.99 Title Flamingo Seat	Revision 2	Revision by PER Revision date 15-03-2017
		Project	Item no. 79725

Position	No.	Description	Quantity	Unit of Measure	Ean code
1	110200	Snap Lock R82-25 mobile,W ø6,2	PCS	4	5707292579508
2	110201	Snap Lock R82-25 mobile, white	PCS	4	5707292579492
34	71136	Thread spindle, Flamingo	PCS	2	5707292104410
35	71133	T-flange, Flamingo	PCS	8	5707292104380
36	71128	Thread nut for angle adj.	PCS	2	5707292104359
43	71164	Knob Ø23, 5 for handle	PCS	2	5707292104540
45	1811112	Sheet metal screw 4,8*19 phx	PCS	2	5707292452184
91	71302	Flamingo Seat shell, size 1	PCS	1	5707292104984
91	71305-1	Flamingo sæde 4	PCS	1	5707292629432
91	71304	Flamingo Seat shell, size 3	PCS	1	5707292105004
91	71303	Flamingo Seat shell, size 2	PCS	1	5707292104991
92	71312	Flamingo back shell, size 1	PCS	1	5707292105028
92	71315	Flamingo back shell, size 4	PCS	1	5707292105059
92	71314	FLAMINGO BACK SHELL, SIZE 3	PCS	1	5707292105042
92	71313	Flamingo back shell, size 2	PCS	1	5707292105035
95	102429	Seatlock for Flamingo	PCS	2	5707292562036
96	71146	T-flange, Flamingo	PCS	2	5707292104434
97	71144	HAND WHEEL, FLAMINGO	PCS	2	5707292104427
98	0310124	Rf.spring 0.5*5.2*11mm	PCS	8	5707292135568
99	1940038	Eco-syn uh Ø4*14	PCS	8	5707292465597
100	0659200	Button head screw 6*12 stainl.	PCS	2	5707292139436
101	1310536	Covering button,white 13*5,0mm	PCS	16	5707292397911
101	1310536	Covering button,white 13*5,0mm	PCS	20	5707292397911
102	81138-90	Mounting for neck support	PCS	1	5707292227157
103	0609727	Stainless insex 6x12 uh	PCS	4	5707292138484
104	112708	Knob for Seat m6x23	PCS	2	5707292604286
105	3501050	Star grap Ø40 M8*15mm white	PCS	1	5707292146717
106	0631952	Button head M5*6, rf tuflok	PCS	4	5707292503725
107	110202	Snap Lock R82-25 With ø6	PCS	4	5707292575913

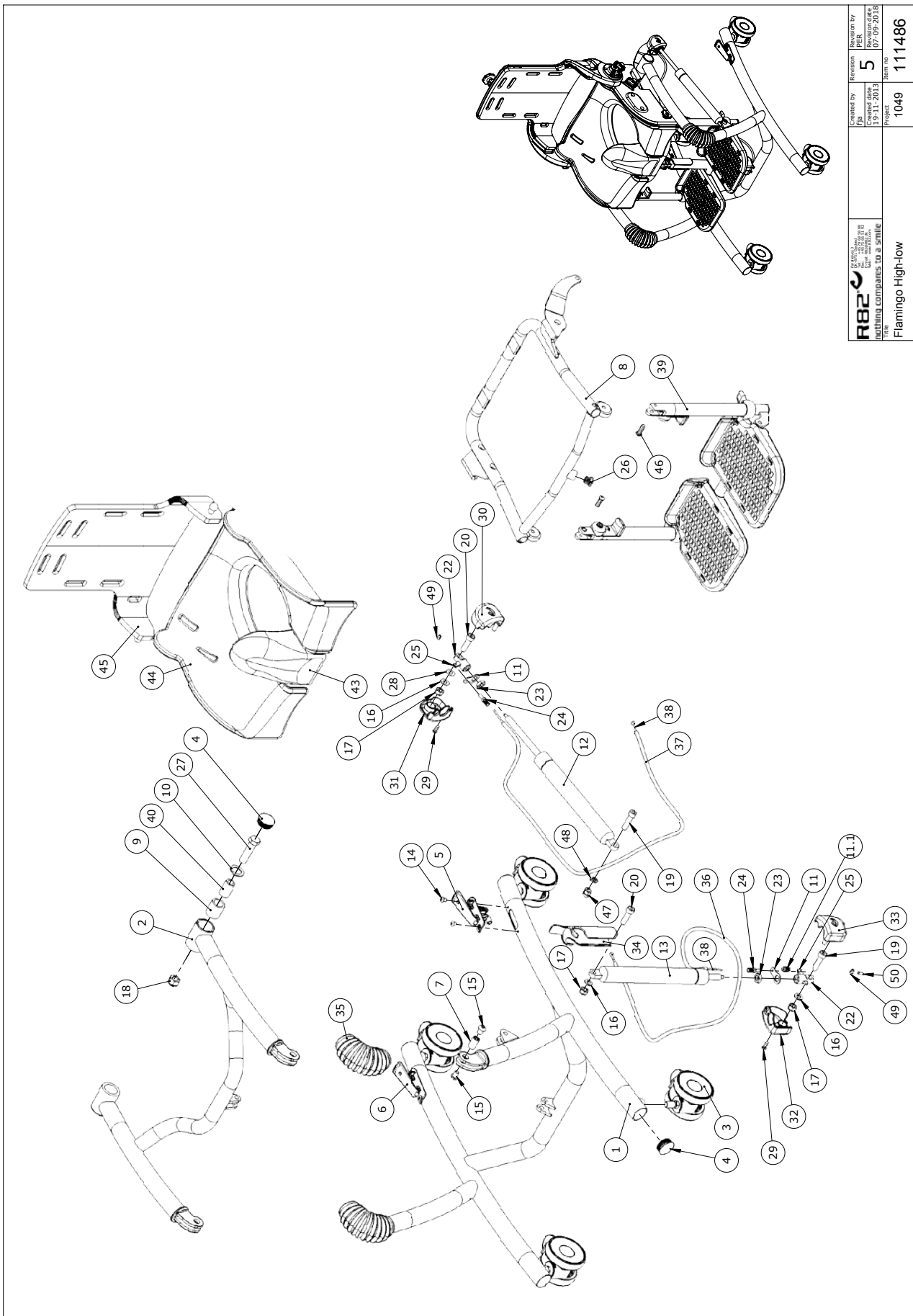

R82 <small>Flamingo</small> <small>Flamingo COMPARTMENTS TO A SEAT</small>	<small>Created by</small> <small>Project</small>	<small>Revision</small> 5	<small>Revision date</small> 17/09/2018	<small>Item no.</small> 111486

Position	No.	Description	Quantity	Unit of Measure	Ean code
1	111383-10	Frame bottom, size 3+4	PCS	1	5707292601605
1	111382-10	Frame bottom, size 1+2	PCS	1	5707292601599
2	111464-10	Frame top, size 3+4	PCS	1	5707292602510
2	111463-10	Frame top, size 1+2	PCS	1	5707292602527
3	3006830	75MM WHITE STEINCO WHEEL	PCS	4	5707292368560
4	2507006	COVERSHOE FOR PIPE GRAY Ø32	PCS	6	5707292580917
5	111869	Release pedal complete f. tilt	PCS	1	5707292604200
6	114288	Release pedal komplet for lift	PCS	1	5707292604477
7	100236	Bush ø12 x M8 x 22,5	PCS	2	5707292559746
8	112413-10	Seatframe complete size 4	PCS	1	5707292602817
8	112412-10	Seatframe complete size 3	PCS	1	5707292602800
8	112411-10	Seatframe complete size 2	PCS	1	5707292602794
8	112410-10	Seatframe complete size 1	PCS	1	5707292602787
9	100234	Bearing ø25x18x28	PCS	2	5707292560001
10	100233	Disk ø27x18,2x2	PCS	2	5707292560025
11	82268	Wire mounting by pedal	PCS	2	5707292123039
11.1	0310100	Pressure spring 0,9*8*23*10,5v	PCS	1	5707292135513
12	2206510	Lockable Gas traction spring	PCS	1	5707292603838
12	2206500	Lockable Gas Traction spring	PCS	1	5707292603821
13	2206160	Bloc-o-lift 833175, 800N	PCS	1	5707292603852
13	2206150	Bloc-o-lift 832697, 600N	PCS	1	5707292603845
14	0631952	Button head M5*6, rf tufflok	PCS	4	5707292503725
15	0661153	Button head screw 8*10 stainl.	PCS	4	5707292616654
16	1105080	Stainless washer 8mm	PCS	3	5707292140364
17	1631080	Stainless lock nut 8mm	PCS	3	5707292141453
18	1631120	Stainless lock nut 12mm	PCS	2	5707292502049
19	0609832	Stainless insex screw 8*30	PCS	2	5707292606037
20	0609832	Stainless insex screw 8*30	PCS	2	5707292606037
22	138241	House for gasspring	PCS	2	5707292839923
23	1761005	Counter nut 10*0.1	PCS	2	5707292509925
24	0380000	Cable tighter M5*23	PCS	2	5707292135933
25	2200050	Small release arm	PCS	2	5707292143075
26	2501600	Ferrule d15*1,0 black	PCS	1	5707292143969
27	0607999	Stainless Insex screw 12*50	PCS	2	5707292482013
28	1310645	Washer, nature Ø16*8*0,5mm	PCS	1	5707292446015
29	1810050	Plate screw 7981c st 2,9*13	PCS	2	5707292404671
30	113085	Cover, outside	PCS	1	5707292604279
31	113086	Cover, inside	PCS	1	5707292604262
32	113099	Cover, right	PCS	1	5707292604255
33	113098	Cover, left	PCS	1	5707292604248
34	113080	Chock absorber	PCS	1	5707292604293
35	112354	Bellow for frame/adjust. arm	PCS	2	5707292602039
36	113783	Outher casing for lift, 3-4	PCS	1	5707292604347
36	113782	Outher casing for lift, 1-2	PCS	1	5707292604323
37	113785	Outher casing for tilt, 3-4	PCS	1	5707292604354
37	113784	Outher Casing for tilt 1-2	PCS	1	5707292604316
38	0371854-1	Soft inner cable for Stingray	PCS	2	5707292536945
39	116969-4	Flamingo fodstøtte,kompl.str4	PCS	1	5707292609588
39	116969-3	Flamingo fodstøtte,kompl.str3	PCS	1	5707292609571

39	116969-1	Flamingo fodstøtte,kompl.str1	PCS	1	5707292609557
39	71050	Flamingo footrest size 1, H/L	PCS	1	5707292605146
39	116969-2	Flamingo fodstøtte,kompl.str2	PCS	1	5707292609564
40	100229	Bush	PCS	2	5707292560018
40	100229	Bush	PCS	1	5707292560018
43	880192	Splashguard grey PU, high, size 2-4	PCS	1	5707292134547
43	880191	Splashguard grey PU, high, size 1	PCS	1	5707292134530
43	880191-61	Splashguard, high blue PU, size 1	PCS	1	5707292615978
43	880192-61	Splashguard, high blue PU, size 2-4	PCS	1	5707292615985
44	880164	Upholstery grey PU for Seat size 4	PCS	1	5707292134448
44	880163	Upholstery grey PU Flamingo Seat sz.3	PCS	1	5707292134431
44	880162	Upholstery grey PU Flamingo Seat sz.2	PCS	1	5707292134424
44	880161	Upholstery grey PU Flamingo Seat sz.1	PCS	1	5707292134417
44	880161-61	Upholstery blue PU Flamingo seat sz.1	PCS	1	5707292615787
44	880162-61	Upholstery blue PU Flamingo seat sz.2	PCS	1	5707292615794
44	880163-61	Upholstery blue PU Flamingo seat sz.3	PCS	1	5707292615800
44	880164-61	Upholstery blue PU Flamingo seat sz.4	PCS	1	5707292615817
45	880174	Upholstery grey PU sz.4,back,flamingo	PCS	1	5707292134486
45	880173	Upholstery grey PU Flamingo back sz.3	PCS	1	5707292134479
45	880172	Upholstery grey PU Flamingo back sz.2	PCS	1	5707292134462
45	880171	Upholstery grey PU Flamingo back sz.1	PCS	1	5707292134455
45	880171-61	Upholstery blue PU Flamingo back sz.1	PCS	1	5707292615855
45	880172-61	Upholstery blue PU Flamingo back sz.2	PCS	1	5707292615862
45	880173-61	Upholstery blue PU Flamingo back sz.3	PCS	1	5707292615879
45	880174-61	Upholstery blue PU Flamingo back sz.4	PCS	1	5707292615886
46	0659555	Button head M6*20, A2 w/ ES-Lo	PCS	2	5707292612120
47	1781080	Stainless top nut 8mm	PCS	1	5707292141576
48	1100500	Spring washer 8mm, stainless	PCS	1	5707292140142
49	0388006	Screwnipple 5*7,5mm	PCS	2	5707292136114
50	0387000	End nipple for outer casing	PCS	1	5707292136091




Position	No.	Description	Quantity	Unit of Measure	Ean code
129	880184-61H	Side upholstery, sz.4, right, blue PU	PCS	1	5707292615954
129	880182-61H	Side upholstery, sz.2, right, blue PU	PCS	1	5707292615916
129	880182-H	Upholst. F/right side, flam. 2	PCS	1	5707292254481
129	880181-61V	Side upholstery, sz.1, left, blue PU	PCS	1	5707292615909
129	880181-H	Upholst. F/right side,flam 1, grey PU	PCS	1	5707292254467
129	880184-H	Upholst. F/right side,flam. 4, grey PU	PCS	1	5707292254528
129	880183-61H	Side upholstery, sz.3, right, blue PU	PCS	1	5707292615930
129	880183-H	Upholst. F/right side,flam. 3, grey PU	PCS	1	5707292254504
130	880184-61V	Side upholstery, sz.4, left, blue PU	PCS	1	5707292615961
130	880182-61V	Side upholstery, sz.2, left, blue PU	PCS	1	5707292615923
130	880182-V	Upholst. F/left side, flam. 2	PCS	1	5707292254498
130	880181-61H	Side upholstery, sz.1, right, blue PU	PCS	1	5707292615893
130	880181-V	Upholst.f/left side flam. Sz.1, grey PU	PCS	1	5707292254474
130	880184-V	Upholst f/left side,flam. 4, grey PU	PCS	1	5707292254535
130	880183-61V	Side upholstery, sz.3, left, blue PU	PCS	1	5707292615947
130	880183-V	Upholst. F/left side, flam. 3, grey PU	PCS	1	5707292254511
139	71325	Backside size 4,right-Flamingo	PCS	1	5707292105097
139	71323	Backside size 2,right-Flamingo	PCS	1	5707292105073
139	71322	Backside size 1,right-Flamingo	PCS	1	5707292105066
139	71324	Backside size 3,right-Flamingo	PCS	1	5707292105080
140	71335	Backside size 4, left-Flamingo	PCS	1	5707292105134
140	71333	Backside size 2, left-Flamingo	PCS	1	5707292105110
140	71332	Backside size 1, left-Flamingo	PCS	1	5707292105103
140	71334	Backside size 3, left,flamingo	PCS	1	5707292105127
141	0609727	Stainless insex 6x12 uh	PCS	2	5707292138484
160	880129	Comode pan, Flamingo size 3+4	PCS	1	5707292134318
160	880128	Comode pan, Flamingo size 1+2	PCS	1	5707292134301
161	71344	Commode pan holder, left	PCS	1	5707292105202
161	71343	Commode pan holder, right	PCS	1	5707292105196
161	109114	Pottebøjle str.2, venstre	PCS	1	5707292576873
161	71338	Commode pan holder, left	PCS	1	5707292105141
162	71345	Commode pan holder, right	PCS	1	5707292105219
162	71342	Commode pan holder, left	PCS	1	5707292105189
162	109113	Pottebøjle str.2, Højre	PCS	1	5707292576866
162	71339	Commode pan holder, right	PCS	1	5707292105158
163	0659200	Button head screw 6*12 stainl.	PCS	4	5707292139436
164	71134	Distancepart for suction cup	PCS	4	5707292104397
165	2522015	Suction cup Ø80mm M6*12mm	PCS	4	5707292144461
166	1105060	Stainless washer 6mm	PCS	4	5707292140357
170	71371	Support rod STR.1 VENSTRE	PCS	1	5707292105394
170	71369	Support rod size 4, left	PCS	1	5707292436757
170	71367	Support rod STR.3 VENSTRE	PCS	1	5707292436771
170	71373	Support rod for size 2	PCS	1	5707292105417
171	71370	Support rod STR.1 HØJRE	PCS	1	5707292105387
171	71368	Support rod size 4, right	PCS	1	5707292436740
171	71366	Support rod STR. 3 HØJRE	PCS	1	5707292436764
171	71372	Support rod STR.2 HØJRE	PCS	1	5707292105400
172	6175361	Armrest table small, left	PCS	1	5707292189707
172	6175365	Armrest table large, left	PCS	1	5707292189721

173	6175359	Armrest table small, right	PCS	1	5707292189691
173	6175363	Armrest table large, right	PCS	1	5707292189714
174	71130-90	Bushing, white	PCS	2	5707292223593
175	71132	Locking socket,white	PCS	1	5707292104373
176	104377	Washer Ø15,5Ø5,3, h10,0	PCS	4	5707292568878
177	72134-90	Mounting for armrest, white	PCS	2	5707292223760
178	82408-90	Underlying plate armrest flam	PCS	2	5707292231482
179	0506519	Stainless blind rivet 4*12	PCS	2	5707292137388
180	6153700	Knob spring f.activity centre	PCS	2	5707292188601
181	1310552	Cover plugs k 5 natur	PCS	2	5707292581747
182	0603522	Insex ch M5*22 rf	PCS	4	5707292490988


R82 <small>nothing compares to a smile</small> Title	R82 A/S Parallelvej 3 8751 Gedved Denmark	Checked by Admin Created date 31-03-2011	Revision by PER Revision date 28-01-2016	Item no. 104369
		Project		

Armrest for Flamingo

C:\PROMOTION\Vispove\flamingo\104369

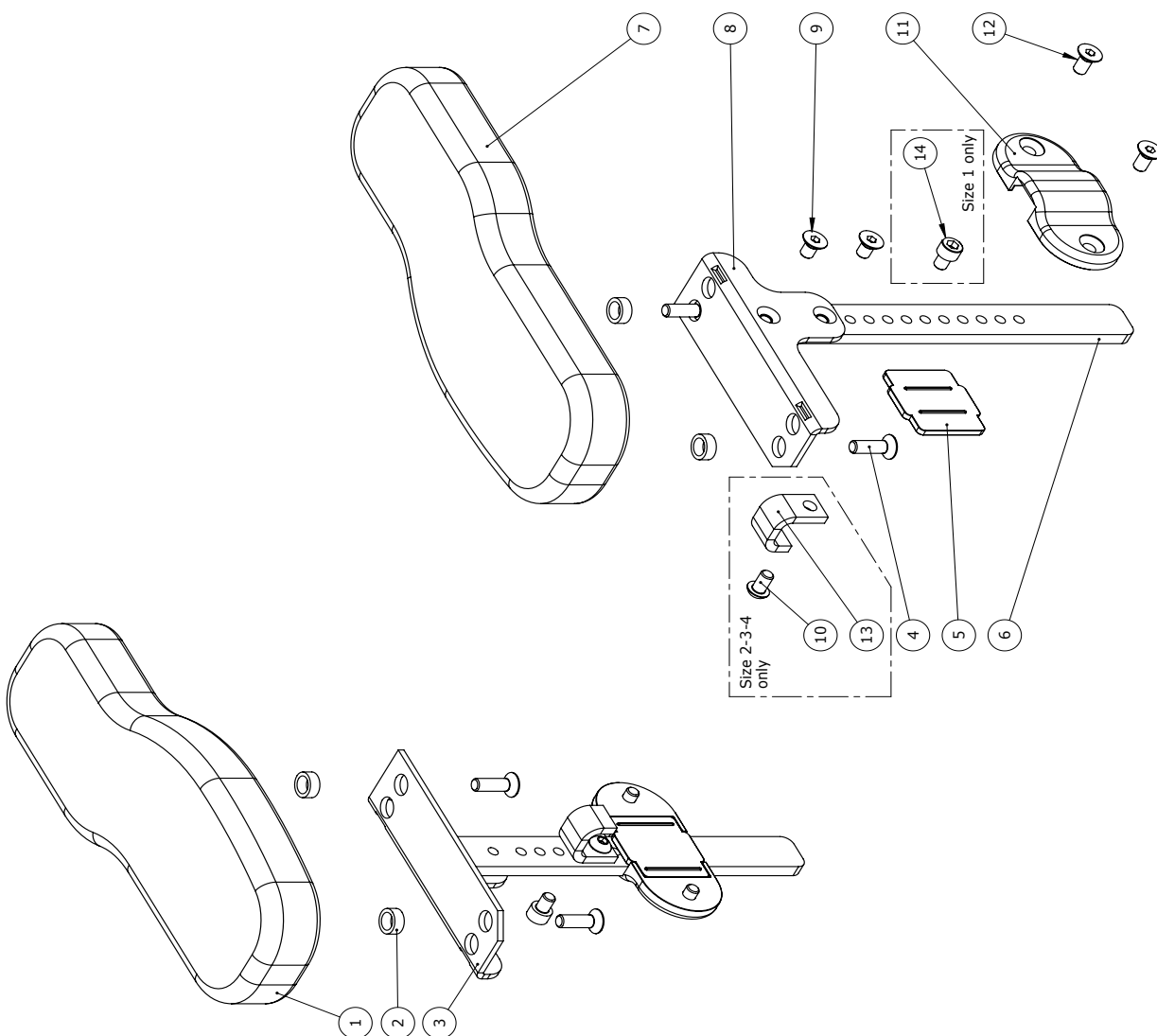

Position	Art. No.	Description	Unit of Measure	Quantity	Ean code
1	6175150	Armrest cushion, size 1, right	PCS	1	5707292189226
2	1310539	Distance piece 12/8,4*5mm	PCS	4	5707292423474
3	104360	Armrest bracket, right	PCS	1	5707292570703
4	0609730	Stainless insex 6*20 uh.	PCS	4	5707292138507
6	117054	Amrest bar	PCS	2	5707292611383
7	6175152	Armrest cushion, size 1, left	PCS	1	5707292189233
8	104359	Armrest bracket, left	PCS	1	5707292570697
9	0609760	Rustfri insex 6*10 uh. tuflok	PCS	4	5707292570796
10	0659005	Insex button head M6*10,stainl	PCS	2	5707292423740
13	117055	Brace	PCS	2	5707292611390
14	0602479	Insex M6*8, stainless	PCS	2	5707292465412
5+11+12	5001013	Plastic bag enclosing small	SET	1	5707292154378

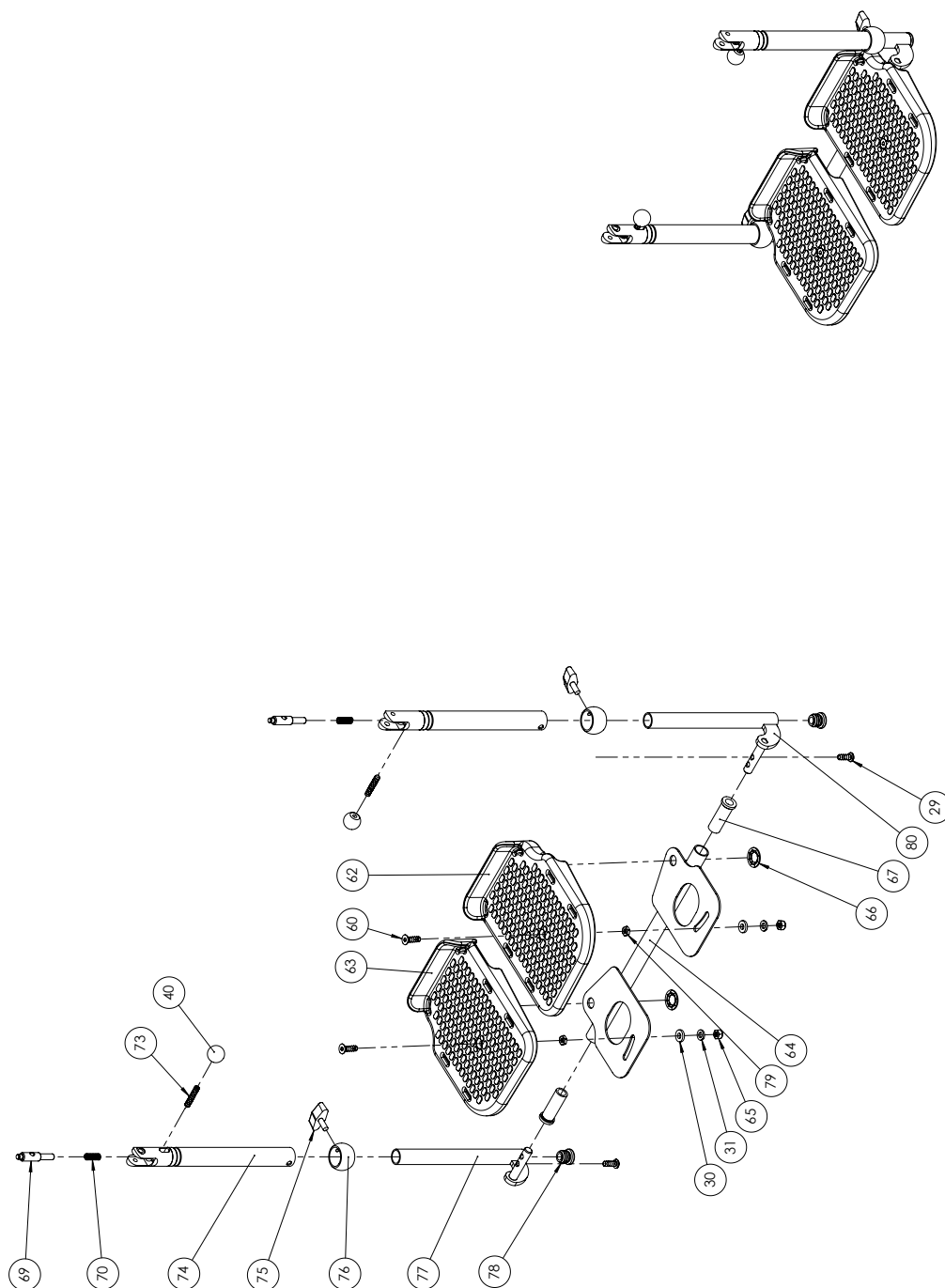

 Nothing compares to a smile	<div>For R82 A/S 1620 Svanevej 2650 Lyngby Denmark</div>	Created by Admin	Revision C	Revision by R82
		Created date 31-03-2011	Revision date 08-10-2014	
Project		Item no		
Footrest for Flamingo		79724		

C:\Users\TJBA\Documents\R82\Flamingo\79724

Position	No.	Description	Quantity	Unit of Measure	Ean code
30	1310717	NYLON WASHER, NAT O17-8,4*2MM	2	PCS	5707292140821
40	3502008	KNOB HANDLE Ø20 M6	2	PCS	5707292146830
60	0609732	INSEX 6*22, STAINLESS	2	PCS	5707292409669
62*	71207	FOOTPLATE LEFT, FLAM. SZ 1-2	1	PCS	5707292104625
	71209	FOOTPLATE LEFT, FLAM. SZ 3-4	1	PCS	5707292104649
63*	71208	FOOTPLATE RIGHT, FLAM. SZ 1-2	1	PCS	5707292104632
	71210	FOOTPLATE RIGHT, FLAM. SZ 3-4	1	PCS	5707292104656
64*	71282	FOOTPLATE INLAY FLAM. SIZE 1	1	PCS	5707292104816
	71283	FOOTPLATE INLAY FLAM. SIZE 2	1	PCS	5707292104823
	71284	FOOTPLATE INLAY FLAM. SIZE 3	1	PCS	5707292104830
	71285	FOOTPLATE INLAY FLAM. SIZE 4	1	PCS	5707292104847
65	1631060	STAINLESS LOCK NUT 6MM	2	PCS	5707292141446
66	1109000	STARLOCK SKIVE M12, U/KAP, RF.	2	PCS	5707292140432
67	71119	FICTIONBUSH FOR FLAMINGO	2	PCS	5707292104267
69	71123	LOCK RATCHET	2	PCS	5707292104304
70	0310147	STAINLESS SPRING1*9,3*20MM*7,5	2	PCS	5707292135636
73	0806030	STAINLESS STOP SCREW 6*30MM	2	PCS	5707292140029
74*	71242	TELESCOPIC PIPE SZ. 1	2	PCS	5707292104687
	71243	TELESCOPIC PIPE SZ. 2-4	2	PCS	5707292104694
75	3503210	FINGER SCREW,WHITE M6X6	2	PCS	5707292147004
76	71118	BUSHING Ø22-18 FLAM.	2	PCS	5707292104250
77*	71272	TELESCOPE EXTENSION LEFT, SZ 1	1	PCS	5707292104755
	71273	TELESCOPE ARM LEFT 2	1	PCS	5707292104762
	71274	TELESCOPE EXT. LEFT, SZ 3+4	1	PCS	5707292104779
78	2502005	FERRULE D18*1,0-2,0 INT.WHITE	2	PCS	5707292144010
79	1750600	LOCK NUT, STAINLESS M6	2	PCS	5707292141477
80*	71277	TELESCOPE EXTENSION RIGHT	1	PCS	5707292104786
	71278	TELESCOPE EXTENSION RIGHT,SZ 2	1	PCS	5707292104793
	71279	TELESCOPE EXTENSION RIGHT,SZ3	1	PCS	5707292104809

*		CHOOSE SIZE
---	--	-------------

Position	Art. No.	Description	Unit of Measure	Quantity	Ean code
141	0609727	Stainless insex 6x12 uh	PCS	4	5707292138484
170	71371	Support rod STR.1 VENSTRE	PCS	1	5707292105394
170	71369	Support rod size 4, left	PCS	1	5707292436757
170	71367	Support rod STR.3 VENSTRE	PCS	1	5707292436771
170	71373	Support rod for size 2	PCS	1	5707292105417
171	71370	Support rod STR.1 HØJRE	PCS	1	5707292105387
171	71368	Support rod size 4, right	PCS	1	5707292436740
171	71366	Support rod STR. 3 HØJRE	PCS	1	5707292436764
171	71372	Support rod STR.2 HØJRE	PCS	1	5707292105400
172	6175361	Armrest table small, left	PCS	1	5707292189707
172	6175365	Armrest table large, left	PCS	1	5707292189721
173	6175359	Armrest table small, right	PCS	1	5707292189691
173	6175363	Armrest table large, right	PCS	1	5707292189714
174	71130-90	Bushing, white	PCS	2	5707292223593
175	71132	Locking socket,white	PCS	1	5707292104373
176	104377	Washer Ø15,5Ø5,3, h10,0	PCS	4	5707292568878
177	72134-90	Mounting for armrest, white	PCS	2	5707292223760
178	82408-90	Underlaying plate armrest flam	PCS	2	5707292231482
179	0506519	Stainless blind rivet 4*12	PCS	2	5707292137388
180	6153700	Knob spring f.activity centre	PCS	2	5707292188601
181	1310552	Cover plugs k 5 natur	PCS	2	5707292581747
182	0603522	Insex ch M5*22 rf	PCS	4	5707292490988