

high-low

Exploded views / Part lists


Rev. date (09.2014)

On the following pages you will find the parts used to assemble this product. Please note that some spare parts cannot be purchased individually but are part of a complete kit. Please also note that for some spare parts extra delivery time may occur.



Contents


High-low indoor frame	3
High-low outdoor frame.....	6
High-low indoor with power	10

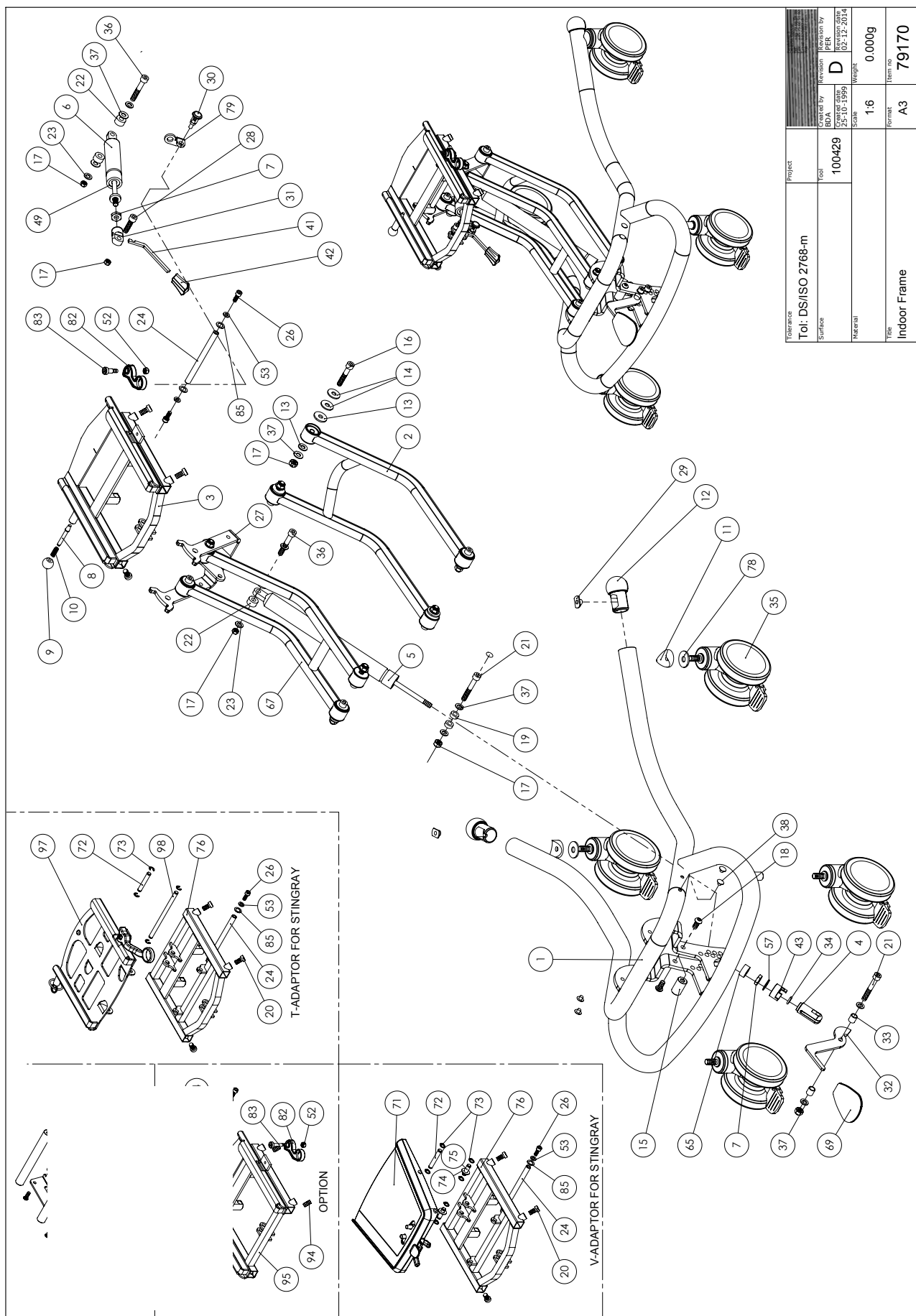

Position	No.	Description	Quantity	Unit of Measure	Ean code
1*	82820-60	FRAME, MOTHER OF PEARL BLACK	1	PCS	
	82820-01	FRAME, BLACK	1	PCS	5707292231659
	82820-20	FRAME, BLUE	1	PCS	5707292231666
	82820-21	FRAME, GREEN	1	PCS	5707292231673
2*	82822-60	ARM FOR SUPPORT HIGHLOW, MOP.BLACK	1	PCS	
	82822-01	ARM FOR SUPPORT HIGHLOW, BLACK	2	PCS	5707292231697
	82822-20	ARM FOR SUPPORT HIGHLOW, BLUE	2	PCS	5707292231703
	82822-21	ARM FOR SUPPORT HIGHLOW, GREEN	2	PCS	5707292231710
3*	82825-60	T-FRAME, MOTHER OF PEARL BLACK	1	PCS	
	82825-01	T-FRAME, BLACK	1	PCS	5707292231772
	82825-20	T-FRAME, BLUE	1	PCS	5707292231789
	82825-21	T-FRAME, GREEN	1	PCS	5707292231796
4	6118822	HORIZONTAL RELEASE	1	PCS	5707292186744
5	5040050	GASSPRING FOR HIGH LOW	1	PCS	5707292397669
6	2201500	BLOC-O-LIFT 9127MP 500 N	1	PCS	5707292143181
7	1761000	COUNTER NUT 10*1,0 FINE THREAD	1	PCS	5707292141538
8	6113000	SPINDLE, T-PART/FRAME	1	PCS	5707292186362
9	3502008	KNOB HANDLE Ø20 M6	1	PCS	5707292146830
10	0310100	PRESSURESPRING 25 MM F.SPINDEL	1	PCS	5707292135513
11	82894	WASHER	2	PCS	5707292387578
12	82829-30	BALL Ø40, GREEN	2	PCS	5707292231857
13	1310705	NYLON WASHER, WHITE, Ø25*8,4*2,5	16	PCS	5707292140791
14	1108000	WASHER 8MM	16	PCS	5707292140388
15	82808	COLLECTING BUSH	1	PCS	5707292125224
16	0603500	INSEX SCREW 8*45	8	PCS	5707292138231
17	1607000	LOCK NUT 8MM, LARGE PACK	13	PCS	5707292141378
18	0661500	BUTTON HEAD SCREW 8*16 STAINL.	2	PCS	5707292139559
19	1350000	DISTANCEPART BLACK 6*15*8,2 MM	2	PCS	5707292140876
20	3502516	TIGHTENING HANDLE 45MM M8*16	1	PCS	5707292146915
21	0603600	INSEX SCREW 8*55	3	PCS	5707292138262
22	1350100	DISTANCE PART BLACK 8,2*18*10	4	PCS	5707292140937
23	1103200	FAZET WASHER 8MM	4	PCS	5707292140296
24	82806	AXLE	1	PCS	5707292125200
26	0602200	INSEX SCREW 6*16	2	PCS	5707292138095
27	76800-60	BRACE FOR TILT, BLACK	1	PCS	5707292497116
28	0615010	INSEX M8x30 FZB SHORTENED	1	PCS	5707292367990
29	1600525	STAINL. WELDING NUT W/FLANGE	2	PCS	5707292141149
30	0380184	SPEC. LOCKING HOUSE M10*1, 28MM	1	PCS	5707292366030
31	6118820	HOUSE FOR GASSPRING, BLACK	1	PCS	5707292186720
32	82851	ARM FOR FOOT PEDAL	1	PCS	5707292125507
33	82859	BUSH FOR FOOT PEDAL	2	PCS	5707292125552
34	0108000	O-RINGS O13*2,5 QUALITY XP 70	1	PCS	5707292135124
35	3006937	100MM DOUBLE WHEEL WITH BRAKE	4	PCS	5707292548030
36	0603550	INSEX SCREW 8*50	2	PCS	5707292138255
37	1102000	PLANING WASHER 8MM	16	PCS	5707292140227
38	1310540	COVERING PUSHBUTTON, BLACK	4	PCS	5707292140692
40	2504000	FERRULE E15*1,0-2,0 INT.BLACK	4	PCS	5707292144102
41	2200013	ARM F. RELEASE, F.INDOOR FRAME	1	PCS	5707292142993
42	2200030	BUTTON FOR RELEASE ARM	1	PCS	5707292143051

43	82866	LOCK FOR RELEASE HEAD	1	PCS	5707292125606
49	1310712	NYLON WASHER, NATURE	1	PCS	5707292356345
52	1606500	LOCK NUT 6 MM	1	PCS	5707292141361
53	1103000	FAZET WASHER 6MM	2	PCS	5707292140289
57	1102500	PLANNING WASHER 10 MM	1	PCS	5707292140234
65	1310630	DISTANCE PIECE, Ø10,3*22*10MM	1	PCS	5707292140753
67*	82888-60	CARRYING ARM HIGH/LOW, MOP.BLACK	1	PCS	
	82888-01	CARRYING ARM HIGH/LOW, BLACK	1	PCS	5707292369369
	82888-20	CARRYING ARM HIGH/LOW, BLUE	1	PCS	5707292369376
	82888-21	CARRYING ARM HIGH/LOW, GREEN	1	PCS	5707292369383
69	3470335	RELEASE BUTTON Ø8, 5*8MM	1	PCS	5707292146632
78	1108050	WASHER M10, DIN9021	2	PCS	5707292140401
79	82481	SAFETY MOUNTING FOR GASSPRING	1	PCS	5707292124500
82	82880	HANDLE F/ADAPTOR T/HIGH-LOW	1	PCS	5707292361318
83	1106000	WASHER 6MM	2	PCS	5707292140371
85	1310640	WASHER, NATURE Ø16*10*0,5MM	2	PCS	5707292140760
88	1310640	WASHER, NATURE Ø16*10*0,5MM	4	PCS	5707292140760
91	0683803	PASBOLT M/INDV.6KT. 10*25 (M8)	2	PCS	5707292460066
92	1607000	LOCK NUT 8MM, LARGE PACK	2	PCS	5707292141378
93	1102000	PLANING WASHER 8MM	2	PCS	5707292140227
94	0603540	INSEX LOW HEAD M8X40 STAINLESS	4	PCS	5707292412485
95	75600-60	T-FRAME, X:PANDA 1+2	1	PCS	5707292459541
130	1101000	CENTER SPRING WASHER 8MM	1	PCS	5707292140173

*		CHOOSE COLOUR
---	--	---------------

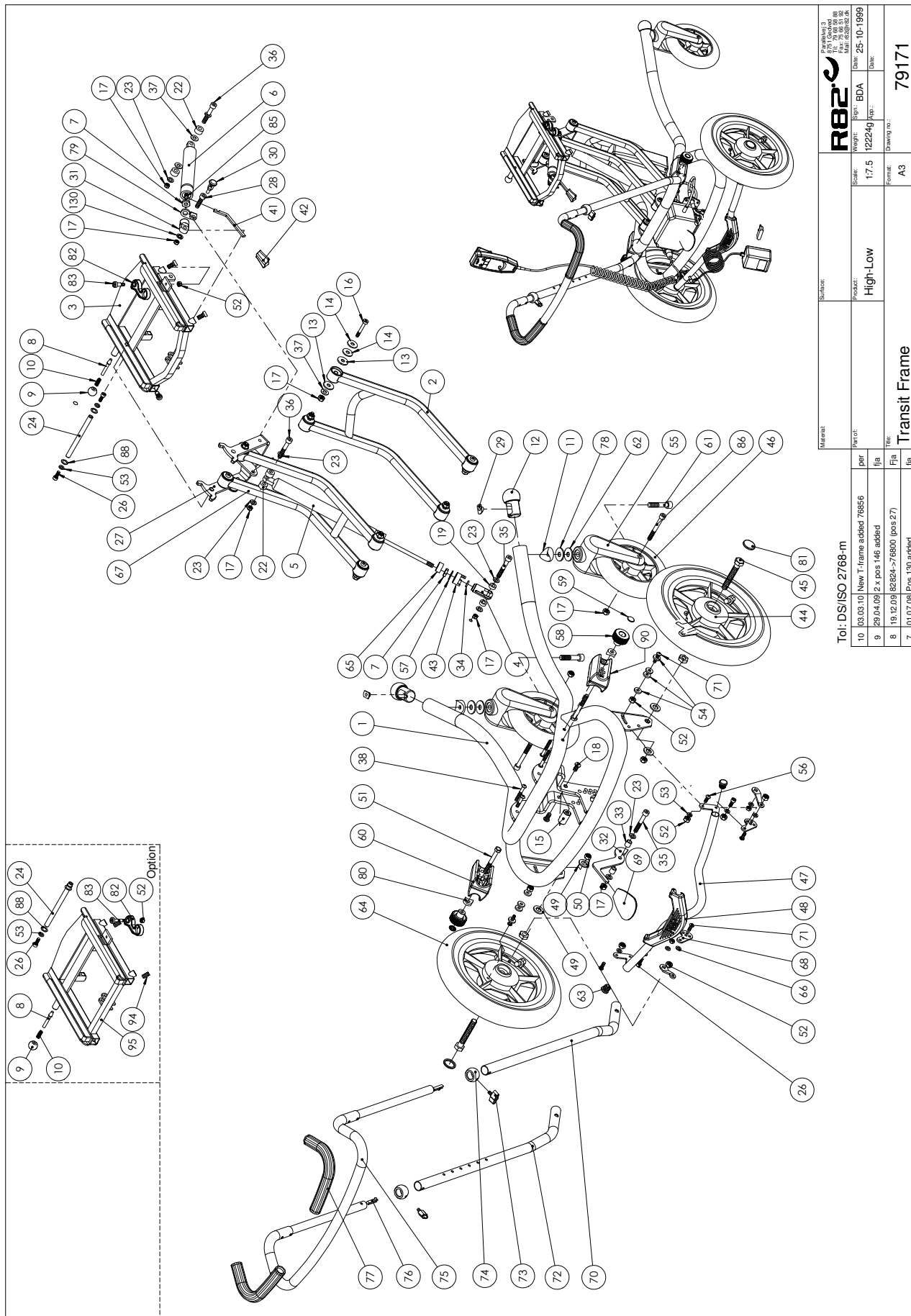


Position	No.	Description	Quantity	Unit of Measure	Ean code
1*	82656-60	TRANSITFRAME, MOTHER OF PEARL BLACK	1	PCS	
	82656-01	TRANSITFRAME, BLACK	1	PCS	5707292397690
	82656-20	TRANSITFRAME, BLUE	1	PCS	5707292397706
	82656-21	TRANSITFRAME, GREEN	1	PCS	5707292397713
2*	82822-60	ARM FOR SUPPORT HIGHLOW, MOP.BLACK	1	PCS	
	82822-01	ARM FOR SUPPORT HIGHLOW, BLACK	2	PCS	5707292231697
	82822-20	ARM FOR SUPPORT HIGHLOW, BLUE	2	PCS	5707292231703
	82822-21	ARM FOR SUPPORT HIGHLOW, GREEN	2	PCS	5707292231710
4	6118822	HORIZONTAL RELEASE	1	PCS	5707292186744
5	5040050	GASSPRING FOR HIGH LOW	1	PCS	5707292397669
7	1761000	COUNTER NUT 10*1,0 FINE THREAD	1	PCS	5707292141538
8	6113000	SPINDLE, T-PART/FRAME	1	PCS	5707292186362
9	3502008	KNOB HANDLE Ø20 M6	1	PCS	5707292146830
10	0310100	PRESSURESPRING 0,9*8,23*10,5V	1	PCS	5707292135513
11	82894	WASHER	2	PCS	5707292387578
12	82829-30	BALL Ø40, GREEN	2	PCS	5707292231857
13	1310705	NYLON WASHER, WHITE, Ø25*8,4*2,5	16	PCS	5707292140791
14	1108000	WASHER 8MM	16	PCS	5707292140388
15	82808	COLLECTING BUSH	1	PCS	5707292125224
16	0603500	INSEX SCREW 8*45	8	PCS	5707292138231
17	1607000	LOCK NUT 8MM, LARGE PACK	16	PCS	5707292141378
18	0661500	BUTTON HEAD SCREW 8*16 STAINL.	2	PCS	5707292139559
19	1350000	DISTANCEPART BLACK 6*15*8,2 MM	2	PCS	5707292140876
22	1350100	DISTANCE PART BLACK 8,2*18*10	2	PCS	5707292140937
23	1103200	FAZET WASHER 8MM	4	PCS	5707292140296
26	0602200	INSEX SCREW 6*16	4	PCS	5707292138095
27	76800-60	BRACE FOR TILT, BLACK	1	PCS	5707292497116
29	1600525	STAINL. WELDING NUT W/FLANGE	2	PCS	5707292141149
32	82851	ARM FOR FOOT PEDAL	1	PCS	5707292125507
33	82859	BUSH FOR PEDAL	2	PCS	5707292125552
34	0108000	O-RINGS O13*2,5 QUALITY XP 70	1	PCS	5707292135124
35	0603600	INSEX SCREW 8*55	1	PCS	5707292138262
36	0603550	INSEX SCREW 8*50	2	PCS	5707292138255
37	1102000	PLANING WASHER 8MM	14	PCS	5707292140227
38	0645645	BUTTON HEAD M6*45 ZN/BLACK	4	PCS	5707292139351
43	82866	LOCK FOR RELEASE HEAD	1	PCS	5707292125606
40	2504000	FERRULE E15*1,0+2,0 INT.BLACK	4	PCS	5707292144102
44*	3003510	12½*2 1/4" BRAKE/RIGHT, HIGH-LOW	1	PCS	5707292397492
	3004010	12½*2 1/4" BRAKE/LEFT, HIGH-LOW	1	PCS	5707292397508
	3003510	12½*2 1/4" BRAKE/RIGHT, HIGH-LOW	1	PCS	5707292397492
	3002500	12½*2 1/4 BRAKE/LEFT, HIGH LOW	1	PCS	5707292145642
45	1902380	STEEL SET SCREW 12*80MM	2	PCS	5707292142030
46	3007910	175*40 BLACK/PNEUM. WHEEL	2	PCS	5707292146045
47	82840	BRAKE BRACE	1	PCS	5707292125415
48	82830	BRAKEPEDAL PANDA INDOORFRAME	1	PCS	5707292125347
49	1310712	NYLON WASHER, NATURE	1	PCS	5707292356345
50	1604500	STEEL NUT 12MM, LARGE PACK	2	PCS	5707292141309
51	1902320	STEEL SET SCREW M8*70, FZB	2	PCS	5707292142023
52	1606500	LOCK NUT 6 MM	2	PCS	5707292141361

53	1106000	WASHER 6MM	2	PCS	5707292140371
54	1125618	SCREENWASHER FZ6*18*3MM	8	PCS	5707292140494
55	2965155	FRONTFORK 200MM, BLACK	2	PCS	5707292144935
56	0641620	BUTTON HEAD SCREW 6*20	4	PCS	5707292139276
57	1102500	PLANNING WASHER 10 MM	1	PCS	5707292140234
58	3470400	HAND GRIP WITHOUT COVER	2	PCS	5707292146663
59	1109500	STARLOCK SKIVE M/KAPPE Ø6	2	PCS	5707292140449
60	82828	HAND BRACE HOLDER, HIGH-LOW	1	PCS	5707292125330
61*	0606000	INSEX SCREW 10*50	2	PCS	5707292138408
	1101600	CENTER SPRING WASHER 12MM	4	PCS	5707292140197
62	1103400	FAZET WASHER 10 MM	2	PCS	5707292140319
63	2502002	FERRULE D18*0,8-2,5 BLACK	2	PCS	5707292433947
64*	3004010	12½*2 1/4" BRAKE/LEFT, HIGH-LOW	1	PCS	5707292397508
	3003510	12½*2 1/4" BRAKE/RIGHT, HIGH-LOW	1	PCS	5707292397492
	3004010	12½*2 1/4" BRAKE/LEFT, HIGH-LOW	1	PCS	5707292397508
	3003000	12½*2 1/4" BRAKE/RIGHT, HIGH-LOW	1	PCS	5707292145659
65	1310630	DISTANCE PIECE, Ø10,3*22*10MM	1	PCS	5707292140753
66	1310628	DISTANCE PCS NATUREL Ø6,4*10*2,8MM	2	PCS	5707292140746
67*	82888-60	CARRYING ARM HIGH/LOW, MOP.BLACK	1	PCS	
	82888-01	CARRYING ARM HIGH/LOW, BLACK	1	PCS	5707292369369
	82888-20	CARRYING ARM HIGH/LOW, BLUE	1	PCS	5707292369376
	82888-21	CARRYING ARM HIGH/LOW, GREEN	1	PCS	5707292369383
68	82838	LINKARM	2	PCS	5707292125392
69	3470335	RELEASE BUTTON Ø8, 5*8MM	1	PCS	5707292146632
70*	82856-60	ROD FOR HAND BRACE, RIGHT, MOP.BLACK	1	PCS	
	82856-01	ROD FOR HAND BRACE, RIGHT, BLACK	1	PCS	5707292232007
	82856-20	ROD FOR HAND BRACE, RIGHT, BLUE	1	PCS	5707292232014
	82856-21	ROD FOR HAND BRACE, RIGHT, GREEN	1	PCS	5707292232021
71	0641625	BUTTON HEAD SCREW 6*25 MM	2	PCS	5707292139283
72*	82855-60	ROD FOR HAND BRACE, LEFT, MOP.BLACK	1	PCS	
	82855-01	ROD FOR HAND BRACE, LEFT, BLACK	1	PCS	5707292231970
	82855-20	ROD FOR HAND BRACE, LEFT, BLUE	1	PCS	5707292231987
	82855-21	ROD FOR HAND BRACE, LEFT, GREEN	1	PCS	5707292231994
73	3652500	WING SCREW 28*11 M6*16MM BLACK	2	PCS	5707292147172
74*	71137	BUSH BALL FOR HANDBRACE	2	PCS	5707292361813
	71118	BUSHING Ø22-18 FLAM.	2	PCS	5707292104250
75*	82858-60	HANDLE, MOTHER OF PEARL BLACK	1	PCS	
	82858-01	HANDLE, BLACK	1	PCS	5707292232038
	82858-20	HANDLE, BLUE	1	PCS	5707292232045
	82858-21	HANDLE, GREEN	1	PCS	5707292232052
76	6153500	KNOB SPRING 0,5*6,5 MM.	2	PCS	5707292188571
77	2500560	GRIP F/Ø22 HANDBRACE 800MM	1	PCS	5707292388537
78	1108050	WASHER M10, DIN9021	2	PCS	5707292140401
80	1357500	DISTANCE PCS BLACK 20*8,4*4MM	2	PCS	5707292141026
81	2704300	BLACK HUB CAP FOR ROLKO WHEEL	2	PCS	5707292144515
82	82880	HANDLE FOR ADAPTOR	1	PCS	5707292361318
83	0683810	PASBOLT M/INDV.6KT. 8*12 (M6)	1	PCS	5707292361929
84	0659500	BUTTON HEAD SCREW	4	PCS	5707292139450

85	1310640	WASHER, NATURE Ø16*10*0,5MM	2	PCS	5707292140760
86	0603700	INNSEX SCREW 8*70	2	PCS	5707292138293
88	70800	SEAT CONTROL TIGER 2000/SERVAL	2	PCS	5707292102782
89*	82827-60	ADAÅTPR FPR TOGER 2000 SEAT, MOP.BLACK	1	PCS	
	82827-01	ADAPTOR FOR TIGER 2000 SEAT, BLACK	1	PCS	5707292231819
	82827-20	ADAPTOR FOR TIGER 2000 SEAT, BLUE	1	PCS	5707292231826
	82827-21	ADAPTOR FOR TIGER 2000 SEAT, GREEN	1	PCS	5707292231833
90	82899	HANDBRACE HOLDER, LEFT	1	PCS	5707292401397
91	0683803	PASBOLT M/INDV.6KT. 10*25 (M8)	2	PCS	5707292460066
92	1607000	LOCK NUT 8MM, LARGE PACK	2	PCS	5707292141378
93	1102000	PLANING WASHER 8MM	2	PCS	5707292140227
94	0603540	INSEX LOW HEAD M8X40 STAINLESS	2	PCS	5707292412485
95	75600-60	T-FRAME, X:PANDA 1+2	1	PCS	5707292459541
130	1101000	CENTER SPRING WASHER 8MM	1	PCS	5707292140173

*		CHOOSE COLOUR
---	--	---------------

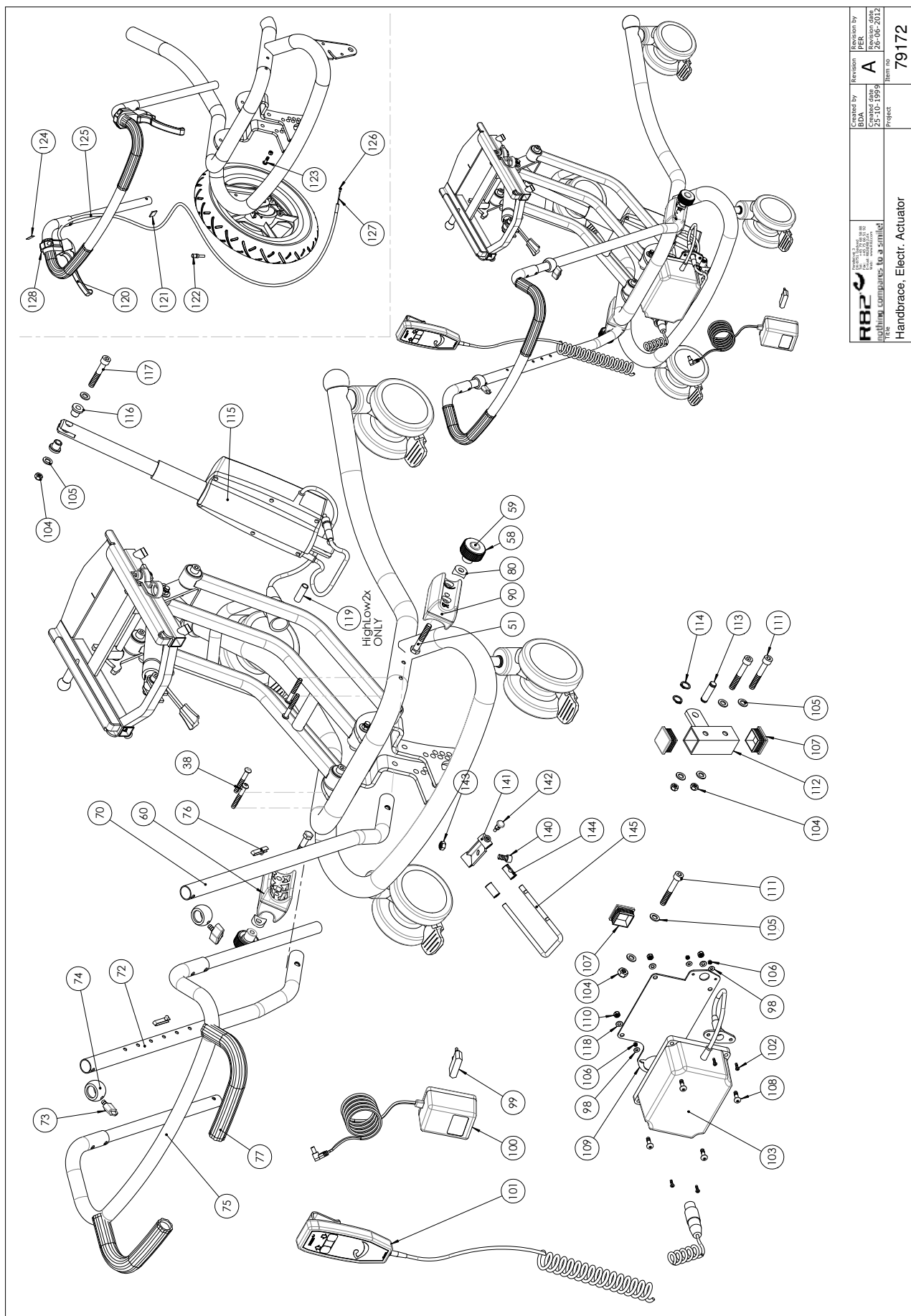

R82 Nothing compares to a smile!	Created by	Revision	Revision by
	Created date	A	Revision date
Title		Item no.	
Handbrace Electr. Actuator		79172	

Position	No.	Description	Quantity	Unit of Measure	Ean code
8	6113000	SPINDLE, T-PART/FRAME	1	PCS	5707292186362
9	3502008	KNOB HANDLE O20 M6	1	PCS	5707292146830
10	0310100	PRESSURESPRING 0,9*8*23*10,5V	1	PCS	5707292135513
38	1310540	COVERING PUSHBUTTON, BLACK	4	PCS	5707292140692
40	2504000	FERRULE E15*1,0-2,0 INT.BLACK	2	PCS	5707292144102
51	1902320	STEEL SET SCREW 8*70MM	2	PCS	5707292142023
52	1606500	LOCK NUT 6 MM	1	PCS	5707292141361
58	3470400	HAND GRIP WITHOUT COVER	2	PCS	5707292146663
59	1109500	STARLOCK WASHER WITH CAPE 6	2	PCS	5707292140449
60	82899	HAND BRACE HOLDER, LEFT	1	PCS	5707292401397
70*	82855-60	ROD FOR HAND BRACE, LEFT, MOP.BLACK	1	PCS	
	82855-01	ROD FOR HAND BRACE, LEFT, BLACK	1	PCS	5707292231970
	82855-20	ROD FOR HAND BRACE, LEFT, BLUE	1	PCS	5707292231987
	82855-21	ROD FOR HAND BRACE, LEFT, GREEN	1	PCS	5707292231994
72*	82856-60	ROD FOR HAND BRACE, RIGHT, MOP.BLACK	1	PCS	
	82856-01	ROD FOR HAND BRACE, RIGHT, BLACK	1	PCS	5707292232007
	82856-20	ROD FOR HAND BRACE, RIGHT, BLUE	1	PCS	5707292232014
	82856-21	ROD FOR HAND BRACE, RIGHT, GREEN	1	PCS	5707292232021
73	3652501	R82 HANDLE M6*16MM SCREW	2	PCS	5707292413963
74	71137	BUSH BALL FOR HANDBRACE	2	PCS	5707292361813
75*	82858-60	HANDLE, MOTHER OF PEARL BLACK	1	PCS	
	82858-01	HANDLE, BLACK	1	PCS	5707292232038
	82858-20	HANDLE, BLUE	1	PCS	5707292232045
	82858-21	HANDLE, GREEN	1	PCS	5707292232052
76	6153500	KNOB SPRING 0,5*6,5 MM.	2	PCS	5707292188571
77	2500560	GRIP F/ø22 HANDBRACE 800MM	1	PCS	5707292388537
80	1357500	DISTANCE PCS 20*8,4*4MM	2	PCS	5707292141026
82	82880	HANDLE FOR ADAPTOR	1	PCS	5707292361318
83	0683810	PASBOLT M/INDV.6KT. 8*12 (M6)	1	PCS	5707292361929
88	1310640	WASHER, NATURE O16*10*0,5MM	4	PCS	5707292140760
90	82828	HAND BRACE HOLDER, HIGH-LOW	1	PCS	5707292125330
91	0683803	PASBOLT M/INDV.6KA.10*25 (M8)	2	PCS	5707292460066
92	1607000	LOCK NUT 8MM, LARGE PACK	2	PCS	5707292141378
93	1102000	PLANING WASHER 8MM	2	PCS	5707292140227
94	0603540	INSEX LOW HEAD M8X40 STAINLESS	4	PCS	5707292412485
95	75600-60	T-FRAME, X:PANDA 1+2	1	PCS	5707292459541
99	2222165	PLUG F/DK	1	PCS	5707292492388
99	2222166	PLUG F/USA+CANADA+TAIWAN	1	PCS	5707292492395
99	2222167	PLUG FOR UK	1	PCS	5707292492401
99	2222169	PLUG F/NEW ZEALAND + AUS	1	PCS	5707292492425
100	2222160	CHARGER WO/PLUG	1	PCS	5707292492296
101	2220600	HANDSET W/R82 PRINT & CHARGING	1	PCS	5707292492241
102	1902590	CROSSHEAD SCREW M3*10, FZB	4	PCS	5707292408389
103	2222110	BATTERIBOKS BA18027	1	PCS	5707292492258
104	1607000	LOCK NUT 8MM, LARGE PACK	3	PCS	5707292141378
105	1102000	PLANING WASHER 8MM	6	PCS	5707292140227
106	1606250	SELF-LOCKING NUT 3MM	4	PCS	5707292408396
107	2505000	FERRULE E30*301,0-2,0 INT.BLAC	3	PCS	5707292144140
108	1902760	CROSS-HEADED SCREW 5*16 MM	4	PCS	5707292142146

109	77576	BRACKET FOR BATTERY	1	PCS	5707292125477
110	1606300	LOCK NUT 5 MM	4	PCS	5707292141354
111	0603600	INSEX SCREW 8*55	3	PCS	5707292138262
112	82844	AC MOUNTING FOR INDOOR FRAME	1	PCS	5707292125446
113	82849	RATCHET Ø10	1	PCS	5707292125491
114	1100100	STAINLESS LOCKRING Ø10 AXLE	2	PCS	5707292140111
115	2222130	ACTUATOR LA28 F/HIGHLOW	1	PCS	5707292492265
116	82848	BUSH	2	PCS	5707292125484
117	0603550	INSEX SCREW 8*50	1	PCS	5707292138255
120	3503535	BRAKE PARKING LEVER WO/CLAMP	2	PCS	5707292413536
121	2521854	KABELKLIPS 4MM, NATUR	2	PCS	5707292144416
122	0380010	KABELSTRAMMER M5*23 M.RUND MØT	2	PCS	5707292135971
123	0380040	CABLE BENCH SCREW M5	2	PCS	5707292135988
124	0494000	LABEL "BRAKE" MET. FILM	2	PCS	5707292137159
125	5001200	CABLE FOR TB-PANDA	2	PCS	5707292154446
126	0387000	END NIPPLE FOR OUTER CASING	2	PCS	5707292136091
127	0371853	STRAINED WIRES 1,5MM 19T.	2	PCS	5707292135889
128	3503536	CLAMP W/SCREWS	2	PCS	5707292427762
140	0604000	INSEX SCREW 10*20	1	PCS	5707292138361
141	82488	STEER FOR ANTI TIP DEVICE	1	PCS	5707292357847
142	0380182	LOCKING HOUSE, COMPLETE M10*1	1	PCS	5707292357953
143	1607500	LOCK NUT 10 MM	1	PCS	5707292141385
144	82375	PVC-BUSHING FOR PELOT	2	PCS	5707292123763
145	82484	BRACE FOR ANTI TIP DEVICE	1	PCS	5707292357892

*		CHOOSE COLOUR
**		CHOOSE COUNTRY