

kuduTM

Exploded views / Part lists


Rev. date (08.2015)

On the following pages you will find the parts used to assemble this product. Please note that some spare parts cannot be purchased individually but are part of a complete kit. Please also note that for some spare parts extra delivery time may occur.



Contents


Kudu with 20"/22"/24" rear wheels (101182)	3 - 6
Kudu with 12½" drum brake rear wheels (111504)	7 - 10
Kudu with 16" drum brake rear wheels (111505)	11 - 14
Frame assembly 12½" & 16" (111718)	15 - 16
Frame assembly 22" - 24" (107354)	17 - 18
Seat assembly size 1-2 (115022)	19 - 20
Seat assembly size 3-4 (115023)	21 - 22
Back assembly (107345)	23 - 24
Foot plate, centrally mounted, size 1-2 (111561)	25 - 26
Foot plate, centrally mounted, size 3-4 (111637)	27 - 28
Foot rests, parted, size 1-2 (111647)	29 - 30
Foot rests, parted, size 3-4 (109288)	31 - 32
Push brace, angle adjustable (111629)	33 - 34
Push handles (108370)	35 - 36
Anti-tip device, short (107379)	37 - 38
Anti-tip device, long (112320)	39 - 40
Wheel brake, right, size 1-2 (111859)	41 - 42
Wheel brake, left, size 1-2 (111860)	43 - 44
Wheel brake, right, size 3-4 (101994)	45 - 46
Wheel brake, left, size 3-4 (102562)	47 - 48

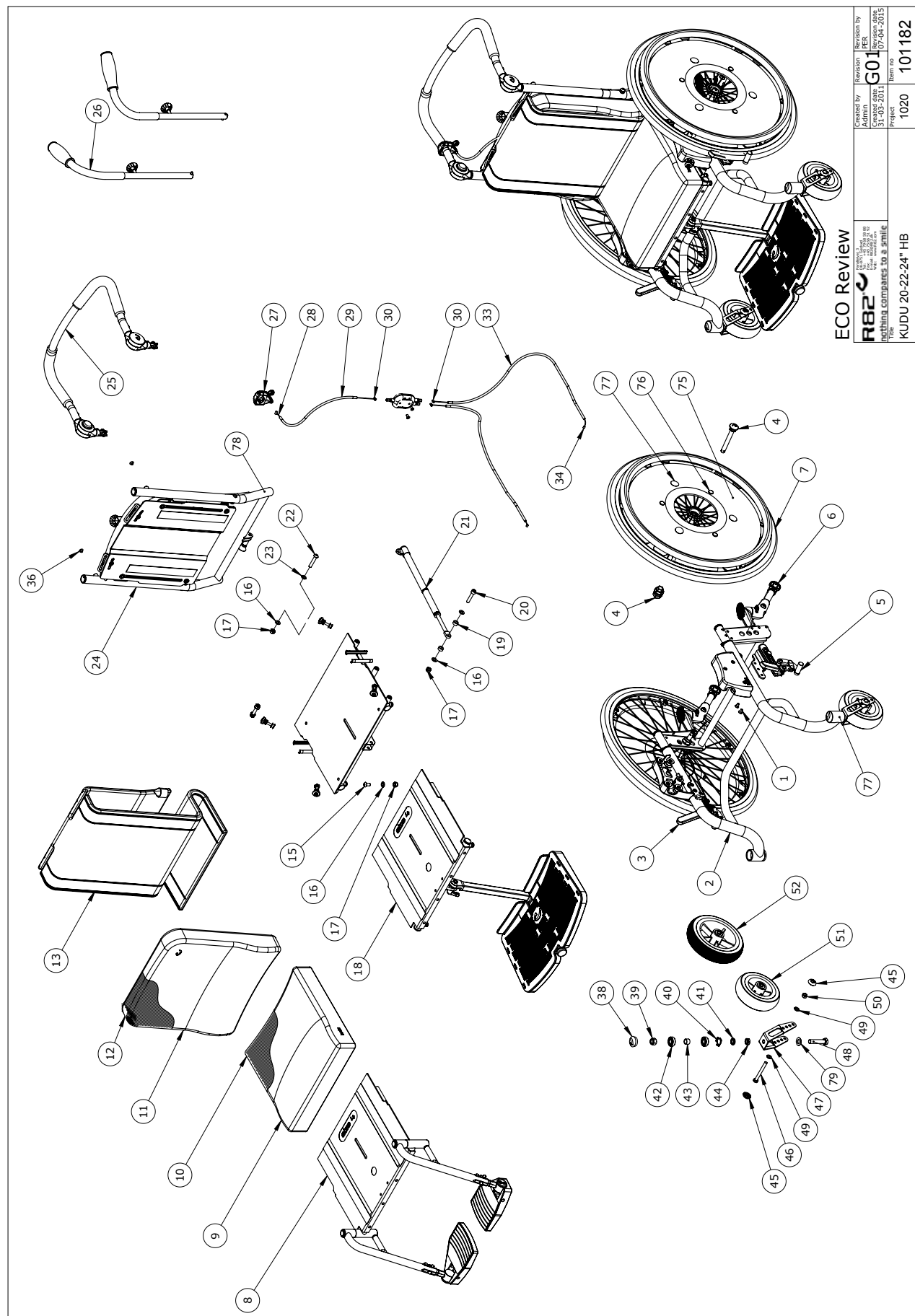

Position	Art nr.	Description	Quantity	Unit of measure	Ean code
9512121321 9531112221 9511111021 9542112933 9522112612 9533112832 9541111121 9542122531 9532122431 9531111733					
1	0609729	Stainless inox 6*16	4,00	Pieces	5707292435330
2	111718-110	Frame assembly size 1, white	1,00	Pieces	5707292585615
2	111718-131	Frame assembly size 1, red	1,00	Pieces	5707292585622
2	111718-135	Frame assembly size 1, grey	1,00	Pieces	5707292585646
2	111718-210	Frame assembly size 2, white	1,00	Pieces	5707292585653
2	111718-231	Frame assembly size 2, red	1,00	Pieces	5707292585677
2	107354-235	Frame assembly size 2, grey	1,00	Pieces	5707292585684
2	107354-310	Frame assembly size 3, white	1,00	Pieces	5707292585714
2	107354-331	Frame assembly size 3, red	1,00	Pieces	5707292585721
2	107354-335	Frame assembly size 3, grey	1,00	Pieces	5707292585691
2	111718-410	Frame assembly size 4, white	1,00	Pieces	5707292585738
2	107354-431	Frame assembly size 4, red	1,00	Pieces	5707292585745
2	107354-435	Frame assembly size 4, grey	1,00	Pieces	5707292585752
3	111859	Wheel brake 1-2, right	1,00	Pieces	5707292587688
3	101994	Wheel brake 3-4, right	1,00	Pieces	5707292583086
4	2985065	Compl. Stainless quick-release	2,00	Pieces	5707292145345
5	111860	Wheel brake 1-2, left	1,00	Pieces	5707292587671
5	102562	Wheel brake 3-4, left	1,00	Pieces	5707292583154
6	950953	Anti-tip device for Kudu	1,00	Set	5707292580931
6	950958	Anti-tip device for Kudu, long	1,00	Set	5707292589378
7	3202010	20" wheel, without brake, air, black	2,00	Pieces	5707292584045
7	3202015	20" wheel, without brake, solid, black	2,00	Pieces	5707292584052
7	3202020	20" wheel, with drum brake, air, black	2,00	Pieces	5707292584069
7	3202025	20" wheel, with drum brake, solid, black	2,00	Pieces	5707292584083
7	3202210	22" wheel, without brake, air, black	2,00	Pieces	5707292584090
7	3202215	22" wheel, without brake, solid, black	2,00	Pieces	5707292584106
7	3202220	22" wheel, with drum brake, air, black	2,00	Pieces	5707292584113
7	3202225	22" wheel, with drum brake, solid, black	2,00	Pieces	5707292584120
7	3202410	24" wheel, without brake, air, black	2,00	Pieces	5707292584137
7	3202415	24" wheel, without brake, solid, black	2,00	Pieces	5707292584144
7	3202420	24" wheel, with drum brake, air, black	2,00	Pieces	5707292584151
7	3202425	24" wheel, with drum brake, solid, black	2,00	Pieces	5707292584168
8	P-95151-260	Foot rests parted, size 2	1,00	Set	5707292588258
8	P-95151-360	Foot rests parted, size 3	1,00	Set	5707292588326
8	P-95151-460	Foot rests parted, size 4	1,00	Set	5707292588333
9+10	95742-160	Seat cushion size 1	1,00	Pieces	5707292586582
9+10	95742-260	Seat cushion size 2	1,00	Pieces	5707292586599
9+10	95742-360	Seat cushion size 3	1,00	Pieces	5707292586605
9+10	95742-460	Seat cushion size 4	1,00	Pieces	5707292586612
11+12	95741-160	Back cushion size 1	1,00	Pieces	5707292586667
11+12	95741-260	Back cushion size 2	1,00	Pieces	5707292586674
11+12	95741-360	Back cushion size 3	1,00	Pieces	5707292586681
11+12	95741-460	Back cushion size 4	1,00	Pieces	5707292586698
13	95744-160	Cushion covering size 1	1,00	Pieces	5707292586827
13	95744-260	Cushion covering size 2	1,00	Pieces	5707292586834
13	95744-360	Cushion covering size 3	1,00	Pieces	5707292586841
13	95744-460	Cushion covering size 4	1,00	Pieces	5707292586858
14	115022-1	Seat assembly size 1	1,00	Pieces	5707292605207
14	115022-2	Seat assembly size 2	1,00	Pieces	5707292605245

14	105023-3	Seat assembly size 3	1,00	Pieces	5707292581792
14	105023-4	Seat assembly size 4	1,00	Pieces	5707292581808
15	0661500	Button head screw 8*16 stainl.	1,00	Pieces	5707292139559
16	1105080	Stainless washer 8mm	5,00	Pieces	5707292140364
17	1631080	Stainless lock nut 8mm	4,00	Pieces	5707292141453
18	P-95152-160	Foot plate, centrally mounted, size 1	1,00	Pieces	5707292588340
18	P-95152-260	Foot plate, centrally mounted, size 2	1,00	Pieces	5707292588364
18	P-95152-360	Foot plate, centrally mounted, size 3	1,00	Pieces	5707292588371
18	P-95152-460	Foot plate, centrally mounted, size 4	1,00	Pieces	5707292588388
19	1350015	Dist.piece,black 7*Ø15*Ø8,2mm	2,00	Pieces	5707292467225
20	0609840	Stainless inox screw 8*40	1,00	Pieces	5707292138613
21	2207060	Spring Stopper Adjuster, for size 1	1,00	Pieces	5707292584175
21	2207070	Spring stopper adjuster, for size 2	1,00	Pieces	5707292584199
21	2207080	Spring Stopper Adjuster, for size 3+4	1,00	Pieces	5707292584205
22	0661145	Button head 8*45 FZB	2,00	Pieces	5707292139535
23	1310711	Nylon washer 15*8,4*1,5mm	2,00	Pieces	5707292429933
24	107345-110	Back assembly size 1, White	1,00	Pieces	5707292580450
24	107345-131	Back assembly size 1, Red	1,00	Pieces	5707292585394
24	107345-135	Back assembly size 1, Grey	1,00	Pieces	5707292585400
24	107345-210	Back assembly size 2, White	1,00	Pieces	5707292580467
24	107345-231	Back assembly size 2, Red	1,00	Pieces	5707292585547
24	107345-235	Back assembly size 2, Grey	1,00	Pieces	5707292585554
24	107345-310	Back assembly size 3, White	1,00	Pieces	5707292580474
24	107345-331	Back assembly size 3, Red	1,00	Pieces	5707292585561
24	107345-335	Back assembly size 3, Grey	1,00	Pieces	5707292585585
24	107345-410	Back assembly size 4, White	1,00	Pieces	5707292580481
24	107345-431	Back assembly size 4, Red	1,00	Pieces	5707292585592
24	107345-435	Back assembly size 4, Grey	1,00	Pieces	5707292585608
25	95629-1	Push brace, size 1	1,00	Pieces	5707292586513
25	95629-2	Push brace size 2	1,00	Pieces	5707292586537
25	95629-3	Push brace size 3	1,00	Pieces	5707292586544
25	95629-4	Push brace size 4	1,00	Pieces	5707292586551
26	95370-10	Push handles	1,00	Set	5707292586568
27	3503531	Gas pump lever with clamp	1,00	Pieces	5707292413529
28	0371853	Strained wires 1,5mm 190 cm.	1,00	Pieces	5707292135889
29	5002998	Cable for new rabbit, 39 cm	1,00	Pieces	5707292436016
30	0388006	Screwnipple 5*7,5mm	1,00	Pieces	5707292136114
33	5001600	Cable for back, 52 cm	2,00	Pieces	5707292154491
33	5002756	Cable for back, 62 cm	2,00	Pieces	5707292413673
34	0371854-1	Soft inner cable for Stingray	2,00	Pieces	5707292536945
35	0388010	Screwnippel 7*7mm	2,00	Pieces	5707292136121
36	1310528	Covering stud, black 13mm	2,00	Pieces	5707292470348
38	2985000	Pressureplug black soft rubber	2,00	Pieces	5707292145314
39	1608000	Lock nut 12mm, large pack	2,00	Pieces	5707292141392
40	6108150	Spring screw 29,57*17,48	2,00	Pieces	5707292378200
41	111853	Washer Ø12,2 x Ø20 x 3,7	2,00	Pieces	5707292587374
42	2985210	Ball bearing	4,00	Pieces	5707292379412
43	100643	Distance Bush	2,00	Pieces	5707292552105
44	1760012	Kontranut M12,fzb	2,00	Pieces	5707292519078
45	109378	Wheel protector	4,00	Pieces	5707292581303
46	1907150	Steel bolt 8*75mm	2,00	Pieces	5707292142474


47	77980	FRONT FORK 20 - 22"	2,00	Pieces	5707292534132
48	1902372	Steel bolt M12*55 FZB	2,00	Pieces	5707292519436
49	1105080	Stainless washer 8mm	2,00	Pieces	5707292140364
50	1631080	Stainless lock nut 8mm	2,00	Pieces	5707292141453
51	3007736	Wheel 140*40mm with rubber	2,00	Pieces	5707292558213
52	3007901	Wheel, 7" solid, black	2,00	Pieces	5707292587343
52	3007902	Wheel, 7" air, black	2,00	Pieces	5707292587350
75	2983500	20" Kudu spoke protector	2,00	Pieces	5707292575975
75	2983510	22" Kudu spoke protector	2,00	Pieces	5707292575906
75	2983520	24" Kudu spoke protector	2,00	Pieces	5707292575982
76	2982401	Clips for spoke guards, black	6,00	Pieces	5707292427670
77	0494522	Reflex 22mm, grey	2,00	Pieces	5707292137234
78	0493437	Label "exclamation mark"	2,00	Pieces	5707292136909

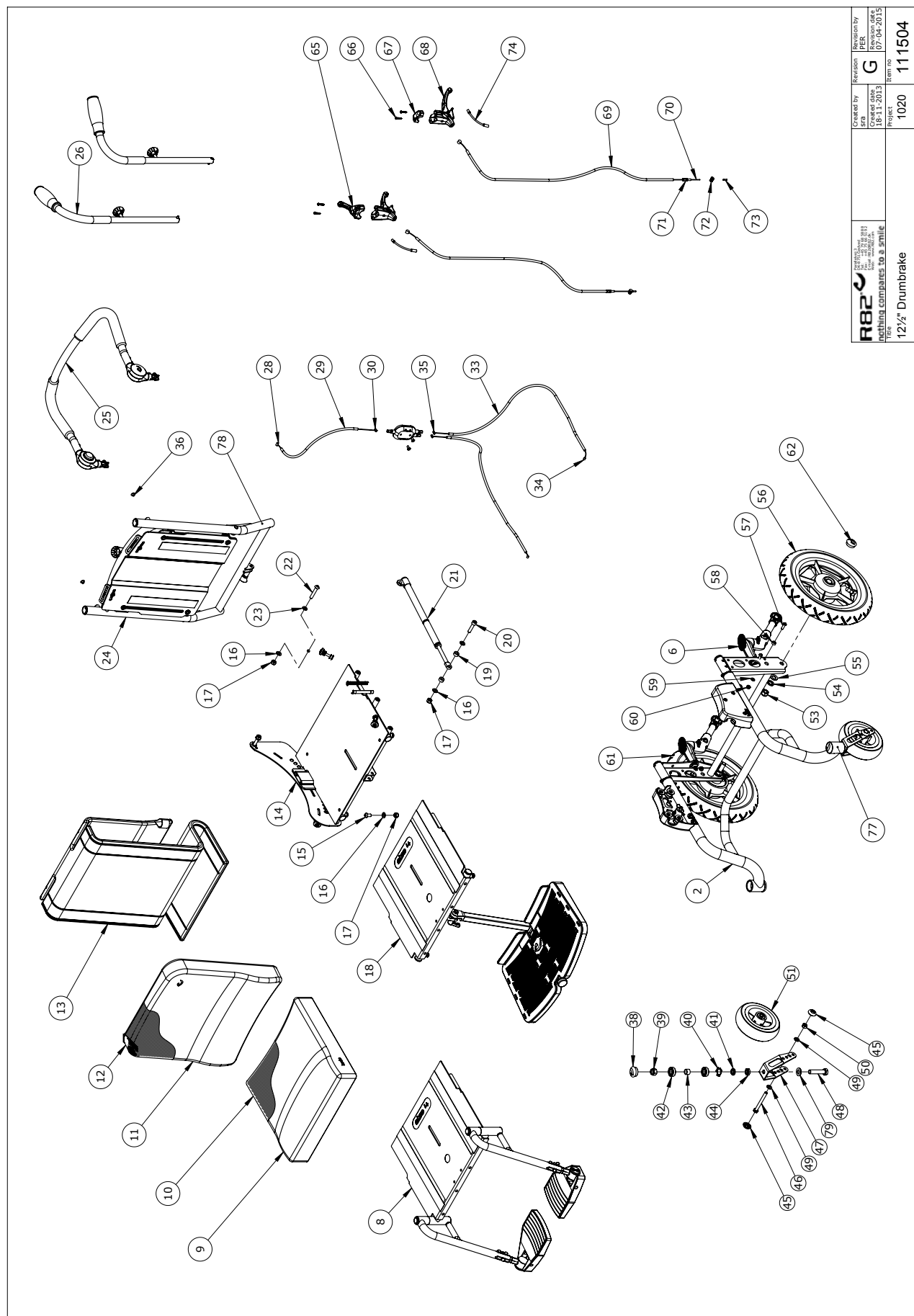

Position	Art nr.	Description	Quantity	Unit of measure	Ean code
9512121321 9531112221 9511111021 9542112933 9522112612 9533112832 9541111121 9542122531 9532122431 9531111733					
2	111718-110	Frame assembly size 1, white	1,00	Pieces	5707292585615
2	111718-131	Frame assembly size 1, red	1,00	Pieces	5707292585622
2	111718-135	Frame assembly size 1, grey	1,00	Pieces	5707292585646
2	111718-210	Frame assembly size 2, white	1,00	Pieces	5707292585653
2	111718-231	Frame assembly size 2, red	1,00	Pieces	5707292585677
2	107354-235	Frame assembly size 2, grey	1,00	Pieces	5707292585684
2	107354-310	Frame assembly size 3, white	1,00	Pieces	5707292585714
2	107354-331	Frame assembly size 3, red	1,00	Pieces	5707292585721
2	107354-335	Frame assembly size 3, grey	1,00	Pieces	5707292585691
2	111718-410	Frame assembly size 4, white	1,00	Pieces	5707292585738
2	107354-431	Frame assembly size 4, red	1,00	Pieces	5707292585745
2	107354-435	Frame assembly size 4, grey	1,00	Pieces	5707292585752
8	P-95151-260	Foot rests parted, size 2	1,00	Set	5707292588258
8	P-95151-360	Foot rests parted, size 3	1,00	Set	5707292588326
8	P-95151-460	Foot rests parted, size 4	1,00	Set	5707292588333
9+10	95742-160	Seat cushion size 1	1,00	Pieces	5707292586582
9+10	95742-260	Seat cushion size 2	1,00	Pieces	5707292586599
9+10	95742-360	Seat cushion size 3	1,00	Pieces	5707292586605
9+10	95742-460	Seat cushion size 4	1,00	Pieces	5707292586612
11+12	95741-160	Back cushion size 1	1,00	Pieces	5707292586667
11+12	95741-260	Back cushion size 2	1,00	Pieces	5707292586674
11+12	95741-360	Back cushion size 3	1,00	Pieces	5707292586681
11+12	95741-460	Back cushion size 4	1,00	Pieces	5707292586698
13	95744-160	Cushion covering size 1	1,00	Pieces	5707292586827
13	95744-260	Cushion covering size 2	1,00	Pieces	5707292586834
13	95744-360	Cushion covering size 3	1,00	Pieces	5707292586841
13	95744-460	Cushion covering size 4	1,00	Pieces	5707292586858
14	115022-1	Seat assembly size 1	1,00	Pieces	5707292605207
14	115022-2	Seat assembly size 2	1,00	Pieces	5707292605245
14	105023-3	Seat assembly size 3	1,00	Pieces	5707292581792
14	105023-4	Seat assembly size 4	1,00	Pieces	5707292581808
15	0661500	Button head screw 8*16 stainl.	1,00	Pieces	5707292139559
16	1105080	Stainless washer 8mm	5,00	Pieces	5707292140364
17	1631080	Stainless lock nut 8mm	4,00	Pieces	5707292141453
18	P-95152-160	Foot plate, centrally mounted, size 1	1,00	Pieces	5707292588340
18	P-95152-260	Foot plate, centrally mounted, size 2	1,00	Pieces	5707292588364
18	P-95152-360	Foot plate, centrally mounted, size 3	1,00	Pieces	5707292588371
18	P-95152-460	Foot plate, centrally mounted, size 4	1,00	Pieces	5707292588388
19	1350015	Dist.piece,black 7*Ø15*Ø8,2mm	2,00	Pieces	5707292467225
20	0609840	Stainless insex screw 8*40	1,00	Pieces	5707292138613
21	2207060	Spring Stopper Adjuster, for size 1	1,00	Pieces	5707292584175
21	2207070	Spring stopper adjuster, for size 2	1,00	Pieces	5707292584199
21	2207080	Spring Stopper Adjuster, for size 3+4	1,00	Pieces	5707292584205
22	0661145	Button head 8*45 FZB	2,00	Pieces	5707292139535
23	1310711	Nylon washer 15*8,4*1,5mm	2,00	Pieces	5707292429933
24	107345-110	Back assembly size 1, White	1,00	Pieces	5707292580450
24	107345-131	Back assembly size 1, Red	1,00	Pieces	5707292585394
24	107345-135	Back assembly size 1, Grey	1,00	Pieces	5707292585400
24	107345-210	Back assembly size 2, White	1,00	Pieces	5707292580467

24	107345-231	Back assembly size 2, Red	1,00	Pieces	5707292585547
24	107345-235	Back assembly size 2, Grey	1,00	Pieces	5707292585554
24	107345-310	Back assembly size 3, White	1,00	Pieces	5707292580474
24	107345-331	Back assembly size 3, Red	1,00	Pieces	5707292585561
24	107345-335	Back assembly size 3, Grey	1,00	Pieces	5707292585585
24	107345-410	Back assembly size 4, White	1,00	Pieces	5707292580481
24	107345-431	Back assembly size 4, Red	1,00	Pieces	5707292585592
24	107345-435	Back assembly size 4, Grey	1,00	Pieces	5707292585608
25	95629-1	Push brace, size 1	1,00	Pieces	5707292586513
25	95629-2	Push brace size 2	1,00	Pieces	5707292586537
25	95629-3	Push brace size 3	1,00	Pieces	5707292586544
25	95629-4	Push brace size 4	1,00	Pieces	5707292586551
26	95370-10	Push handles	1,00	Set	5707292586568
28	0371853	Strained wires 1,5mm 190 cm.	1,00	Pieces	5707292135889
29	5002998	Cable for new rabbit, 39 cm	1,00	Pieces	5707292436016
30	0388006	Screwnipple 5*7,5mm	1,00	Pieces	5707292136114
33	5001600	Cable for back, 52 cm	2,00	Pieces	5707292154491
33	5002756	Cable for back, 62 cm	2,00	Pieces	5707292413673
34	0371854-1	Soft inner cable for Stingray	2,00	Pieces	5707292536945
35	0388010	Screwnippel 7*7mm	2,00	Pieces	5707292136121
36	1310528	Covering stud, black 13mm	2,00	Pieces	5707292470348
38	2985000	Pressureplug black soft rubber	2,00	Pieces	5707292145314
39	1608000	Lock nut 12mm, large pack	2,00	Pieces	5707292141392
40	6108150	Spring screw 29,57*17,48	2,00	Pieces	5707292378200
41	111853	Washer Ø12,2 x Ø20 x 3,7	2,00	Pieces	5707292587374
42	2985210	Ball bearing	4,00	Pieces	5707292379412
43	100643	Distance Bush	2,00	Pieces	5707292552105
44	1760012	Kontranut M12,fzb	2,00	Pieces	5707292519078
45	109378	Wheel protector	4,00	Pieces	5707292581303
46	1907150	Steel bolt 8*75mm	2,00	Pieces	5707292142474
47	77980	FRONT FORK 20 - 22"	2,00	Pieces	5707292534132
48	1902372	Steel bolt M12*55 FZB	2,00	Pieces	5707292519436
49	1105080	Stainless washer 8mm	2,00	Pieces	5707292140364
50	1631080	Stainless lock nut 8mm	2,00	Pieces	5707292141453
51	3007736	Wheel 140*40mm with rubber	2,00	Pieces	5707292558213
53	1608000	Lock nut 12mm, large pack	2,00	Pieces	5707292141392
54	1101600	Center spring washer 12mm	2,00	Pieces	5707292140197
55	1103500	Fazet washer 12mm	2,00	Pieces	5707292140326
56+71+72	3009526	Wheel, 12½" drum brake, air, left	1,00	Pieces	5707292589590
56+71+72	3009528	Wheel, 12½" drum brake, solid, left	1,00	Pieces	5707292589613
57	0641620	Button head screw 6*20	2,00	Pieces	5707292139276
57	0659550	Button head M6*20, stainless	2,00	Pieces	5707292139467
58	104387	Spacer	2,00	Pieces	5707292582553
59	1105060	Stainless washer 6mm	2,00	Pieces	5707292140357
60	1606500	Lock nut 6 mm	2,00	Pieces	5707292141361
60	1631060	Stainless lock nut 6mm	2,00	Pieces	5707292141446
61+71+72	3009525	Wheel, 12½" drum brake, air, right	1,00	Pieces	5707292589583
61+71+72	3009527	Wheel, 12½" drum brake, solid, right	1,00	Pieces	5707292589606
62	2704300	Black hub cap for rolko wheel	2,00	Pieces	5707292144515
65+66	3503532	Gas pump lever without clamp	1,00	Pieces	5707292427755
67	3503536	Clamp w/screws	1,00	Pieces	5707292427762

68	3503535	Brake parking lever wo/clamp	2,00	Pieces	5707292413536
69	5002751	Cable for brake, size 3+4	2,00	Pieces	5707292413642
69	5002750	Cable for brake, size 1+2	2,00	Pieces	5707292413635
70	0371853	Strained wires 1,5mm 190 cm	2,00	Pieces	5707292135889
73	0387000	End nipple for outer casing	2,00	Pieces	5707292136091
74	0380060	Flexible v-brake pipe.	2,00	Pieces	5707292425195
77	0494522	Reflex 22mm, grey	2,00	Pieces	5707292137234
78	0493437	Label "exclamation mark"	2,00	Pieces	5707292136909

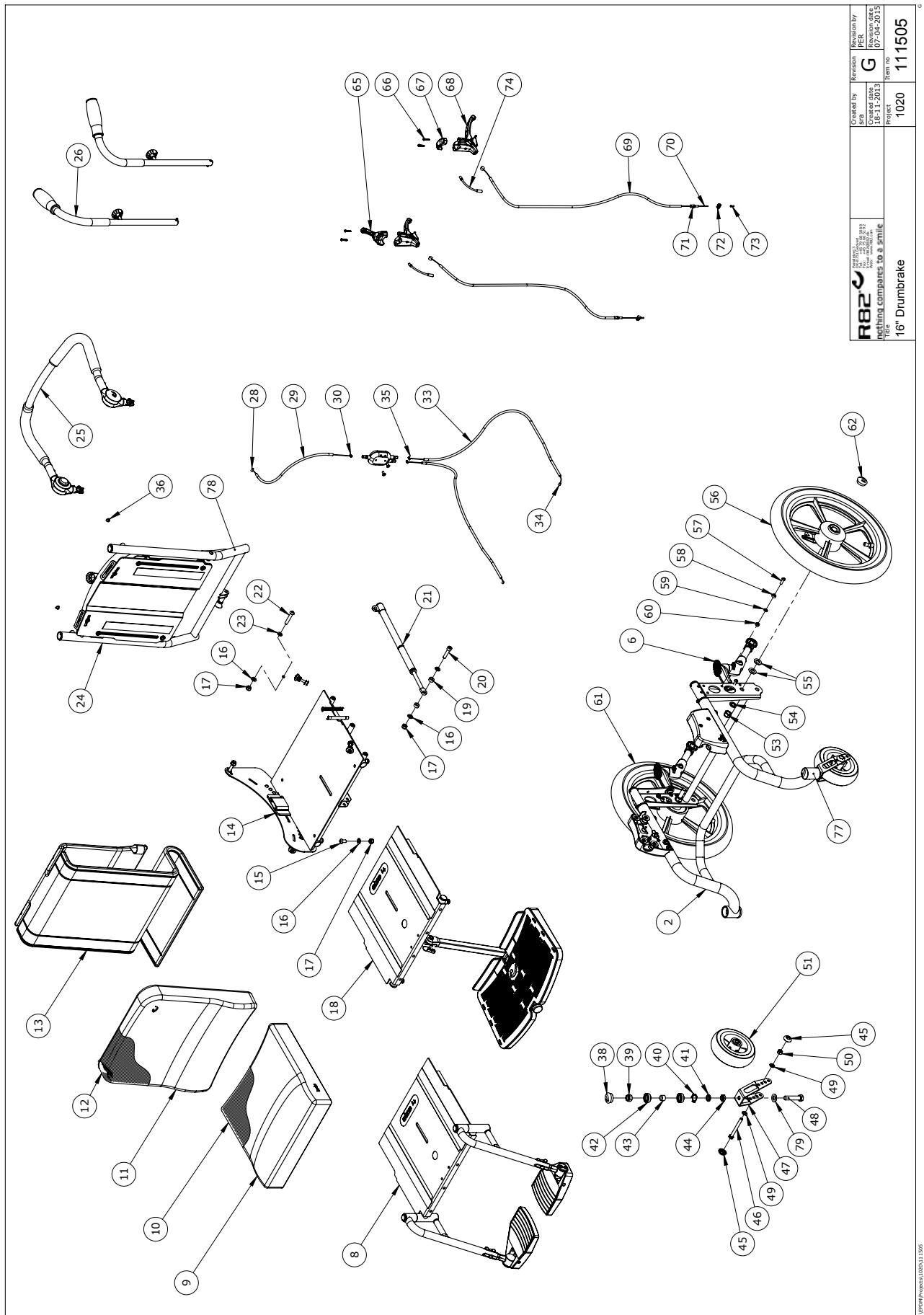


R82 <small>Nothing Compares to a Smile</small>	Created by	Revision	Revision by
	18-11-2013	G	107-04-2015
Project 1020	Item no		111505
	Title		16" Drumbrake

Position	Art nr.	Description	Quantity	Unit of measure	Ean code
9512121321 9531112221 9511111021 9542112933 9522112612 9533112832 9541111121 9542122531 9532122431 9531111733					
2	111718-110	Frame assembly size 1, white	1,00	Pieces	5707292585615
2	111718-131	Frame assembly size 1, red	1,00	Pieces	5707292585622
2	111718-135	Frame assembly size 1, grey	1,00	Pieces	5707292585646
2	111718-210	Frame assembly size 2, white	1,00	Pieces	5707292585653
2	111718-231	Frame assembly size 2, red	1,00	Pieces	5707292585677
2	107354-235	Frame assembly size 2, grey	1,00	Pieces	5707292585684
2	107354-310	Frame assembly size 3, white	1,00	Pieces	5707292585714
2	107354-331	Frame assembly size 3, red	1,00	Pieces	5707292585721
2	107354-335	Frame assembly size 3, grey	1,00	Pieces	5707292585691
2	111718-410	Frame assembly size 4, white	1,00	Pieces	5707292585738
2	107354-431	Frame assembly size 4, red	1,00	Pieces	5707292585745
2	107354-435	Frame assembly size 4, grey	1,00	Pieces	5707292585752
6	950953	Anti-tip device for Kudu	1,00	Set	5707292580931
6	950958	Anti-tip device for Kudu, long	1,00	Set	5707292589378
8	P-95151-260	Foot rests parted, size 2	1,00	Set	5707292588258
8	P-95151-360	Foot rests parted, size 3	1,00	Set	5707292588326
8	P-95151-460	Foot rests parted, size 4	1,00	Set	5707292588333
9+10	95742-160	Seat cushion size 1	1,00	Pieces	5707292586582
9+10	95742-260	Seat cushion size 2	1,00	Pieces	5707292586599
9+10	95742-360	Seat cushion size 3	1,00	Pieces	5707292586605
9+10	95742-460	Seat cushion size 4	1,00	Pieces	5707292586612
11+12	95741-160	Back cushion size 1	1,00	Pieces	5707292586667
11+12	95741-260	Back cushion size 2	1,00	Pieces	5707292586674
11+12	95741-360	Back cushion size 3	1,00	Pieces	5707292586681
11+12	95741-460	Back cushion size 4	1,00	Pieces	5707292586698
13	95744-160	Cushion covering size 1	1,00	Pieces	5707292586827
13	95744-260	Cushion covering size 2	1,00	Pieces	5707292586834
13	95744-360	Cushion covering size 3	1,00	Pieces	5707292586841
13	95744-460	Cushion covering size 4	1,00	Pieces	5707292586858
14	115022-1	Seat assembly size 1	1,00	Pieces	5707292605207
14	115022-2	Seat assembly size 2	1,00	Pieces	5707292605245
14	105023-3	Seat assembly size 3	1,00	Pieces	5707292581792
14	105023-4	Seat assembly size 4	1,00	Pieces	5707292581808
15	0661500	Button head screw 8*16 stainl.	1,00	Pieces	5707292139559
16	1105080	Stainless washer 8mm	5,00	Pieces	5707292140364
17	1631080	Stainless lock nut 8mm	4,00	Pieces	5707292141453
18	P-95152-160	Foot plate, centrally mounted, size 1	1,00	Pieces	5707292588340
18	P-95152-260	Foot plate, centrally mounted, size 2	1,00	Pieces	5707292588364
18	P-95152-360	Foot plate, centrally mounted, size 3	1,00	Pieces	5707292588371
18	P-95152-460	Foot plate, centrally mounted, size 4	1,00	Pieces	5707292588388
19	1350015	Dist.piece,black 7*Ø15*Ø8,2mm	2,00	Pieces	5707292467225
20	0609840	Stainless insex screw 8*40	1,00	Pieces	5707292138613
21	2207060	Spring Stopper Adjuster, for size 1	1,00	Pieces	5707292584175
21	2207070	Spring stopper adjuster, for size 2	1,00	Pieces	5707292584199
21	2207080	Spring Stopper Adjuster, for size 3+4	1,00	Pieces	5707292584205
22	0661145	Button head 8*45 FZB	2,00	Pieces	5707292139535
23	1310711	Nylon washer 15*8,4*1,5mm	2,00	Pieces	5707292429933
24	107345-110	Back assembly size 1, White	1,00	Pieces	5707292580450
24	107345-131	Back assembly size 1, Red	1,00	Pieces	5707292585394


24	107345-135	Back assembly size 1, Grey	1,00	Pieces	5707292585400
24	107345-210	Back assembly size 2, White	1,00	Pieces	5707292580467
24	107345-231	Back assembly size 2, Red	1,00	Pieces	5707292585547
24	107345-235	Back assembly size 2, Grey	1,00	Pieces	5707292585554
24	107345-310	Back assembly size 3, White	1,00	Pieces	5707292580474
24	107345-331	Back assembly size 3, Red	1,00	Pieces	5707292585561
24	107345-335	Back assembly size 3, Grey	1,00	Pieces	5707292585585
24	107345-410	Back assembly size 4, White	1,00	Pieces	5707292580481
24	107345-431	Back assembly size 4, Red	1,00	Pieces	5707292585592
24	107345-435	Back assembly size 4, Grey	1,00	Pieces	5707292585608
25	95629-1	Push brace, size 1	1,00	Pieces	5707292586513
25	95629-2	Push brace size 2	1,00	Pieces	5707292586537
25	95629-3	Push brace size 3	1,00	Pieces	5707292586544
25	95629-4	Push brace size 4	1,00	Pieces	5707292586551
26	95370-10	Push handles	1,00	Set	5707292586568
28	0371853	Strained wires 1,5mm 190 cm.	1,00	Pieces	5707292135889
29	5002998	Cable for new rabbit, 39 cm	1,00	Pieces	5707292436016
30	0388006	Screwnipple 5*7,5mm	1,00	Pieces	5707292136114
33	5001600	Cable for back, 52 cm	2,00	Pieces	5707292154491
33	5002756	Cable for back, 62 cm	2,00	Pieces	5707292413673
34	0371854-1	Soft inner cable for Stingray	2,00	Pieces	5707292536945
35	0388010	Screwnippel 7*7mm	2,00	Pieces	5707292136121
36	1310528	Covering stud, black 13mm	2,00	Pieces	5707292470348
38	2985000	Pressureplug black soft rubber	2,00	Pieces	5707292145314
39	1608000	Lock nut 12mm, large pack	2,00	Pieces	5707292141392
40	6108150	Spring screw 29,57*17,48	2,00	Pieces	5707292378200
41	111853	Washer Ø12,2 x Ø20 x 3,7	2,00	Pieces	5707292587374
42	2985210	Ball bearing	4,00	Pieces	5707292379412
43	100643	Distance Bush	2,00	Pieces	5707292552105
44	1760012	Kontranut M12,fzb	2,00	Pieces	5707292519078
45	109378	Wheel protector	4,00	Pieces	5707292581303
46	1907150	Steel bolt 8*75mm	2,00	Pieces	5707292142474
47	77980	FRONT FORK 20 - 22"	2,00	Pieces	5707292534132
48	1902372	Steel bolt M12*55 FZB	2,00	Pieces	5707292519436
49	1105080	Stainless washer 8mm	2,00	Pieces	5707292140364
50	1631080	Stainless lock nut 8mm	2,00	Pieces	5707292141453
51	3007736	Wheel 140*40mm with rubber	2,00	Pieces	5707292558213
53	1608000	Lock nut 12mm, large pack	2,00	Pieces	5707292141392
54	1101600	Center spring washer 12mm	2,00	Pieces	5707292140197
55	1103500	Fazet washer 12mm	2,00	Pieces	5707292140326
56+71+72	3201630	Wheel, 16" black, left	1,00	Pieces	5707292587282
56+71+72	3201610	Wheel 16" with drum brake, black, left	1,00	Pieces	5707292587268
57	0641620	Button head screw 6*20	2,00	Pieces	5707292139276
57	0659550	Button head M6*20, stainless	2,00	Pieces	5707292139467
58	104387	Spacer	2,00	Pieces	5707292582553
59	1105060	Stainless washer 6mm	2,00	Pieces	5707292140357
60	1606500	Lock nut 6 mm	2,00	Pieces	5707292141361
60	1631060	Stainless lock nut 6mm	2,00	Pieces	5707292141446
61+71+72	3201620	Wheel, 16" black, right	1,00	Pieces	5707292587275
61+71+72	3201600	Wheel 16" with drum brake, black, right	1,00	Pieces	5707292587237
62	2704300	Black hub cap for rolko wheel	2,00	Pieces	5707292144515

65+66	3503532	Gas pump lever without clamp	1,00	Pieces	5707292427755
67	3503536	Clamp w/screws	1,00	Pieces	5707292427762
68	3503535	Brake parking lever wo/clamp	2,00	Pieces	5707292413536
69	5002751	Cable for brake, size 3+4	2,00	Pieces	5707292413642
69	5002750	Cable for brake, size 1+2	2,00	Pieces	5707292413635
70	0371853	Strained wires 1,5mm 190 cm	2,00	Pieces	5707292135889
73	0387000	End nipple for outer casing	2,00	Pieces	5707292136091
74	0380060	Flexible v-brake pipe.	2,00	Pieces	5707292425195
77	0494522	Reflex 22mm, grey	2,00	Pieces	5707292137234
78	0493437	Label "exclamation mark"	2,00	Pieces	5707292136909

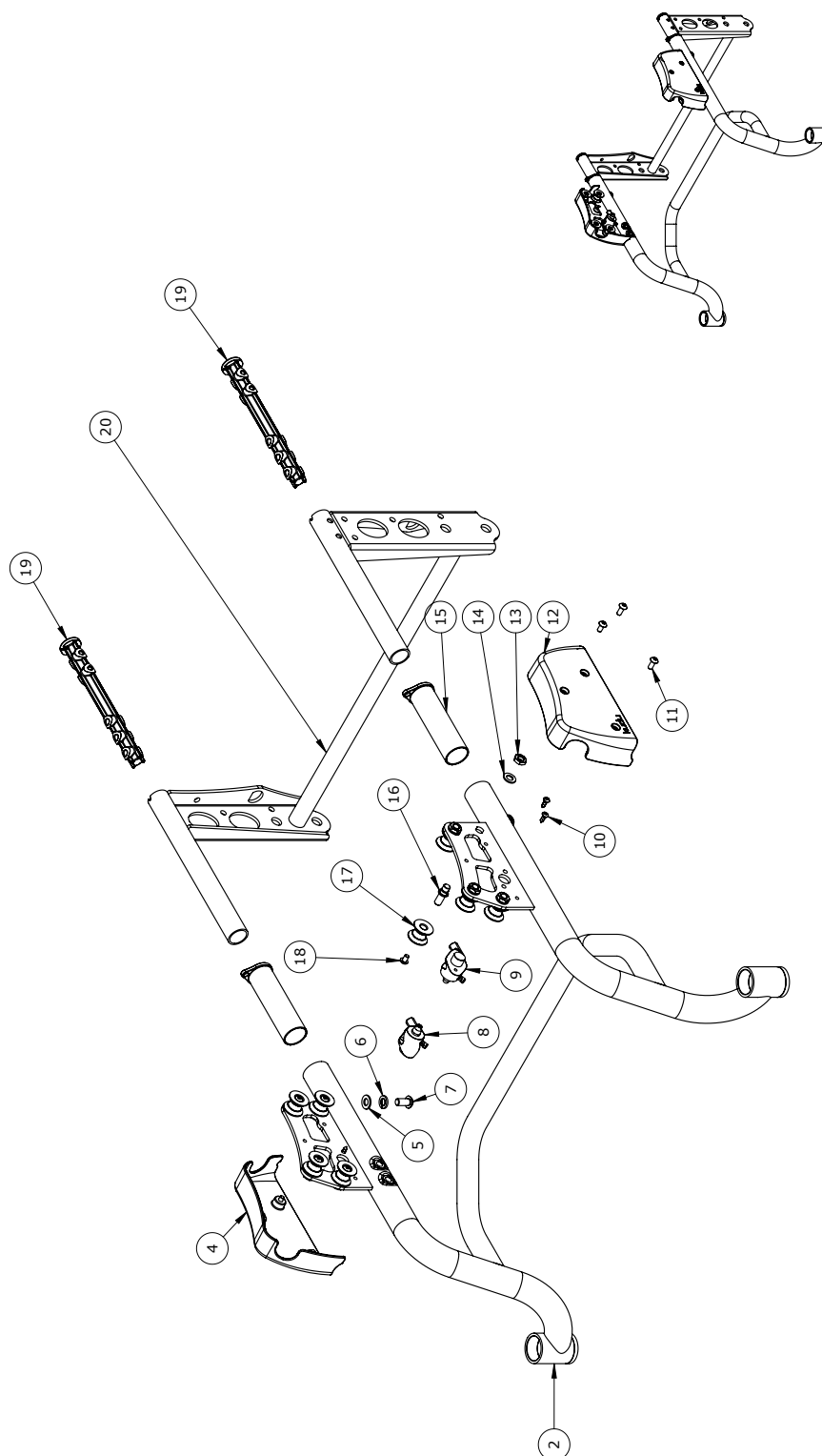


 nothing companies to a smile R82 245 79 75 15 245 79 75 15 Email: info@r82.com Phone: +31 (0) 45 79 75 15	Created by	Revision	Revision by
	SFA	SFA	SFA
	Created date	Created date	Created date
19-12-2013	19-12-2013	22-12-2014	
Project	Item no		
1020	111718		
Frame Assembly 12 1/2" & 16"			

[illegible]

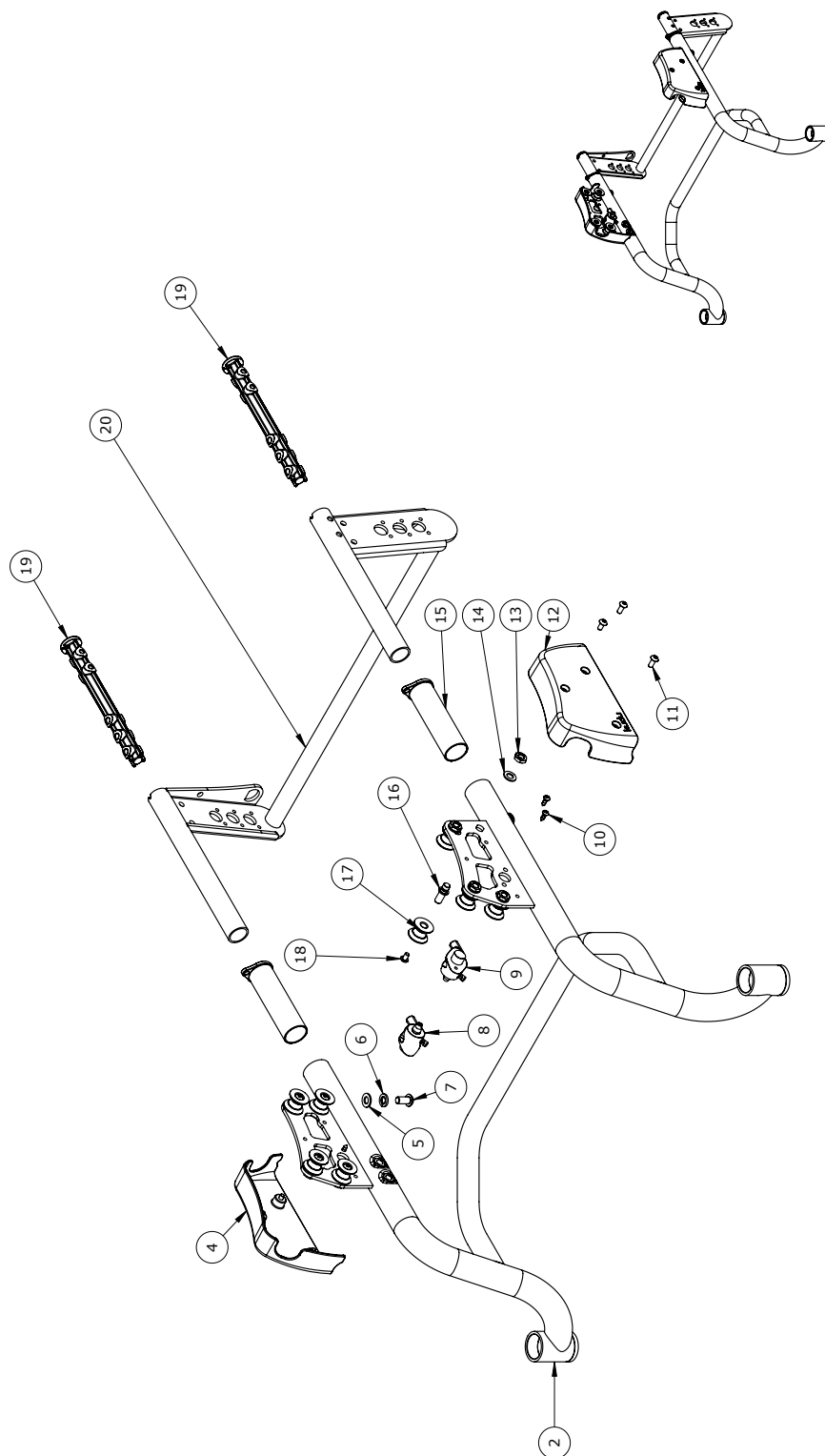

R82 R82 A/S Røddingvej 3 8751 Gedved Denmark	Created by Admin	Revision C	Revision by Admin
	Created date 31-03-2011	Revision date 24-03-2015	Revision no. 107354
Project Frame Assembly 22" - 24"		Item no. 1020	


[illegible]

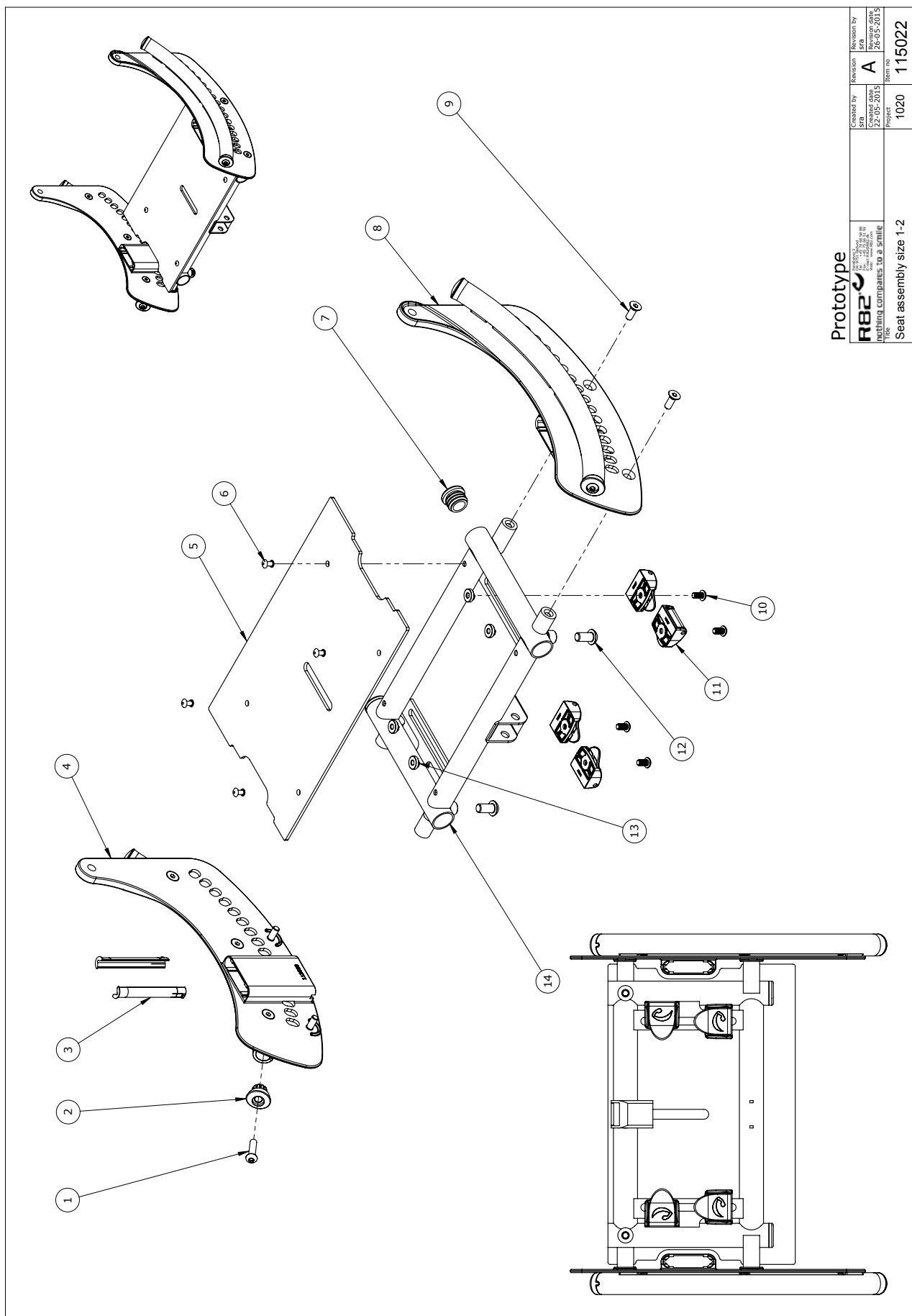


Prototype

R82 [®] nothing compares to a smile	Created by SFB	Revision A	Reviewed by SFB
	Created date 22-05-2015	Review date 26-05-2015	Item no. 115022
Project 1020		Seat assembly size 1-2	

[illegible]


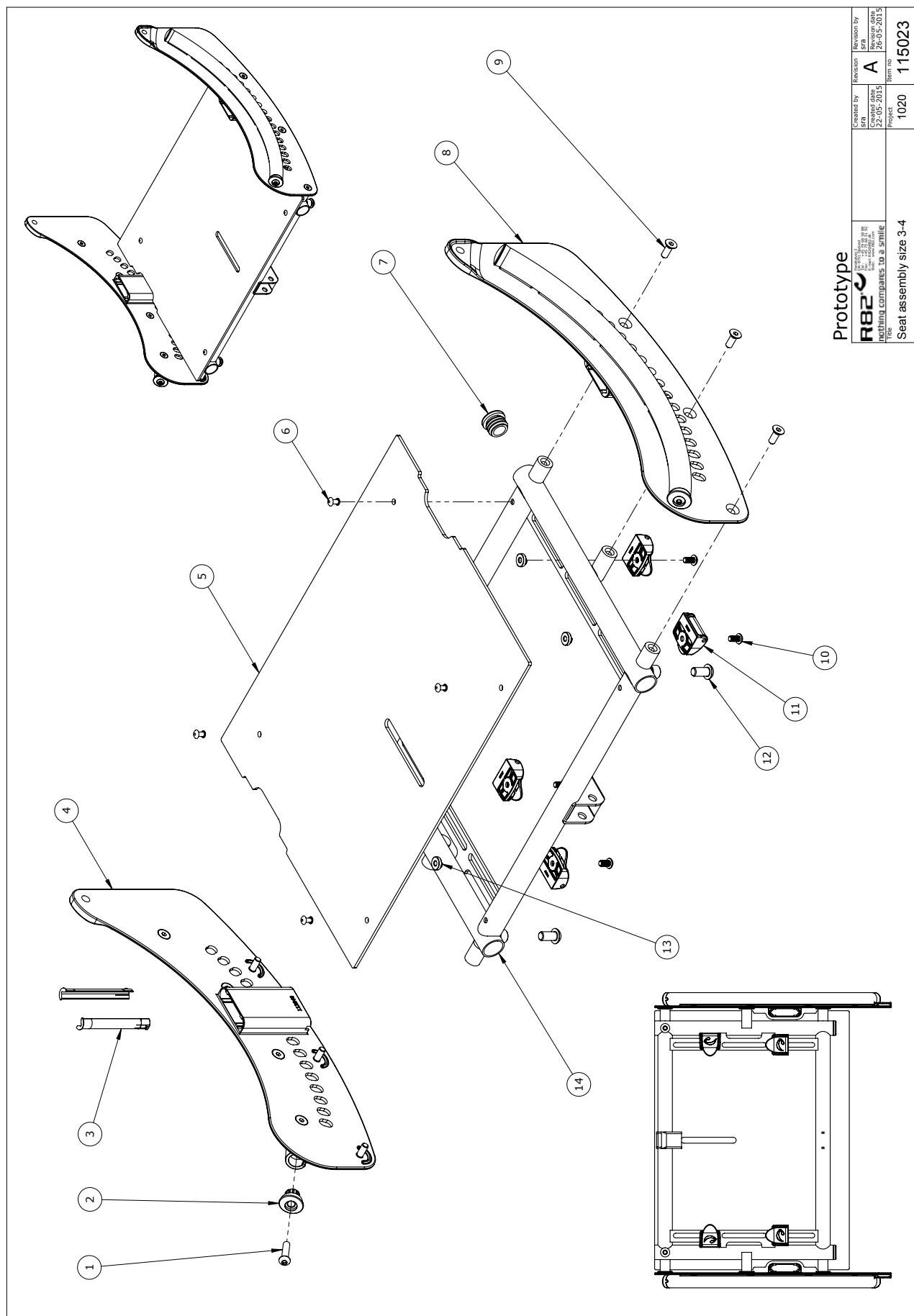


[illegible]

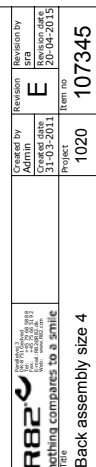


R82 A/S
Parallelvej 3
8751 Gedved
Denmark


1	1	113904	Seat extension plate size 2
2	2	30mm velcro	
3	3	0506509	RT4x10-7.5-SS
4	2	2501100	FRR22-W
5	1	3502516	Handle M8-45x15
6	1	1100495	W8*12*2-SP-SS
7	1	108743	Seat extension size 2
8	1	1607000	NUT8*8-FZB-LC
9	2	0661500	M8*16-BH-SS-A-A2
10	1	75261	Rod size 1 for footrest
11	1		Footsupport size 2
12	1	2504500	FRS20x20-B

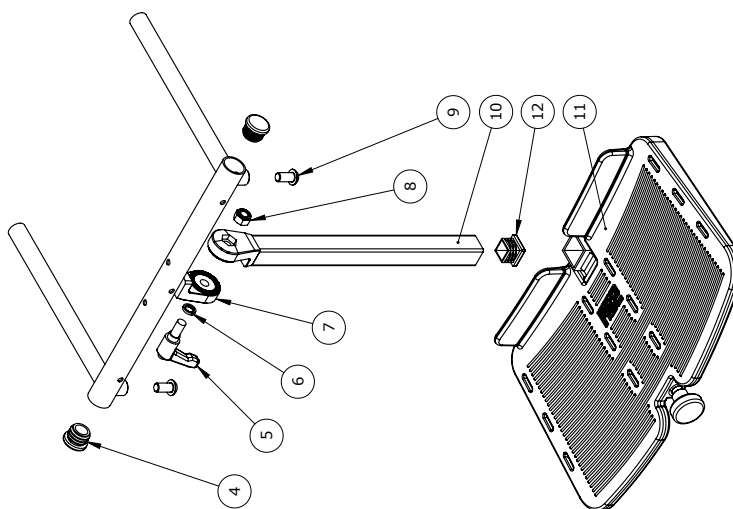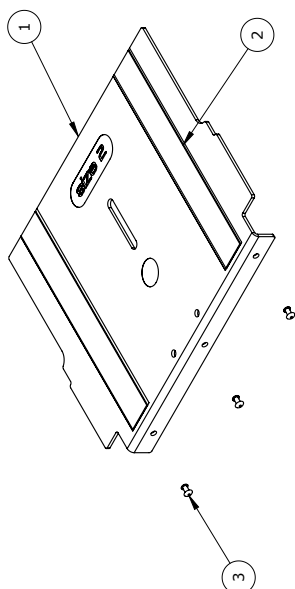


 <small>nothing compares to a smile</small>	Created by	Revision	Revised by
	10-12-2013	D	PH
	Project	Item no	
	Foot support model Wombat size 1-2	1020	111561


R82 A/S
Parallelvej 3
8751 Gedved
Denmark

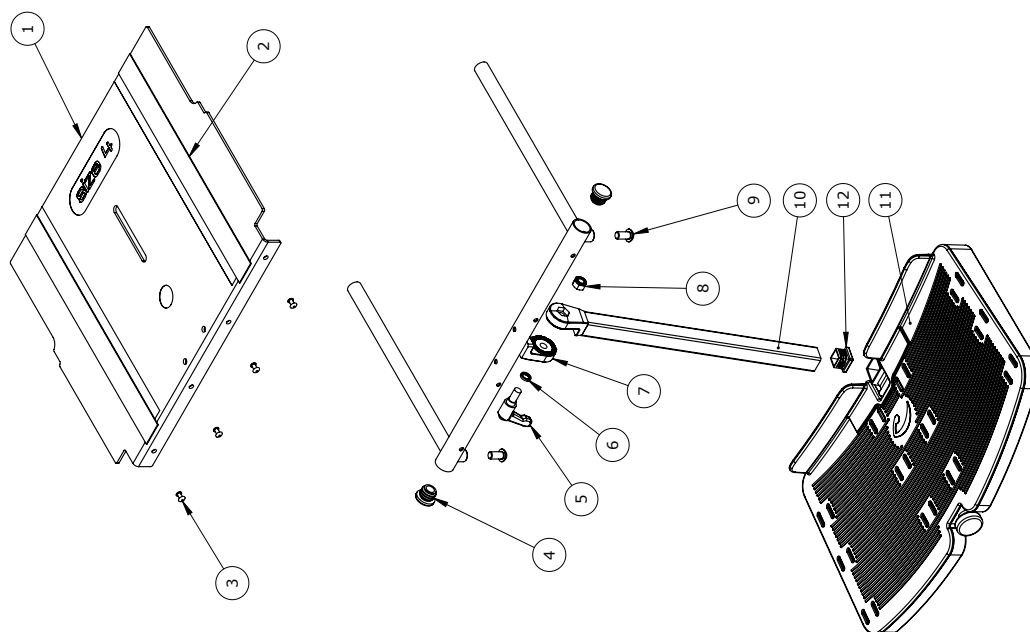


 R82 <small>© 2011 R82 LLC 10000 R82 Blvd Suite 100 Dallas, TX 75243 Tel: 214-343-8888 Fax: 214-343-8889 Email: info@r82.com</small>	nothing compares to a smile <small>© 2011 R82 LLC</small>	Created by Admin	Revision PER	Revision by D
		Created date 31-03-2011	Revision date 07-04-2015	
		Title Foot support model Wombat size 3-4		Project 1020


[illegible]

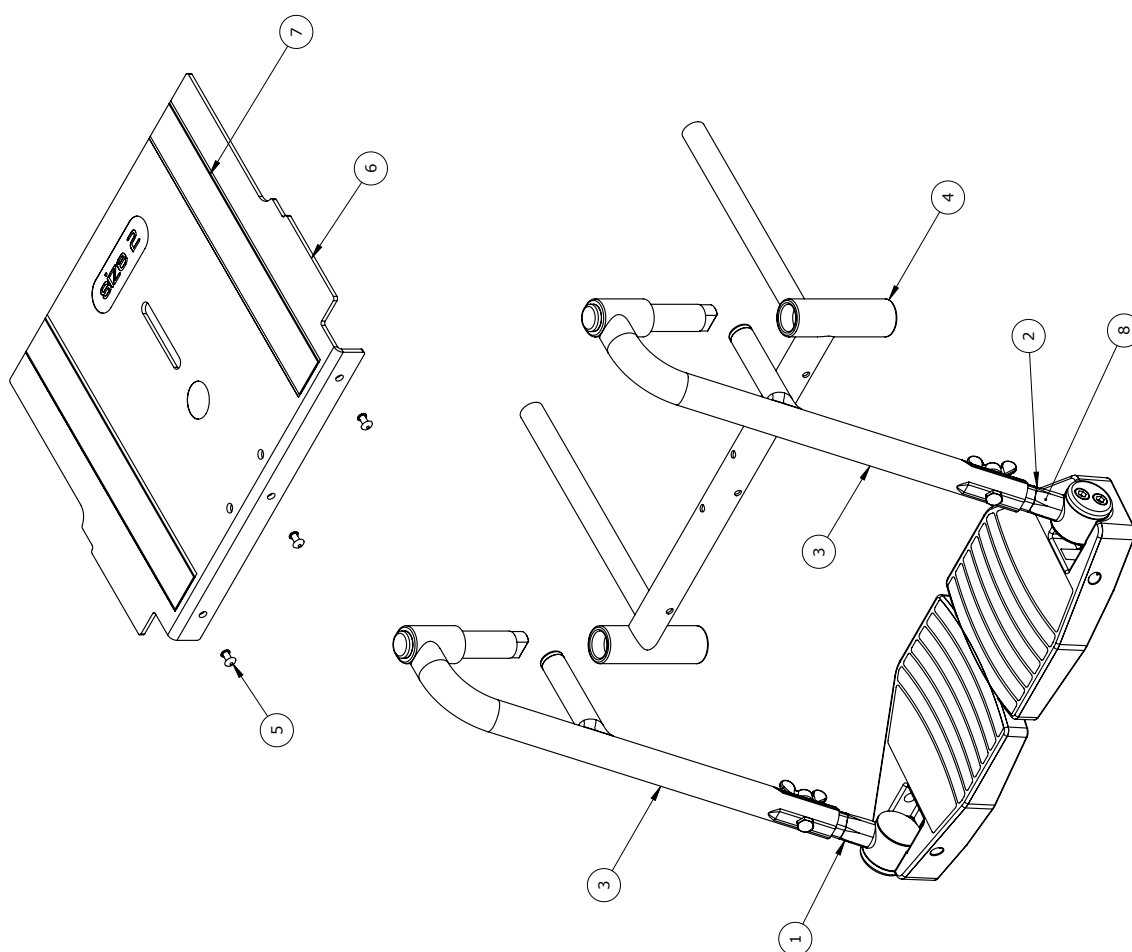


 R82 <small>© 2013 R82 LLC 10000 R82 Blvd Suite 100 Dallas, TX 75243-1000 Tel: 214-776-8812 Fax: 214-776-8813 Email: info@r82.com Web: www.r82.com</small>	nothing compares to a smile	Created by	Revision	Revision by
		sra	PER	E
Created date		Revision date		
28-05-2013		07-04-2015		
Title		Project	Item no	
Foot support model ETAC size 2		1020	111647	

[illegible]

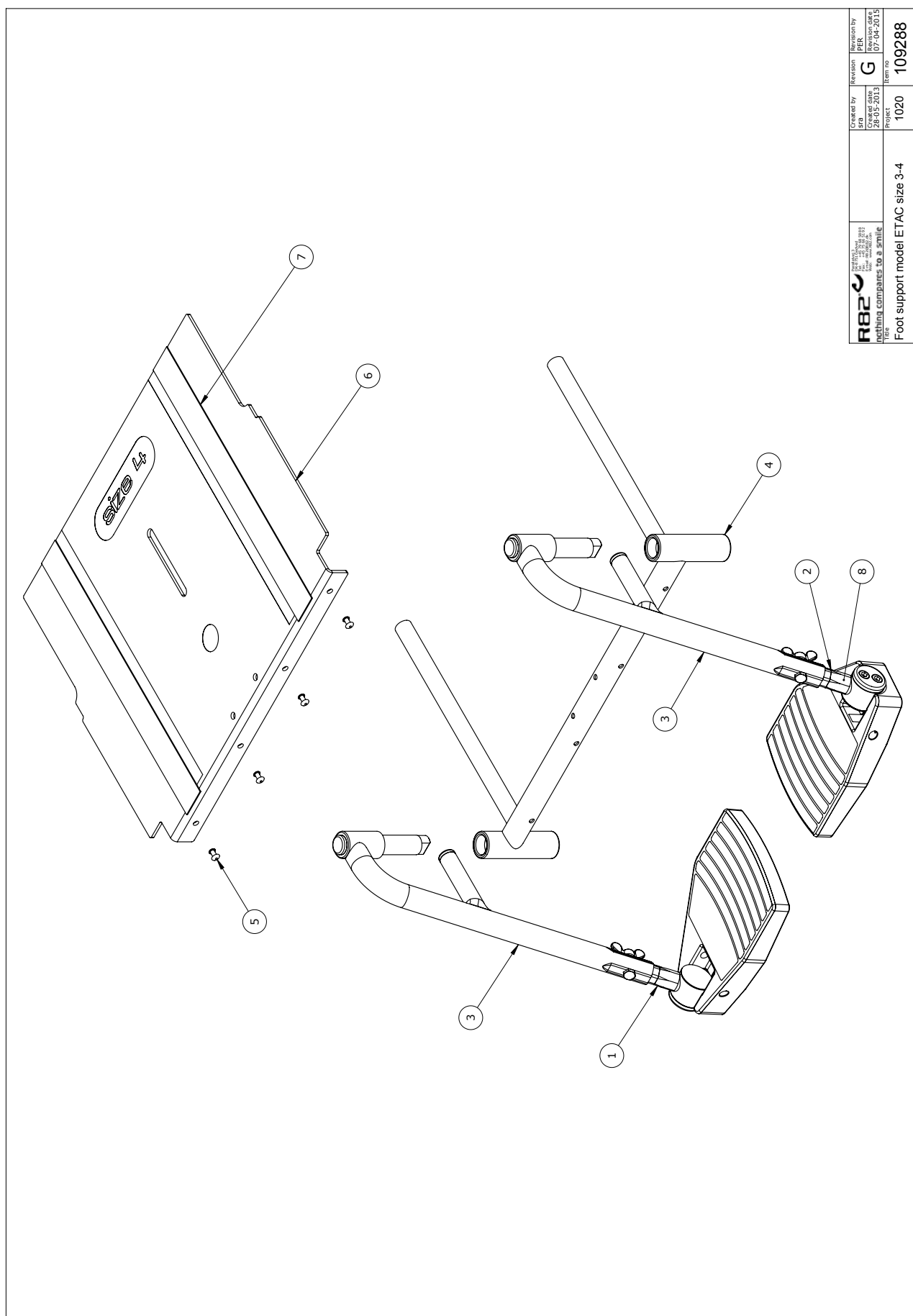



 Designing companies to a smile	Created by SFB	Revision G	Revision by PER	Revision date 07-04-2013	Item no 109288
Foot support model ETAC size 3-4					

R82 A/S
Parallelvej 3
8751 Gedved
Denmark

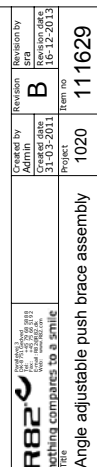


[illegible]

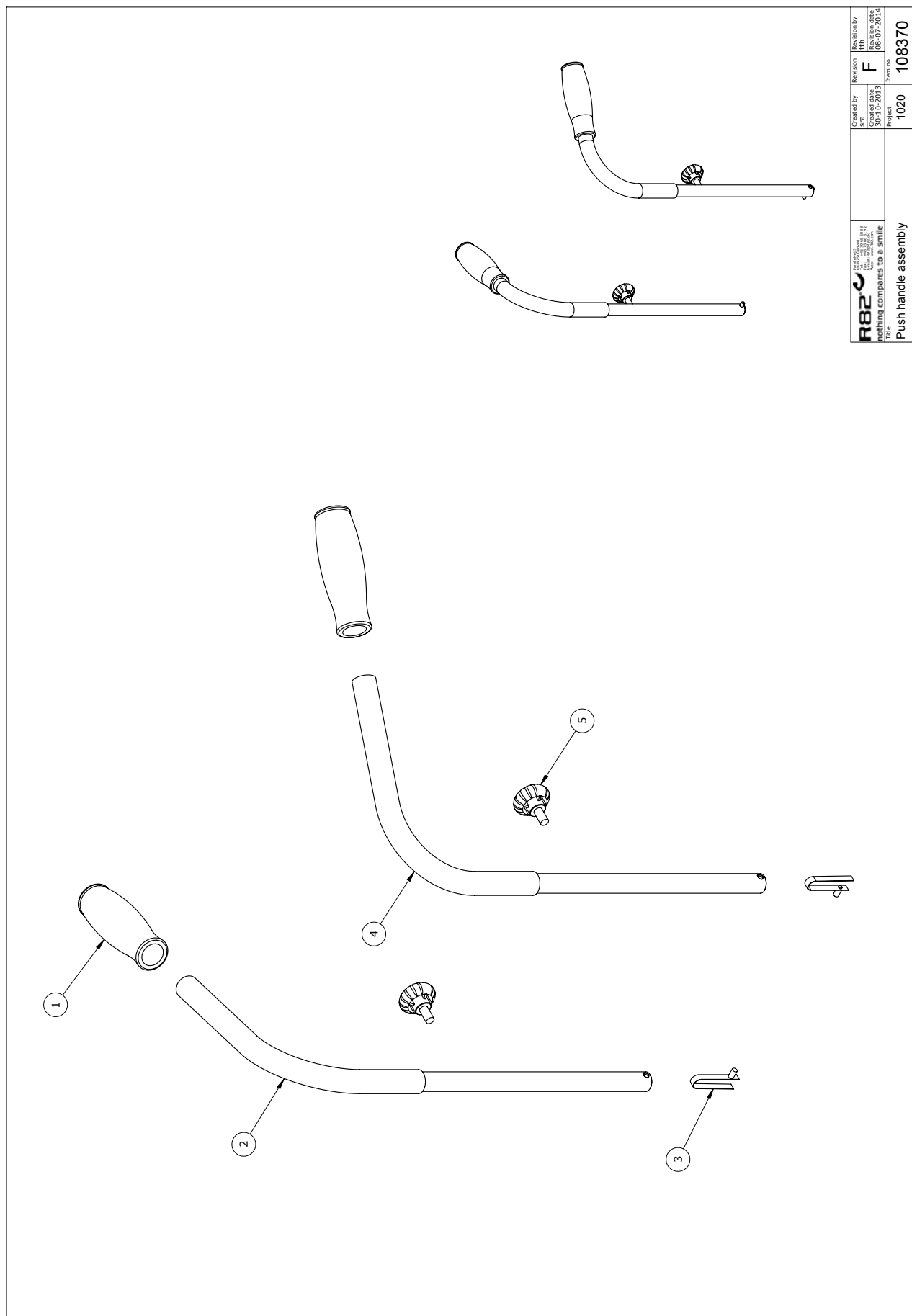


 nothing compares to a smile	Created by sra	Revision F	Revision by tth
	Created date 30-10-2013	Revision date 08-07-2014	
Title	Project	Item no	
Push handle assembly	1020	108370	


[illegible]

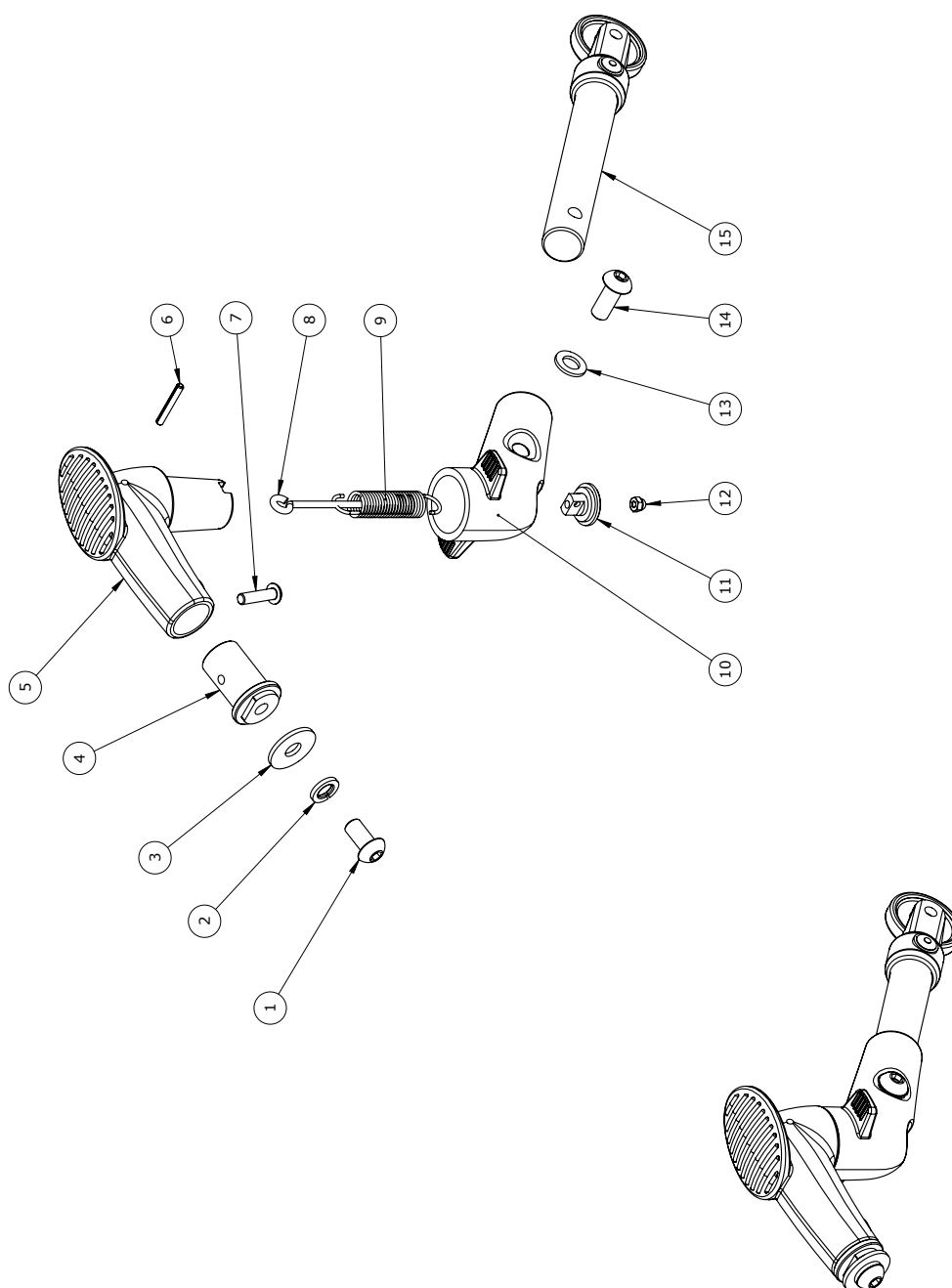


R82 <small>nothing compares to a smile</small> Anti tip device	Created by JYBA	Revision E	Revision by JYBA
	Created date 29-05-2013	Revision date 27-05-2015	Item no. 107379
	Project 1020		

[illegible]

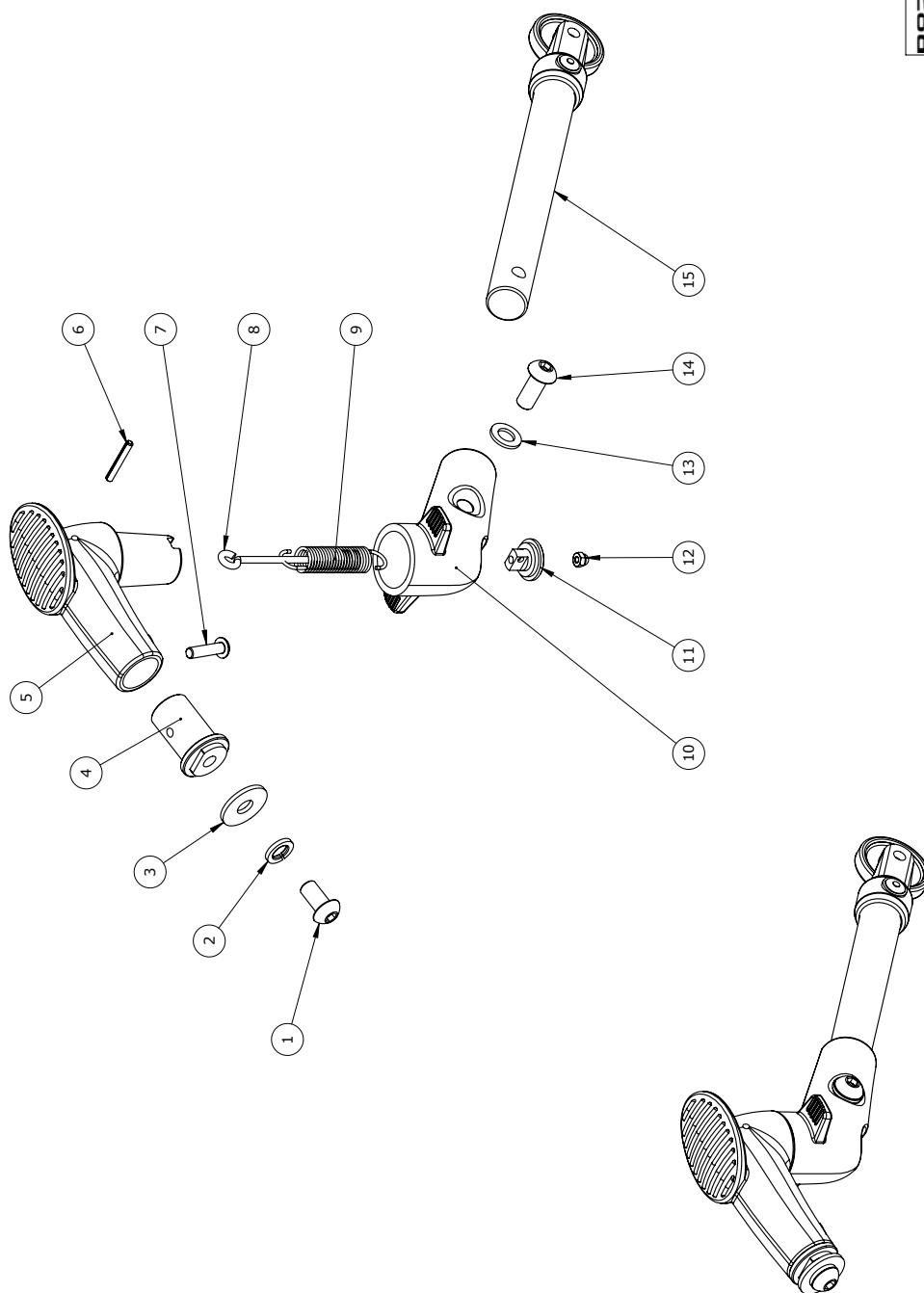


 nothing compares to a smile	 nothing compares to a smile	Created by		Revision	Revision by
		SFA		SFA	SFA
Title		Created date		Revision date	
Anti tip device (long-180mm)		29-05-2013		27-05-2015	
		Project		Item no	
		1020		112320	

R82 A/S
Parallelvej 3
8751 Gedved
Denmark

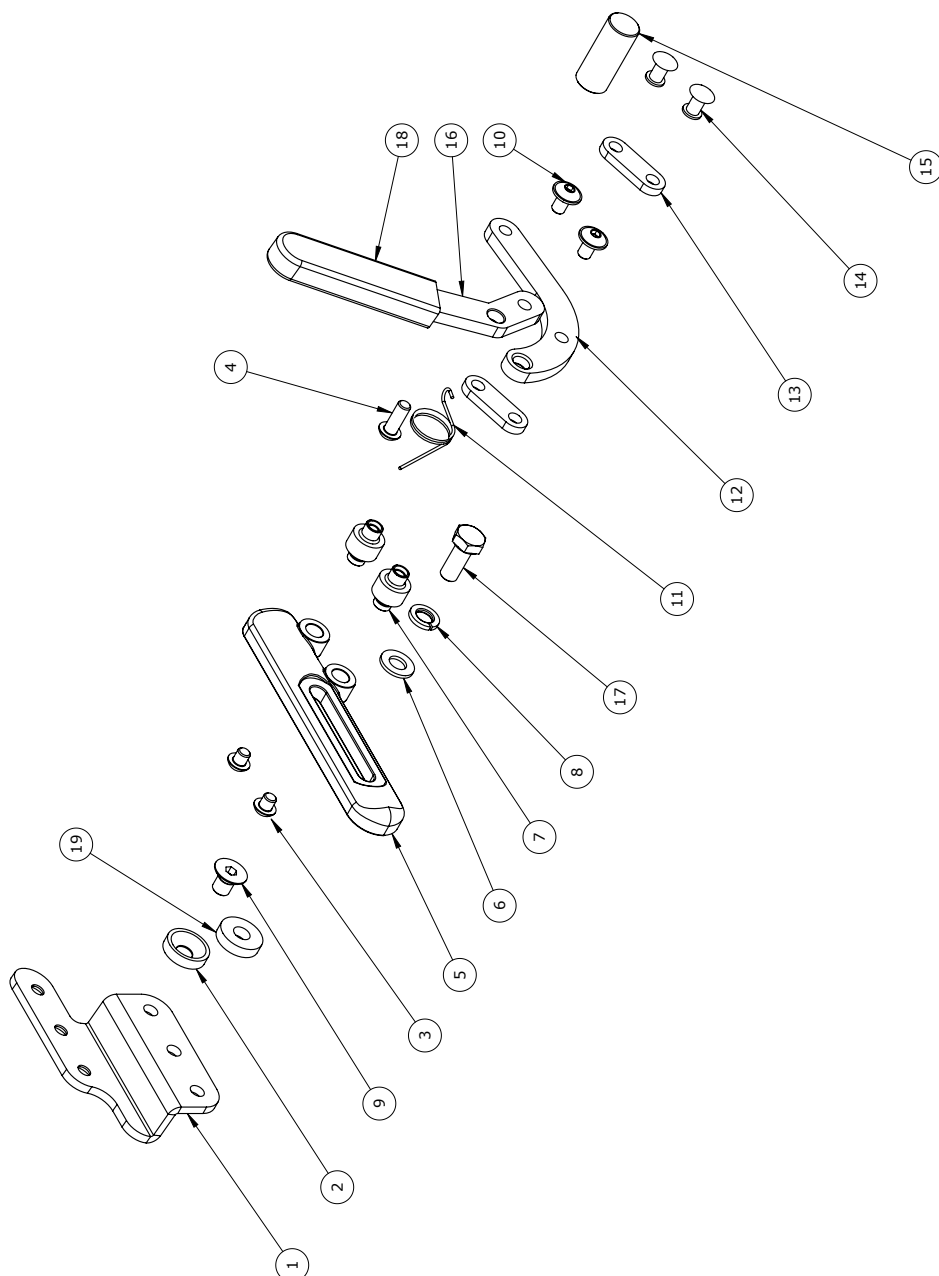


R82[®] <small>nothing compares to a smile</small>	R82 A/S 12000, 12000, 12000 12000, 12000, 12000	Checked by Admin	Revision C	Drawn by Era
		Created date 31-03-2011	Revision date 15-04-2015	Item no. 111859
Project Wheel brake size 1-2, right		Project no. 1020		


[illegible]

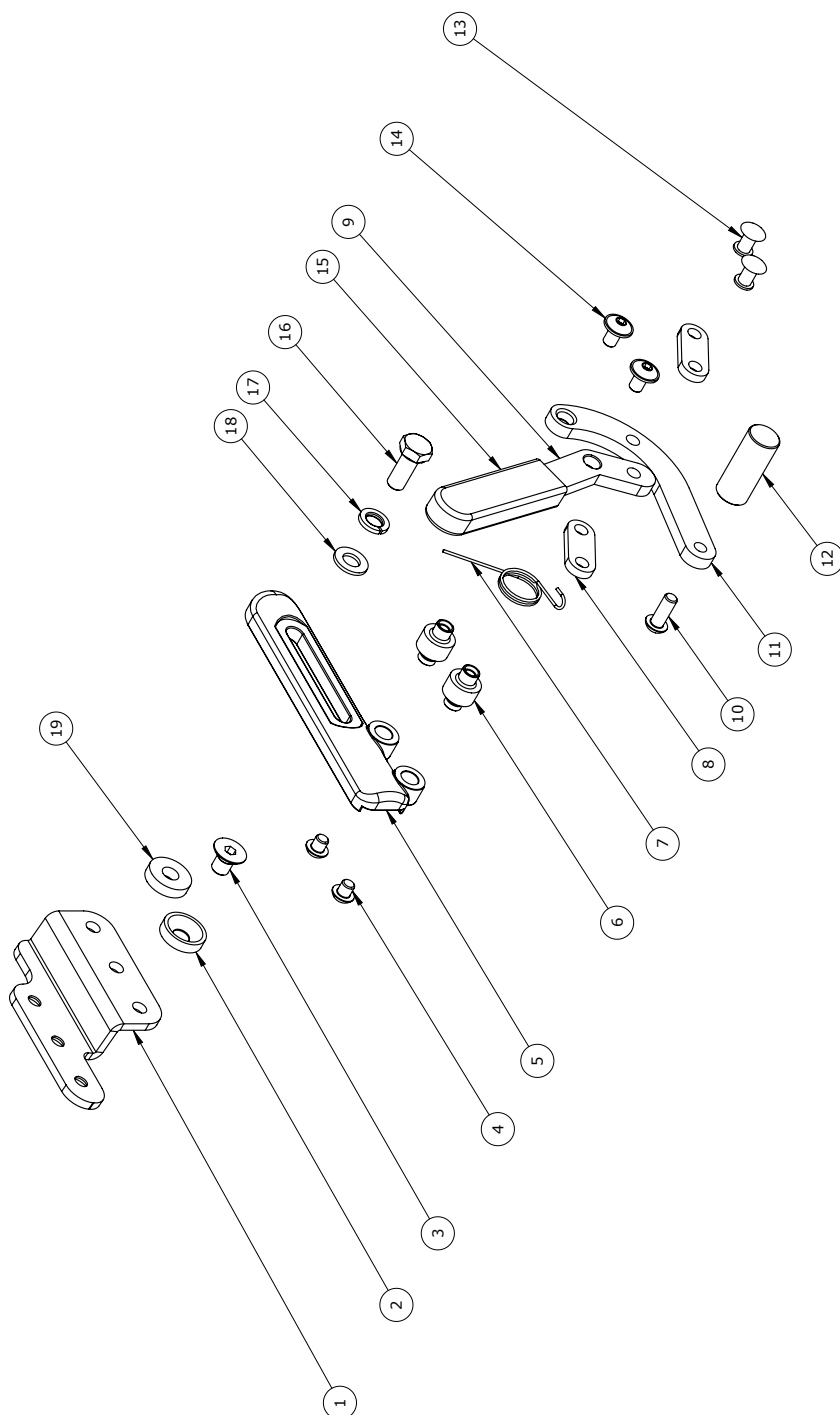


R82 [®] Nothing compares to a smile	Created by SFB	Revision C	Revision by SFB
Wheel brake size 1-2, left	1020	111860	13-04-2015

44

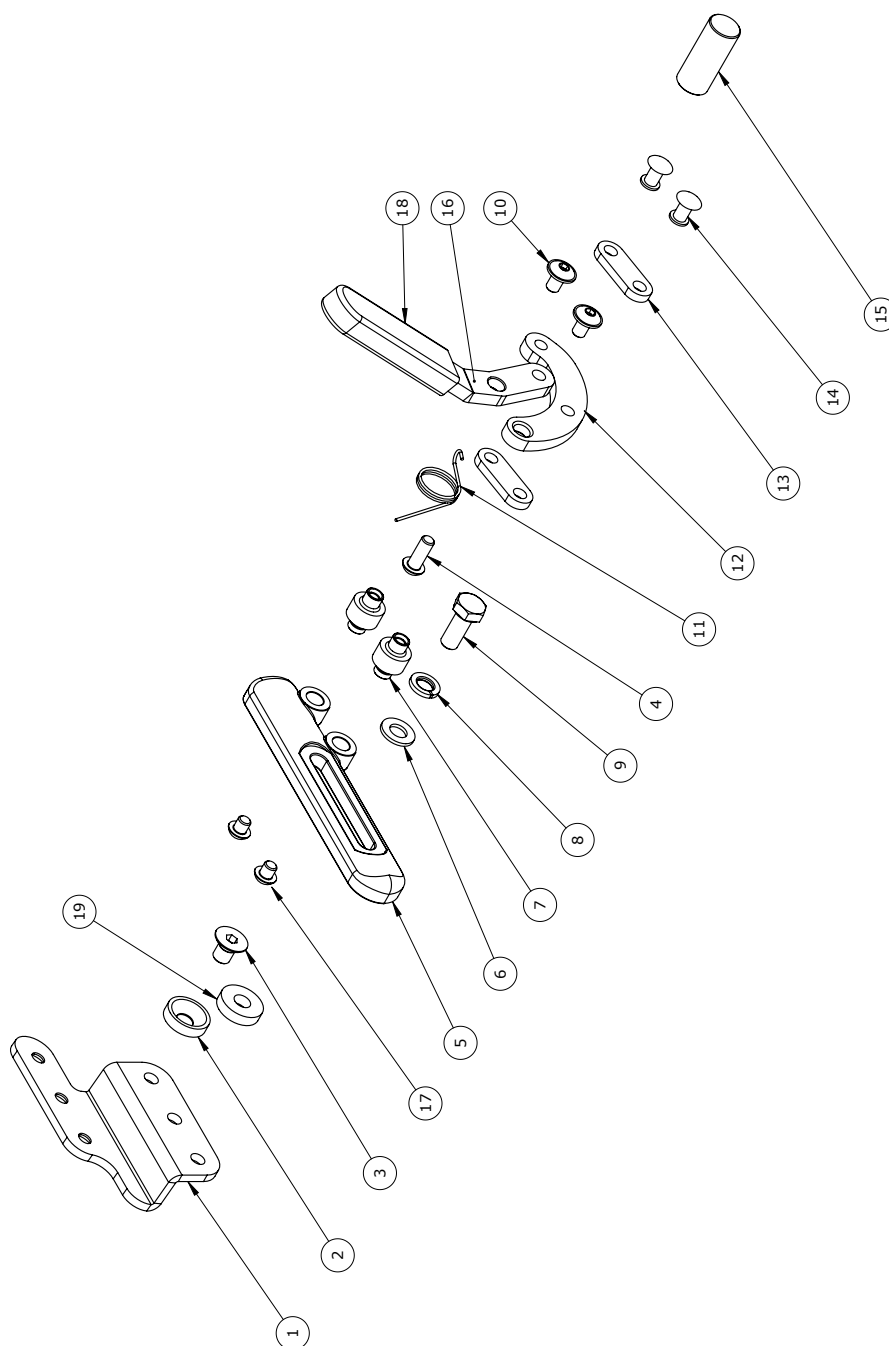

 Nothing compares to a smile	Created by Admin	Revision G	Revisions by Admin
			Revision date 15-04-2015
Wheel brake size 3-4, right	Project 1020	Item no 101994	

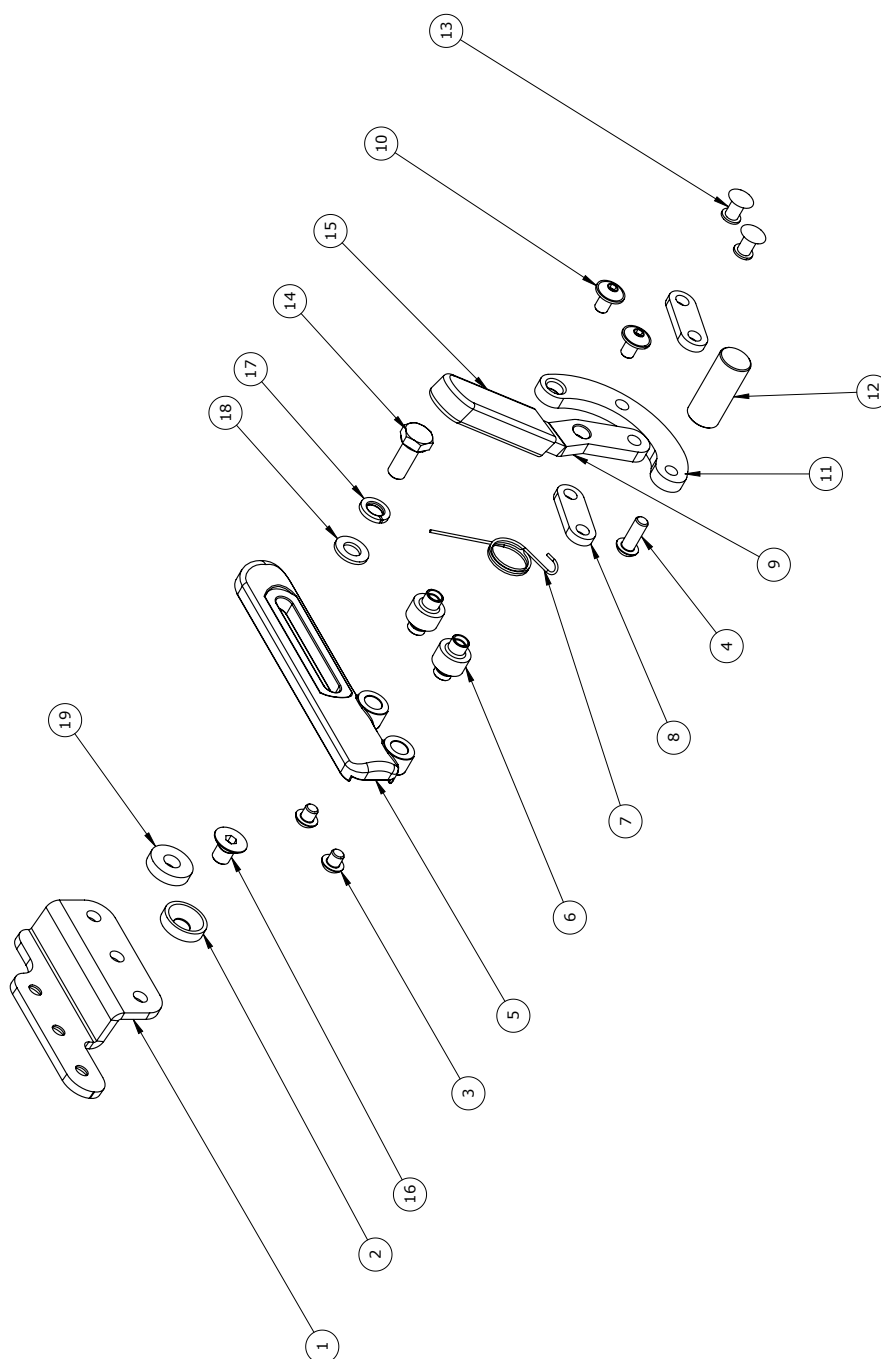

R82[®] <small>nothing compares to a smile</small>	<small>Product No.</small> 1025562	<small>Created by</small> Admin	<small>Revision</small> G	<small>Revision by</small> Era
	<small>Wheel brake size 3-4, left</small>	<small>Create date</small> 31-03-2011	<small>Revision date</small> 13-04-2013	<small>Item no.</small> 102562

[illegible]