180-400 mm (7-15½")

<40 kg (88 lb)

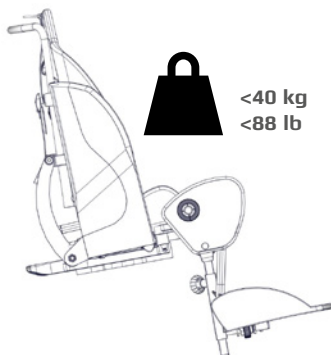
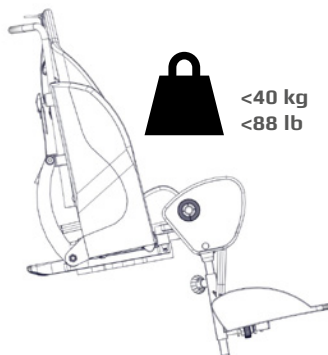
## Stingray seat User Guide

EN 12182

EN 12183


2017-11-rev.5

	3	ENGLISH DEUTSCH	28 - 30 31 - 33
	4	NEDERLANDS DANSK	34 - 36 37 - 39
	5	NORSK SVENSKA	40 - 42 43 - 45
	6 - 7	SUOMI ESPAÑOL	46 - 48 49 - 51
	8	FRANÇAIS ITALIANO	52 - 54 55 - 57
	9	PORTUGUÊS PORTUGUÊS DO BRASIL	58 - 60 61 - 63
	10 - 11	РОССИЯ POLSKI	64 - 66 67 - 69
	12	ČESKY 中國	70 - 72 73 - 75
	13	日本 ΕΛΛΗΝΙΚΗ	76 - 78 79 - 81
	14 - 15		
	16		
	17		
	18		
	19		
	20		
	21 - 25		
mm (inch)	26		
	27		


**Stingray 1****Stingray 2**

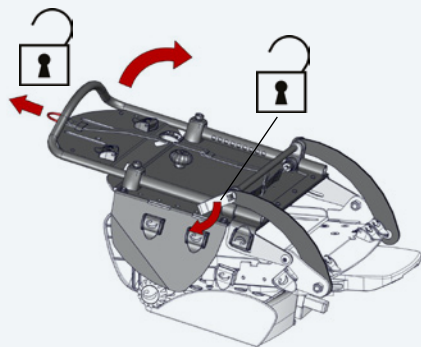
1.

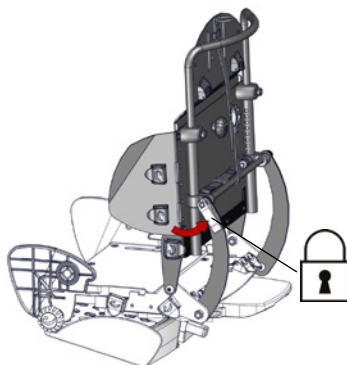
2.

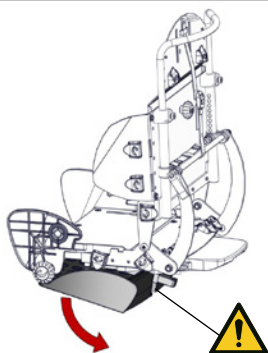
3.

 05

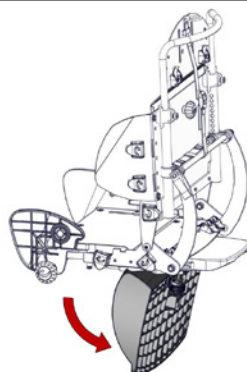
4.

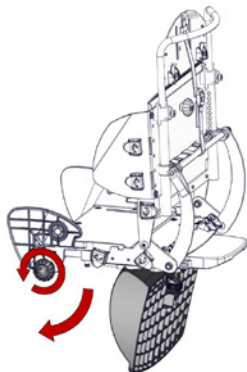5.

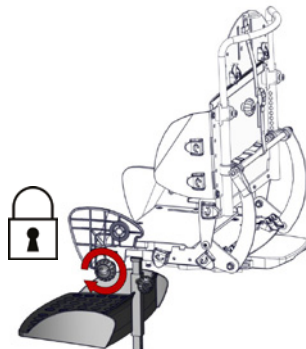
6.

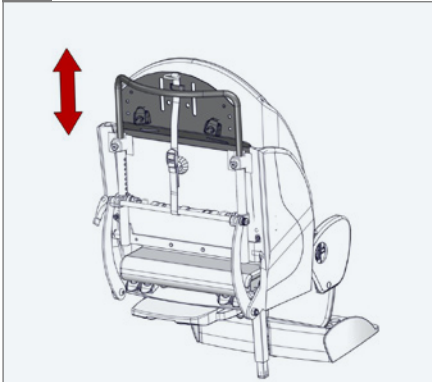
1.

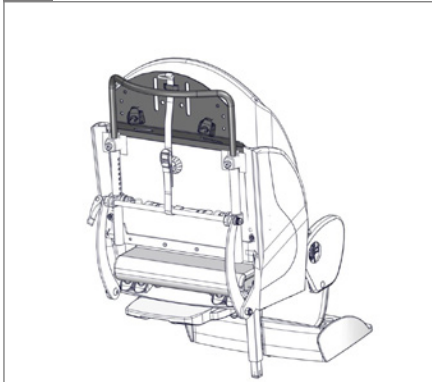
2.


3.

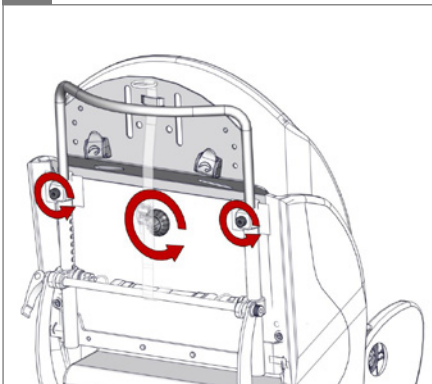⌀ 5 mm


4.

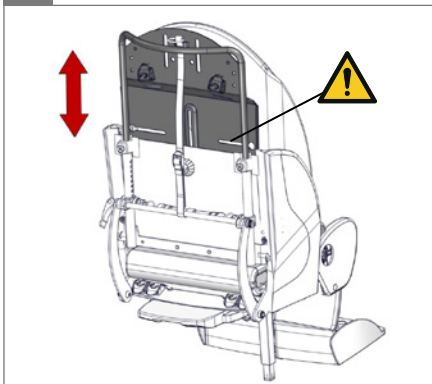⚠ 05


5.

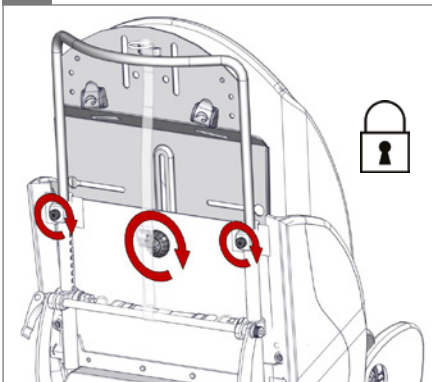⌀ 5 mm

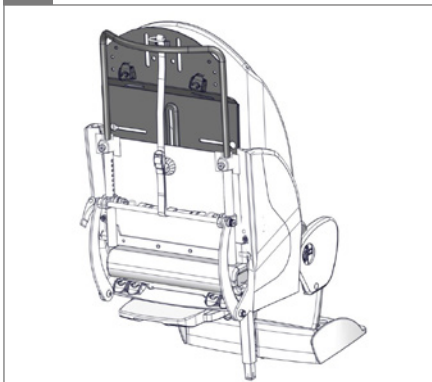
6.

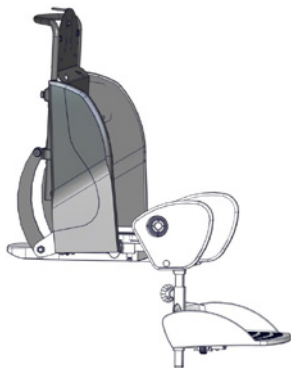
1.


2.

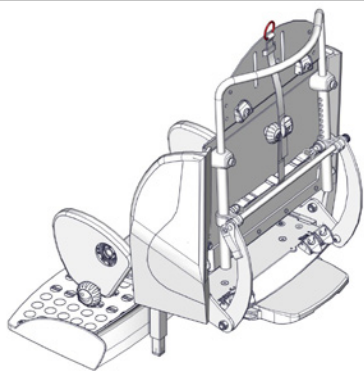
3.

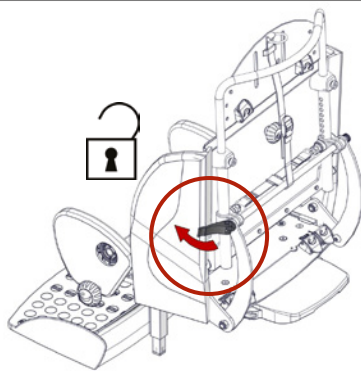
4.

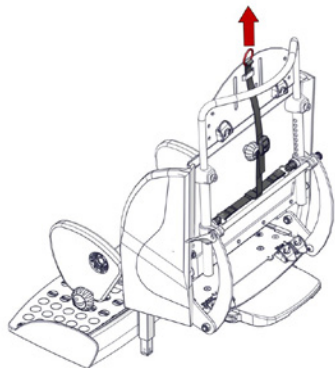5.

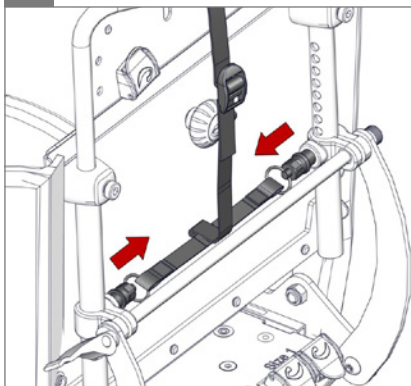
6.


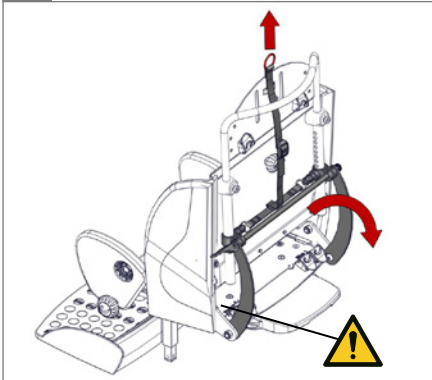
1.

 05

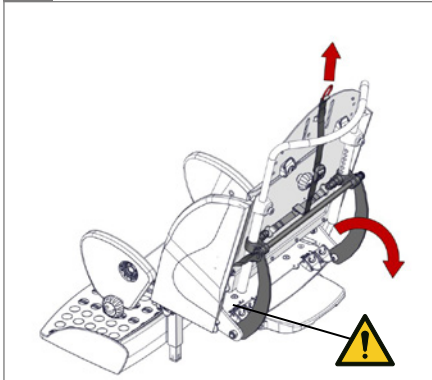
2.

 05

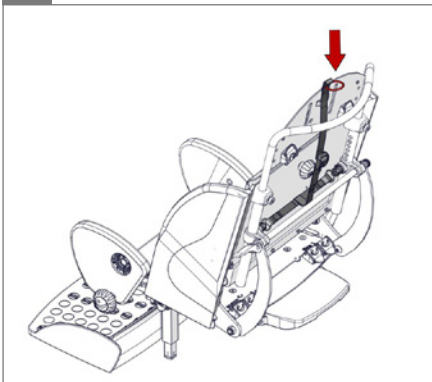
3.

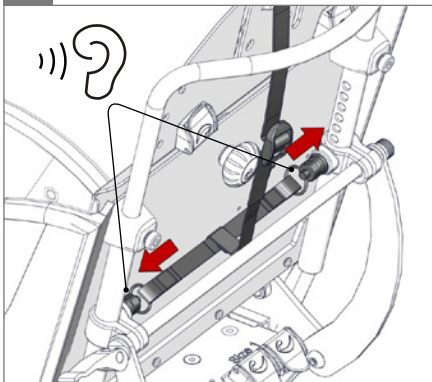
4.

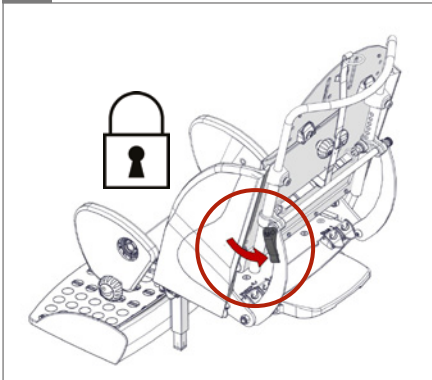5.

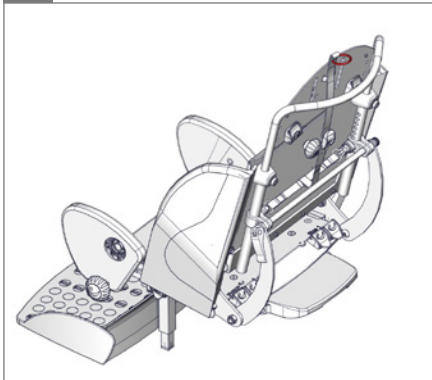
6.

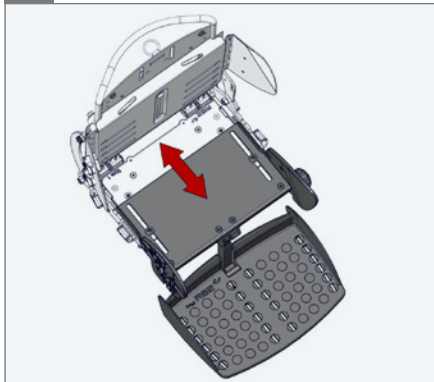1.


2.

⬡ 5 mm

3.

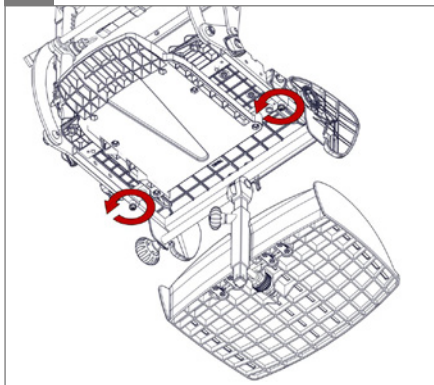⬡ 5 mm


4.


5.

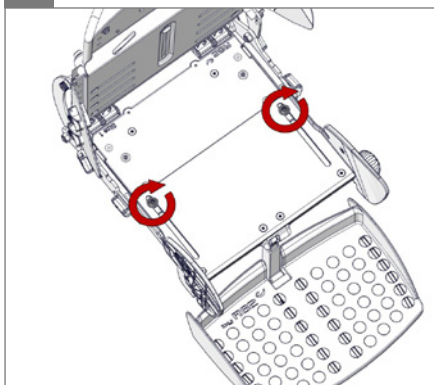⬡ 5 mm

6.

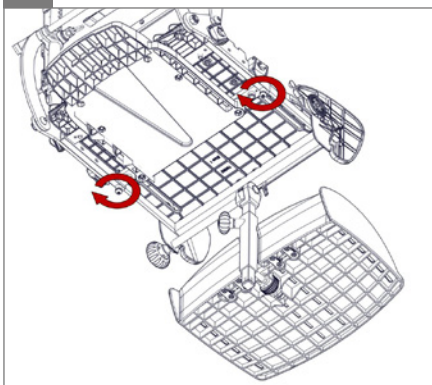
⬡ 5 mm


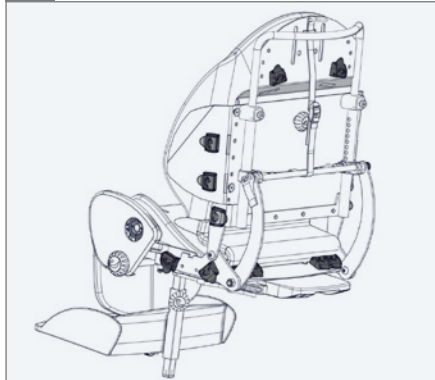
1.

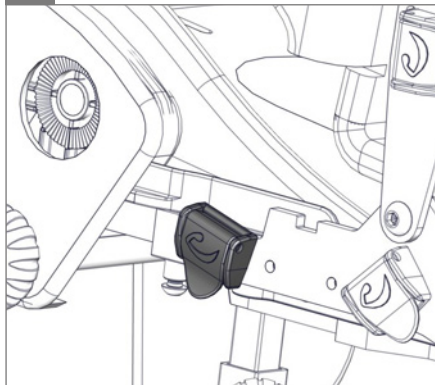
2.

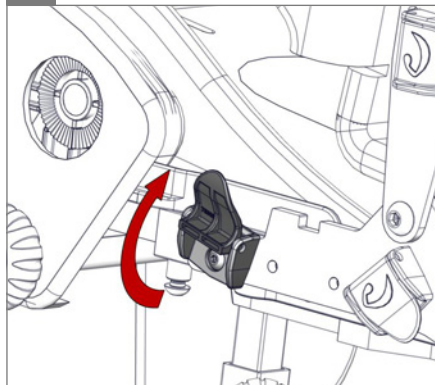
3.

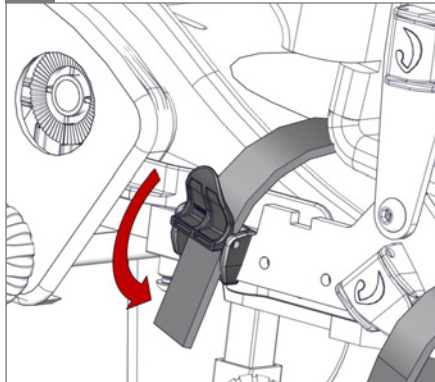4.

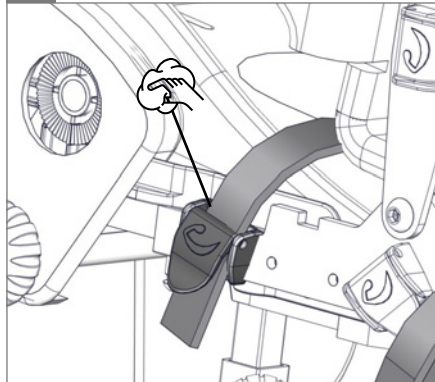5.



1.

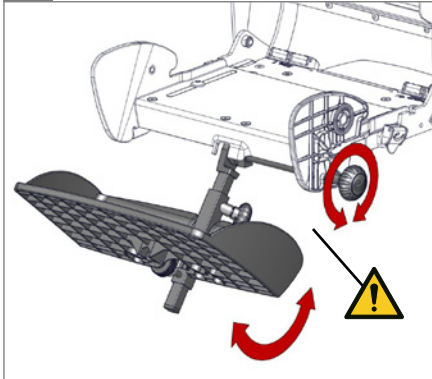
2.

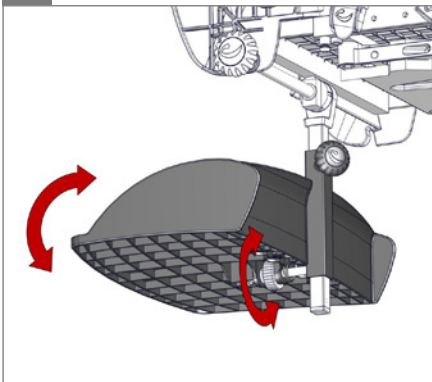
3.

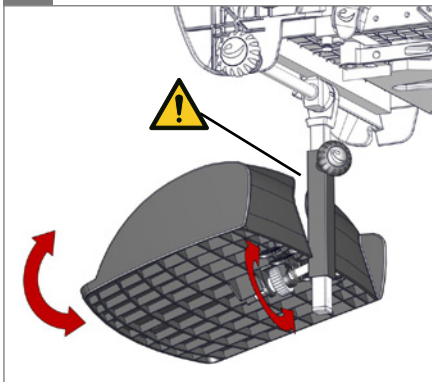
4.

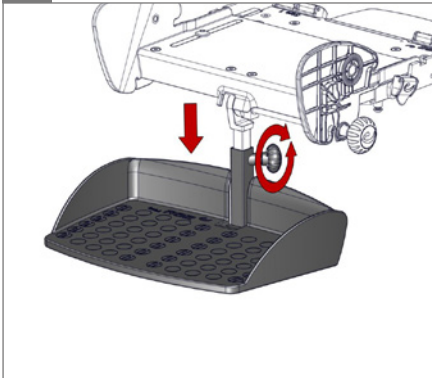5.

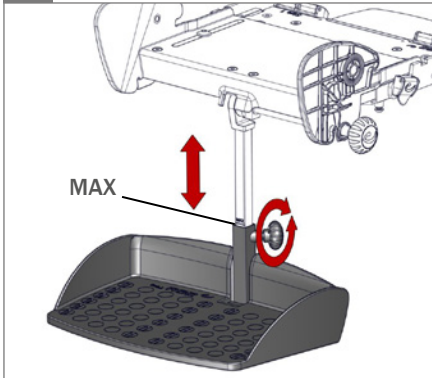
6.


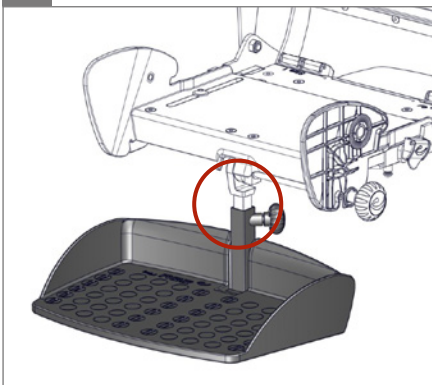
7.

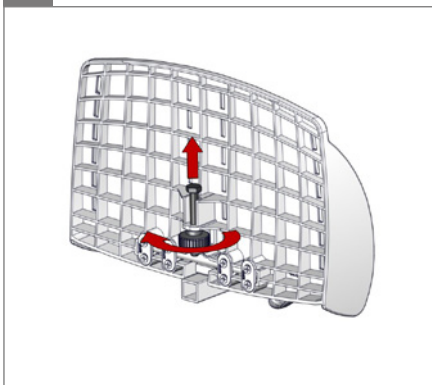
8.

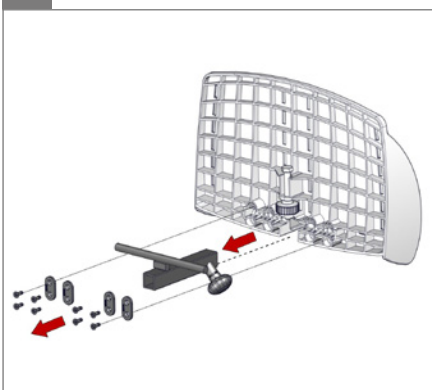
9.

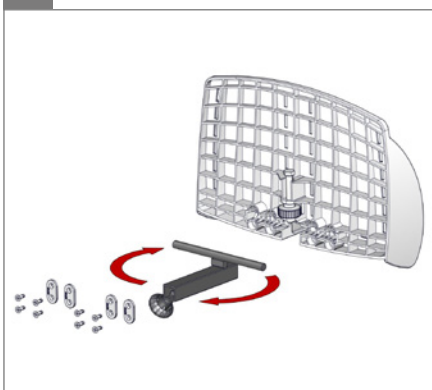
10.

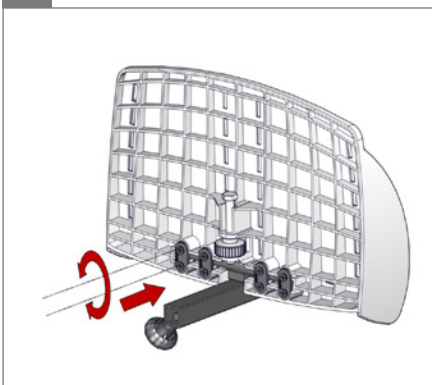11.

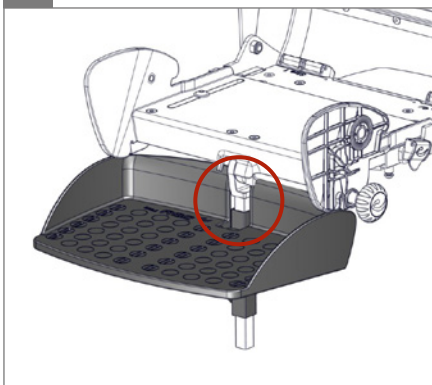12.


1.

40


2.

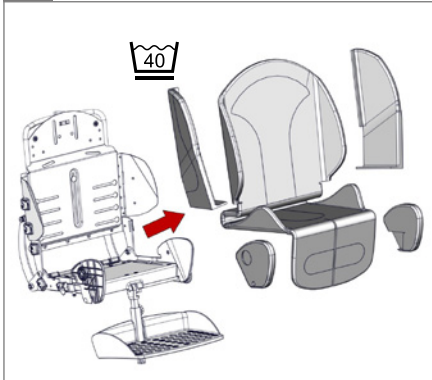
40

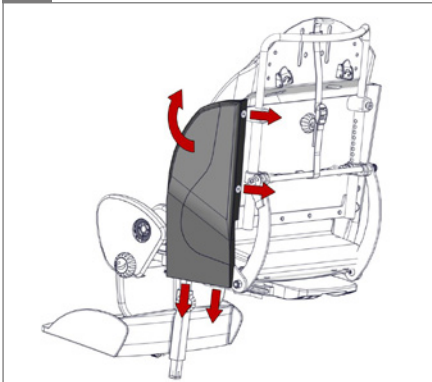
3.

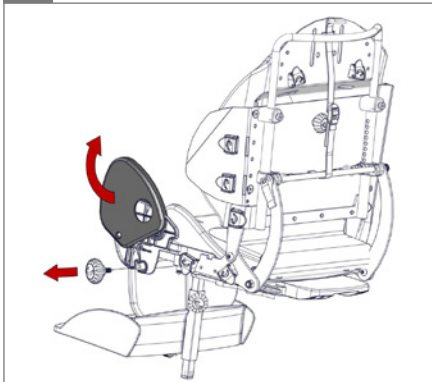
4.

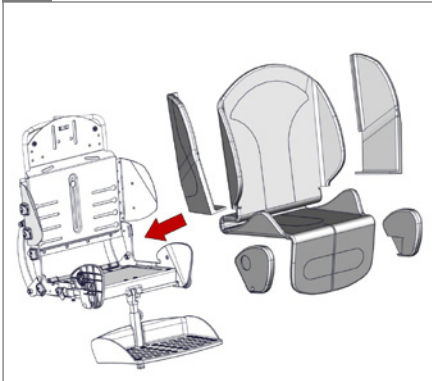5.

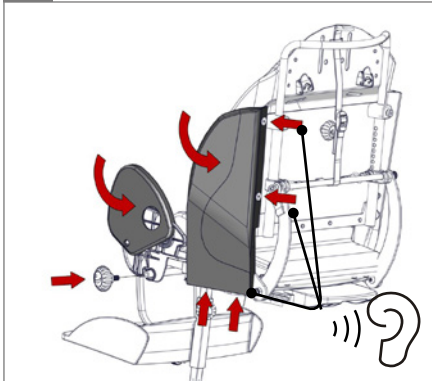6.

1.

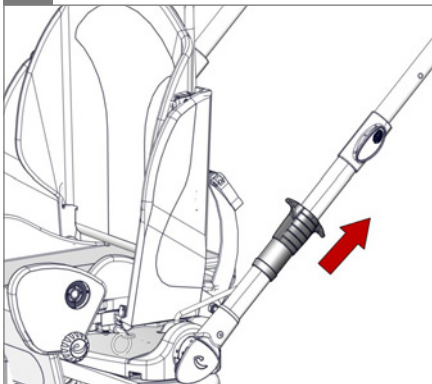
2.

3.


4.

5.

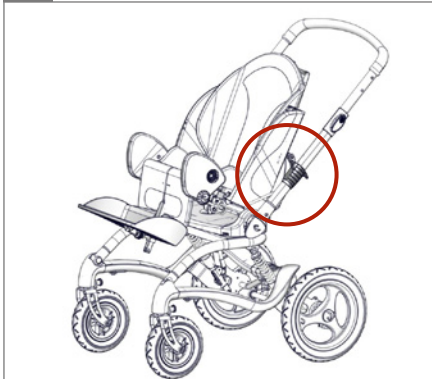6.


1.

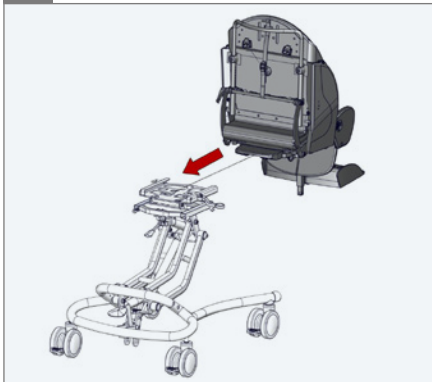
2.

3.

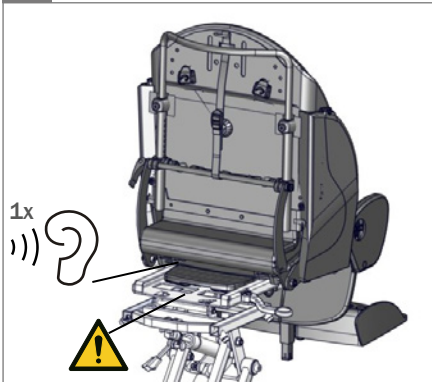
02


4.

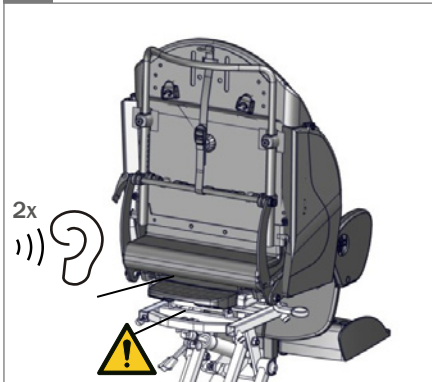02

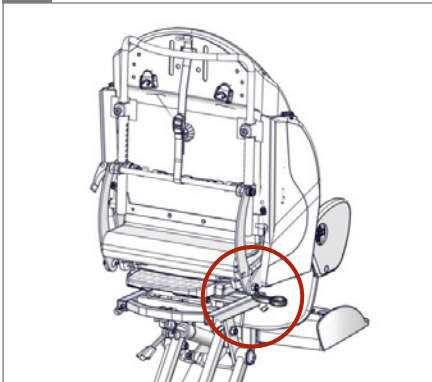
5.

6.

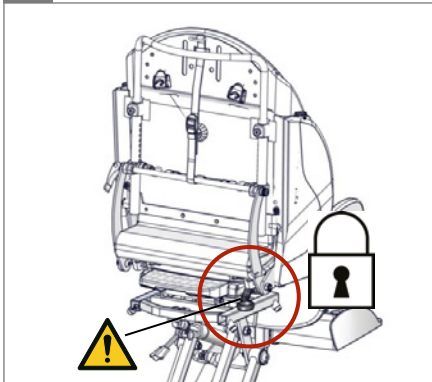05


1.

2.

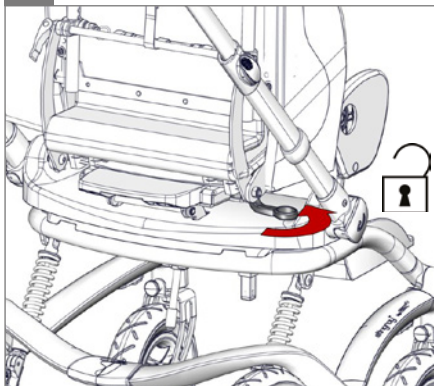
3.

4.

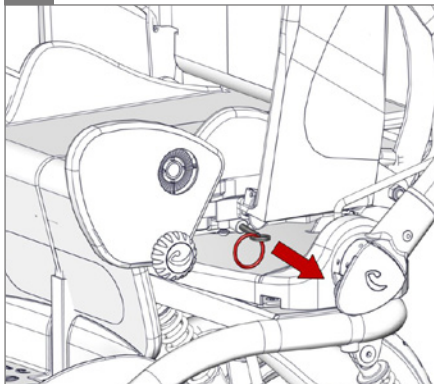5.


6.

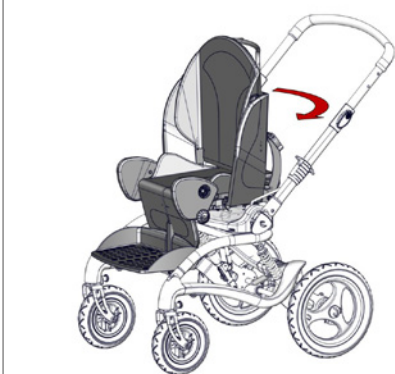
1.

2.

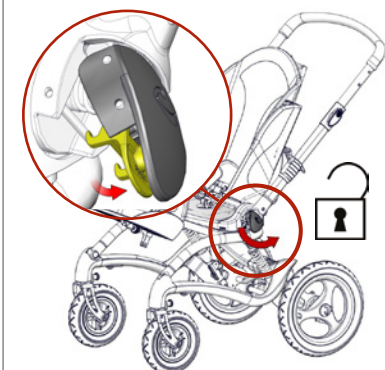3.

4.

5.

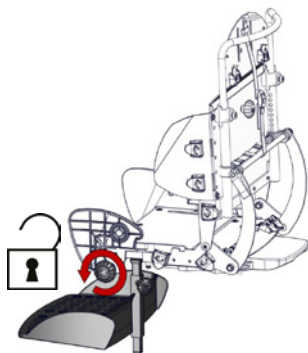
6.


1.

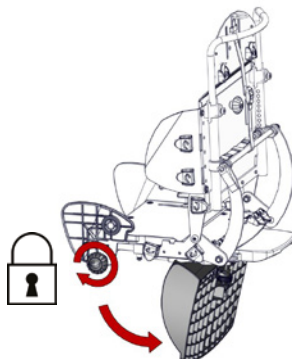
2.


3.

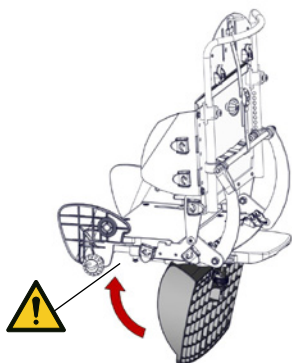05

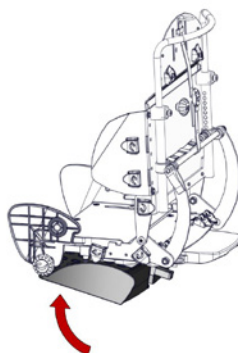
4.


5.

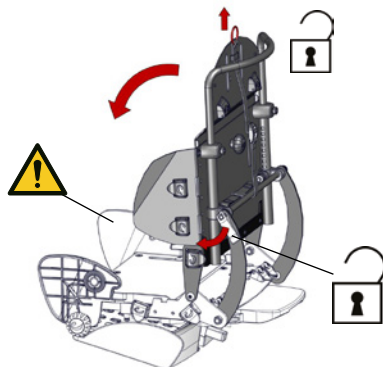05

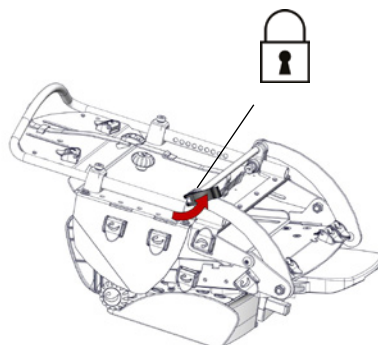6.


1.


<40 kg  
<88 lb

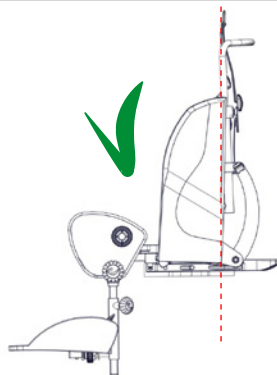
2.

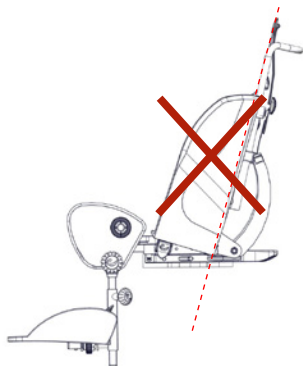
3.

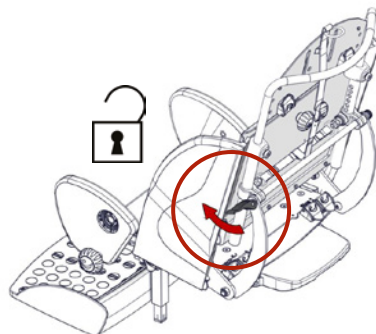
4.


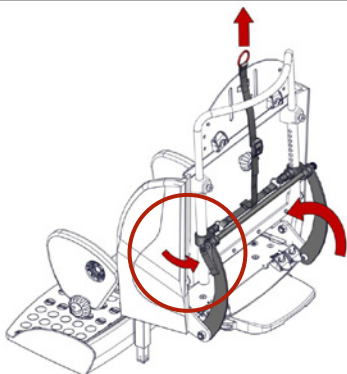
5.

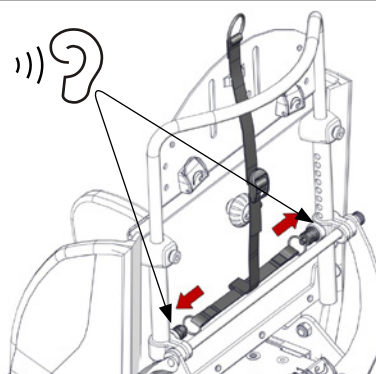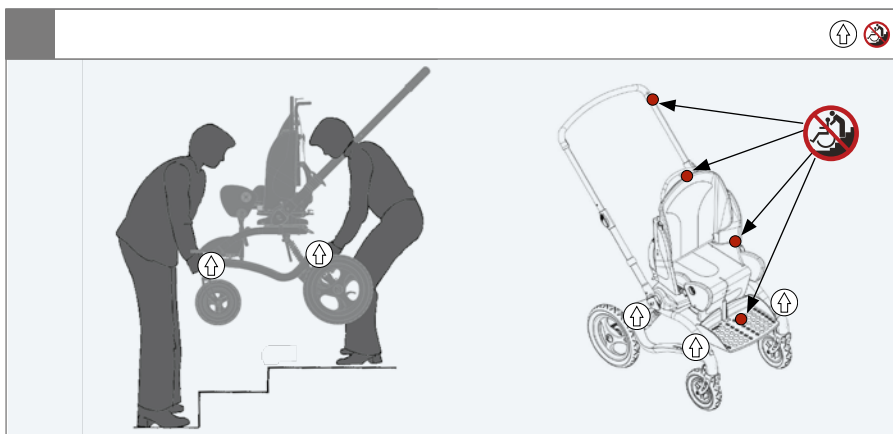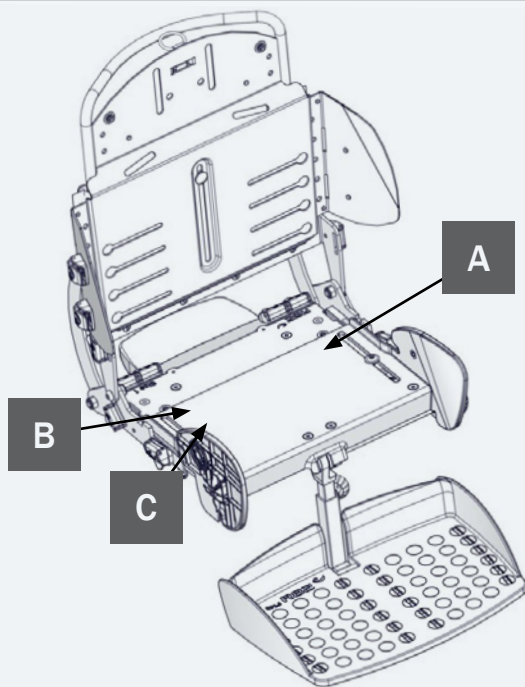6.

A

Max load: xxx kg  
DMR ver.: XXXX  
Art. no.: XXXXXXXX  
Product: XXXX - Size X

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


mm (inch)		
	 1	 2
 Effective seat width	300 (11¾")	350 (13¾")
 Effective seat depth	180-300 (7-11¾")	250-400 (9¾"-15½")
 Backrest height	410-550 (16-21½")	500-650 (19½"-25¼")
 Seat width, folded	420 (16½")	475 (18½")
 Seat length, folded	575 (22½")	575 (22½")
 Seat height, folded	285 (11")	305 (12")
 Footrest to seat distance	140-450 (5½"-17½")	140-450 (5½"-17½")
 Leg to seat surface angle	-30° -> 30°	-30° -> 30°
 Backrest angle	170°	170°
 Total mass	5.5 kg (12 lb)	6.5 kg (14.5 lb)
 Max load	40 kg (88 lb)	40 kg (88 lb)
 Max load at transportation	40 kg (88 lb)	40 kg (88 lb)

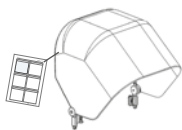
9996097170

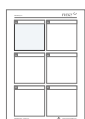
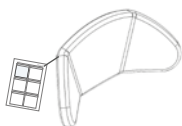
9996097171



9996097172


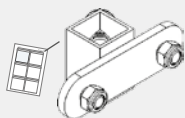
9996097173

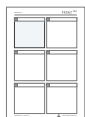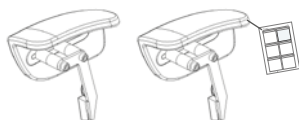
9996097174

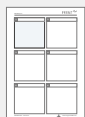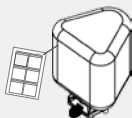9996097175

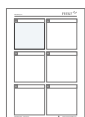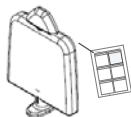9996097177

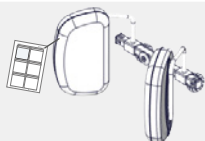9996097185

## ENGLISH

We appreciate your choice of a new product from R82 - a worldwide supplier of technical aids and appliances for children and teenagers with special needs. To fully benefit from the options offered by this product please read this User Guide before use and save it for future reference.

### Intended purpose

The Stingray has been designed for children who need good comfort and support. The seat depth and back height are easy to adjust, making it possible to expand the seat as when the child grows. The Stingray is suitable for users needing practical outdoor transportation, e.g. children with CP, GMFCS level 3-5. The Stingray comes in 2 sizes.

### Enclosed tools:

- 3, 4, and 5 mm Allen key

### Declaration of Conformity

This product conforms to the requirements of the Medical Devices Directive (93/42/EEC). The CE mark must be removed when rebuilding the product, when used in combination with a product from another manufacturer or when using other than original R82 spare parts and fittings. Further, this product conforms to the requirements according to:

- EN 12182
- EN 12183

### Disposal

When a product reaches the end of its lifetime, it should be separated by material type so that the parts can be recycled or disposed of properly. If required, please contact your local dealer for a precise description of each material. Please contact your local authorities to be informed of the options available for recycling in an environmentally-friendly way.

### Accessories and spare parts

The products from R82 can be supplied with a variety of accessories which comply with the needs of the individual user. Spare parts can be ordered on request. Find the specific accessories on our website or contact your local dealer for further information.

### R82 Warranty

R82 offers a 2-year warranty against defects in workmanship and materials and a 5-year warranty on breakage of the metal frame caused by defects in welds. The warranty would be adversely affected if the customer's responsibility of servicing and/or maintenance is not carried out according to the guidelines and intervals prescribed by the supplier and/or stated in the User Guide. For further information, we refer to the R82 homepage/download.

The warranty is only granted in the country where the product was purchased and if the product can be identified by the serial number. The warranty does not cover accidental damage, including damage caused by misuse or neglect. The warranty does not extend to consumable items e.g. tires or upholstery, which are subject to normal wear and tear and need periodic replacement.

The warranty is null and void if other than original parts/accessories are used, or if the product has not been maintained, repaired or altered according to the guidelines and intervals prescribed by the supplier and/or stated in the User Guide. R82 reserves the right to inspect the product being claimed for and the relevant documentation before agreeing to the warranty claim, and to decide upon whether to replace or repair the defective product. It is the customer's responsibility to return the item being claimed for under warranty to the address of purchase.

### Presales information

- [support.r82.org](mailto:support.r82.org)

## Safety

Signs, symbols and instructions placed on this product must never be covered or removed and must remain present and clearly legible throughout the entire lifetime of the product. Immediately replace or repair illegible or damaged signs, symbols and instructions. Contact your local dealer for instructions.

### Caregiver

- Read the "User Guide" carefully before use and keep it for future reference. Incorrect use of this product may cause serious injury to the user
- Information about transportation can be found in the document "Transportation in Motor Vehicles" supplied with the product
- Information regarding frame and seat combinations including max. load in combination can be found at: [support.r82.org](http://support.r82.org)
- Always use the correct lifting techniques and aids for this purpose
- Never leave the user unattended in this product. Ensure permanent supervision by an adult
- Repair/replacements must only be made using new original R82 spare parts and fittings and carried out in accordance with the guidelines and service intervals prescribed by the supplier
- If there is any doubt as to the continued safe use of your R82 product or if any parts should fail, stop using the product immediately and contact your local dealer as soon as possible
- Ensure that moving parts does not squeeze or trap body parts when adjusting them

### Environment

- Observe the surface temperature on the product, before placing an user in it. This goes especially for users with insensitive skin as they cannot feel the heat. If the surface temperature is above 41 °C then allow the product to cool before use

### User

- If the user is close to the max. load limit and/or has a lot of involuntary movements, e.g. rocking, a larger size with a higher max. load or a different R82 product should be considered
- For transportation the product is suitable for children in the weight range of 22 kg to specified max user weight for transportation

### Product

- Make sure the seat unit is securely mounted in the frame interface. Check that the locking system work without obstructions from e.g. fabric or straps. Remove obstructions and activate the locking system
- Carry out all positional adjustments on the product and accessories and ensure that all knobs, screws and buckles are securely fastened before use. Keep all tools out of the reach of children
- Ensure the stability of the product is maintained before placing the user in it
- We recommend you notify the user before angling the seat and back
- Ensure the wheels are fully-functional and secure before use
- Inspect the product and all of its accessories and replace any worn out parts before use
- This product is tested for resistance to ignition at level non-ignition in accordance with EN 1021-1, EN 1021-2 and ISO 7176-16
- The product might lose its flame resistant characteristics if using other than original R82 cushions

Even though the product complies with ISO-7176-19 then:

- Resistance to ignition can change with use, ageing and/or cleaning
- Precautions should be taken to avoid sources of ignition, including flames, cigarettes, etc..
- The environment (such as oxygen, alcohol, etc.) of the product occupant can increase the risk of ignitability

## Service information

Maintenance of a medical device is wholly the responsibility of the owner of that device. Failure to maintain a device in accordance with the instructions may invalidate the device's warranty. Furthermore, failure to maintain a device may compromise the clinical condition or safety of users and/or their caregivers.

### 1st Day

- Read the User Guide carefully before use and store it in a safe place for future reference
- Store the enclosed tools on/at the product

### Daily

- Use a dry cloth to clean the product
- Visual inspection. Repair or replace any damaged or worn-out parts
- Ensure that all fasteners are in place and fitted correctly
- Check all fittings and buckles for signs of worn out parts
- Be aware of any MAX mark indications before adjustment

### Weekly

- Check that all wheels are moving freely and that all wheel locks/brakes are easily activated
- Wipe off any residue and dirt from the product using a cloth with warm water and a mild detergent/soap and dry before use

### Monthly

- All nuts and bolts on this product must be checked and tightened to avoid unnecessary failures
- Oil the swing-away parts. We recommend you use a professional lubricant system

### Annually

- Inspect the frame for cracks or signs for worn-out parts and carry out annual service. Never use a product which appears to be substandard or faulty

### Disinfection

The product without accessories and cushion can be disinfected with a 70% disinfectant IPA solution. It is recommended to wipe off any residue and dirt from the product, using a cloth with warm water and a mild detergent/soap without chlorine and let it dry before disinfection.

### Washing

The cushion and covers can be washed at 40° using a mild detergent in a washing machine.

After washing, leave the washed item(s) to drip-dry after washing

### Warewashing machine

This product can be washed at 60° using a mild detergent for 15-20 minutes in a washing machine designed for medical devices. Use the machine's drying function to dry the product.

Before washing remove the following items and clean them separately:

- Accessories
- Cushioning

### Service Check List:

Check and adjust the following functions as per their intended use. Repair or replace any damaged or worn-out parts. Please check the:

- Seat back is adjustable and can be locked in position
- Adjustment options for the foot rest
- Product can be folded for storage or transportation
- Nuts and bolts on the product are tightened
- Check all fittings and buckles for signs of worn out parts
- Product for cracks or signs for worn-out parts
- All labels on the product are intact

### Service interval

A detailed inspection of the product must be made every 12 months (every 6 months for heavy users) and every time the product is reissued for use. The inspection should be carried out by a technically competent person who has been trained in the use of the product.

The lifetime of this product in normal use is 5 years if all maintenance and servicing is carried out in accordance with the manufacturer's instructions and demonstrably recorded.

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## R82 Nederland B.V.
## R82 Inc.
## R82 au Pty Limited