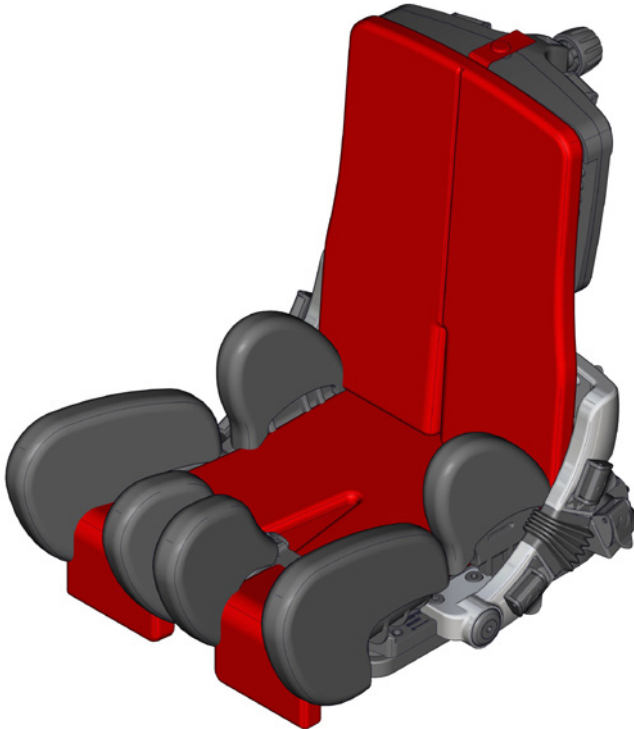
460 mm (18")

max 80 kg (176¼ lb)






x:panda User Guide

EN 12182
EN 12183

2017-08-rev. 18


support.R82.org 

	4	ENGLISH DEUTSCH	25 - 27 28 - 30
	5	NEDERLANDS DANSK	31 - 33 34 - 36
	6	NORSK SVENSKA	37 - 39 40 - 42
	7	SUOMI ESPAÑOL	43 - 45 46 - 48
	8	FRANÇAIS ITALIANO	49 - 51 52 - 54
	9 - 10	PORTUGUÊS PORTUGUÊS DO BRASIL	55 - 57 58 - 60
	11	РОССИЯ POLSKI	61 - 63 64 - 66
	12	ČESKY 中國	67 - 69 70 - 72
	13 - 14	日本 ΕΛΛΗΝΙΚΗ	73 - 75 76 - 78
	15		
	16		
	16		
	17 - 20		
mm (inch)	21		
	22 - 24		

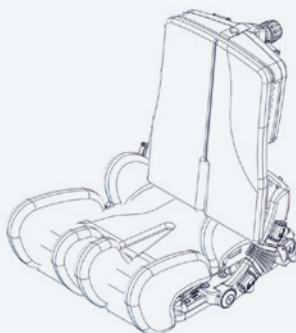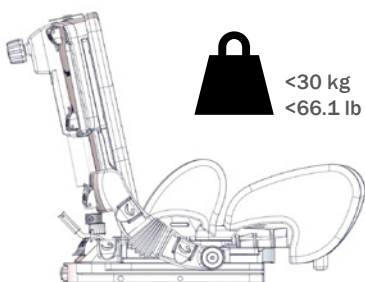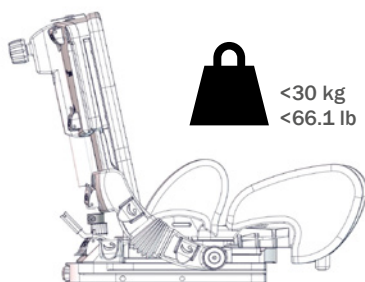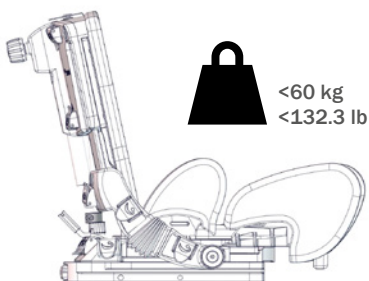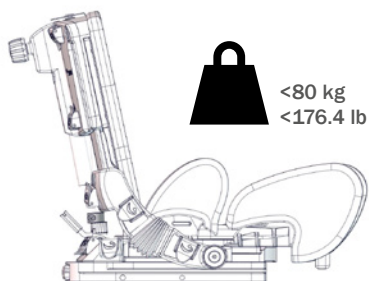
**X:PANDA 1****X:PANDA 2****X:PANDA 3****X:PANDA 4**

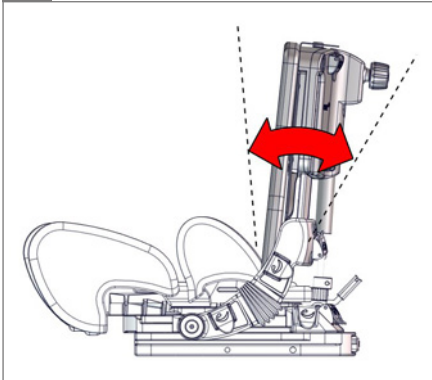
A.


B.

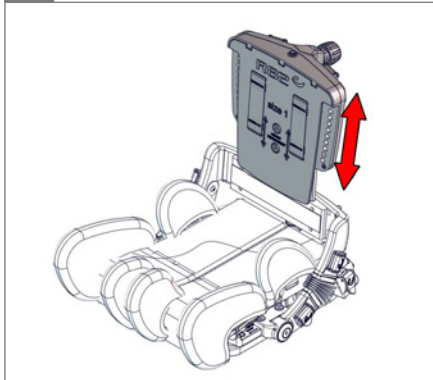
C.

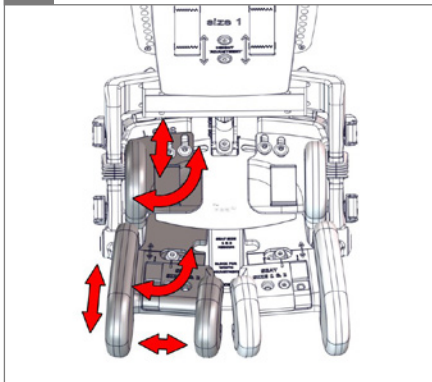D.

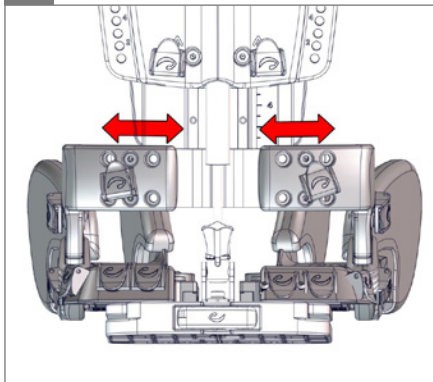
E.


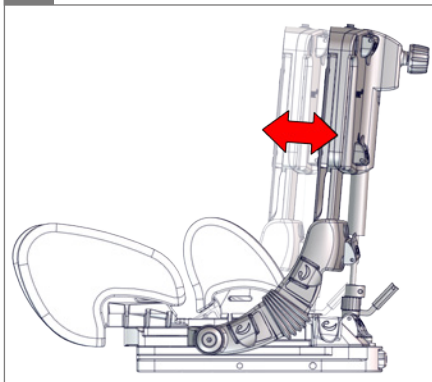
A.

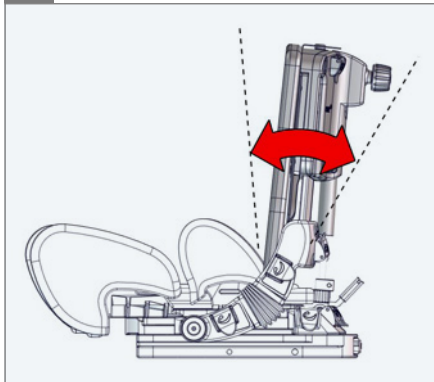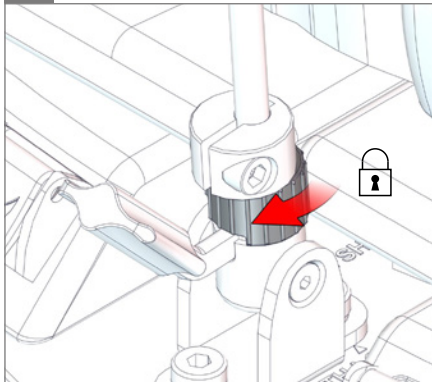1.

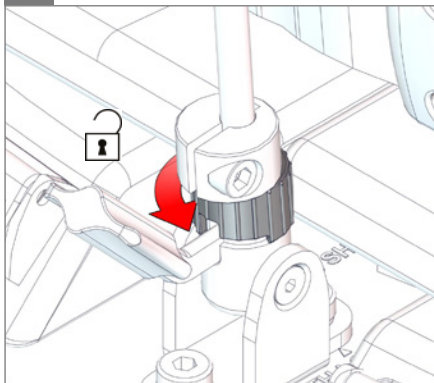
2.


3.

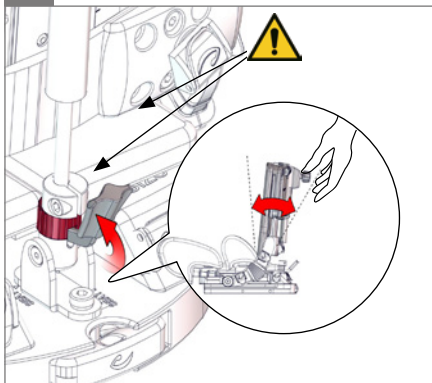⚠ 05

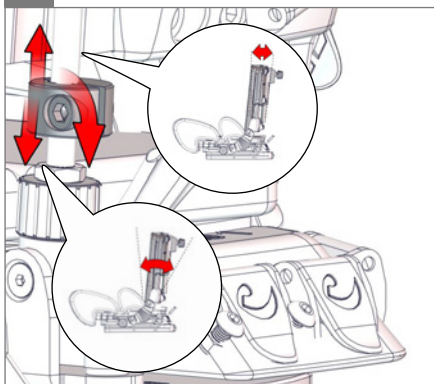5.


W

⚠ 05

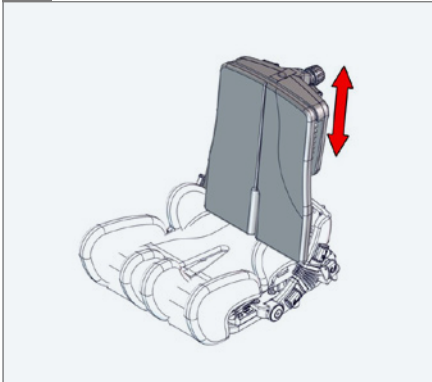
B.


1.

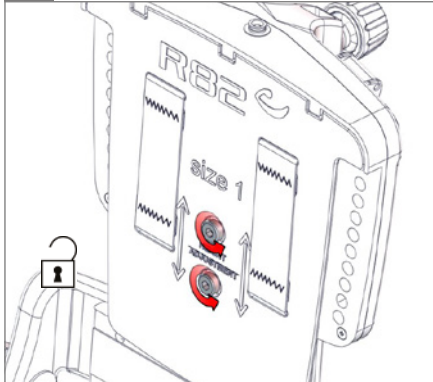○ 5 mm


2.

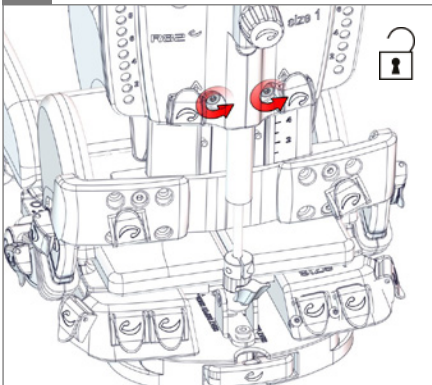○ 5 mm

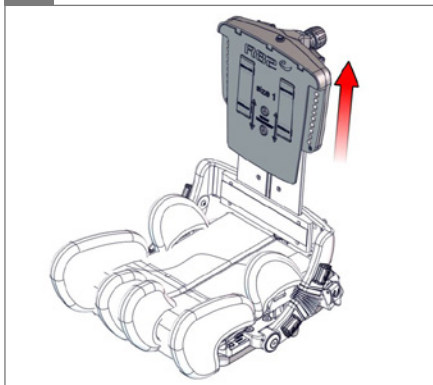
3.


4.

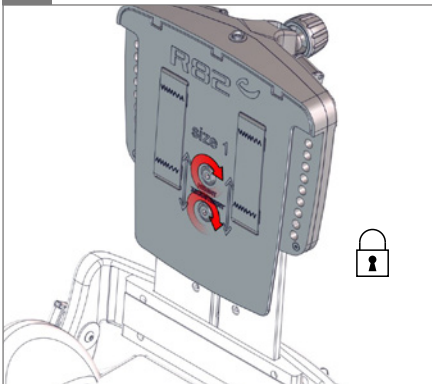○ 5 mm


5.

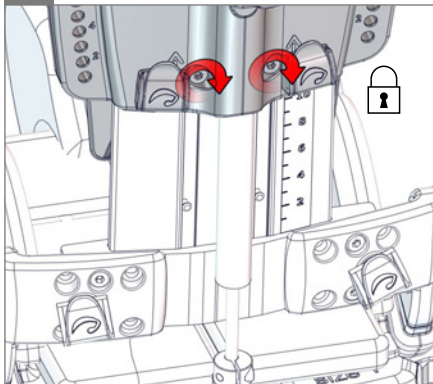○ 5 mm


c.

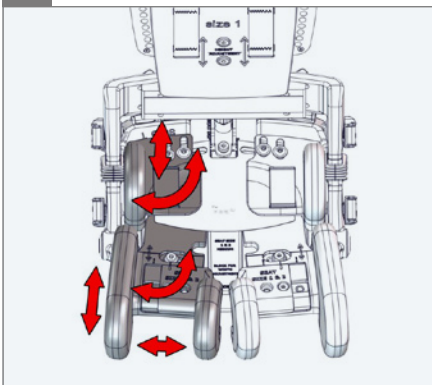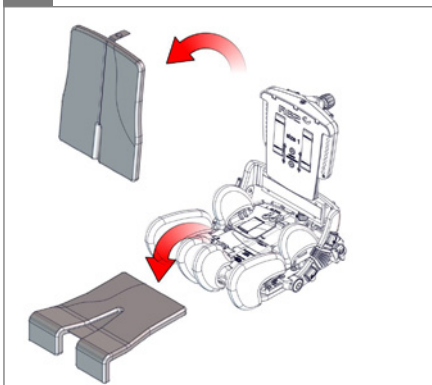
1.


2.

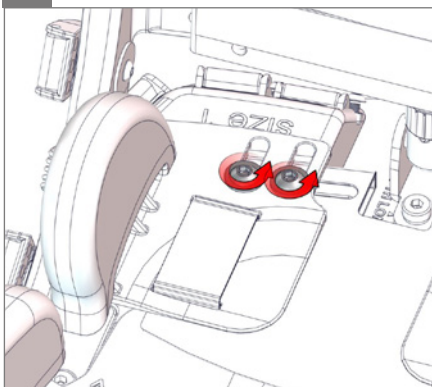○ 5 mm

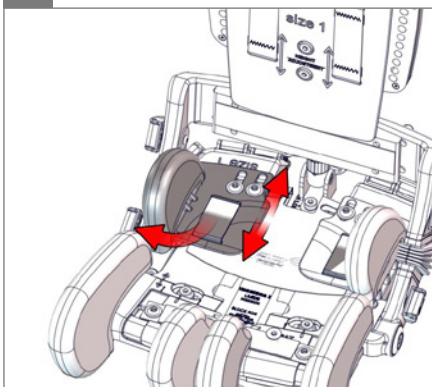
3.


4.

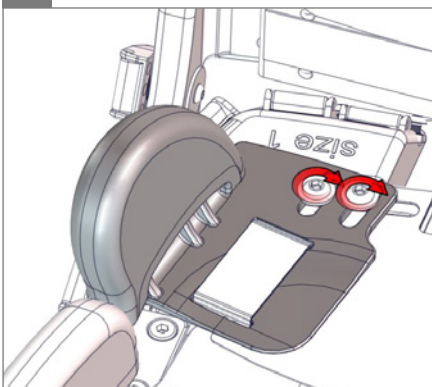○ 5 mm


5.

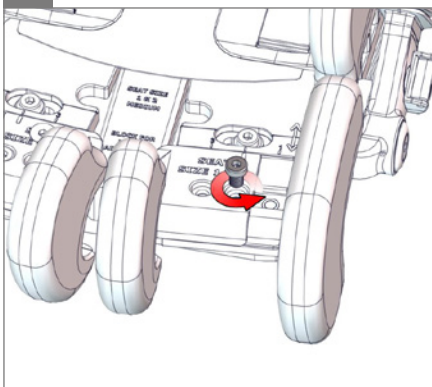○ 5 mm

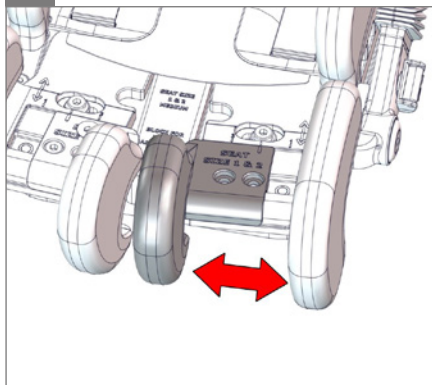
6.


7.

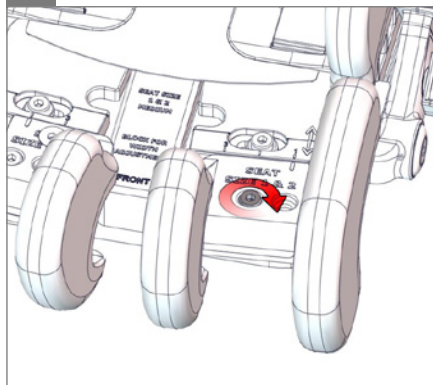○ 5 mm


8.

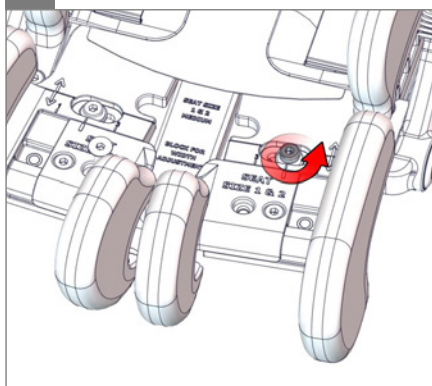
○ 5 mm

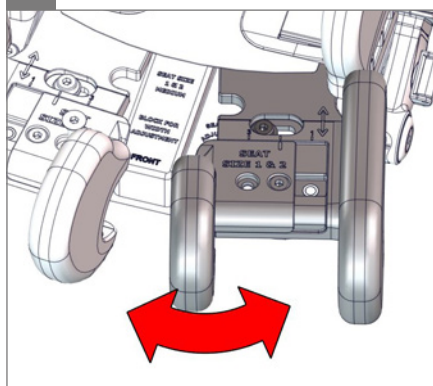9.

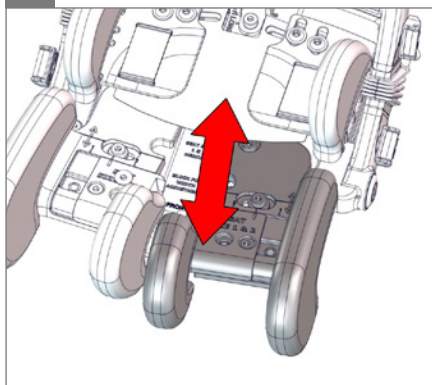
10.


11.

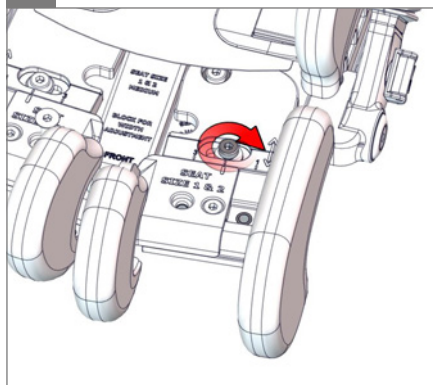○ 5 mm


D.

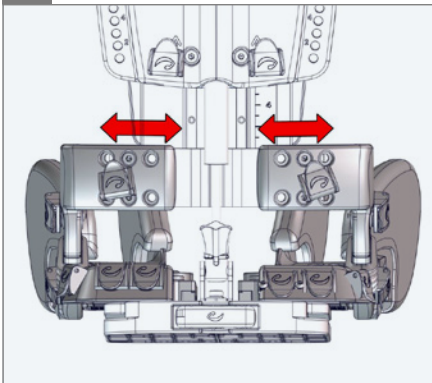
1.

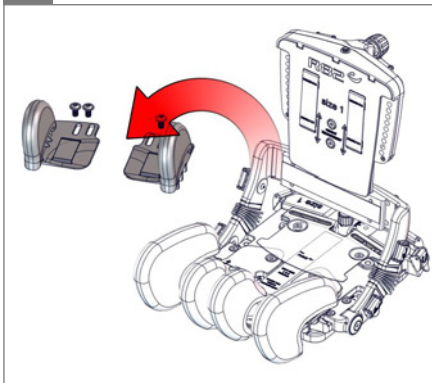○ 5 mm

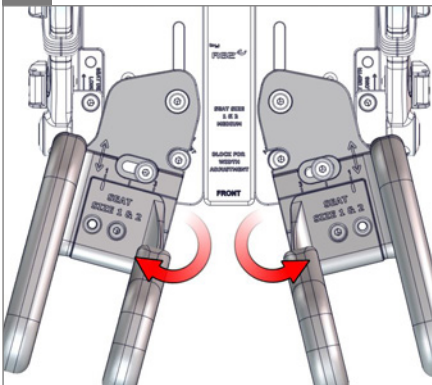
2.


3.

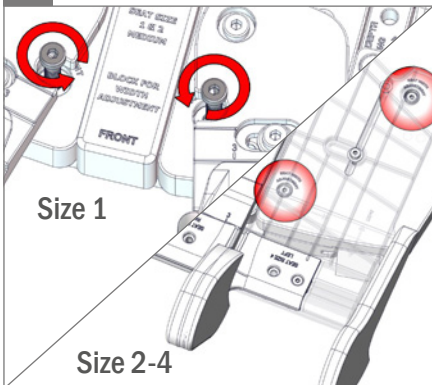○ 5 mm


4.

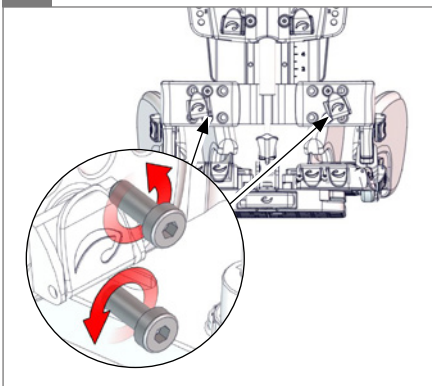○ 5 mm

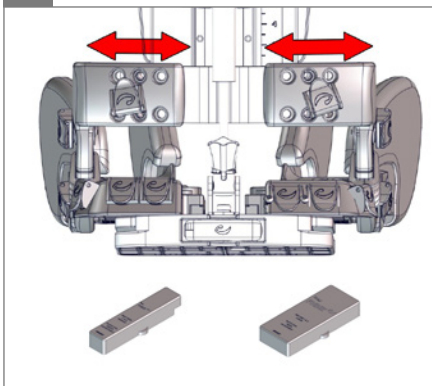
5.


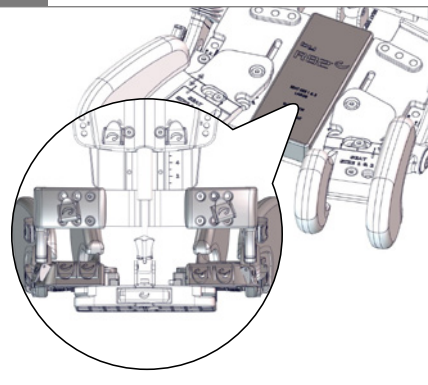
6.

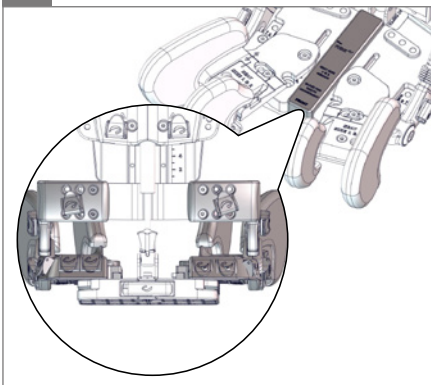7.

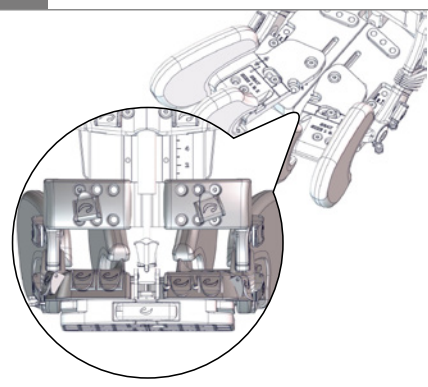
8.


9.

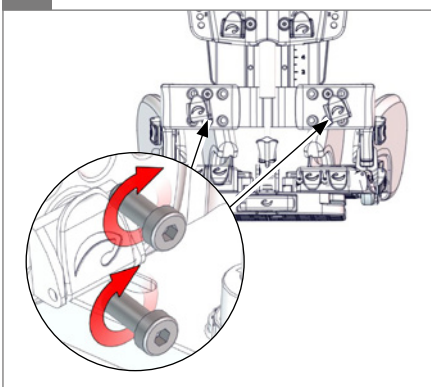○ 5 mm

10.

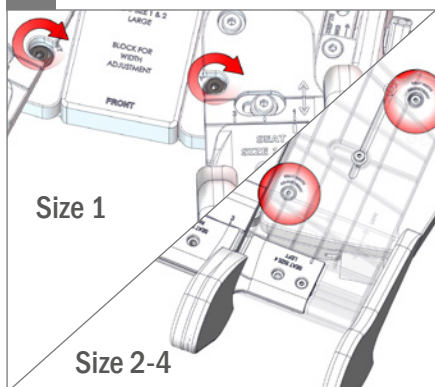○ 5 mm


11.

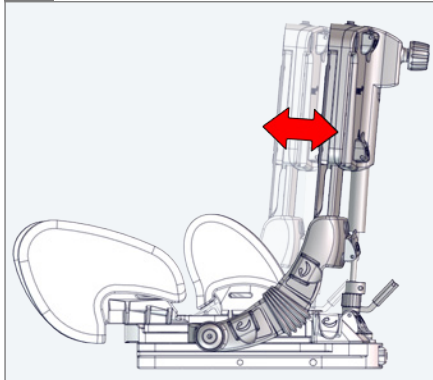
E.


1.

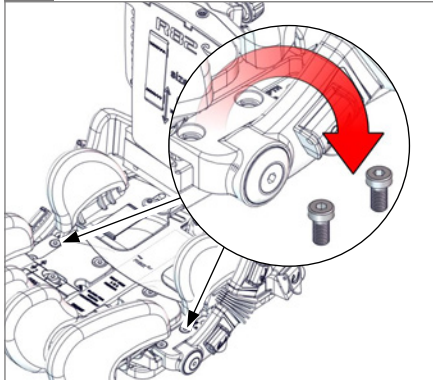⌀ 5 mm


2.

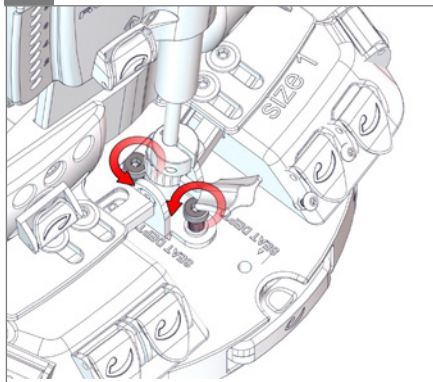⌀ 5 mm

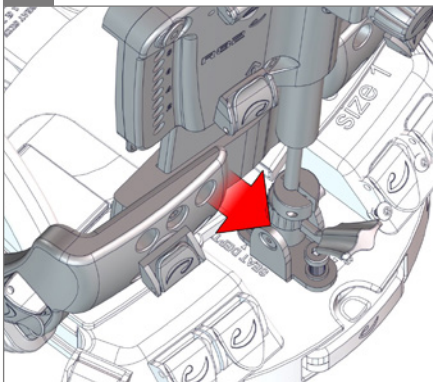
3.


4.

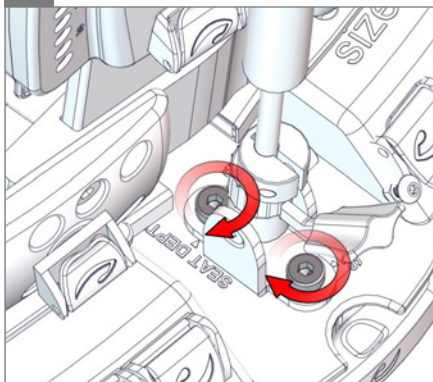⌀ 5 mm


5.

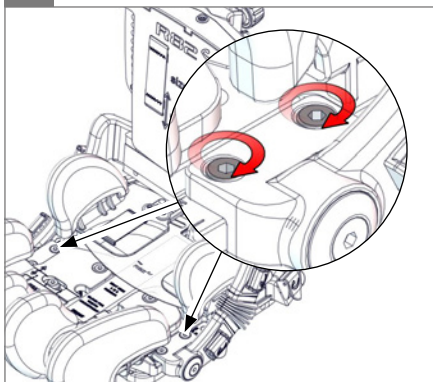
⌀ 5 mm

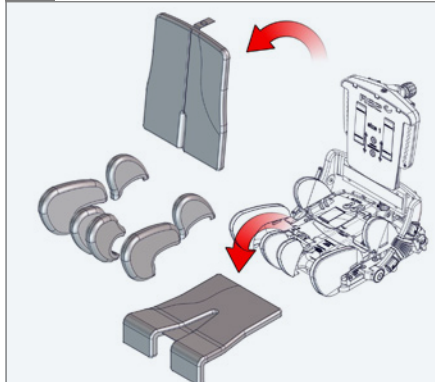
1.

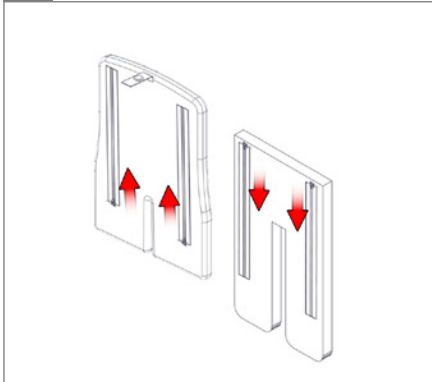2.

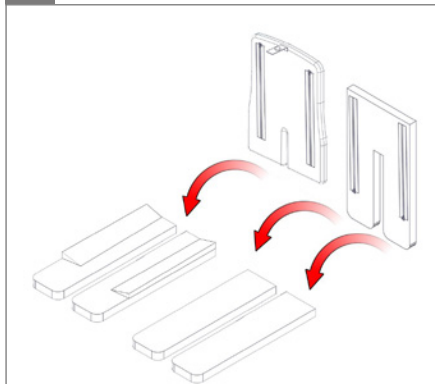
3.


4.

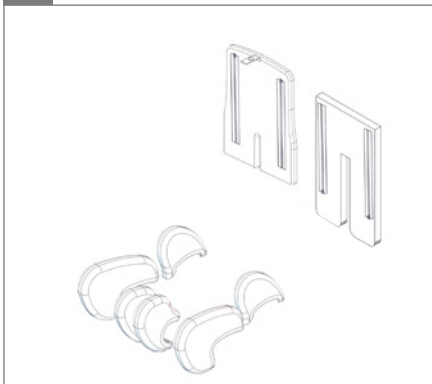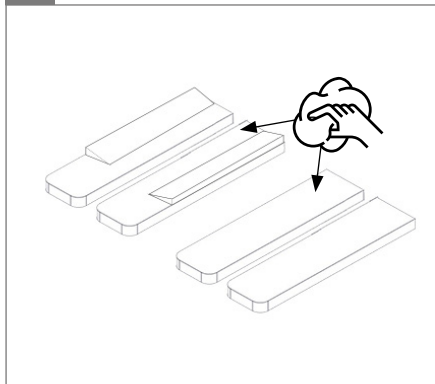40

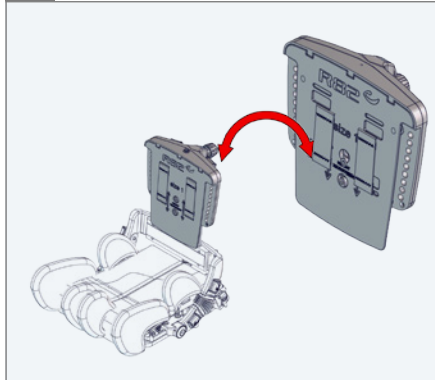
5.

F.

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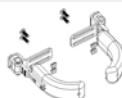
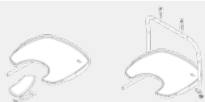
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
9996097901

ENGLISH

We appreciate your choice of a new product from R82 - a worldwide supplier of technical aids and appliances for children and teenagers with special needs. To fully benefit from the options offered by this product please read this User Guide before use and save it for future reference.

Intended purpose

x:panda is a multiadjustable seat, delivered in 4 different sizes. Each seat adjusts in 3 different widths and stepless in seat depth. The back is dynamic, making it possible for the child to extend and push back against relatively light resistance and then recover into the desired sitting position. Furthermore the back angles to make it possible to place the child with either an open or closed hip angle. When angling the seat and the base it is possible to create the desired working or resting position.

The x:panda is suitable for users needing an individual seating system and base, e.g. Children/youngsters with CP at GMFCS level III-V, Strong extensor spasticity and Developmental delays (physical or mentally handicapped). The many adjustment possibilities and wide range of accessories makes it possible to adjust the seat to an individual child's need for comfort and support. The seat fits on a variety of seat bases from R82, making it suitable for both indoor and outdoor use.

For transportation the x:panda is suitable for children in the weight range of 22 kg to specified max user weight for transportation.

Enclosed tools:

- 5 mm Allen key

Declaration of Conformity

This product conforms to the requirements of the Medical Devices Directive (93/42/EEC). The CE mark must be removed when rebuilding the product, when used in combination with a product from another manufacturer or when using other than original R82 spare parts and fittings. Further, this product conforms to the requirements according to:

- EN 12182
- EN 12183

Disposal

When a product reaches the end of its lifetime, it should be separated by material type so that the parts can be recycled or disposed of properly. If required, please contact your local dealer for a precise description of each material. Please contact your local authorities to be informed of the options available for recycling in an environmentally-friendly way.

Accessories and spare parts

The products from R82 can be supplied with a variety of accessories which comply with the needs of the individual user. Spare parts can be ordered on request. Find the specific accessories on our website or contact your local dealer for further information.

R82 Warranty

R82 offers a 2-year warranty against defects in workmanship and materials and a 5-year warranty on breakage of the metal frame caused by defects in welds. The warranty would be adversely affected if the customer's responsibility of servicing and/or maintenance is not carried out according to the guidelines and intervals prescribed by the supplier and/or stated in the User Guide. For further information, we refer to the R82 homepage/download.

The warranty is only granted in the country where the product was purchased and if the product can be identified by the serial number. The warranty does not cover accidental damage, including damage caused by misuse or neglect. The warranty does not extend to consumable items e.g. tires or upholstery, which are subject to normal wear and tear and need periodic replacement.

The warranty is null and void if other than original parts/accessories are used, or if the product has not been maintained, repaired or altered according to the guidelines and intervals prescribed by the supplier and/or stated in the User Guide. R82 reserves the right to inspect the product being claimed for and the relevant documentation before agreeing to the warranty claim, and to decide upon whether to replace or repair the defective product. It is the customer's responsibility to return the item being claimed for under warranty to the address of purchase.

Presales information

- support.r82.org

Safety

Signs, symbols and instructions placed on this product must never be covered or removed and must remain present and clearly legible throughout the entire lifetime of the product. Immediately replace or repair illegible or damaged signs, symbols and instructions. Contact your local dealer for instructions.

Caregiver

- Read the "User Guide" carefully before use and keep it for future reference. Incorrect use of this product may cause serious injury to the user
- Always use the correct lifting techniques and aids for this purpose
- Never leave the user unattended in this product. Ensure permanent supervision by an adult
- Repair/replacements must only be made using new original R82 spare parts and fittings and carried out in accordance with the guidelines and service intervals prescribed by the supplier
- If there is any doubt as to the continued safe use of your R82 product or if any parts should fail, stop using the product immediately and contact your local dealer as soon as possible
- Keep a hand on the handle on the back of the chair when adjusting the back angle.
- Information about transportation can be found in the document "Transportation in Motor Vehicles" supplied with the product
- Information regarding frame and seat combinations including max. load in combination can be found at: support.r82.org

Environment

- Observe the surface temperature on the product, before placing an user in it. This goes especially for users with insensitive skin as they cannot feel the heat. If the surface temperature is above 41 °C then allow the product to cool before use

User

- If the user is close to the max. load limit and/or has a lot of involuntary movements, e.g. rocking, a larger size with a higher max. load or a different R82 product should be considered
- For transportation the product is suitable for children in the weight range of 22 kg to specified max user weight for transportation

Product

- Carry out all positional adjustments on the product and accessories and ensure that all knobs, screws and buckles are securely fastened before use. Keep all tools out of the reach of children
- Ensure the stability of the product is maintained before placing the user in it
- Use the fixlocks mounted on the product to fasten and adjust all vests and belts. Note the prepared slots in the hip and leg supports intended for belt guidance.
- Position the back vertical and tighten the bushing up against the gaspring before transportation.
- Inspect the product and all of its accessories and replace any worn out parts before use
- Do not expose the piston rod to scratches etc. It is important to use a cloth or the like, if tools are used to adjust the gas spring
- This product is tested for resistance to ignition at level non-ignition in accordance with EN 1021-1, EN 1021-2 and ISO 7176-16
- The product might lose its flame resistant characteristics if using other than original R82 cushions

Accessories

Head Supports

- When adjusting the head support, be careful not to get fingers caught between back of the chair and the head support fitting

Arm rest

- Be careful not to get fingers caught between the chair and the armrest
- Do not carry the product by the armrests

Service information

Maintenance of a medical device is wholly the responsibility of the owner of that device. Failure to maintain a device in accordance with the instructions may invalidate the device's warranty. Furthermore, failure to maintain a device may compromise the clinical condition or safety of users and/or their caregivers.

1st Day

- Read the User Guide carefully before use and store it in a safe place for future reference
- Store the enclosed tools on/at the product

Daily

- Use a dry cloth to clean the product
- Visual inspection. Repair or replace any damaged or worn-out parts
- Ensure that all fasteners are in place and fitted correctly
- Check all fittings and buckles for signs of worn out parts
- Be aware of any MAX mark indications before adjustment

Weekly

- Wipe off any residue and dirt from the product using a cloth with warm water and a mild detergent/soap and dry before use
- Disinfect the product. please check the disinfection section in this User Guide for more information

Monthly

- All nuts and bolts on this product must be checked and tightened to avoid unnecessary failures
- Oil the swing-away parts. We recommend you use a professional lubricant system

Annually

- Inspect the frame for cracks or signs for worn-out parts and carry out annual service. Never use a product which appears to be substandard or faulty

Washing

The cushion is removable and the cover can be washed at max. 40 °C using a mild detergent in a washing machine. The upholstery is incontinence proof and can be hand washed in warm water.

Warewashing machine

This product can be washed at 60° using a mild detergent for 15-20 minutes in a washing machine designed for medical devices. Use the machine's drying function to dry the product. Remove all accessories before washing the main product. Clean the accessories separately. Please allow the product to dry before reuse.

Before washing remove the following items and clean them separately:

- Accessories
- Cushions

Disinfection

The product without accessories and cushion can be disinfected with a 70% disinfectant IPA solution. It is recommended to wipe off any residue and dirt from the product, using a cloth with warm water and a mild detergent/soap without chlorine and let it dry before disinfection.

Service interval

A detailed inspection of the product must be made every 12 months (every 6 months for heavy users) and every time the product is reissued for use. The inspection should be carried out by a technically competent person who has been trained in the use of the product.

The lifetime of this product in normal use is 5 years if all maintenance and servicing is carried out in accordance with the manufacturer's instructions and demonstrably recorded.

Service Check List:

Check and adjust the following functions as per their intended use. Repair or replace any damaged or worn-out parts. Please check the:

- Adjustment options for back, hip and legs
- Gas-spring for back recline and the security lock
- Seat back is adjustable and can be locked in position
- Nuts and bolts on the product are tightened
- Check all fittings and buckles for signs of worn out parts
- Product for cracks or signs for worn-out parts
- All labels on the product are intact

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© 2012 R82 A/S All rights reserved The R82 logo and the icons are registered trademarks of R82 A/S
R82 Inc.
R82 au Pty Limited