+40°

<60 kg (132.25 lb)

## High-low & High-low:x User Guide

High-low indoor & High-low:x indoor: EN 12182

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mm (inch)	Length	Overall width	Height floor to interface	Seat plane angle	Total mass	Max load (kg)	Footrest to adaptor plate
							
High-low indoor	690 (27¼")	560 (22")	170-600 (6¾-23½")	+40°	12.3 kg (27.1 lb)	60 kg (132.25 lb)	50-290 (2-11½")
High-low indoor w/ power	690 (27¼")	560 (22")	170-600 (6¾-23½")	+40°	12.8 kg (28.2 lb)	60 kg (132.25 lb)	50-290 (2-11½")
High-low x indoor Size 1	690 (27¼")	560 (22")	170-600 (6¾-23½")	+40°	12.5 kg (27.5 lb)	60 kg (132.25 lb)	50-290 (2-11½")
High-low x indoor Size 2	780 (30¾")	620 (24½")	250-610 (9¾-24")	+40°	14 kg (30.8 lb)	60 kg (132.25 lb)	50-290 (2-11½")
High-low x indoor Size 1 w/ power	690 (27¼")	560 (22")	170-600 (6¾-23½")	+40°	13 kg (28.6 lb)	60 kg (132.25 lb)	50-290 (2-11½")
High-low x indoor Size 2 w/ power	780 (30¾")	620 (24½")	250-620 (9¾-24½")	+40°	16.5 kg (36.3 lb)	60 kg (132.25 lb)	50-290 (2-11½")
High-low x indoor Size 3 w/ power	780 (30¾")	620 (24½")	260-550 (10¼-21¾")	+40°	17 kg (37.4 lb)	100 kg (220.4 lb)	50-290 (2-11½")
High-low x indoor Size 2 w/ hydraulic	780 (30¾")	620 (24½")	220-580 (8¾-22¾")	+40°	16.5 kg (36.5 lb)	60 kg (132.25 lb)	50-290 (2-11½")


mm (inch)	Length	Overall width	Height floor to interface	Seat plane angle	Total mass	Max load (kg)	Max load in transportation	Footrest to adaptor plate
								
High-low outdoor	960 (37 <sup>3</sup> / <sub>4</sub> "")	620 (24 <sup>1</sup> / <sub>2</sub> "")	260-700 (10 <sup>3</sup> / <sub>4</sub> -27 <sup>1</sup> / <sub>2</sub> "")	+40°	17.5 kg (38.5 lb)	60 kg (132.25 lb)	43 kg (94.75 lb)	50-290 (2-11 <sup>1</sup> / <sub>2</sub> "")
High-low outdoor w/ power	960 (37 <sup>3</sup> / <sub>4</sub> "")	620 (24 <sup>1</sup> / <sub>2</sub> "")	260-700 (10 <sup>3</sup> / <sub>4</sub> -27 <sup>1</sup> / <sub>2</sub> "")	+40°	18 kg (39.6 lb)	60 kg (132.25 lb)	43 kg (94.75 lb)	50-290 (2-11 <sup>1</sup> / <sub>2</sub> "")
High-low:x outdoor Size 1	960 (37 <sup>3</sup> / <sub>4</sub> "")	620 (24 <sup>1</sup> / <sub>2</sub> "")	260-700 (10 <sup>3</sup> / <sub>4</sub> -27 <sup>1</sup> / <sub>2</sub> "")	+40°	17.5 kg (38.5 lb)	60 kg (132.25 lb)	43 kg (94.75 lb)	50-290 (2-11 <sup>1</sup> / <sub>2</sub> "")
High-low:x outdoor Size 2	1110 (43 <sup>3</sup> / <sub>4</sub> "")	620 (24 <sup>1</sup> / <sub>2</sub> "")	330-640 (13-25 <sup>1</sup> / <sub>4</sub> "")	+40°	20 kg (44 lb)	60 kg (132.25 lb)	43 kg (94.75 lb)	50-290 (2-11 <sup>1</sup> / <sub>2</sub> "")
High-low:x outdoor Size 1 w/ power	960 (37 <sup>3</sup> / <sub>4</sub> "")	620 (24 <sup>1</sup> / <sub>2</sub> "")	260-700 (10 <sup>3</sup> / <sub>4</sub> -27 <sup>1</sup> / <sub>2</sub> "")	+40°	18 kg (39.6 lb)	60 kg (132.25 lb)	43 kg (94.75 lb)	50-290 (2-11 <sup>1</sup> / <sub>2</sub> "")
High-low:x outdoor Size 2 w/ power	1110 (43 <sup>3</sup> / <sub>4</sub> "")	620 (24 <sup>1</sup> / <sub>2</sub> "")	330-640 (13-25 <sup>1</sup> / <sub>4</sub> "")	+40°	20.5 kg (45.1 lb)	60 kg (132.25 lb)	43 kg (94.75 lb)	50-290 (2-11 <sup>1</sup> / <sub>2</sub> "")
High-low:x outdoor Size 2 w/ hydraulic	1110 (43 <sup>3</sup> / <sub>4</sub> "")	620 (24 <sup>1</sup> / <sub>2</sub> "")	430-590 (17-23 <sup>1</sup> / <sub>4</sub> "")	+40°	20.5 kg (45.1 lb)	60 kg (132.25 lb)	43 kg (94.75 lb)	50-290 (2-11 <sup>1</sup> / <sub>2</sub> "")


9996097100


9996097136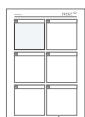

9996097159

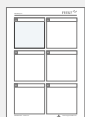
9996097703

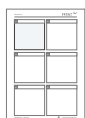9996097709


9996097714

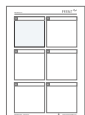
9996097716


9996097718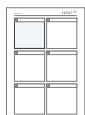


9996097720

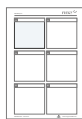
9996097723

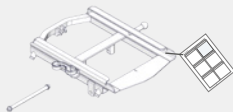
9996097737

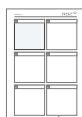9996097118


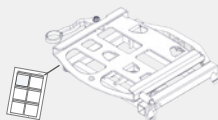
9996097735

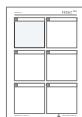9996097731

9996097742

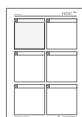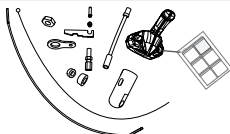9996097743

9996097744

## ENGLISH

We appreciate your choice of a new product from R82 - a worldwide supplier of technical aids and appliances for children and teenagers with special needs. To fully benefit from the options offered by this product please read this User Guide before use and save it for future reference.

### Intended purpose

The product is a frame which along with a seating system or a moulded seat provides a stable base with user-friendly height adjustment from floor to table height.

The frame is available in different configurations where High-low indoor & High-low:x indoor is for indoor use and High-low outdoor & High-low:x outdoor is for both indoor and outdoor use.

The target group for this device is disabled children with cerebral palsy (CP). Also, children with muscular dystrophy, spina bifida, spinal cord injuries and neurological impairments can be subject to the device. The product is suitable for children with GMFCS level 4-5.

### Declaration of Conformity

This product conforms to the requirements of the Medical Devices Directive (93/42/EEC). The CE mark must be removed when rebuilding the product, when used in combination with a product from another manufacturer or when using other than original R82 spare parts and fittings. Further, this product conforms to the requirements according to:

- High-low indoor & High-low:x indoor: EN 12182
- High-low outdoor & High-low:x outdoor: EN 12182 & EN 12183

### Disposal

When a product reaches the end of its lifetime, it should be separated by material type so that the parts can be recycled or disposed of properly. If required, please contact your local dealer for a precise description of each material. Please contact your local authorities to be informed of the options available for recycling in an environmentally-friendly way.

### Accessories and spare parts

The products from R82 can be supplied with a variety of accessories which comply with the needs of the individual user. Spare parts can be ordered on request. Find the specific accessories on our website or contact your local dealer for further information.

### R82 Warranty

R82 offers a 2-year warranty against defects in workmanship and materials and a 5-year warranty on breakage of the metal frame caused by defects in welds. The warranty would be adversely affected if the customer's responsibility of servicing and/or maintenance is not carried out according to the guidelines and intervals prescribed by the supplier and/or stated in the User Guide. For further information, we refer to the R82 homepage/download.

The warranty is only granted in the country where the product was purchased and if the product can be identified by the serial number. The warranty does not cover accidental damage, including damage caused by misuse or neglect. The warranty does not extend to consumable items e.g. tires or upholstery, which are subject to normal wear and tear and need periodic replacement.

The warranty is null and void if other than original parts/accessories are used, or if the product has not been maintained, repaired or altered according to the guidelines and intervals prescribed by the supplier and/or stated in the User Guide. R82 reserves the right to inspect the product being claimed for and the relevant documentation before agreeing to the warranty claim, and to decide upon whether to replace or repair the defective product. It is the customer's responsibility to return the item being claimed for under warranty to the address of purchase.

### Presales information

- [support.r82.org](mailto:support.r82.org)

## Safety

Signs, symbols and instructions placed on this product must never be covered or removed and must remain present and clearly legible throughout the entire lifetime of the product. Immediately replace or repair illegible or damaged signs, symbols and instructions. Contact your local dealer for instructions.

### Caregiver

- Read the "User Guide" carefully before use and keep it for future reference. Incorrect use of this product may cause serious injury to the user
- Always use the correct lifting techniques and aids for this purpose
- Never leave the user unattended in this product. Ensure permanent supervision by an adult
- Repair/replacements must only be made using new original R82 spare parts and fittings and carried out in accordance with the guidelines and service intervals prescribed by the supplier
- If there is any doubt as to the continued safe use of your R82 product or if any parts should fail, stop using the product immediately and contact your local dealer as soon as possible
- Always use the product in a moderate pace. Do not run with the product
- When adjusting the seat angle, keep a hand on the seat mounted on the frame to ensure slow angling
- Ensure that moving parts does not squeeze or trap body parts when adjusting them
- Information about transportation can be found in the document "Transportation in Motor Vehicles" supplied with the product
- Information about roadability can be found in the document "Roadability" supplied with the product
- Information regarding frame and seat combinations including max. load in combination can be found at: [support.r82.org](http://support.r82.org)

### Environment

- Observe the surface temperature on the product, before placing a user in it. This goes especially for users with insensitive skin as they cannot feel the heat. If the surface temperature is above 41°C then allow the product to cool before use
- Use this product on a level surface without any obstacles. Be aware of reduced maneuverability on slippery surfaces
- When using on inclines, please refer to measurements

### Environmental conditions

- Operating temperature: -10°C to +40°C
- Storage temperature: -10°C to +50°C
- Relative humidity: 5% to 90%
- Atmospheric pressure: 800 to 1060 hPa

### User

- If the user is close to the max. load limit and/or has a lot of involuntary movements, e.g. rocking, a larger size with a higher max. load or a different R82 product should be considered
- For transportation the product is suitable for children in the weight range of 22 kg to specified max user weight for transportation
- Do not place more than one user in the product

### Product

- Carry out all positional adjustments on the product and accessories and ensure that all knobs, screws and buckles are securely fastened before use. Keep all tools out of the reach of children
- Activate the brakes before placing the user into the product
- Ensure the stability of the product is maintained before placing the user in it
- When the footplate is close to the floor and the castors, the castors may be unable to swivel
- If the frame has to be used in the lowest position, Remove the footrest before adjustment.
- The frame with power does not have a safety stop. Make sure that the foot support bar does not collide with the cross bar on the frame on High Low:x size 3
- Inspect the product and all of its accessories and replace any worn out parts before use
- Do not expose the gas spring to pressure or high temperatures. DO NOT puncture
- Do not expose the piston rod to scratches etc. It is important to use a cloth or the like, if tools are used to adjust the gas spring
- When mounting a Panda Futura seat size 4 on any High Low or High Low:x frame, you must mount the anti-tip device that comes as an accessory to the product

## Service information

Maintenance of a medical device is wholly the responsibility of the owner of that device. Failure to maintain a device in accordance with the instructions may invalidate the device's warranty. Furthermore, failure to maintain a device may compromise the clinical condition or safety of users and/or their caregivers.

### 1st Day

- Read the User Guide carefully before use and store it in a safe place for future reference
- Store the enclosed tools on/at the product

### Daily

- Use a dry cloth to clean the product
- Visual inspection. Repair or replace any damaged or worn-out parts
- Ensure that all fasteners are in place and fitted correctly
- Check all fittings and buckles for signs of worn out parts
- Be aware of any MAX mark indications before adjustment
- Check that all cables connecting the power system are in good shape

### Weekly

- Check that all wheels are moving freely and that all wheel locks/brakes are easily activated
- Wipe off any residue and dirt from the product using a cloth with warm water and a mild detergent/soap and dry before use

### Monthly

- Check the tire pressure: Front wheels 35 PSI/2.4 bar; Rear wheels 40 PSI/ 2.75 bar
- All nuts and bolts on this product must be checked and tightened to avoid unnecessary failures
- Oil the swing-away parts. We recommend you use a professional lubricant system

### Annually

- Inspect the frame for cracks or signs for worn-out parts and carry out annual service. Never use a product which appears to be substandard or faulty

## Warewashing machine

This product can be washed at 60° using a mild detergent in 15-20 minutes in a washing machine designed for medical devices. Use the machine's drying function to dry the product. Clean the accessories separately.

### Before washing

- Avoid degreasing of the piston rods, the actuators should be retracted to minimum stroke and without load before washing.
- Remove the following items and clean them separately: Seat, Accessories, Power Unit and Handset

### After washing:

- Install power unit and handset
- Check that the handset functionalities works correct

## Disinfection

The product can be disinfected with a 70% disinfectant IPA solution. It is recommended to wipe off any residue and dirt from the product, using a cloth with warm water and a mild detergent/soap without chlorine and let it dry before disinfection.

## Service interval

A detailed inspection of the product must be made every 12 months (every 6 months for heavy users) and every time the product is reissued for use. The inspection should be carried out by a technically competent person who has been trained in the use of the product.

The lifetime of this product in normal use is 5 years if all maintenance and servicing is carried out in accordance with the manufacturer's instructions and demonstrably recorded.

## Service Check List:

Check and adjust the following functions as per their intended use. Repair or replace any damaged or worn-out parts. Please check the:

- Frame is adjustable in height
- Powered versions are able to retain power after charging the battery
- On outdoor versions, the push-brace can be folded over the frame
- Tire pressure
- Seat plane adjustment. The seat plane is adjustable and stays in position after adjustment.
- Wheels are moving freely and that all wheel locks/brakes are easily activated. Adjust if required
- Nuts and bolts on the product are tightened
- Oil the swing-away parts
- Product for cracks or signs for worn-out parts
- All labels on the product are intact

## Power unit

For an optimum lifetime the power unit must be connected to the mains voltages as often as possible (e.g. every night). Charge the batteries 12 hours before use.

The batteries must be charged at least every 3rd month - otherwise there might be damages due to self-discharge. It is recommended to test the battery function at least once every year.

## Recharging the battery:

- Plug the charging lead into the handset
- Plug the charger into the electricity supply and switch ON. A yellow light on the charger will light up
- Charge the batteries until the lamp on the charger turns green

## Warning

### Caregiver

- Do not leave children and youngsters alone with the handset
- Do not let children and youngsters play with the handset
- Do not leave the handset within reach of children and youngsters.
- It is important to remove the transport plastic bag before using the handset. Cables need to be placed in such a way that users or carers cannot trip and injure themselves.

## Product

- The power unit does not have a safety stop
- The system must not be sprayed directly with a highpressure cleaner
- Cleaning with a steam cleaner is not permitted
- Do only use the power unit within specified working limits

- It is recommended to check the handset and cable for damage and holes caused by violent handling at least once a year
- Do not submerge the handset in water
- Do not attempt to repair any part of the power unit.

## Technical Data

### Battery - BA18

- Capacity 1.3 Ah - 24V DC
- Charging current is max. 500 mA
- IPX6
- Duty cycle: Max. 10% or 2 min. use then 18 min. without use

The batteries have a maximum life of 4 years but this can vary depending upon the conditions of use and charging practices

### Battery charger - CH01

- Mains: 100 - 240VAC/50-60 Hz switch mode
- Charging voltage: 27.6 VDC
- Charging current: Max. 500 mA

### Actuator - LA28/LA32

- Input voltage: 24V
- Noise level:  $\leq 45$  dB(A)
- IPX5

### Handset - HB 40


- IPX1

## Detachable parts

- Handset - HB 40
- Battery - BA 18

## Applied part

- Handset - HB 40

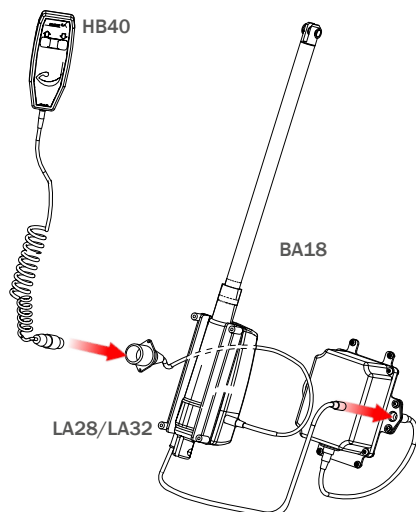## IP-Rating table

IPX0	No protection
IPX1	Protection against vertically falling drops of water e.g. condensation
IPX2	Protection against direct sprays of water up to 15° from the vertical
IPX3	Protected against direct sprays of water up to 60° from the vertical
IPX4	Protection against water sprayed from all directions - limited ingress permitted
IPX5	Protected against low pressure jets of water from all directions - limited ingress
IPX6	Protected against temporary flooding of water, e.g. for use on ship decks - limited ingress permitted
IPX7	Protected against the effect of immersion between 15 cm and 1 m
IPX8	Protects against long periods of immersion under pressure

### Electromagnetic compatibility (EMC) environment

This equipment is for use in home healthcare facility environment.

This equipment is not likely to cause any harmful interference in nearby equipment; however, it can be impaired by electromagnetic (EM) interference emitted by other equipment, which can lead to malfunctions. Do not use this equipment where there is a risk that other electrical and/or electronic equipment can interfere with this equipment.

### This equipment complies with the following standards

IEC 60601-1-2:2014 Medical electrical equipment - Part 1-2: General requirements for basic safety and essential performance - Collateral Standard: Electromagnetic disturbances - Requirements and tests (EN 60601-1-2:2015)

EN 12182:2012, Clause 7, Assistive products for persons with disability - General requirements and test methods

### EMC - WARNINGS

Use of this equipment adjacent to or stacked with other equipment should be avoided because it could result in improper operation. If such use is necessary, this equipment and the other equipment should be observed to verify that they are operating normally.

Use of accessories, transducers and cables other than those specified or provided by R82 could result in increased EM emission or decreased EM immunity of this equipment and result in improper operation. Several power plugs are supplied with this equipment. Please use the correct power plug suitable for your country. Please choose among the supplied power plug adaptor units: English, Australian, EU, US/Japanese plugs.

Portable radio frequency (RF) communication equipment (including peripherals such as antenna cables and external antennas) should be used no closer than 30 cm (12 inches) to any part of the Wombat Living, including cables specified by R82. Otherwise, degradation of the performance of this equipment could result.

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Test modes during emission tests:

1. Movement mode (battery). Actuator running in/out
2. Charging mode, actuator not running.

EM emission standards	Classification/levels
Conducted and radiated RF emission CISPR 11	Group 1 (the product uses RF energy only for its internal function) Class B (the product is suitable for use in all establishments, including domestic establishments and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes)
Harmonic distortion IEC 61000-3-2	Class A. No testing performed, as the rated power for this equipment is less than 75 W. No limits are specified for equipment with a rated power of 75 W or less.
Voltage fluctuations and flicker IEC 61000-3-3	$d_{max} \leq 4\%$ (without additional conditions)

**Test modes during immunity tests:**

Battery mode, standby armed/ready.

Charging mode, actuator not running.

EM immunity standards	IEC 60601-1-2:2014 test levels	Additional test levels
Electrostatic discharge IEC 61000-4-2	± 8 kV contact ± 2 kV, ± 4 kV, ± 8 kV, ± 15 kV air	± 6 kV contact
Radiated RF EM fields IEC 61000-4-3	10 V/m 80 MHz – 2.7 GHz 80 % AM at 1 kHz	20 V/m 800 MHz – 2.5 GHz 80 % AM at 1 kHz
Proximity fields form RF wireless communication equipment IEC 61000-4-3	IEC 60601-1-2:2014 Table 9	N/A
Electrical fast transients / bursts IEC 61000-4-4	± 1 kV, ± 2 kV 100 kHz repetition frequency	± 1 kV, ± 2 kV 5 kHz repetition frequency
Surges, Line-to-line IEC 61000-4-5	± 0,5 kV, ± 1 kV 230 VAC	± 0,5 kV, ± 1 kV 100 VAC and 240 VAC
Conducted disturbance induced by RF fields IEC 61000-4-6	6 Vrms 0.15 MHz – 80 MHz 80% AM at 1 kHz	N/A
Rated power frequency magnetic fields IEC 61000-4-8	30 A/m 50 Hz	30 A/m 50 Hz and 60 Hz
Voltage dips, short interruptions and voltage variations IEC 61000-4-11	0 % $U_T$ for 0,5 cycle at 0°, 45°, 90°, 135°, 180°, 225°, 270° and 315° phase angles 0 % $U_T$ for 1 cycle 70 % $U_T$ for 25 cycles 0 % $U_T$ for 250 cycles 230 VAC	0 % $U_T$ for 1 cycle 40 % $U_T$ for 10 cycles 70 % $U_T$ for 25 cycles 100 VAC and 240 VAC
Variation of power frequency IEC 61000-4-28	N/A	49 Hz and 51 Hz 100 VAC and 240 VAC, 50 Hz 59 Hz and 61 Hz 100 VAC and 240 VAC, 60 Hz

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## R82 Nederland B.V.
## R82 Inc.
## R82 au Pty Limited