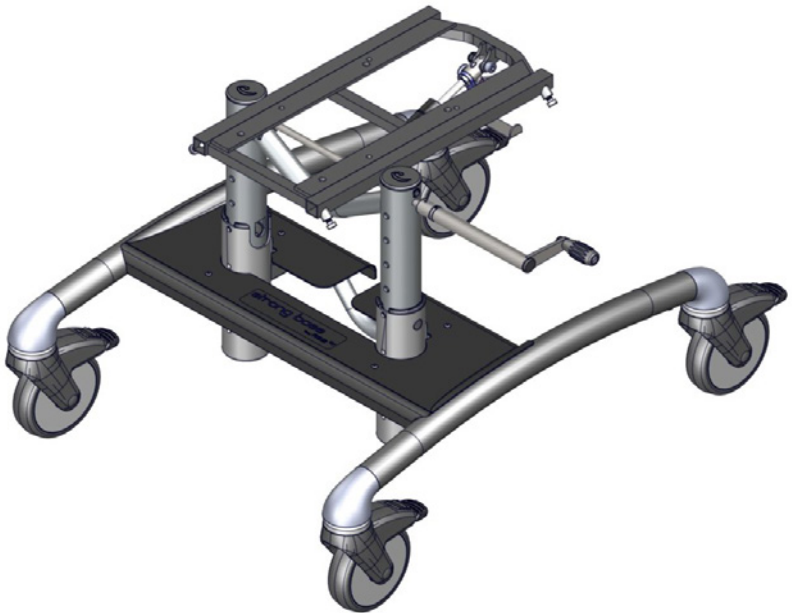
600 mm (23½")

<120 kg (264.5 lb)

## Strong Base User Guide


EN 12182

2018-09-rev.5

support.R82.org

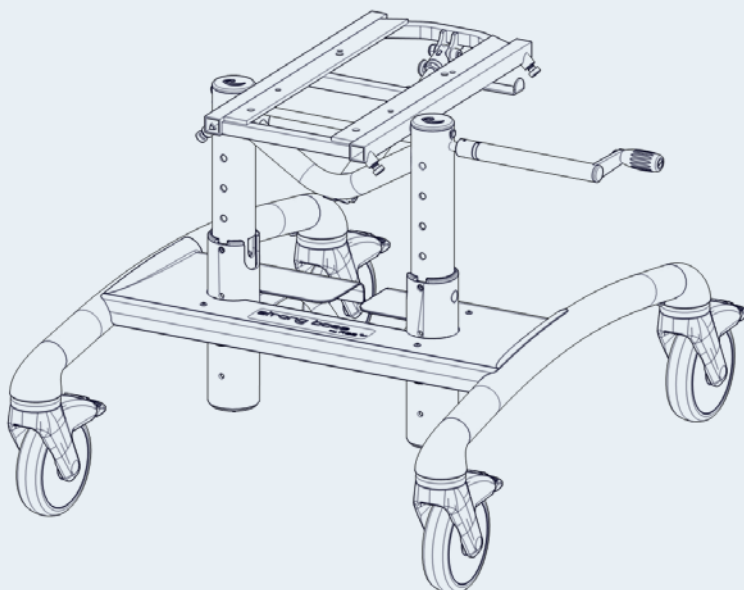
|   |         |                                  |                    |
|---|---------|----------------------------------|--------------------|
|  | 3       | ENGLISH<br>DEUTSCH               | 22 - 24<br>25 - 27 |
|  | 4       | NEDERLANDS<br>DANSK              | 28 - 30<br>31 - 33 |
|  | 5 - 8   | NORSK<br>SVENSKA                 | 34 - 36<br>37 - 39 |
|  | 9       | SUOMI<br>ESPAÑOL                 | 40 - 42<br>43 - 45 |
|  | 10 - 11 | FRANÇAIS<br>ITALIANO             | 46 - 48<br>49 - 51 |
|  | 12 - 14 | PORTUGUÊS<br>PORTUGUÊS DO BRASIL | 52 - 54<br>55 - 57 |
|  | 15      | РОССИЯ<br>POLSKI                 | 58 - 60<br>61 - 63 |
|  | 16      | ČESKY<br>中國                      | 64 - 66<br>67 - 69 |
|  | 17 - 19 | 日本<br>ΕΛΛΗΝΙΚΗ                   | 70 - 72<br>73 - 75 |
| mm (inch)   | 20      |                                  |                    |
|  | 21      |                                  |                    |
|   |         |                                  |                    |
|   |         |                                  |                    |
|   |         |                                  |                    |
|   |         |                                  |                    |
|   |         |                                  |                    |

<120 kg  
<264.5 lb

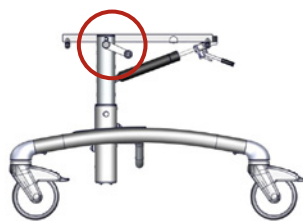
<120 kg  
<264.5 lb

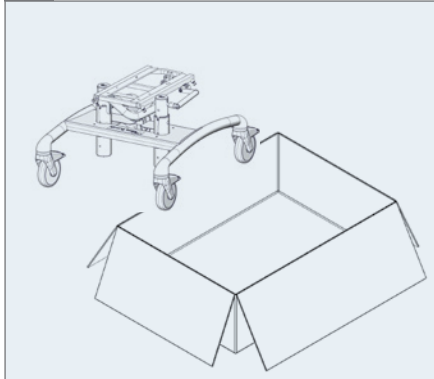
1.

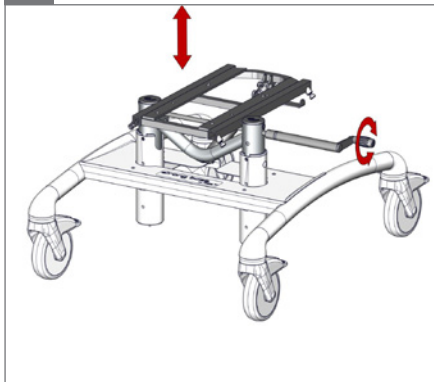2.


3.

4.

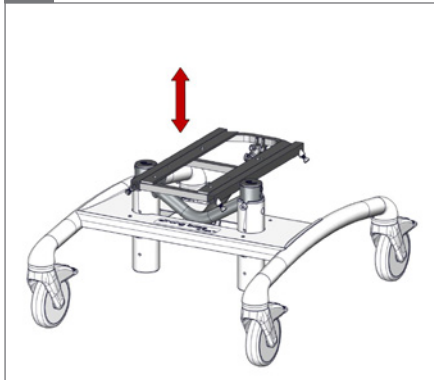⬡ 6 mm, ⚙ 13 mm

5.

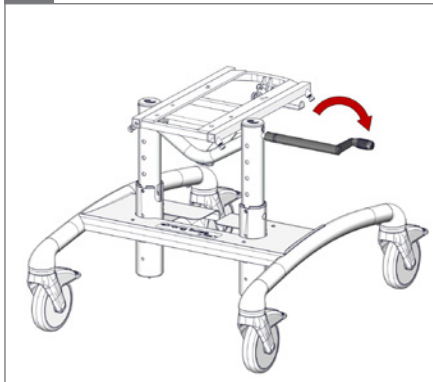
⬡ 6 mm, ⚙ 13 mm


1.


2.

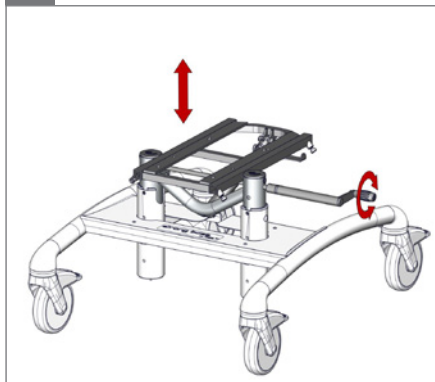3.

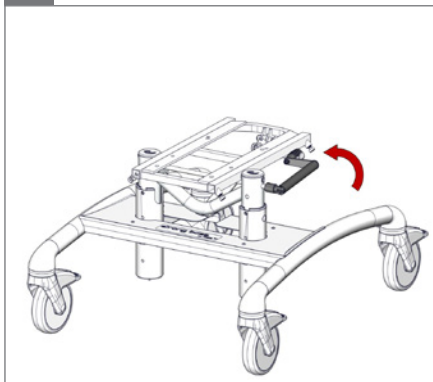
4.


5.

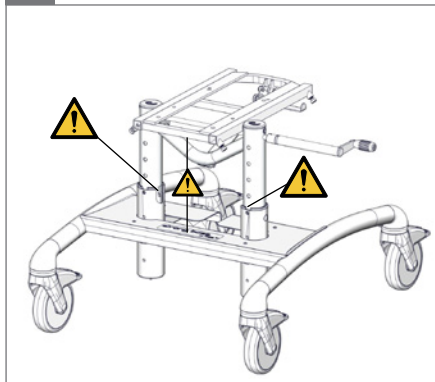05

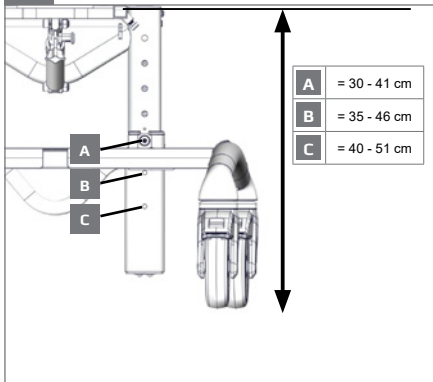6.

1.

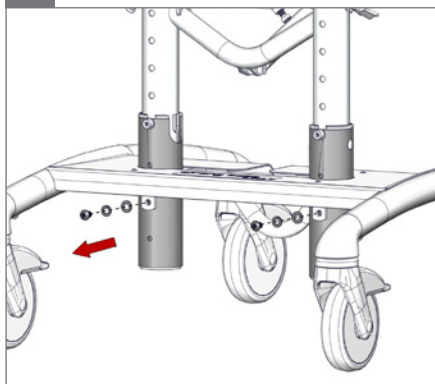
2.


3.

⌀ 5 mm


4.

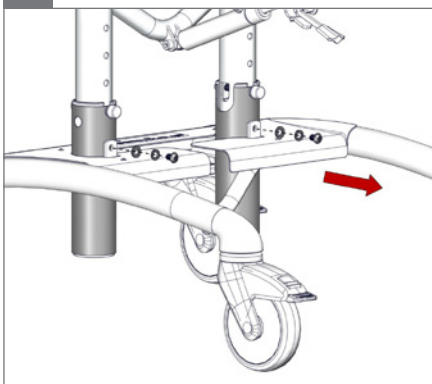⌀ 5 mm

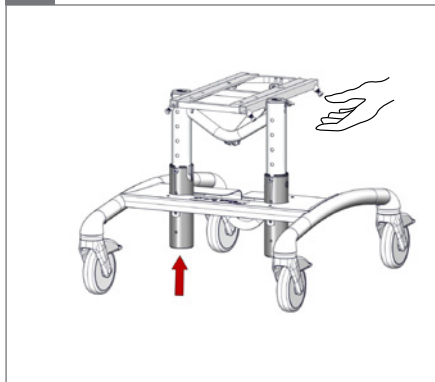5.


6.


7.

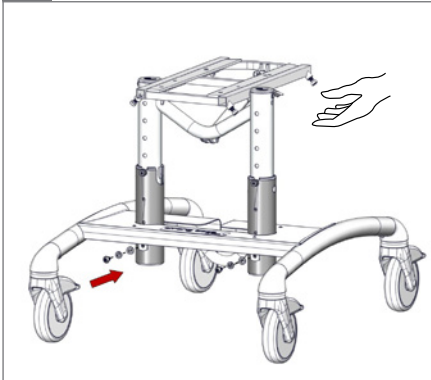⬡ 5+6 mm 2,5 Nm


8.

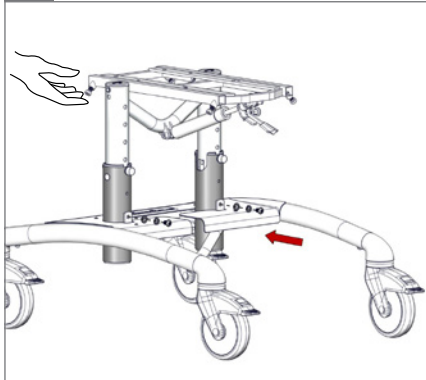⬡ 5 mm, ⚙️ 13 mm - 2,5 Nm


9.

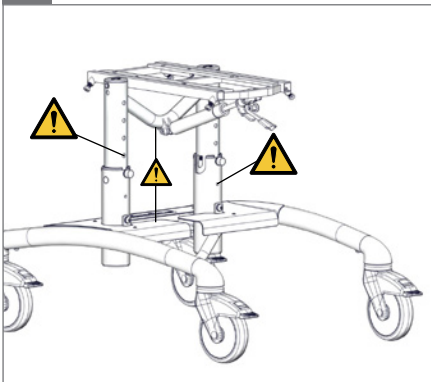⚠️ 05

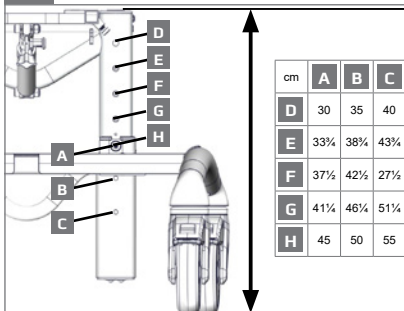10.

1.

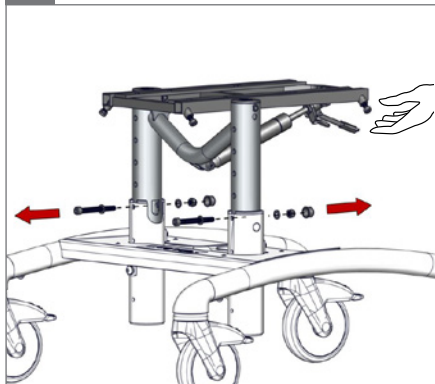
2.


3.

⌀ 6 mm, 13 mm

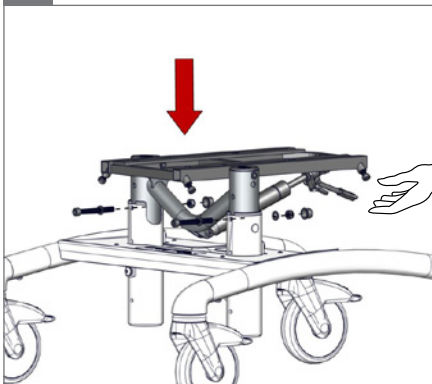
4.


5.

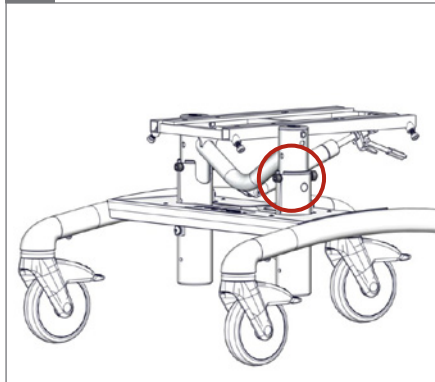⌀ 6 mm, 13 mm - 2,5 Nm

6.

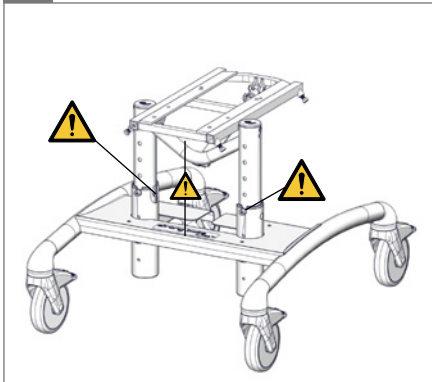⚠ 05



1.

 $-10^{\circ} - +25^{\circ}$ 

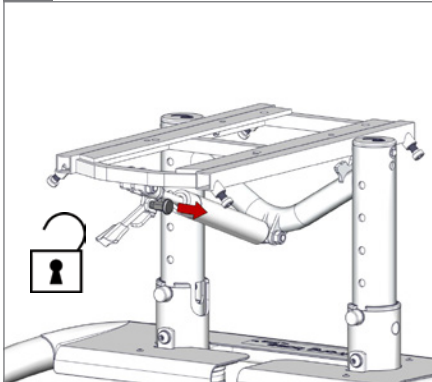
2.

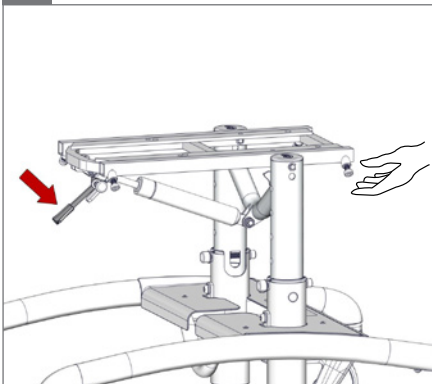
3.

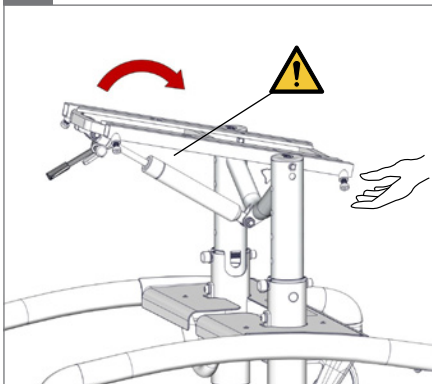
4.

 05

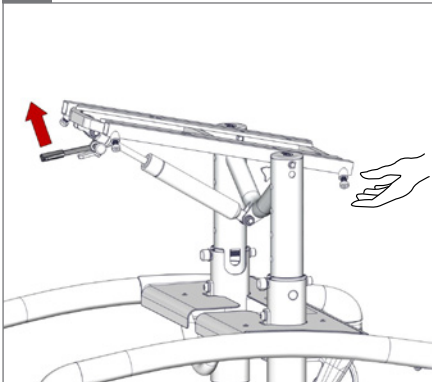5.

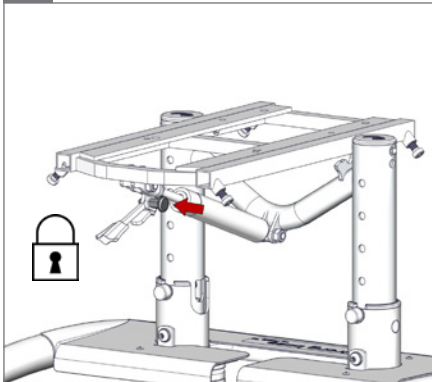6.


1.


2.

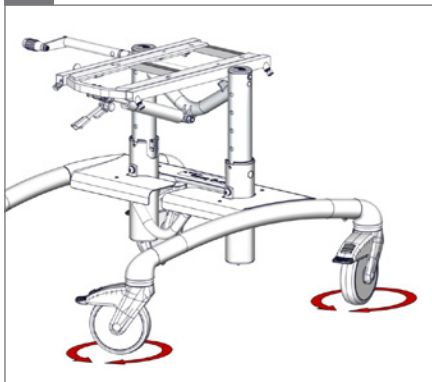
3.

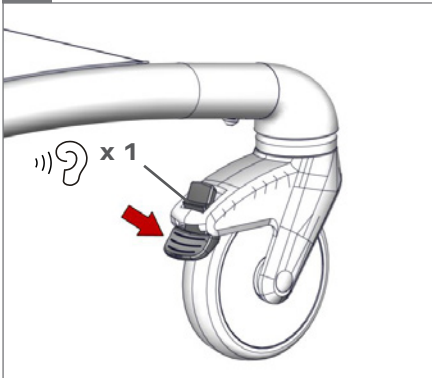4.

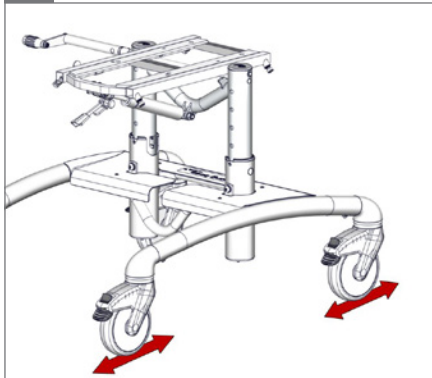5.



1.

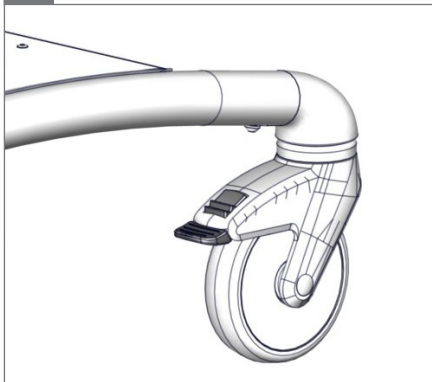
2.

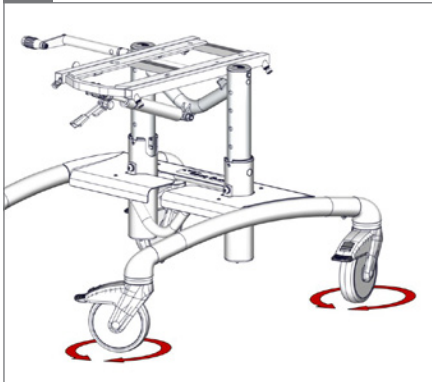
3.

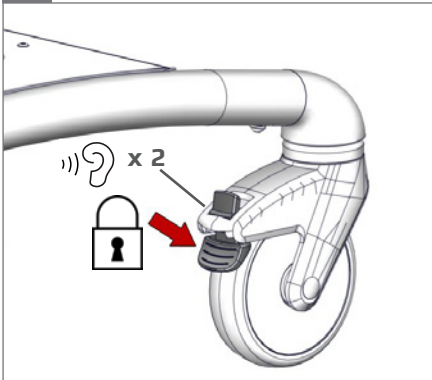4.

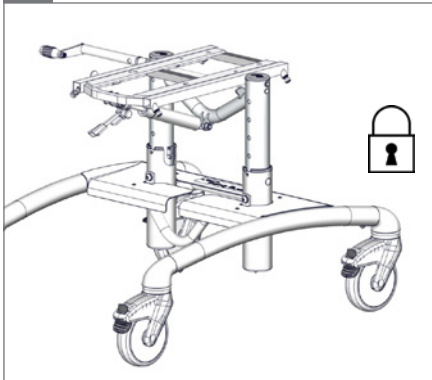5.

1.

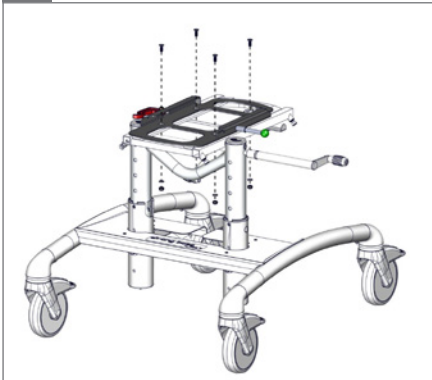
2.


3.


○ 4 mm, ⚙ 10 mm - 2,5 Nm

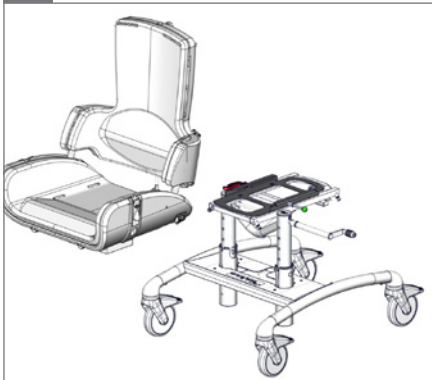4.

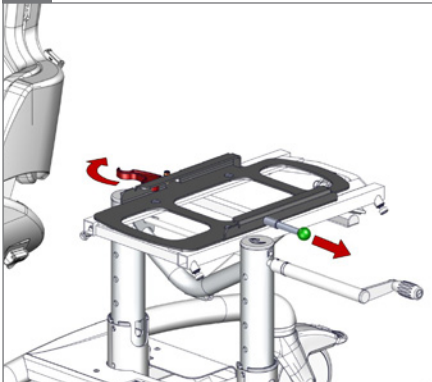5.

6.


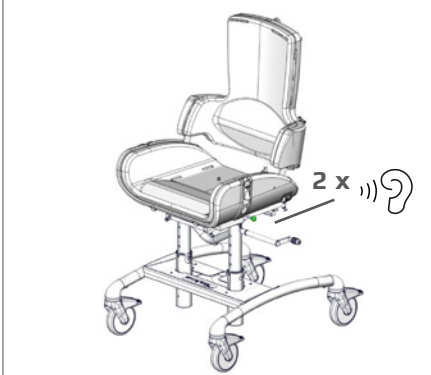
7.


8.

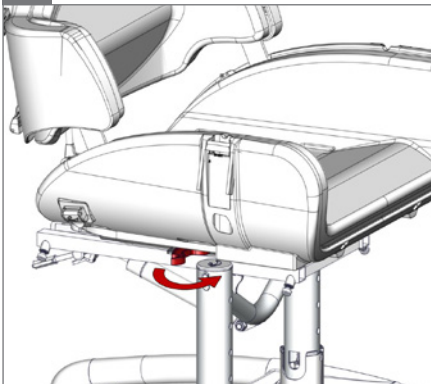9.

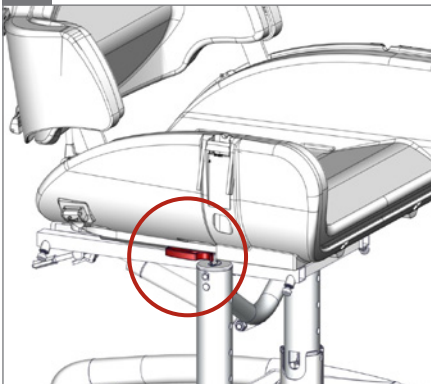10.


1.


2.

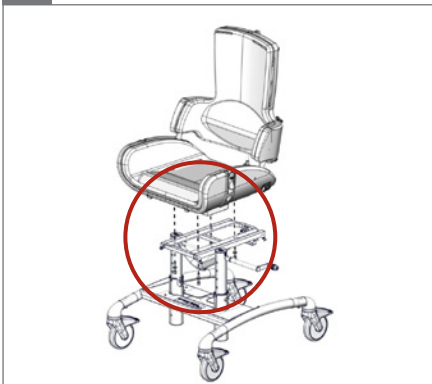⬡ 6 mm - 2,5 Nm

3.

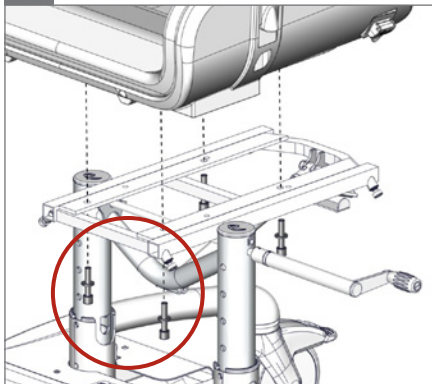⬡ 6 mm - 2,5 Nm

4.

⚠ 05

5.

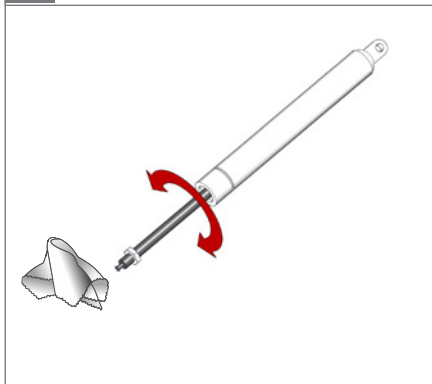
1.

2.




3.

 17 mm

4.

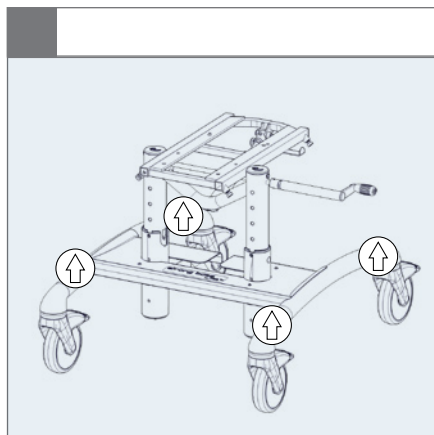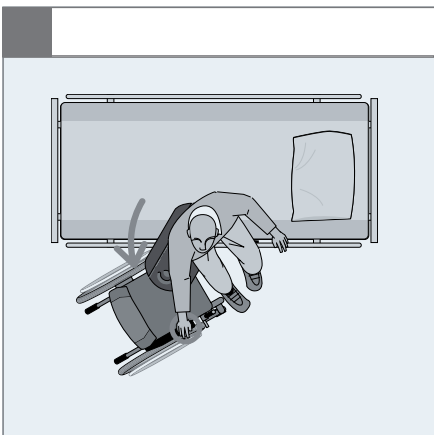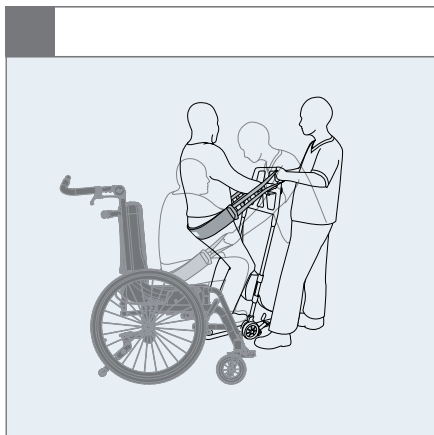 17 mm

**A** **B** **C**

**A**

XXXXXXX  
DMR ver.:XXXX  
Art. no.:XXXXXXX  
Product:XXXX - Size X

Max load: xxx kg

**B**

**R82**

Parallelsvej 3  
DK-8751 Gedved  
R82.org  
Made in XXXXXXXX YYY-MM-DD

CE

**C**

**EN:** For indoor use

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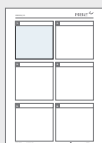
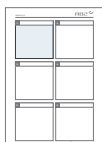
9996097118

## ENGLISH

We appreciate your choice of a new product from R82 - a worldwide supplier of technical aids and appliances for children and teenagers with special needs. To fully benefit from the options offered by this product please read this User Guide before use and save it for future reference.

### Intended purpose

The Strong Base is a frame for larger seat units. It is simple and strong and particularly suitable for Panda Futura 5 and x:panda 4 seats from R82. The Strong Base comes in two version, with either manual or spindle height adjustment. This product is only for indoor use but can in an emergency be taken outside.

### Enclosed tools:

- 5 & 6 mm Allen key and 13 mm spanner.

### Declaration of Conformity

This product conforms to the requirements of the Medical Devices Directive (93/42/EEC). The CE mark must be removed when rebuilding the product, when used in combination with a product from another manufacturer or when using other than original R82 spare parts and fittings. Further, this product conforms to the requirements according to:

- EN 12182

### Disposal

When a product reaches the end of its lifetime, it should be separated by material type so that the parts can be recycled or disposed of properly. If required, please contact your local dealer for a precise description of each material. Please contact your local authorities to be informed of the options available for recycling in an environmentally-friendly way.

### Accessories and spare parts

The products from R82 can be supplied with a variety of accessories which comply with the needs of the individual user. Spare parts can be ordered on request. Find the specific accessories on our website or contact your local dealer for further information.

### R82 Warranty

R82 offers a 2-year warranty against defects in workmanship and materials and a 5-year warranty on breakage of the metal frame caused by defects in welds. The warranty would be adversely affected if the customer's responsibility of servicing and/or maintenance is not carried out according to the guidelines and intervals prescribed by the supplier and/or stated in the User Guide. For further information, we refer to the R82 homepage/download.

The warranty is only granted in the country where the product was purchased and if the product can be identified by the serial number. The warranty does not cover accidental damage, including damage caused by misuse or neglect. The warranty does not extend to consumable items e.g. tires or upholstery, which are subject to normal wear and tear and need periodic replacement.

The warranty is null and void if other than original parts/accessories are used, or if the product has not been maintained, repaired or altered according to the guidelines and intervals prescribed by the supplier and/or stated in the User Guide. R82 reserves the right to inspect the product being claimed for and the relevant documentation before agreeing to the warranty claim, and to decide upon whether to replace or repair the defective product. It is the customer's responsibility to return the item being claimed for under warranty to the address of purchase.

### Presales information

- [support.r82.org](mailto:support.r82.org)

## Safety

Signs, symbols and instructions placed on this product must never be covered or removed and must remain present and clearly legible throughout the entire lifetime of the product. Immediately replace or repair illegible or damaged signs, symbols and instructions. Contact your local dealer for instructions.

### Caregiver

- Read the "User Guide" carefully before use and keep it for future reference. Incorrect use of this product may cause serious injury to the user
- Always use the correct lifting techniques and aids for this purpose
- Never leave the user unattended in this product. Ensure permanent supervision by an adult
- Repair/replacements must only be made using new original R82 spare parts and fittings and carried out in accordance with the guidelines and service intervals prescribed by the supplier
- If there is any doubt as to the continued safe use of your R82 product or if any parts should fail, stop using the product immediately and contact your local dealer as soon as possible

### Environment

- Observe the surface temperature on the product, before placing an user in it. This goes especially for users with insensitive skin as they cannot feel the heat. If the surface temperature is above 41 °C then allow the product to cool before use
- Use this product on a level surface without any obstacles. Be aware of reduced maneuverability on slippery surfaces

### User

- If the user is close to the max. load limit and/or has a lot of involuntary movements, e.g. rocking, a larger size with a higher max. load or a different R82 product should be considered

### Product

- Carry out all positional adjustments on the product and accessories and ensure that all knobs, screws and buckles are securely fastened before use. Keep all tools out of the reach of children
- Activate the brakes before placing the user into the product
- Ensure the stability of the product is maintained before placing the user in it
- We recommend you notify the user before angling the seat and back
- Ensure the castors are fully-functional and secure before use
- Inspect the product and all of its accessories and replace any worn out parts before use
- Do not expose the gas spring to pressure or high temperatures. DO NOT puncture
- Do not expose the piston rod to scratches etc. It is important to use a cloth or the like, if tools are used to adjust the gas spring
- This product is not to be occupied in a motor vehicle. The user must be positioned in a vehicle seat and the product must be stored in a luggage compartment
- The user must not occupy the product while it is being transported into or out of a vehicle by a lift

### Environmental conditions

- Operating temperature: 5 °C to +40 °C
- Relative humidity: 5% to 90%
- Atmospheric pressure: 800 to 1060 hPa

## Service information

Maintenance of a medical device is wholly the responsibility of the owner of that device. Failure to maintain a device in accordance with the instructions may invalidate the device's warranty. Furthermore, failure to maintain a device may compromise the clinical condition or safety of users and/or their caregivers.

### 1st Day

- Read the User Guide carefully before use and store it in a safe place for future reference
- Store the enclosed tools on/at the product

### Daily

- Use a dry cloth to clean the product
- Visual inspection. Repair or replace any damaged or worn-out parts
- Ensure that all fasteners are in place and fitted correctly
- Check all fittings and buckles for signs of worn out parts
- Be aware of any MAX mark indications before adjustment

### Weekly

- Check that all castors are moving freely and that all wheel locks/brakes are easily activated
- Wipe off any residue and dirt from the product using a cloth with warm water and a mild detergent/soap and dry before use

### Monthly

- All nuts and bolts on this product must be checked and tightened to avoid unnecessary failures
- Oil the swing-away parts. We recommend you use a professional lubricant system

### Annually

- Inspect the frame for cracks or signs for worn-out parts and carry out annual service. Never use a product which appears to be substandard or faulty

### Warewashing machine

This product can be washed at 60° using a mild detergent for 10 minutes in a washing machine designed for medical devices. Use the machine's drying function to dry the product.

### Disinfection

The product can be disinfected with a 70% disinfectant IPA solution. It is recommended to wipe off any residue and dirt from the product, using a cloth with warm water and a mild detergent/soap without chlorine and let it dry before disinfection.

### Service interval

A detailed inspection of the product must be made every 12 months (every 6 months for heavy users) and every time the product is reissued for use. The inspection should be carried out by a technically competent person who has been trained in the use of the product.

The lifetime of this product in normal use is 5 years if all maintenance and servicing is carried out in accordance with the manufacturer's instructions and demonstrably recorded.

### Service Check List:

Check and adjust the following functions as per their intended use. Repair or replace any damaged or worn-out parts. Please check the:

- The frame height can be adjusted using the spindle
- Threads for fasteners are in good condition
- Seat plane adjustment. The seat plane is adjustable and stays in position after adjustment.
- Gas-spring for seat tilt is adjusted correctly and is possible to lock the seat plane in position
- Castors are moving freely and all wheel locks/brakes are easily activated
- Nuts and bolts on the product are tightened
- Fittings and buckles for signs of worn out parts
- Product for cracks or signs for worn-out parts
- All labels on the product are intact

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© 2014 R82 A/S All rights reserved The R82 logo and the icons are registered trademarks of R82 A/S
## R82 A/S