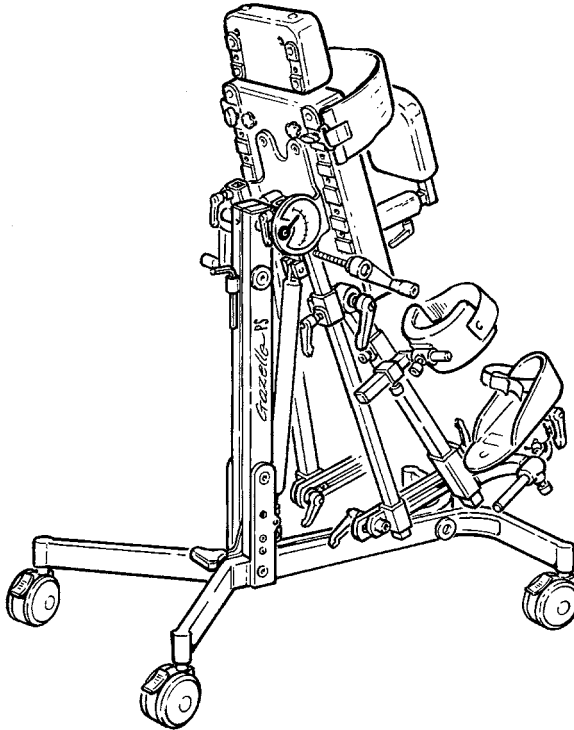
90° - 15°

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
<80 kg 176.4 lb)

Gazelle PS User Guide

EN 12182


2017-06 - rev.9


	3	ENGLISH DEUTSCH	10 - 19 20 - 29
	3	NEDERLANDS DANSK	30 - 39 40 - 49
	4	NORSK SVENSKA	50 - 59 60 - 69
mm (inch)	5	SUOMI FRANÇAIS	70 - 79 80 - 89
	6	ITALIANO ČESKY	90 - 99 100 - 109
	7 - 9		

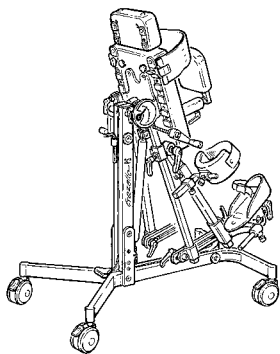
1.

Gazelle PS 1

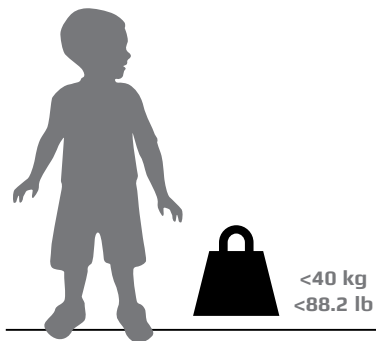Gazelle PS 2

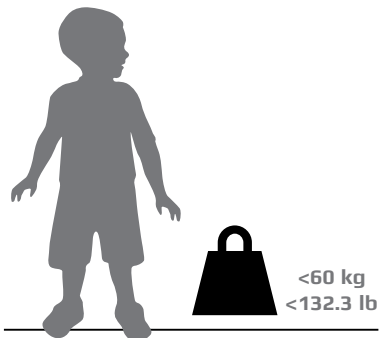Gazelle PS 3

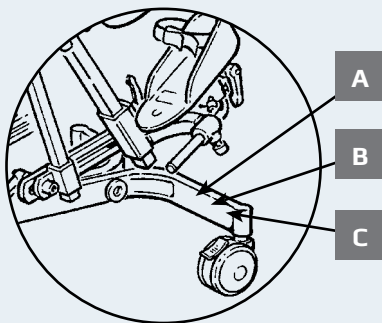
A

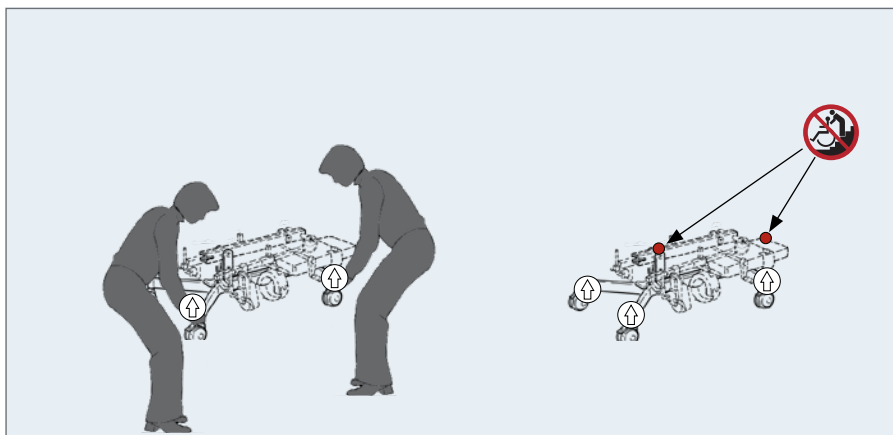
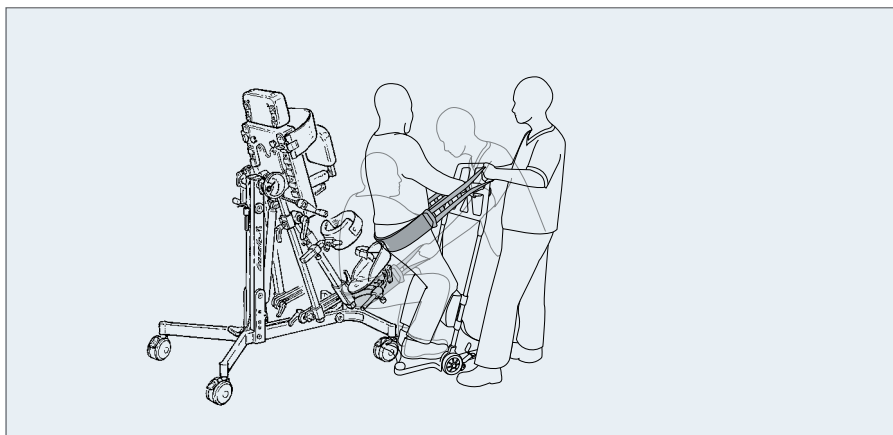
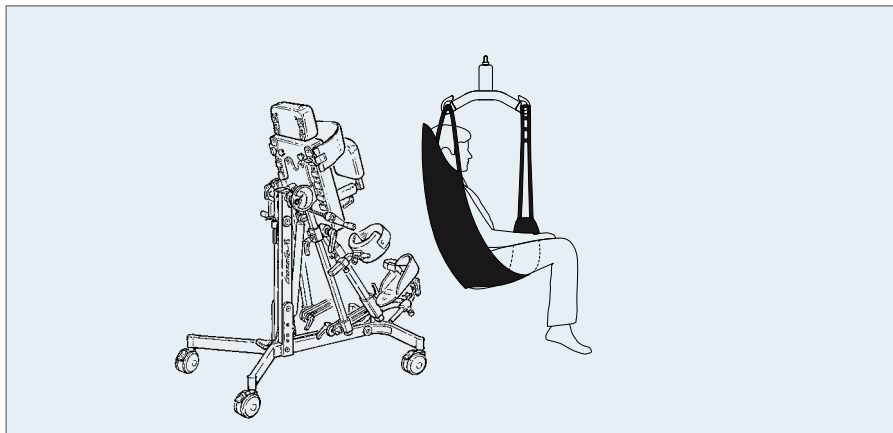
B

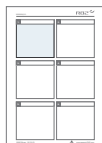
C

mm (inch)


	 1	 2	 3
	480 (19")	580 (23")	650 (26")
	760 (30")	880 (35")	990 (35")
	760 (30")	800 (35")	900 (35")
	450 - 950mm (18 - 37")	600 - 1050 mm (24 - 41")	700 - 1250 mm (28 - 49")
	90 - -15°	90 - -15°	90 - -15°
	40 kg (88.2 lb)	60 kg (132.3 lb)	80 kg (176.4 lb)
	20 kg (44 lb)	22 kg (48 lb)	24 kg (53 lb)

9996097100

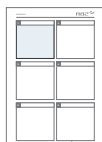9996097119

9996097120

EN: For indoor use

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IT: Per uso interno

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BR: Para uso interno

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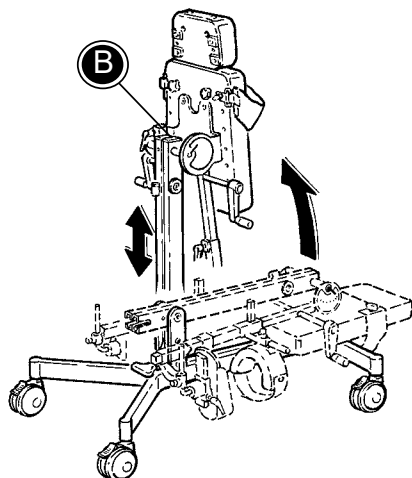
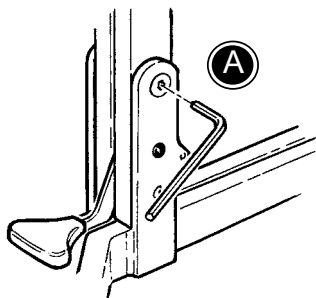
EN

**Head support with facial aperture**

Mounting of Head support with facial aperture:

- A. The two retaining bars go through the adjustable trunk support after removal of two plastic plugs.
- B. Positioning knobs can be screwed into place after making cut-outs in the adjustable trunk support.

It can be helpful to mount the head support with facial aperture before placing the child on the table in horizontal position. Remove the head support when the table is raised into vertical position.

Trunk support and extension

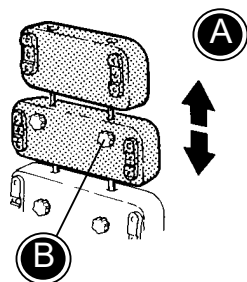
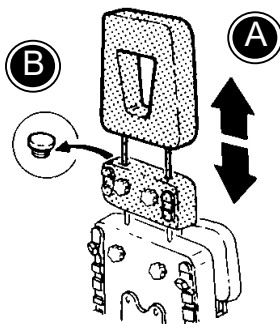
Height adjustable trunk support and extension can be mounted on the Gazelle PS (A).

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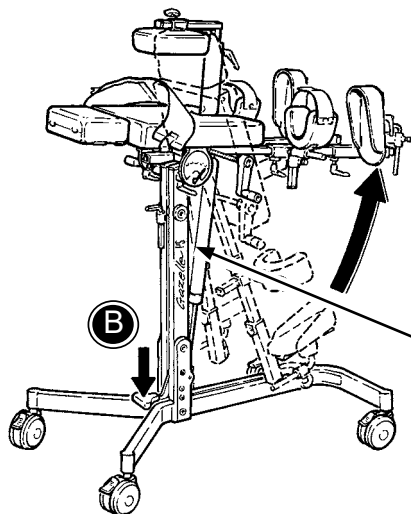
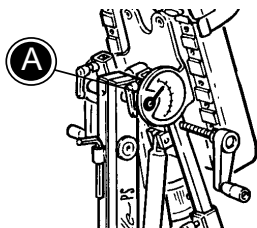
Ready for use

The Gazelle PS is folded when delivered, to obtain as little space as possible, during transportation. Follow the description below, to unfold your Gazelle PS:

1. Make sure that the four wheels are locked.
2. Press against the floor frame with your foot, as the Gazelle PS is raised to vertical position. Make sure the Gazelle PS fits exactly into the fitting on the floor frame.
3. With the Gazelle PS a in vertical position, use the enclosed 6 mm Allen key to fasten the screw (A). The Allen key is placed on the Gazelle PS (B) and is always nearby for adjustments.

Angling

Degree indicator (A)

Adjust the angle of the Gazelle PS from vertical to 15° under horizontal by pressing the foot pedal (B).

As security, the foot pedal can be locked at the wanted angle position. Turn the button at the lower end of the gaspring and the Gazelle PS locks (C).

- 05 Squeezing Hazard

Independent abduction

Adjusting the abduction 0° - 30° by turning the handle in both sides (A).

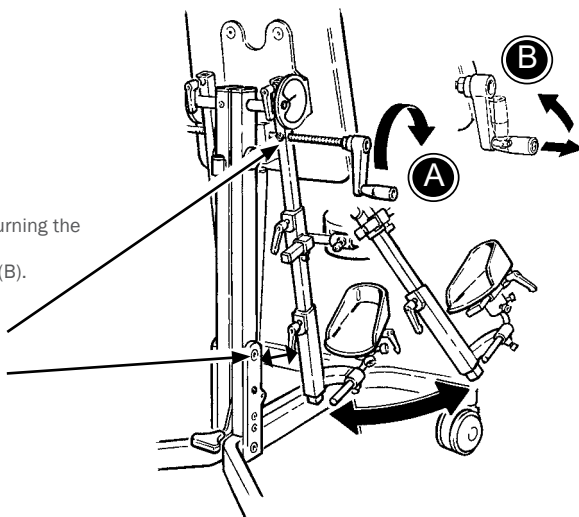
Retractable handle for tilt adjustment (B).

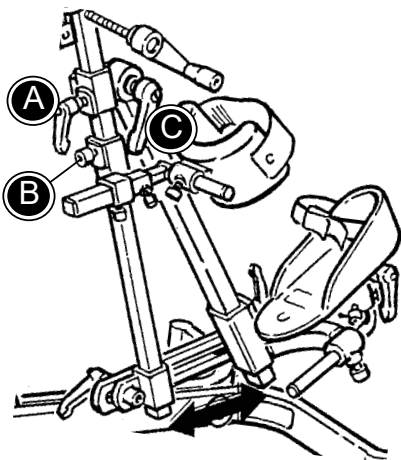
- 05 Squeezing hazard

- 05 Squeezing hazard

Knee support

As standard, the Gazelle PS is delivered with knee supports mounted to be used by front-standing children. Turn around the knee supports, to be able to use it for backstanding children:

1. Loosen the handle (A) to remove the foot supports.
2. Loosen the handle (B) to remove the complete knee support unit.
3. Mount the complete knee support unit on the opposite side and fasten with the handle.
4. Turn the fitting for the knee support 180°, by loosening the screws (C) and (D).
5. Loosen the screw (E) and turn around the knee support, for backstanding children.

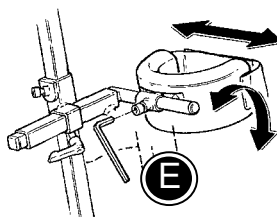
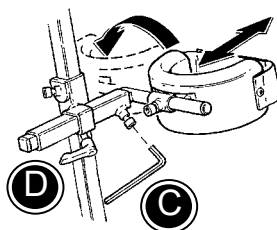
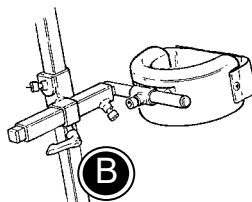
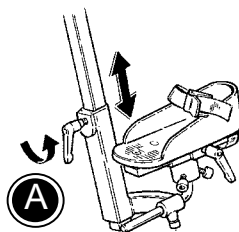
Adjust the knee support with the enclosed 6 mm Allen key. Loosen the screws (C) and (D) and the knee supports are depthadjustable. By loosening the screw (E) the knee supports adjust in width and angles. Use the handle (B) to adjust the height of the complete knee support unit.

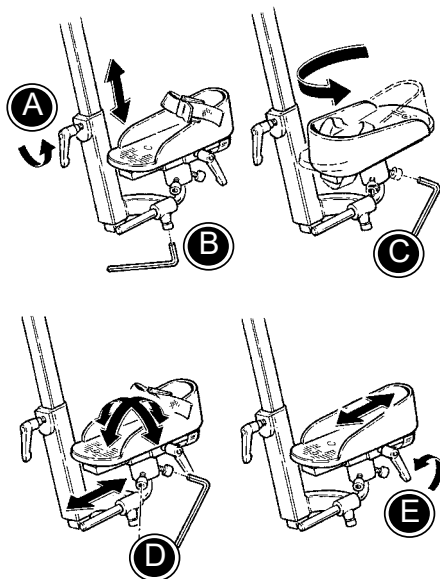
Angling of lower leg

Mount the bar for the lower leg angling on the abduction bar, as follows:

1. Loosen the handle (A) to remove the support.
2. Loosen the Allen screw (B) to remove the knee supports.
3. Mount the knee support on the bar for the lower leg angling and tighten the handle (A).
4. Mount the foot support on the bar for the lower leg angling.

Adjust the height of the lower leg angling by turning the handle (B) and adjust the angling by turning the handle (C). To adjust the foot support see page 16.

Foot support

For use of back standing:

When the knee support is turned, the foot supports must be mounted in the same side as they were demounted.

1. Mount the foot supports and tighten the handle. (A).
2. Loosen the screw (B) and adjust the depth of the foot support.
3. Loosen the screws (C) and (D) and turn the foot support 180°.
4. Tighten the screws in the requested position.

The foot plate mounting is mounted on the joint. By means of this, the footplate can be adjusted in different angles by loosening the screws (C) og (D).

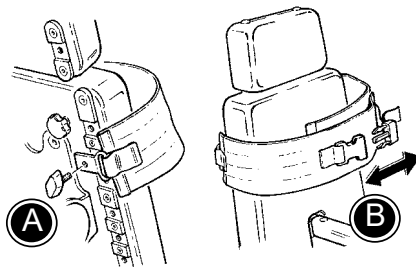
The plate can be adjusted in the depth by loosening the handle (E), or the unbracho screw (B).

Strap

Mount the strap on the trunk support or on the basic plate considering where the support is needed.

The strap is mounted on the back (A) and fastened in front with velcro and a snap-lock fitting (B).

Back support

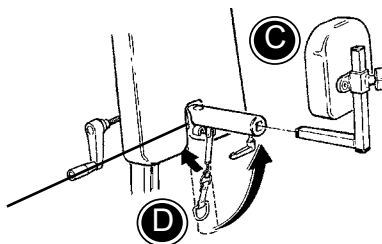
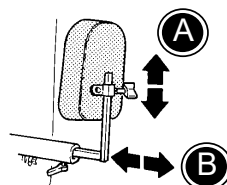
The back support is adjustable in height (A) and in depth (B).

Before placing the child:

1. Remove the back support (C)
2. Fold down the fitting for back support by pulling the release chain (D).

Fasten with the handles.

The child may not sit on the mounting for the back support. There must always be a couple of centimetres between the child and the mounting.

05 Squeezing hazard!

Take care when refitting the back support.

Tray with / without bowl

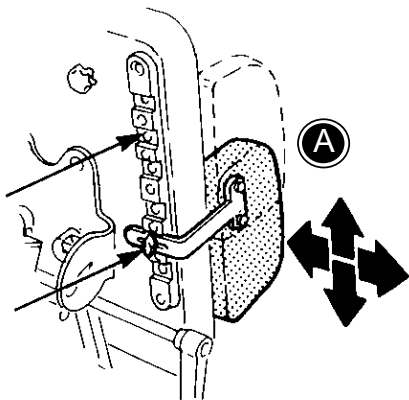
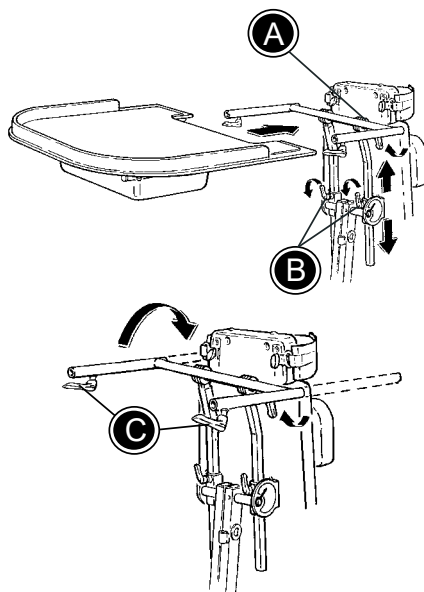
Loosen the big handles (A) and (B) completely, and place the tray-mounting in the two tubes on the back. Fasten the big handles (A) and (B) again.

Loosen the big handle (A) and the tray-mounting can be turned through 180 degree's for either prone or supine use.

Place the tray in the tray-mounting tubes. When the required position is achieved, fasten the big handles (C)


Loosen the handles (B) to adjust the height of the tray. Loosen the handle (A) to adjust the angle for the tray.

The tray with bowl is delivered with an extra cover plate, so it can be used as a normal tray.

Side support

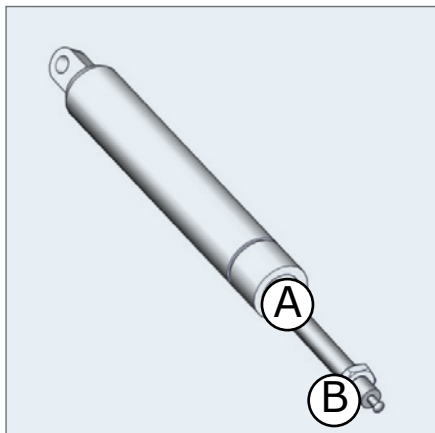Mount the side supports on the trunk support to give the rightsupport to the upper part of the body or on the basic plate to support the body and hips. The side supports are adjustable in height and in width (A).


Adjusting the gas spring

Gazelle PS is equipped with a gas spring for angle adjustment. After a while it might need adjustment. The adjustment is necessary when:

The gas spring is releasing itself
The gas spring can not be released

If the gas spring releases itself, loosen the nut (B) with a 17 mm wrench. Turn the bar (A) anti-clockwise to loosen. Tighten the nut (B) again.

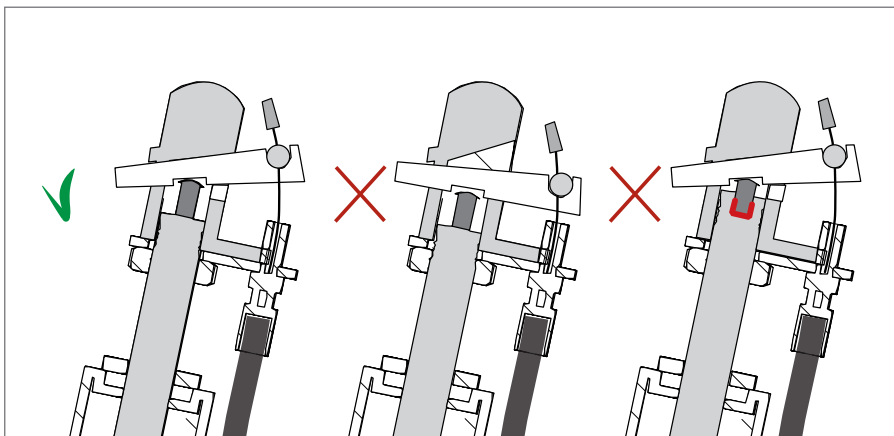If the gas spring will not release, loosen the nut (B) with a 17 mm wrench. Turn the bar (A) clockwise to tighten. Tighten the nut (B) again.

We appreciate your choice of a new product from R82 - a worldwide supplier of technical aids and appliances for children and teenagers with special needs. To fully benefit from the options offered by this product please read this User Guide before use and save it for future reference.

Intended purpose

The product is primarily for prone standing, however, supine standing is also possible. It tilts infinitely from vertical to 15° past horizontal. The product is available in 3 sizes and is for indoor use only.

The target group for this device is disabled children with cerebral palsy (CP) at ages from 1 to 14 years, GMFCS level III - IV. The consequence of CP is either spasticity, reduced walking or standing skills and/or impairments in addition to their motor disability. Other effects with no relation to this device is intellectual, vision and hearing impairment. Also children with muscular dystrophy, spina bifida, spinal cord injuries and neurological impairments can be subject to the device.

Enclosed tools:

- 6 mm Allen key

Declaration of Conformity

This product conforms to the requirements of the Medical Devices Directive (93/42/EEC). The CE mark must be removed when rebuilding the product, when used in combination with a product from another manufacturer or when using other than original R82 spare parts and fittings. Further, this product conforms to the requirements according to:

Disposal

When a product reaches the end of its lifetime, it should be separated by material type so that the parts can be recycled or disposed of properly. If required, please contact your local dealer for a precise description of each material. Please contact your local authorities to be informed of the options available for recycling in an environmentally-friendly way.

Accessories and spare parts

The products from R82 can be supplied with a variety of accessories which comply with the needs of the individual user. Spare parts can be ordered on request. Find the specific accessories on our website or contact your local dealer for further information.

R82 Warranty

R82 offers a 2-year warranty against defects in workmanship and materials and a 5-year warranty on breakage of the metal frame caused by defects in welds. The warranty would be adversely affected if the customer's responsibility of servicing and/or maintenance is not carried out according to the guidelines and intervals prescribed by the supplier and/or stated in the User Guide. For further information, we refer to the R82 homepage/download.

The warranty is only granted in the country where the product was purchased and if the product can be identified by the serial number. The warranty does not cover accidental damage, including damage caused by misuse or neglect. The warranty does not extend to consumable items e.g. tires or upholstery, which are subject to normal wear and tear and need periodic replacement.

The warranty is null and void if other than original parts/accessories are used, or if the product has not been maintained, repaired or altered according to the guidelines and intervals prescribed by the supplier and/or stated in the User Guide. R82 reserves the right to inspect the product being claimed for and the relevant documentation before agreeing to the warranty claim, and to decide upon whether to replace or repair the defective product. It is the customer's responsibility to return the item being claimed for under warranty to the address of purchase.

Presales information

- support.r82.org

Safety

Signs, symbols and instructions placed on this product must never be covered or removed and must remain present and clearly legible throughout the entire lifetime of the product. Immediately replace or repair illegible or damaged signs, symbols and instructions. Contact your local dealer for instructions.

Caregiver

- Read the "User Guide" carefully before use and keep it for future reference. Incorrect use of this product may cause serious injury to the user
- Always use the correct lifting techniques and aids for this purpose
- Never leave the user unattended in this product. Ensure permanent supervision by an adult
- Repair/replacements must only be made using new original R82 spare parts and fittings and carried out in accordance with the guidelines and service intervals prescribed by the supplier
- If there is any doubt as to the continued safe use of your R82 product or if any parts should fail, stop using the product immediately and contact your local dealer as soon as possible
- Ensure that moving parts does not squeeze or trap body parts when adjusting them
- Transportation of the product whilst the user is in it is not allowed.

Environment

- Observe the surface temperature on the product, before placing an user in it. This goes especially for users with insensitive skin as they cannot feel the heat. If the surface temperature is above 41 °C then allow the product to cool before use
- Use this product on a level surface without any obstacles. Be aware of reduced maneuverability on slippery surfaces
- When using on inclines, please refer to measurements

User

- If the user is close to the max. load limit and/or has a lot of involuntary movements, e.g. rocking, a larger size with a higher max. load or a different R82 product should be considered

Product

- Carry out all positional adjustments on the product and accessories and ensure that all knobs, screws and buckles are securely fastened before use. Keep all tools out of the reach of children
- Activate the brakes before placing the user into the product
- Ensure the stability of the product is maintained before placing the user in it
- Ensure the castors are fully-functional and secure before use
- Inspect the product and all of its accessories and replace any worn out parts before use
- Do not expose the gas spring to pressure or high temperatures. DO NOT puncture
- Do not expose the piston rod to scratches etc. It is important to use a cloth or the like, if tools are used to adjust the gas spring
- The footplate is for support only! The foot support does not have a weight bearing function
- This product is tested for resistance to ignition at level non-ignition in accordance with EN 1021-1 and EN 1021-2
- The product might lose its flame resistant characteristics if using other than original R82 cushioning
- This product is not to be occupied in a motor vehicle. The user must be positioned in a vehicle seat and the product must be stored in a luggage compartment
- The user must not occupy the product while it is being transported into or out of a vehicle by a lift

Environmental conditions

- Operating temperature: 5 °C to +40 °C
- Storage temperature: -10 °C to +50 °C
- Relative humidity: 5% to 90%
- Atmospheric pressure: 800 to 1060 hPa

Service information

Maintenance of a medical device is wholly the responsibility of the owner of that device. Failure to maintain a device in accordance with the instructions may invalidate the device's warranty. Furthermore, failure to maintain a device may compromise the clinical condition or safety of users and/or their caregivers.

1st Day

- Read the User Guide carefully before use and store it in a safe place for future reference

Daily

- Use a dry cloth to clean the product
- Visual inspection. Repair or replace any damaged or worn-out parts
- Ensure that all fasteners are in place and fitted correctly
- Check all fittings and buckles for signs of worn out parts
- Be aware of any MAX mark indications before adjustment

Weekly

- Check that all castors are moving freely and that all wheel locks/brakes are easily activated
- Wipe off any residue and dirt from the product using a cloth with warm water and a mild detergent/soap and dry before use
- Disinfect the product. please check the disinfection section in this User Guide for more information

Monthly

- All nuts and bolts on this product must be checked and tightened to avoid unnecessary failures
- Oil the swing-away parts. We recommend you use a professional lubricant system

Annually

- Inspect the frame for cracks or signs for worn-out parts and carry out annual service. Never use a product which appears to be substandard or faulty

Warewashing machine

This product can be washed at 60 ° using a mild detergent for 10 minutes in a washing machine designed for medical devices. Use the machine's drying function to dry the product.

Disinfection

The product can be disinfected with a 70% disinfectant IPA solution. It is recommended to wipe off any residue and dirt from the product, using a cloth with warm water and a mild detergent/soap without chlorine and let it dry before disinfection.

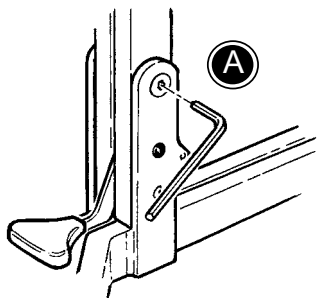
Service interval

A detailed inspection of the product must be made every 12 months (every 6 months for heavy users) and every time the product is reissued for use. The inspection should be carried out by a technically competent person who has been trained in the use of the product.

The lifetime of this product in normal use is 5 years if all maintenance and servicing is carried out in accordance with the manufacturer's instructions and demonstrably recorded.

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R82 Nederland B.V.
R82 Inc.
R82 au Pty Limited