0 - 87°

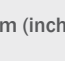
<100 kg (220.4 lb)

## Caribou User Guide

EN 12182

2018-05-rev.6

	3	ENGLISH DEUTSCH	24 - 29 30 - 35
	4 - 5	NEDERLANDS DANSK	36 - 41 42 - 47
	6	NORSK SVENSKA	48 - 53 54 - 59
	7-9	SUOMI ESPAÑOL	60 - 65 66 - 71
	10	FRANÇAIS ITALIANO	72 - 77 78 - 83
	11	PORTUGUÊS PORTUGUÊS DO BRASIL	84 - 89 90 - 95
	12	РОССИЯ POLSKI	96 - 101 102 - 107
	13	ČESKY 中國	108 - 113 114 - 119
	14	日本 ΕΛΛΗΝΙΚΗ	120 - 125 126 - 131
	15		
	16		
mm (inch)	17		
	18-21		
	22-23		


1.


2.

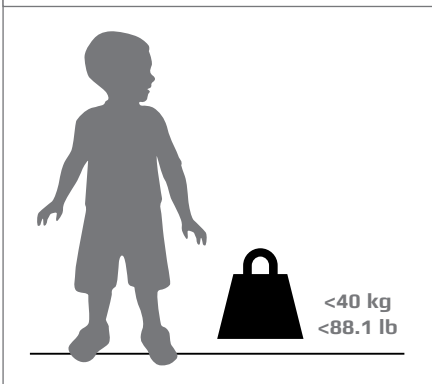
CARIBOU 1


CARIBOU 2

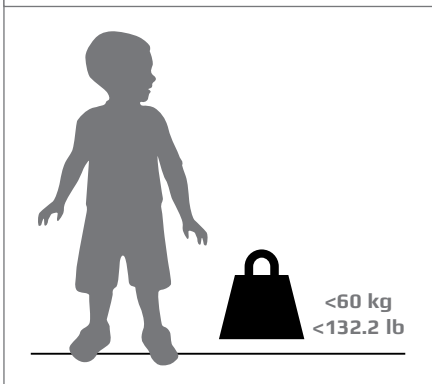CARIBOU 3

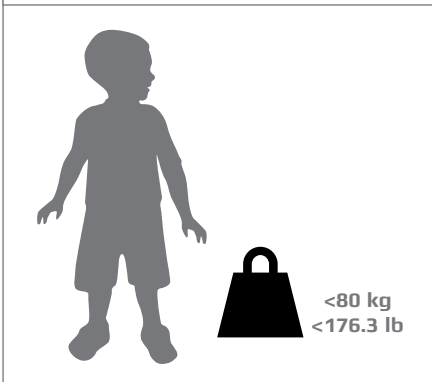
CARIBOU 4

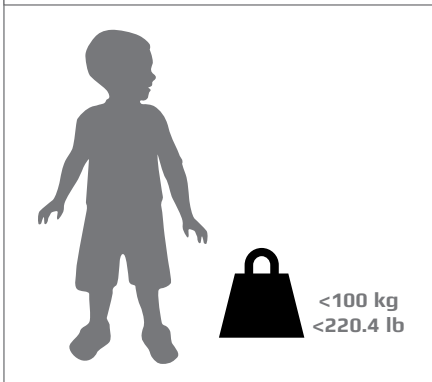
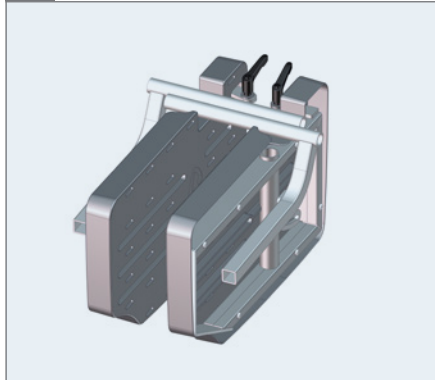
1.

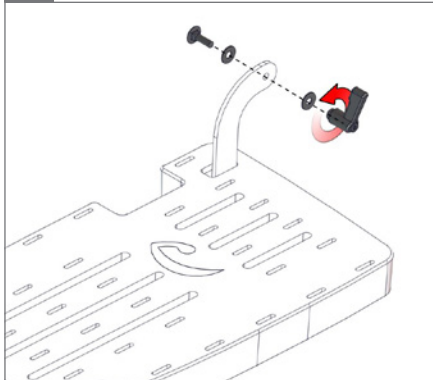
2.

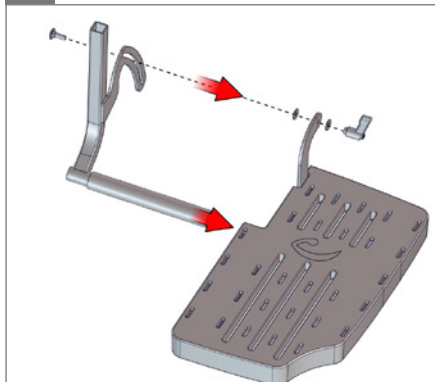
3.

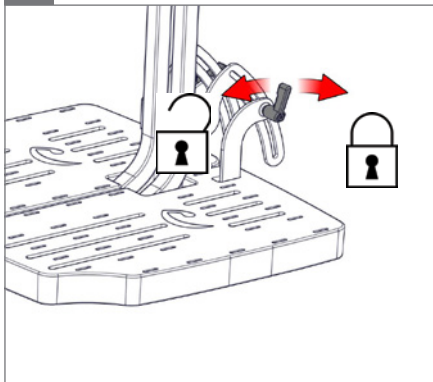
4.

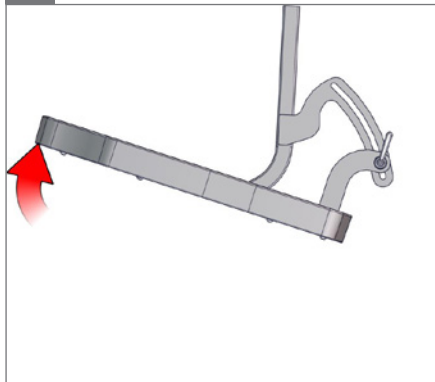5.

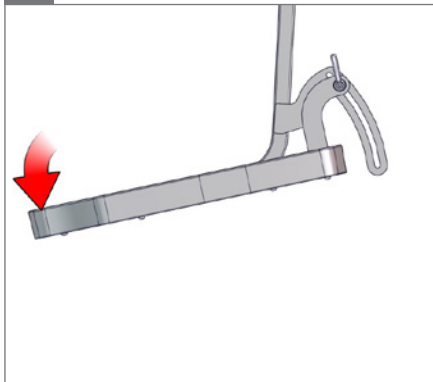6.

1a

1b

2a

⊙ 5 mm

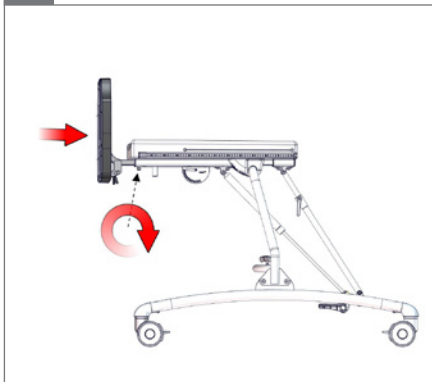
2b

⊙ 5 mm


3a

⊙ 5 mm

3b

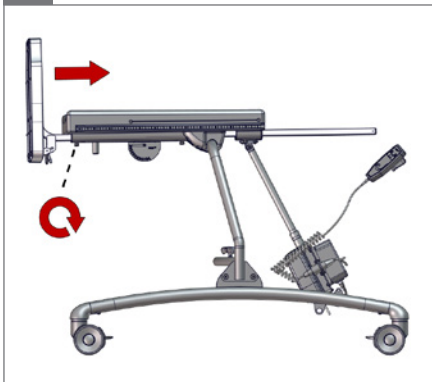⊙ 5 mm


1.

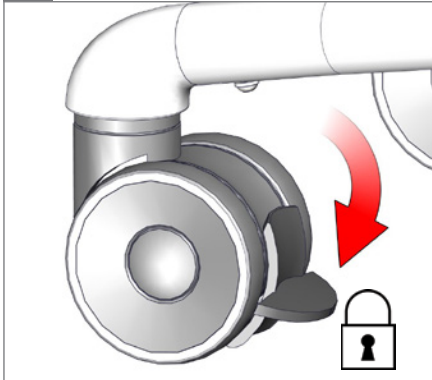2.

3.

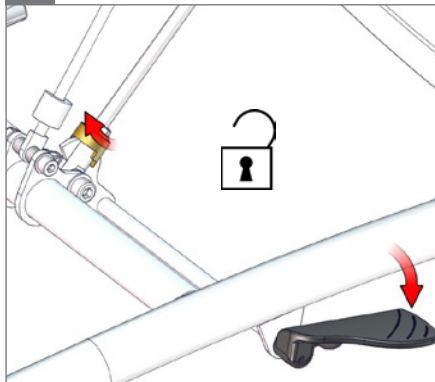
1.

2.


3.


4.

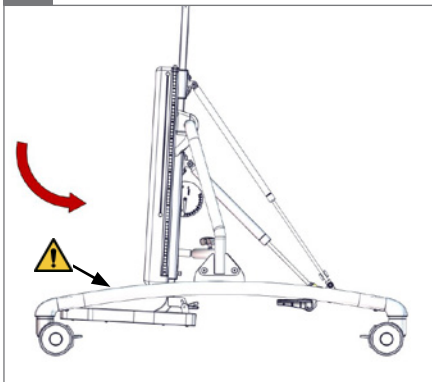05

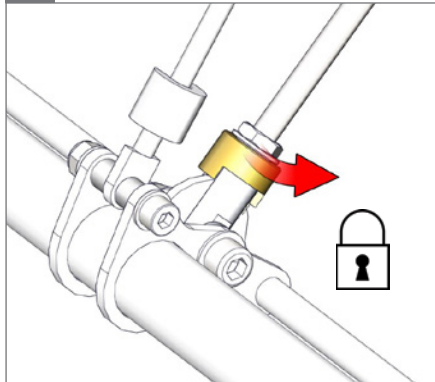5.


1.


2.

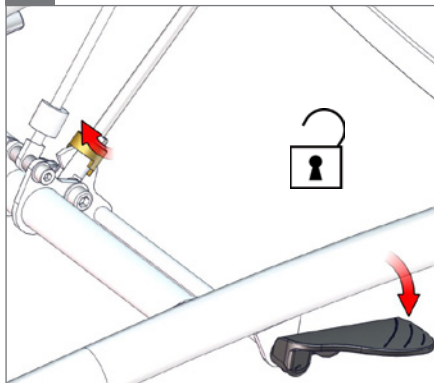3.

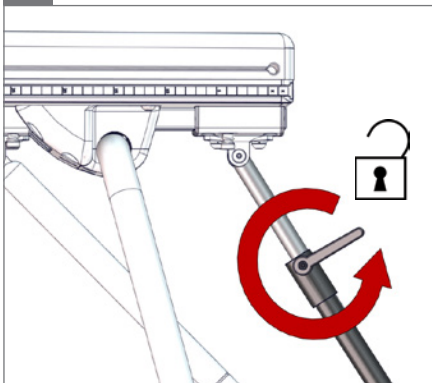
4.


5.

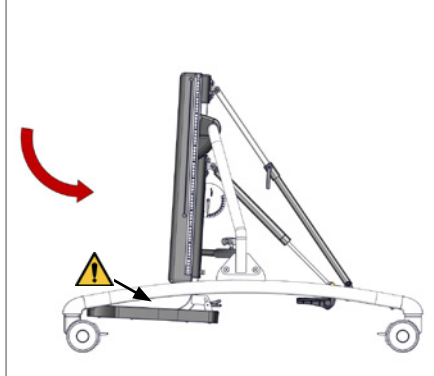05

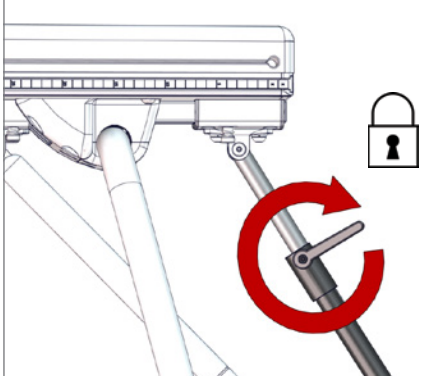
6.

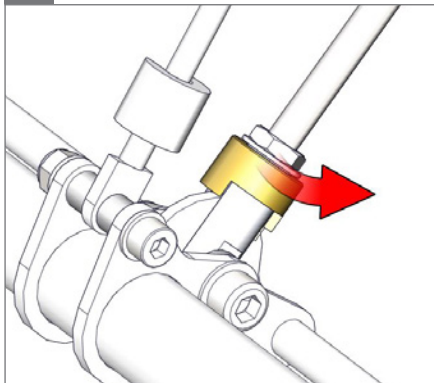7.

8.


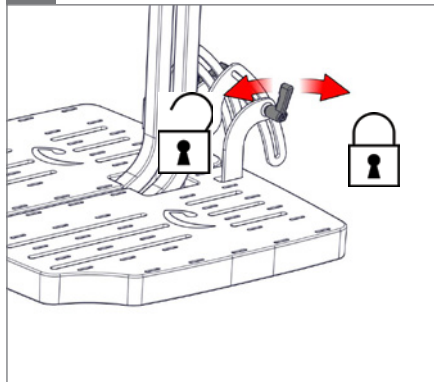
1.

2.

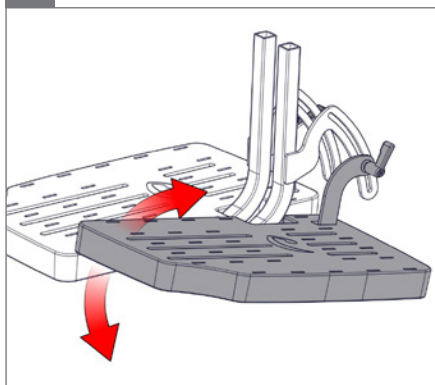
3.


4.

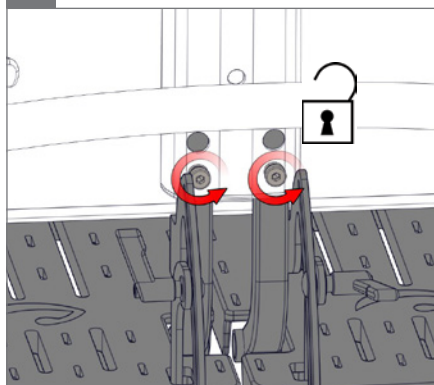○ 5 mm


5.

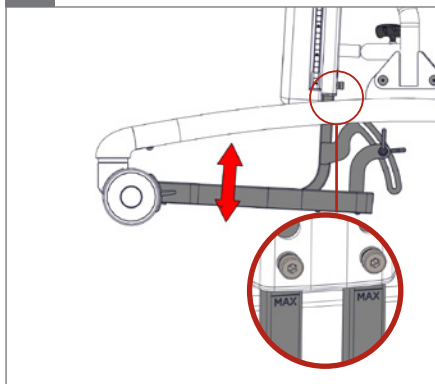○ 5 mm

6.

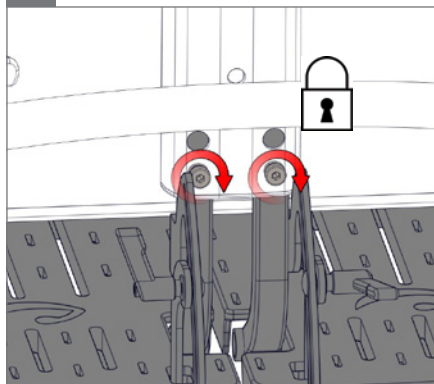○ 5 mm



1.

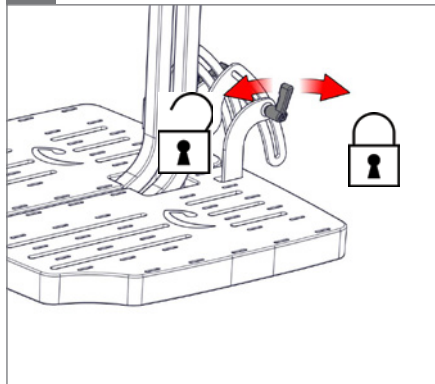2.

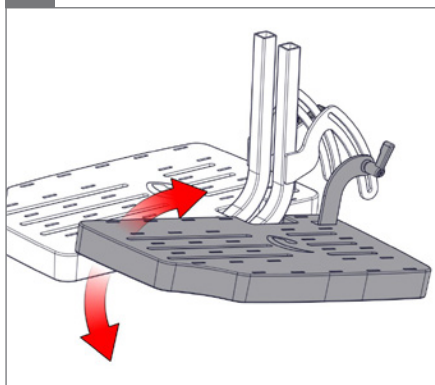
3.


4.

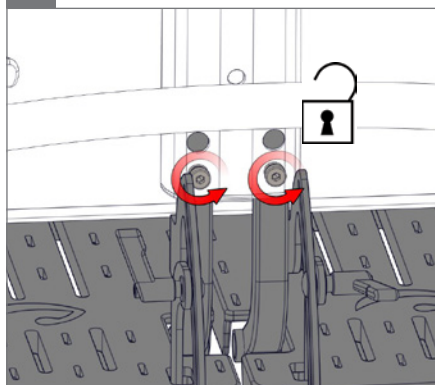○ 5 mm


5.

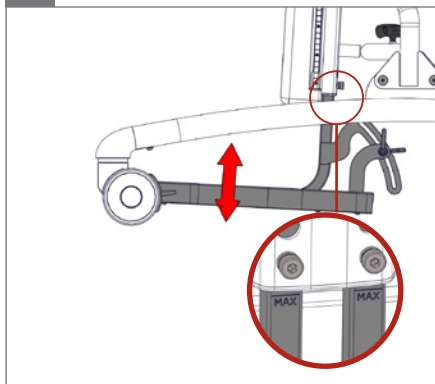○ 5 mm

6.

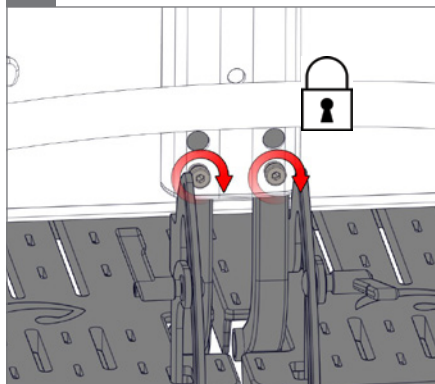○ 5 mm


1.

2.

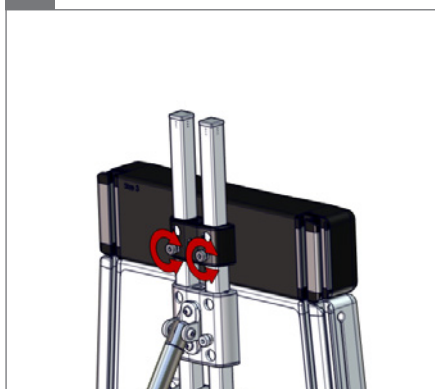○ 5 mm


3.

○ 5 mm


4.

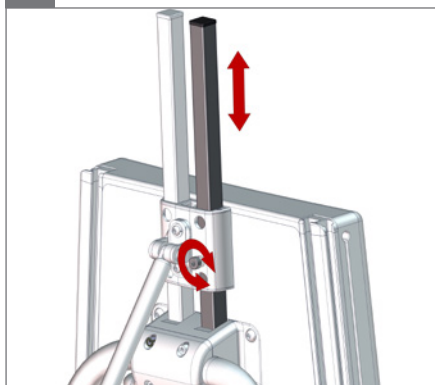○ 5 mm

5.

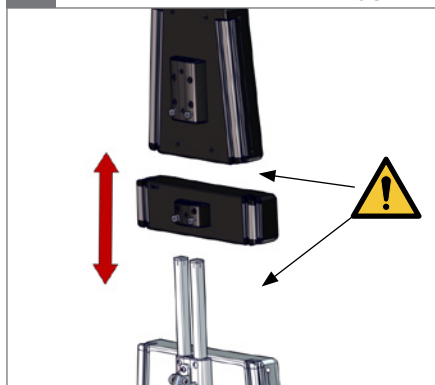05 ⚠


1.

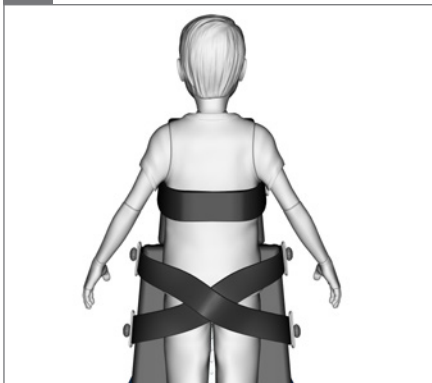
2.

3.

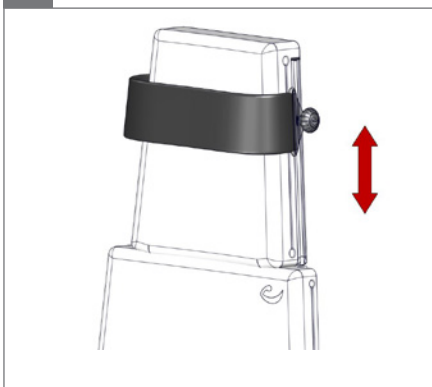
4.

5.

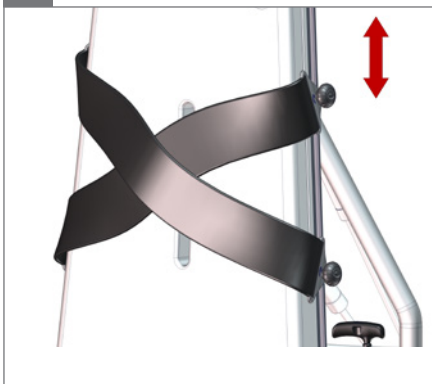
6.


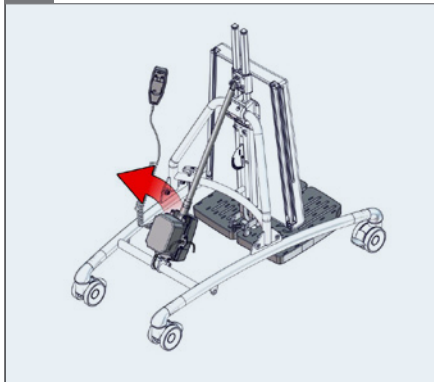
1.

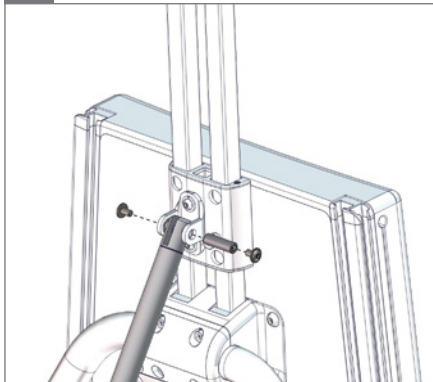2.

2 x  $\odot$  4 mm


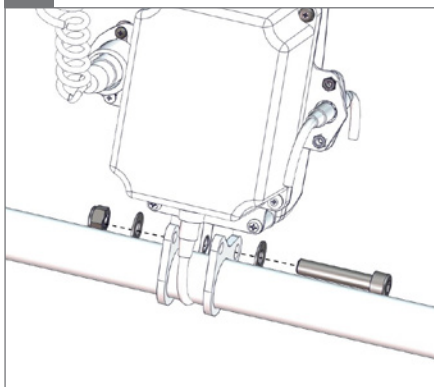
3.

 $\odot$  6 mm




4.

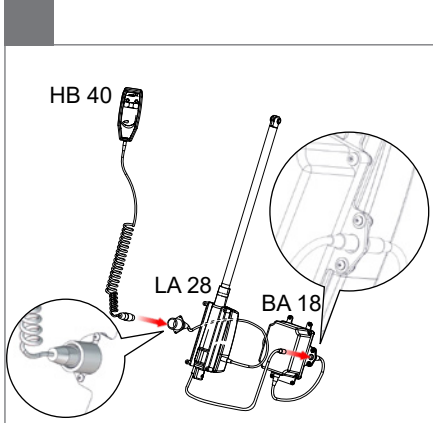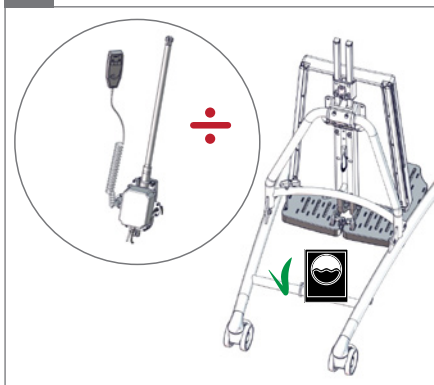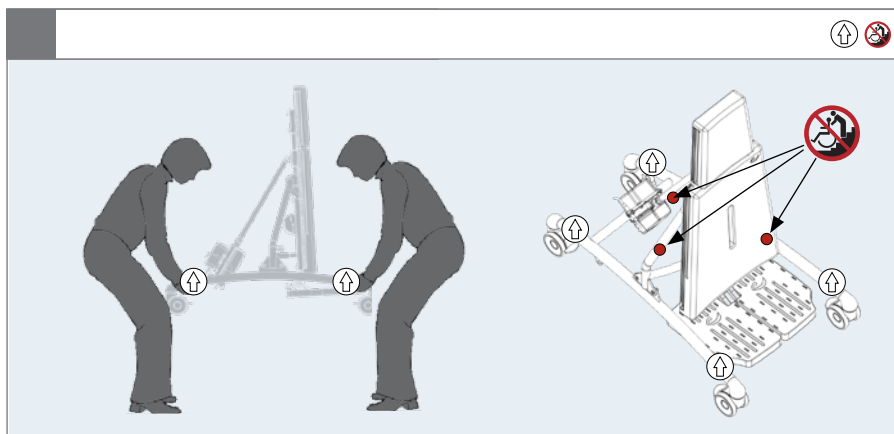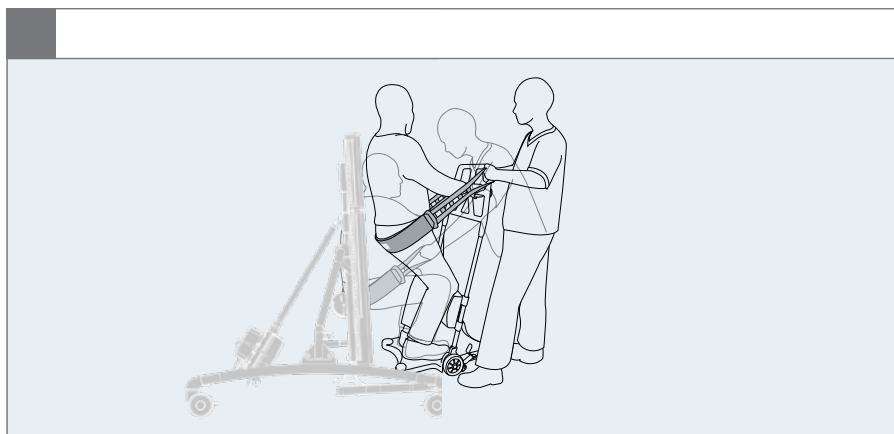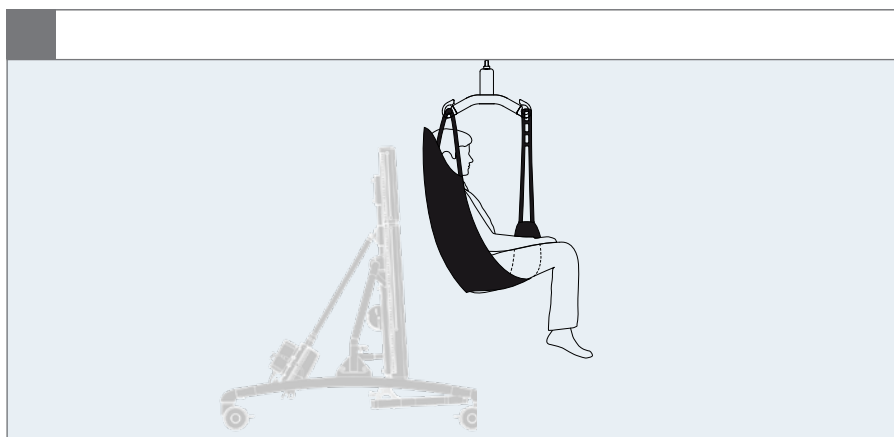5.

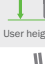
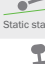
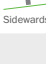
## mm (inch)

	 1	 2	 3	 4
 Overall width	510 (20")	570 (22½")	650 (25½")	750 (29½")
 Length	750 (29½")	920 (36¼")	1110 (43¾")	1110 (43¾")
 Overall height, vertical	460-570 (18-22½")	530-640 (20¾-25¼")	680-790 (26¾-31")	880-990 (34¾-39")
 Overall height, horizontal	700 (27½")	700 (27½")	800 (31½")	800 (31½")
 Angling	0-87°	0-87°	0-87°	0-87°
 Total mass	20,5 kg (45.1 lb)	24 kg (52.9 lb)	30,5 kg (67.2 lb)	35 kg (77.1 lb)
 Total mass, with power	22,5 kg (49.6 lb)	25,5 kg (56.2 lb)	32,5 kg (71.7 lb)	37 kg (81.6 lb)
 Max load	40 kg (88.1 lb)	60 kg (132.2 lb)	80 kg (176.3 lb)	100 kg (220.4 lb)
 User height	850-1120 (33½-44")	980-1290 (38½-50¾")	1250-1560 (49¼-61½")	1530-1840 (60¼-72½")
 Static stability upwards	max. 5°	max. 5°	max. 5°	max. 5°
 Static stability downwards	max. 5°	max. 5°	max. 5°	max. 5°
 Sideways stability sideways	max. 5°	max. 5°	max. 5°	max. 5°

**EN:** For indoor use

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**IT:** Per uso interno

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**BR:** Para uso interno

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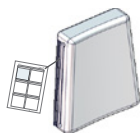
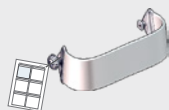
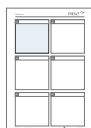
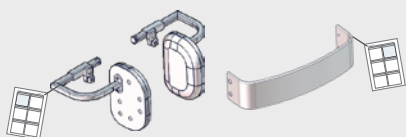
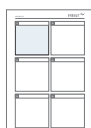
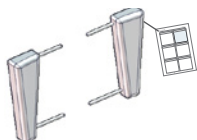
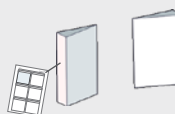
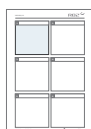
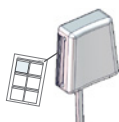
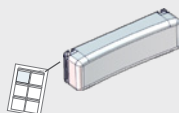
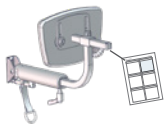
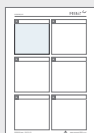
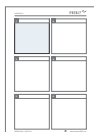
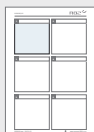
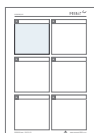
9996097243

## ENGLISH

We appreciate your choice of a new product from R82 - a worldwide supplier of technical aids and appliances for children and teenagers with special needs. To fully benefit from the options offered by this product please read this User Guide before use and save it for future reference.

### Intended purpose

This product is a prone and supine stander providing the user very important physical and psychological benefits. It is developed for users suffering from muscular atrophy e.g. children with CP, GMFCS level 3-5. It is built with an optimum of security in mind and is only to be used on a level, non tilting surface. It comes in 4 sizes, with the possibility for either manual or power operated tilt. This product is only for indoor use but can in an emergency be taken outside.

### Enclosed tools:

- 5 mm Allen key

### Declaration of Conformity

This product conforms to the requirements of the Medical Devices Directive (93/42/EEC). The CE mark must be removed when rebuilding the product, when used in combination with a product from another manufacturer or when using other than original R82 spare parts and fittings. Further, this product conforms to the requirements according to:

- EN 12182

### Disposal

When a product reaches the end of its lifetime, it should be separated by material type so that the parts can be recycled or disposed of properly. If required, please contact your local dealer for a precise description of each material. Please contact your local authorities to be informed of the options available for recycling in an environmentally-friendly way.

### Accessories and spare parts

The products from R82 can be supplied with a variety of accessories which comply with the needs of the individual user. Spare parts can be ordered on request. Find the specific accessories on our website or contact your local dealer for further information.

### R82 Warranty

R82 offers a 2-year warranty against defects in workmanship and materials and a 5-year warranty on breakage of the metal frame caused by defects in welds. The warranty would be adversely affected if the customer's responsibility of servicing and/or maintenance is not carried out according to the guidelines and intervals prescribed by the supplier and/or stated in the User Guide. For further information, we refer to the R82 homepage/download.

The warranty is only granted in the country where the product was purchased and if the product can be identified by the serial number. The warranty does not cover accidental damage, including damage caused by misuse or neglect. The warranty does not extend to consumable items e.g. tires or upholstery, which are subject to normal wear and tear and need periodic replacement.

The warranty is null and void if other than original parts/accessories are used, or if the product has not been maintained, repaired or altered according to the guidelines and intervals prescribed by the supplier and/or stated in the User Guide. R82 reserves the right to inspect the product being claimed for and the relevant documentation before agreeing to the warranty claim, and to decide upon whether to replace or repair the defective product. It is the customer's responsibility to return the item being claimed for under warranty to the address of purchase.

### Presales information

- [support.r82.org](http://support.r82.org)

## Safety

Signs, symbols and instructions placed on this product must never be covered or removed and must remain present and clearly legible throughout the entire lifetime of the product. Immediately replace or repair illegible or damaged signs, symbols and instructions. Contact your local dealer for instructions.

### Caregiver

- Read the "User Guide" carefully before use and keep it for future reference. Incorrect use of this product may cause serious injury to the user
- Always use the correct lifting techniques and aids for this purpose
- Never leave the user unattended in this product. Ensure permanent supervision by an adult
- Repair/replacements must only be made using new original R82 spare parts and fittings and carried out in accordance with the guidelines and service intervals prescribed by the supplier
- If there is any doubt as to the continued safe use of your R82 product or if any parts should fail, stop using the product immediately and contact your local dealer as soon as possible
- Transportation of the product whilst the user is in it is not allowed. It is allowed to use the product with the user safely strapped in it for transfers over short distances. Do not cross any obstacles during transfer.

### Environment

- Observe the surface temperature on the product, before placing an user in it. This goes especially for users with insensitive skin as they cannot feel the heat. If the surface temperature is above 41 °C then allow the product to cool before use
- Use this product on a level surface without any obstacles. Be aware of reduced maneuverability on slippery surfaces
- When using on inclines, please refer to measurements

### User

- If the user is close to the max. load limit and/or has a lot of involuntary movements, e.g. rocking, a larger size with a higher max. load or a different R82 product should be considered

### Product

- Carry out all positional adjustments on the product and accessories and ensure that all knobs, screws and buckles are securely fastened before use. Keep all tools out of the reach of children
- Activate the brakes before placing the user into the product
- Ensure the stability of the product is maintained before placing the user in it
- Ensure the castors are fully-functional and secure before use
- Inspect the product and all of its accessories and replace any worn out parts before use
- Do not expose the gas spring to pressure or high temperatures. DO NOT puncture
- Do not expose the piston rod to scratches etc. It is important to use a cloth or the like, if tools are used to adjust the gas spring
- The footplate is for support only! The foot support does not have a weight bearing function
- This product is tested for resistance to ignition at level non-ignition in accordance with EN 1021-1 and EN 1021-2
- The product might lose its flame resistant characteristics if using other than original R82 cushioning
- This product is not to be occupied in a motor vehicle. The user must be positioned in a vehicle seat and the product must be stored in a luggage compartment
- The user must not occupy the product while it is being transported into or out of a vehicle by a lift

## Service information

Maintenance of a medical device is wholly the responsibility of the owner of that device. Failure to maintain a device in accordance with the instructions may invalidate the device's warranty. Furthermore, failure to maintain a device may compromise the clinical condition or safety of users and/or their caregivers.

### 1st Day

- Read the User Guide carefully before use and store it in a safe place for future reference
- Store the enclosed tools on/at the product

### Daily

- Use a dry cloth to clean the product
- Visual inspection. Repair or replace any damaged or worn-out parts
- Ensure that all fasteners are in place and fitted correctly
- Check all fittings and buckles for signs of worn out parts
- Be aware of any MAX mark indications before adjustment
- Check that all cables connecting the power system are in good shape

### Weekly

- Check that all castors are moving freely and that all wheel locks/brakes are easily activated
- Wipe off any residue and dirt from the product using a cloth with warm water and a mild detergent/soap and dry before use
- Disinfect the product. please check the disinfection section in this User Guide for more information

### Monthly

- All nuts and bolts on this product must be checked and tightened to avoid unnecessary failures
- Oil the swing-away parts. We recommend you use a professional lubricant system

### Annually

- Inspect the frame for cracks or signs for worn-out parts and carry out annual service. Never use a product which appears to be substandard or faulty

### Warewashing machine

This product can be washed at:

- 60 °C in 10 minutes
- 85 °C in 3 minutes

...using a mild detergent in a washing machine designed for medical devices. Use the machine's drying function to dry the product. Clean the accessories separately.

Before washing

Remove the following items and clean them separately:

- Accessories
- Power unit including: Actuator, battery and handset

After washing:

- Replace the power column going backwards through the procedure give on page 14
- Check that the handset functionalities works correct

### Disinfection

The product can be disinfected with a 70% disinfectant IPA solution. It is recommended to wipe off any residue and dirt from the product, using a cloth with warm water and a mild detergent/soap without chlorine and let it dry before disinfection.

### Service interval

A detailed inspection of the product must be made every 12 months (every 6 months for heavy users) and every time the product is reissued for use. The inspection should be carried out by a technically competent person who has been trained in the use of the product.

The lifetime of this product in normal use is 5 years if all maintenance and servicing is carried out in accordance with the manufacturer's instructions and demonstrably recorded.

### Power unit

For an optimum lifetime the power unit must be connected to the mains voltages as often as possible (e.g. every night). Charge the batteries 12 hours before use.

The batteries must be charged at least every 3rd month - otherwise there might be damages due to self-discharge. It is recommended to test the battery function at least once every year.

### Recharging the battery:

- Plug the charging lead into the handset
- Plug the charger into the electricity supply and switch ON. A yellow light on the charger will light up
- Charge the batteries until the lamp on the charger turns green

### Caregiver

- Do not leave children and youngsters alone with the handset
- Do not let children and youngsters play with the handset
- Do not leave the handset within reach of children and youngsters.

### Product

- The power unit does not have a safety stop
- The system must not be sprayed directly with a highpressure cleaner
- Cleaning with a steam cleaner is not permitted
- Do only use the power unit within specified working limits
- It is recommended to check the handset and cable for damage and holes caused by violent handling at least once a year
- Do not submerge the handset in water
- Do not attempt to repair any part of the power unit.

### Environmental conditions

- Operating temperature: +5 °C to +40 °C
- Storage temperature: -10 °C to +50 °C
- Relative humidity: 20% to 90%
- Atmospheric pressure: 800 to 1060 hPa

### Technical Data

#### Battery - BA18

- Capacity 1.3 Ah - 24V DC
- Charging current is max. 500 mA
- IPX6
- Duty cycle: Max. 10% or 2 min. use then 18 min. without use

The batteries have a maximum life of 4 years but this can vary depending upon the conditions of use and charging practices

#### Battery charger - CH01

- Mains: 100 - 240VAC/50-60 Hz switch mode
- Charging voltage: 27.6 VDC
- Charging current: Max. 500 mA

#### Actuator - LA28

- Input voltage: 24V
- Noise level: ≤ 45 dB(A)
- IPX1

#### Handset - HB 40

- IPX1

#### Detachable parts

- Handset - HB 40
- Battery - BA 18

#### Applied part

- Handset - HB 40

#### IP-Rating table

IPX0	No protection
IPX1	Protection against vertically falling drops of water e.g. condensation
IPX2	Protection against direct sprays of water up to 15° from the vertical
IPX3	Protected against direct sprays of water up to 60° from the vertical
IPX4	Protection against water sprayed from all directions - limited ingress permitted
IPX5	Protected against low pressure jets of water from all directions - limited ingress
IPX6	Protected against temporary flooding of water, e.g. for use on ship decks - limited ingress permitted
IPX7	Protected against the effect of immersion between 15 cm and 1 m
IPX8	Protects against long periods of immersion under pressure

### Electromagnetic compatibility (EMC) environment

This equipment is for use in home healthcare facility environment.

This equipment is not likely to cause any harmful interference in nearby equipment; however, it can be impaired by electromagnetic (EM) interference emitted by other equipment, which can lead to malfunctions. Do not use this equipment where there is a risk that other electrical and/or electronic equipment can interfere with this equipment.

### This equipment complies with the following standards

IEC 60601-1-2:2014 Medical electrical equipment - Part 1-2: General requirements for basic safety and essential performance - Collateral Standard: Electromagnetic disturbances - Requirements and tests (EN 60601-1-2:2015)

EN 12182:2012, Clause 7, Assistive products for persons with disability - General requirements and test methods

### EMC - WARNINGS

Use of this equipment adjacent to or stacked with other equipment should be avoided because it could result in improper operation. If such use is necessary, this equipment and the other equipment should be observed to verify that they are operating normally.

Use of accessories, transducers and cables other than those specified or provided by R82 could result in increased EM emission or decreased EM immunity of this equipment and result in improper operation. Several power plugs are supplied with this equipment. Please use the correct power plug suitable for your country. Please choose among the supplied power plug adaptor units: English, Australian, EU, US/ Japanese plugs.

Portable radio frequency (RF) communication equipment (including peripherals such as antenna cables and external antennas) should be used no closer than 30 cm (12 inches) to any part of the Caribou, including cables specified by R82. Otherwise, degradation of the performance of this equipment could result.

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Test modes during emission tests:

1. Movement mode (battery). Actuator running in/out
2. Charging mode, actuator not running.

EM emission standards	Classification/levels
Conducted and radiated RF emission CISPR 11	Group 1 (the product uses RF energy only for its internal function) Class B (the product is suitable for use in all establishments, including domestic establishments and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes)
Harmonic distortion IEC 61000-3-2	Class A. No testing performed, as the rated power for this equipment is less than 75 W. No limits are specified for equipment with a rated power of 75 W or less.
Voltage fluctuations and flicker IEC 61000-3-3	$d_{\max} \leq 4\%$ (without additional conditions)

**Test modes during immunity tests:**

- Battery mode, standby armed/ready.
- Charging mode, actuator not running.

EM immunity standards	IEC 60601-1-2:2014 test levels	Additional test levels
Electrostatic discharge IEC61000-4-2	$\pm 8$ kV contact $\pm 2$ kV, $\pm 4$ kV, $\pm 8$ kV, $\pm 15$ kV air	$\pm 6$ kV contact
Radiated RF EM fields IEC 61000-4-3	10 V/m 80 MHz – 2.7 GHz 80 % AM at 1 kHz	20 V/m 800 MHz – 2.5 GHz 80 % AM at 1 kHz
Proximity fields from RF wireless communication equipment IEC 61000-4-3	IEC 60601-1-2:2014 Table 9	N/A
Electrical fast transients / bursts IEC 61000-4-4	$\pm 1$ kV, $\pm 2$ kV 100 kHz repetition frequency	$\pm 1$ kV, $\pm 2$ kV 5 kHz repetition frequency
Surges, Line-to-line IEC 61000-4-5	$\pm 0,5$ kV, $\pm 1$ kV 230 VAC	$\pm 0,5$ kV, $\pm 1$ kV 100 VAC and 240 VAC
Conducted disturbance induced by RF fields IEC 61000-4-6	6 Vrms 0.15 MHz – 80 MHz 80% AM at 1 kHz	N/A
Rated power frequency magnetic fields IEC 61000-4-8	30 A/m 50 Hz	30 A/m 50 Hz and 60 Hz
Voltage dips, short interruptions and voltage variations IEC 61000-4-11	0 % $U_T$ for 0,5 cycle at 0°, 45°, 90°, 135°, 180°, 225°, 270° and 315° phase angles 0 % $U_T$ for 1 cycle 70 % $U_T$ for 25 cycles 0 % $U_T$ for 250 cycles 230 VAC	0 % $U_T$ for 1 cycle 40 % $U_T$ for 10 cycles 70 % $U_T$ for 25 cycles 100 VAC and 240 VAC
Variation of power frequency IEC 61000-4-28	N/A	49 Hz and 51 Hz 100 VAC and 240 VAC, 50 Hz 59 Hz and 61 Hz 100 VAC and 240 VAC, 60 Hz

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## R82 Nederland B.V.
## R82 Inc.
## R82 au Pty Limited