<180 cm (39 1/4")

<80 kg (176 1/4 lb)

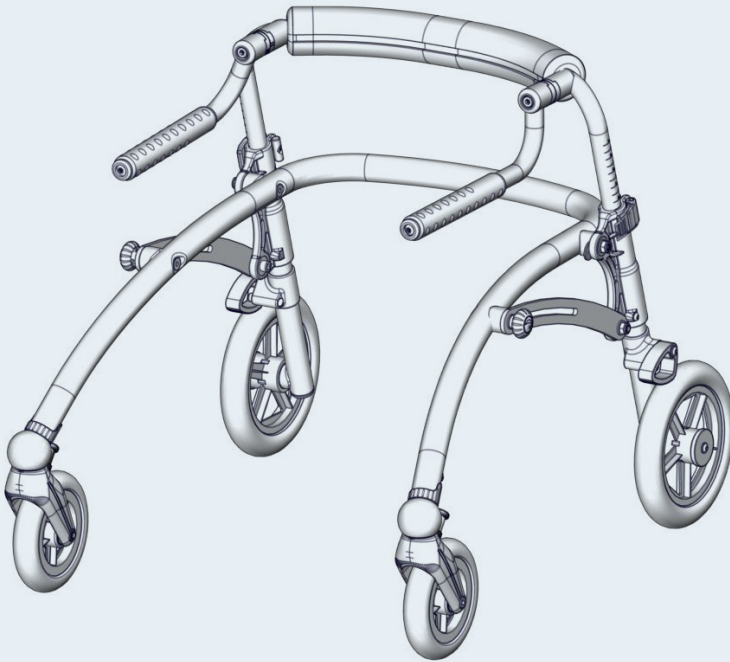
Crocodile User Guide

EN 12182


2018-10-rev.06

CE
support.R82.org


	CROCODILE 1 & 2	CROCODILE 3		
	3	13	ENGLISH GERMAN	29 - 31 32 - 34
	4 - 5	14 - 15	NEDERLANDS DANSK	35 - 37 38 - 40
	6	16	NORSK SVENSKA	41 - 43 44 - 46
	7	17	SUOMI ESPAÑOL	47 - 49 50 - 52
	8	18	FRANÇAIS ITALIANO	53 - 55 56 - 58
	9	19	PORTUGUÊS PORTUGUÊS DO BRASIL	59 - 61 62 - 64
	10	20	РОССИЯ POLSKI	65 - 67 68 - 70
	11	21	ČESKY 中國	71 - 73 74 - 76
	12	22	日本 ЕЛΛΗΝΙΚΗ	77 - 79 80 - 82
cm (inch)	23	23		
	24 - 25	24 - 25		
	26 - 28	26 - 28		

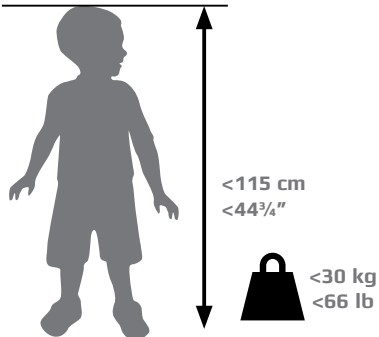CROCODILE 1

CROCODILE 2

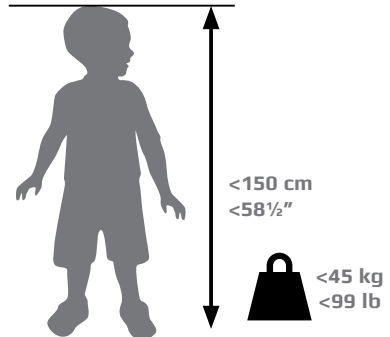
1.

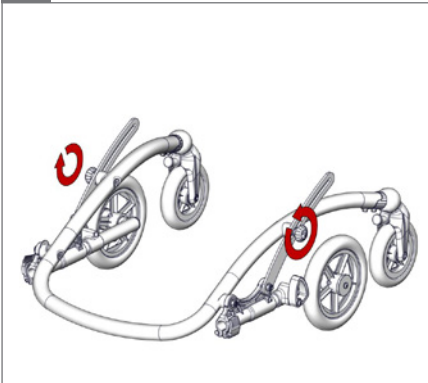
2.


3.

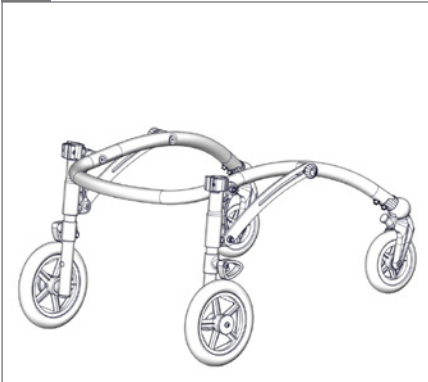4.

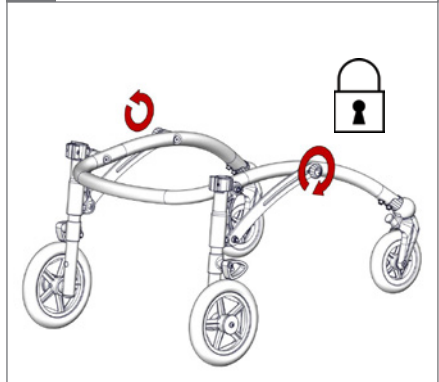
5.


6.

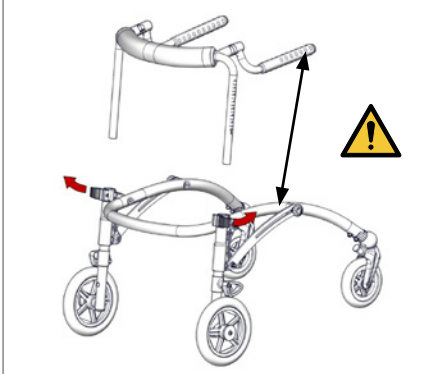
 05

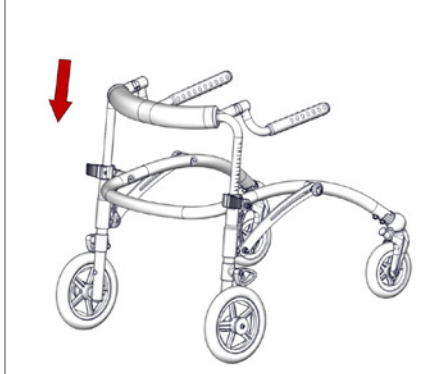7.

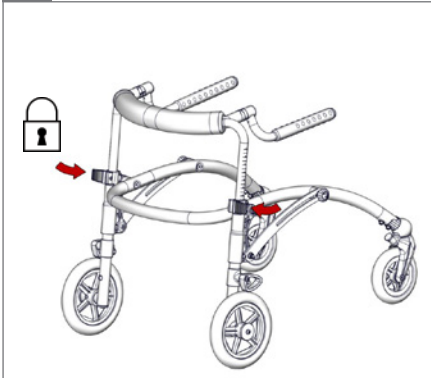8.

9.


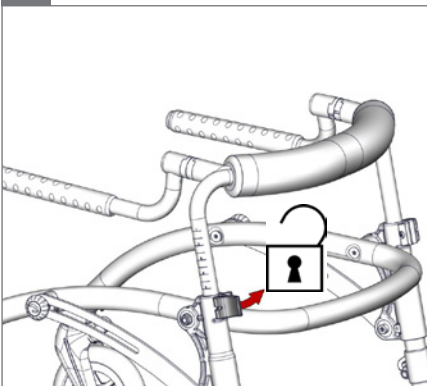
1.


2.

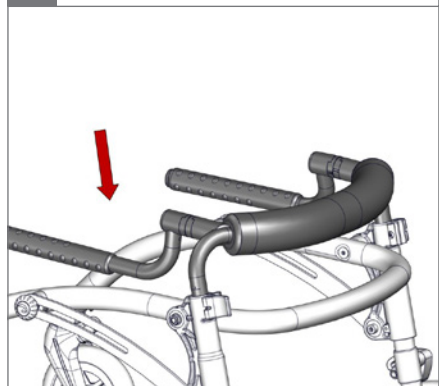
3.

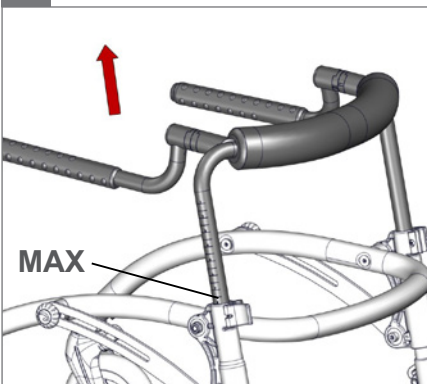4.

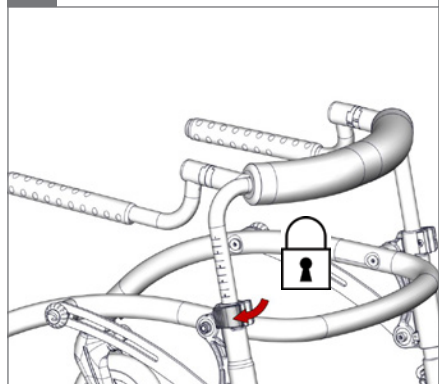5.

1.


2.

6 mm


3.

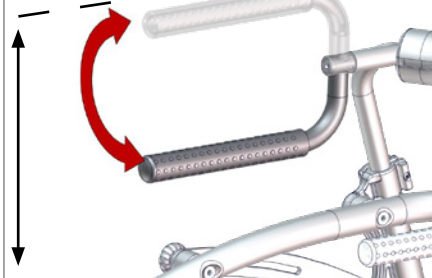Size 1: MAX 560 mm
Size 2: MAX 690 mm


4.

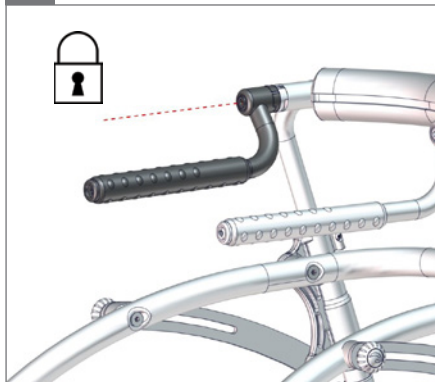6 mm - 2,5 Nm

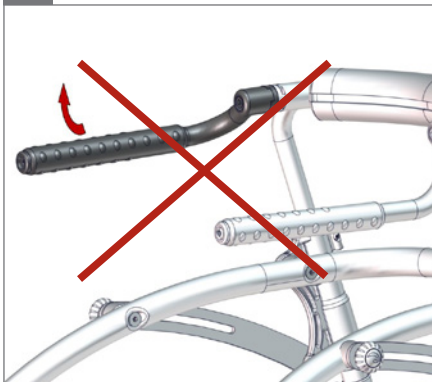
5.

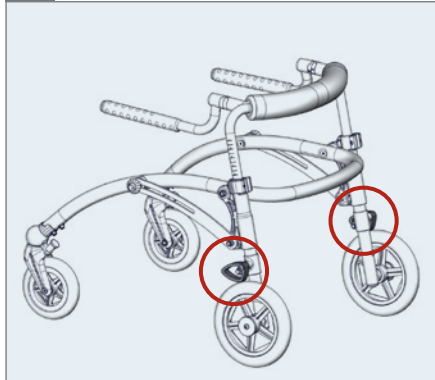
1.

2.

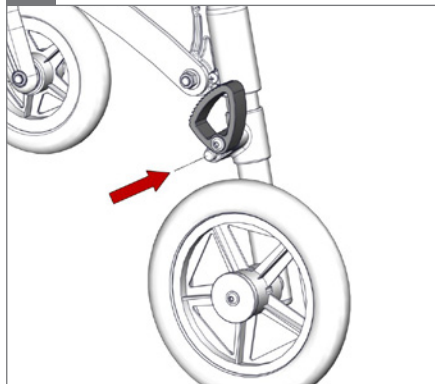
3.


4.

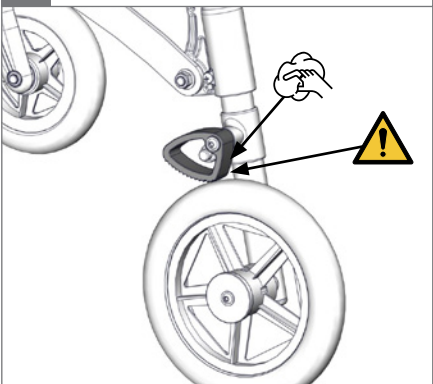 05


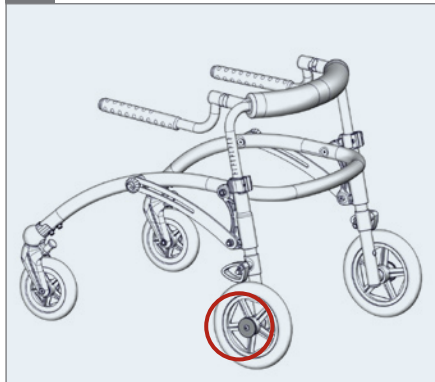
1.

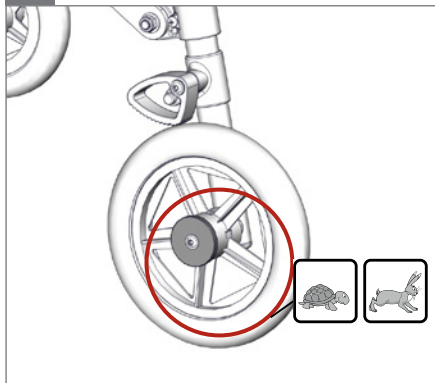
2.

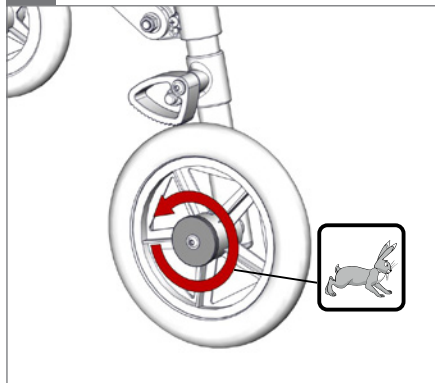3.

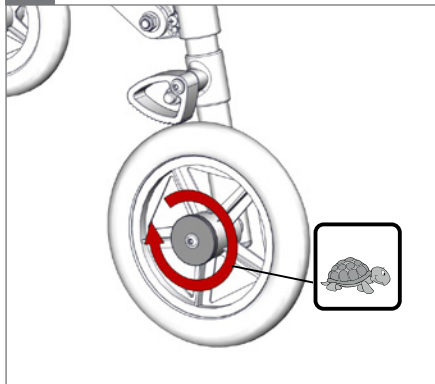4.

5.

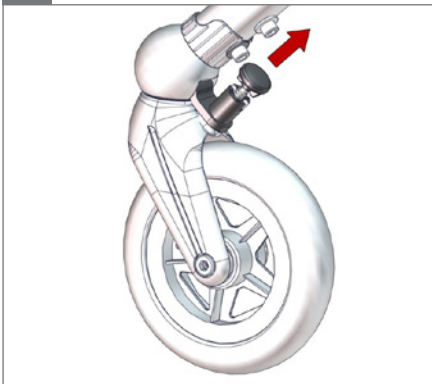
1.

2.


3.

4.

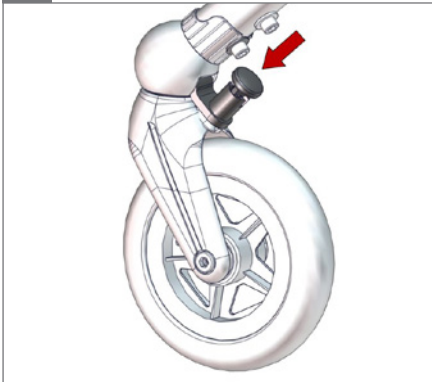5.


6.

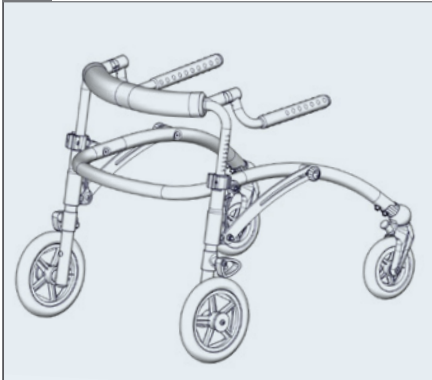
1.

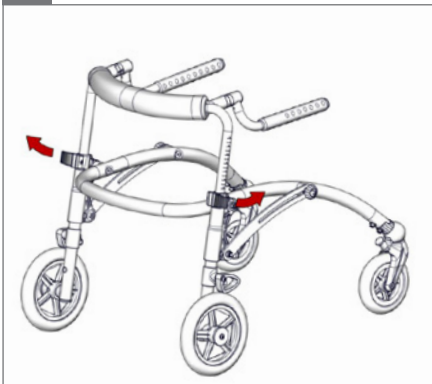
2.


3.

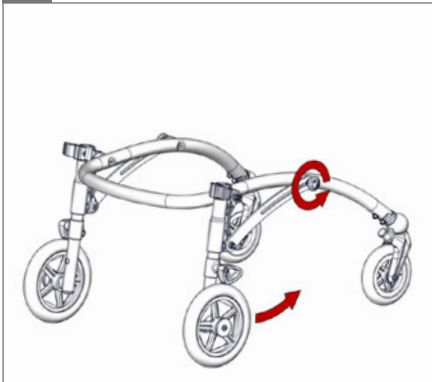
4.

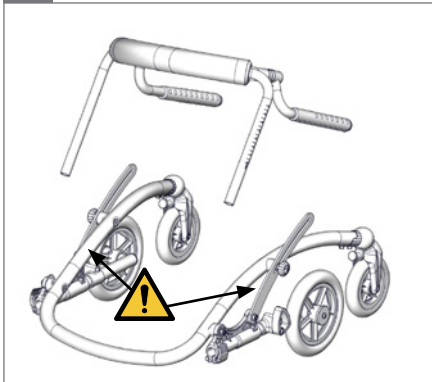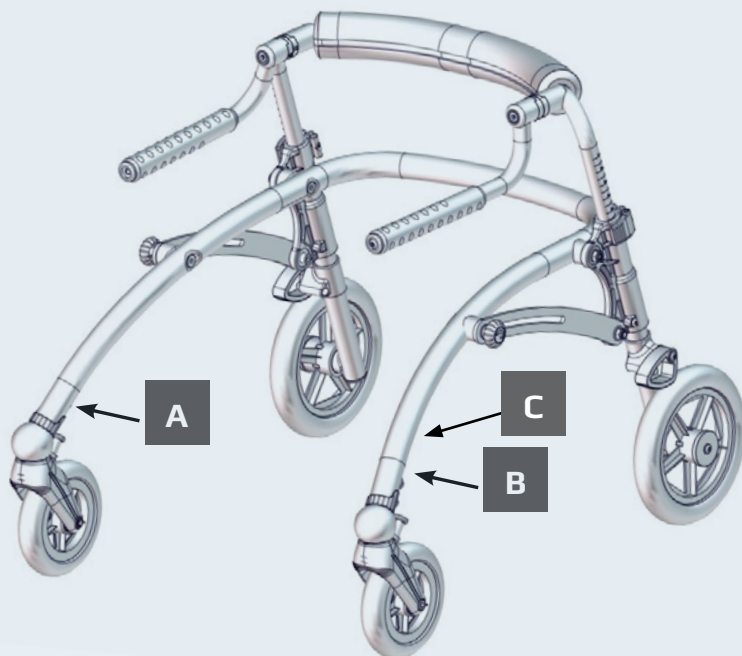5.

A

B

C

R82 

Parallelvej 3
DK-8751 Gedved
R82.org
Made in XXXXXXXX YYYY-MM-DD

SN XXXXXXXX
DMR ver.:XXXX
Art. no.:XXXXXXXX
Product:XXXX - Size X

Max load: xxx kg

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ENGLISH

We appreciate your choice of a new product from R82 - a worldwide supplier of technical aids and appliances for children and teenagers with special needs. To fully benefit from the options offered by this product please read this User Guide before use and save it for future reference.

Intended purpose

The Crocodile is designed to provide users the ability to be mobile. It will grow along with the user and provide many hours of freedom of movement. The Crocodile is light, easy to dismantle and safe and easy to use when the manual instructions are followed. A wide group of children and teenagers with special needs can use the Crocodile as a posterior gait trainer, for example children with CP, GMFCS level 2-4. The Crocodile is constructed for one user only. It is available in three sizes and is suitable for both indoor and outdoor use.

Enclosed tools:

- 2½+3+4+5+6 mm Allen key

Declaration of Conformity

This product conforms to the requirements of the Medical Devices Directive (93/42/EEC). The CE mark must be removed when rebuilding the product, when used in combination with a product from another manufacturer or when using other than original R82 spare parts and fittings. Further, this product conforms to the requirements according to:

- EN 12182

Disposal

When a product reaches the end of its lifetime, it should be separated by material type so that the parts can be recycled or disposed of properly. If required, please contact your local dealer for a precise description of each material. Please contact your local authorities to be informed of the options available for recycling in an environmentally-friendly way.

Accessories and spare parts

The products from R82 can be supplied with a variety of accessories which comply with the needs of the individual user. Spare parts can be ordered on request. Find the specific accessories on our website or contact your local dealer for further information.

R82 Warranty

R82 offers a 2-year warranty against defects in workmanship and materials and a 5-year warranty on breakage of the metal frame caused by defects in welds. The warranty would be adversely affected if the customer's responsibility of servicing and/or maintenance is not carried out according to the guidelines and intervals prescribed by the supplier and/or stated in the User Guide. For further information, we refer to the R82 homepage/download.

The warranty is only granted in the country where the product was purchased and if the product can be identified by the serial number. The warranty does not cover accidental damage, including damage caused by misuse or neglect. The warranty does not extend to consumable items e.g. tires or upholstery, which are subject to normal wear and tear and need periodic replacement.

The warranty is null and void if other than original parts/accessories are used, or if the product has not been maintained, repaired or altered according to the guidelines and intervals prescribed by the supplier and/or stated in the User Guide. R82 reserves the right to inspect the product being claimed for and the relevant documentation before agreeing to the warranty claim, and to decide upon whether to replace or repair the defective product. It is the customer's responsibility to return the item being claimed for under warranty to the address of purchase.

Presales information

- support.r82.org

Safety

Signs, symbols and instructions placed on this product must never be covered or removed and must remain present and clearly legible throughout the entire lifetime of the product. Immediately replace or repair illegible or damaged signs, symbols and instructions. Contact your local dealer for instructions.

Caregiver

- Read the "User Guide" carefully before use and keep it for future reference. Incorrect use of this product may cause serious injury to the user
- Always use the correct lifting techniques and aids for this purpose
- Never leave the user unattended in this product. Ensure permanent supervision by an adult
- Repair/replacements must only be made using new original R82 spare parts and fittings and carried out in accordance with the guidelines and service intervals prescribed by the supplier
- If there is any doubt as to the continued safe use of your R82 product or if any parts should fail, stop using the product immediately and contact your local dealer as soon as possible
- Ensure that moving parts does not squeeze or trap body parts when adjusting them

Environment

- Observe the surface temperature on the product, before placing an user in it. This goes especially for users with insensitive skin as they cannot feel the heat. If the surface temperature is above 41 °C then allow the product to cool before use
- Use this product on a level surface without any obstacles. Be aware of reduced maneuverability on slippery surfaces
- When using on inclines, please refer to measurements

User

- If the user is close to the max. load limit and/or has a lot of involuntary movements, e.g. rocking, a larger size with a higher max. load or a different R82 product should be considered
- Be aware of the child's capacity to exceed walking speed or ability to complete sharp turns, which may cause the product to become unstable

Product

- Carry out all positional adjustments on the product and accessories and ensure that all knobs, screws and buckles are securely fastened before use. Keep all tools out of the reach of children
- Activate the brakes before placing the user into the product
- Activate the anti-tips when the product is being used on inclines
- Ensure the stability of the product is maintained before placing the user in it
- We recommend you notify the user before angling the seat and back
- Ensure the castors are fully-functional and secure before use
- Inspect the product and all of its accessories and replace any worn out parts before use
- This product is not to be occupied in a motor vehicle. The user must be positioned in a vehicle seat and the product must be stored in a luggage compartment
- The user must not occupy the product while it is being transported into or out of a vehicle by a lift

Environmental conditions

- Operating temperature: 5 °C to +40 °C
- Relative humidity: 5% to 90%
- Atmospheric pressure: 800 to 1060 hPa

Service information

Maintenance of a medical device is wholly the responsibility of the owner of that device. Failure to maintain a device in accordance with the instructions may invalidate the device's warranty. Furthermore, failure to maintain a device may compromise the clinical condition or safety of users and/or their caregivers.

1st Day

- Read the User Guide carefully before use and store it in a safe place for future reference
- Store the enclosed tools on/at the product

Daily

- Use a dry cloth to clean the product
- Visual inspection. Repair or replace any damaged or worn-out parts
- Ensure that all fasteners are in place and fitted correctly
- Check all fittings and buckles for signs of worn out parts
- Be aware of any MAX mark indications before adjustment

Weekly

- Check that all castors are moving freely and that all wheel locks/brakes are easily activated
- Wipe off any residue and dirt from the product using a cloth with warm water and a mild detergent/soap and dry before use
- Check that the angle mechanism functions correctly and that the safety lock is working correctly
- Disinfect the product. please check the disinfection section in this User Guide for more information

Monthly

- All nuts and bolts on this product must be checked and tightened to avoid unnecessary failures
- Oil the swing-away parts. We recommend you use a professional lubricant system

Annually

- Inspect the frame for cracks or signs for worn-out parts and carry out annual service. Never use a product which appears to be substandard or faulty

Warewashing machine

This product can be washed at:

- 60 °C in 10 minutes

...using a mild detergent in a washing machine designed for medical devices. Use the machine's drying function to dry the product. Clean the accessories separately.

Before washing

Remove the following items and clean them separately:

- Accessories

...using a mild detergent in a washing machine designed for medical devices. Use the machine's drying function to dry the product. Clean the accessories separately.

Disinfection

The product without accessories can be disinfected with a 70% disinfectant IPA solution. It is recommended to wipe off any residue and dirt from the product, using a cloth with warm water and a mild detergent/soap without chlorine and let it dry before disinfection.

Service interval

A detailed inspection of the product must be made every 12 months (every 6 months for heavy users) and every time the product is reissued for use. The inspection should be carried out by a technically competent person who has been trained in the use of the product.

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© 2013 R82 A/S All rights reserved The R82 logo and the icons are registered trademarks of R82 A/S
R82 A/S