


meerkat

Exploded views / Part lists

Rev. date (07.2018)

On the following pages you will find the parts used to assemble this product. Please note that some spare parts cannot be purchased individually but are part of a complete kit. Please also note that for some spare parts extra delivery time may occur.

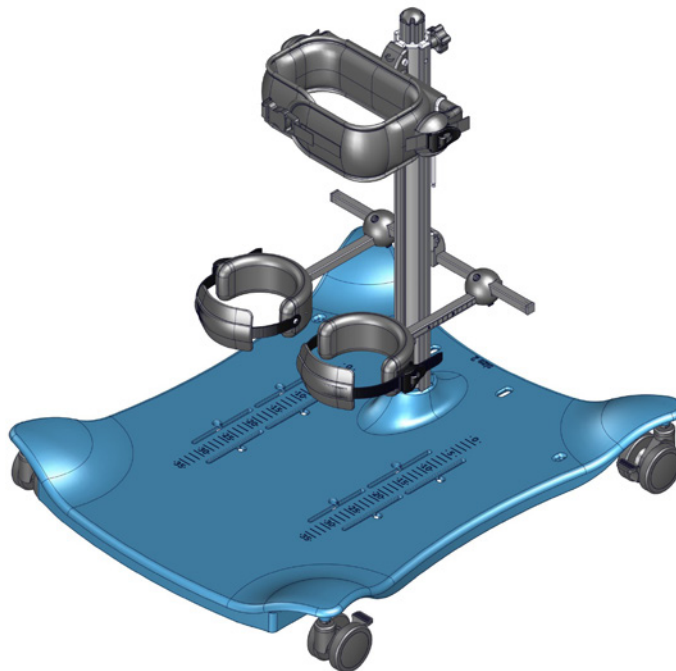

Contents

Meerkat

Base Assembly3

Column Assembly 4 - 5

Meerkat Accessories

Chest Support.....6

Hip Support.....7


Knee Supports size 18

Knee Supports size 2-39


Sandals 10

Rocker 11

Extension Kit..... 12

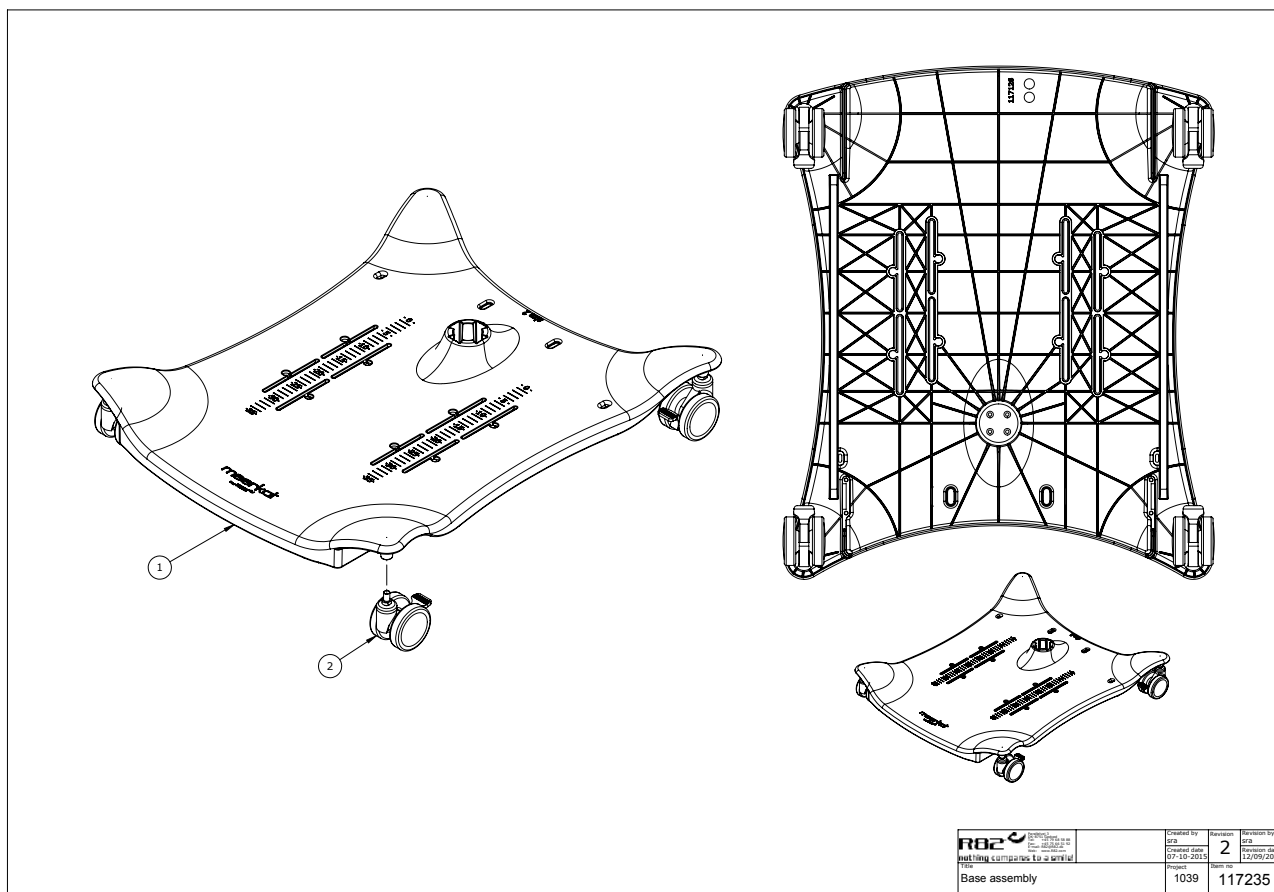


Position	No.	Description	Quantity	Unit of Measure	Ean code
2	3006825	75mm wheel w/brake, black	4	PCS	5707292371393
2	3035000	Wheel - plane - colour grey	4	PCS	5707292790019
2	3035010	Wheel Cow - white	4	PCS	5707292790026

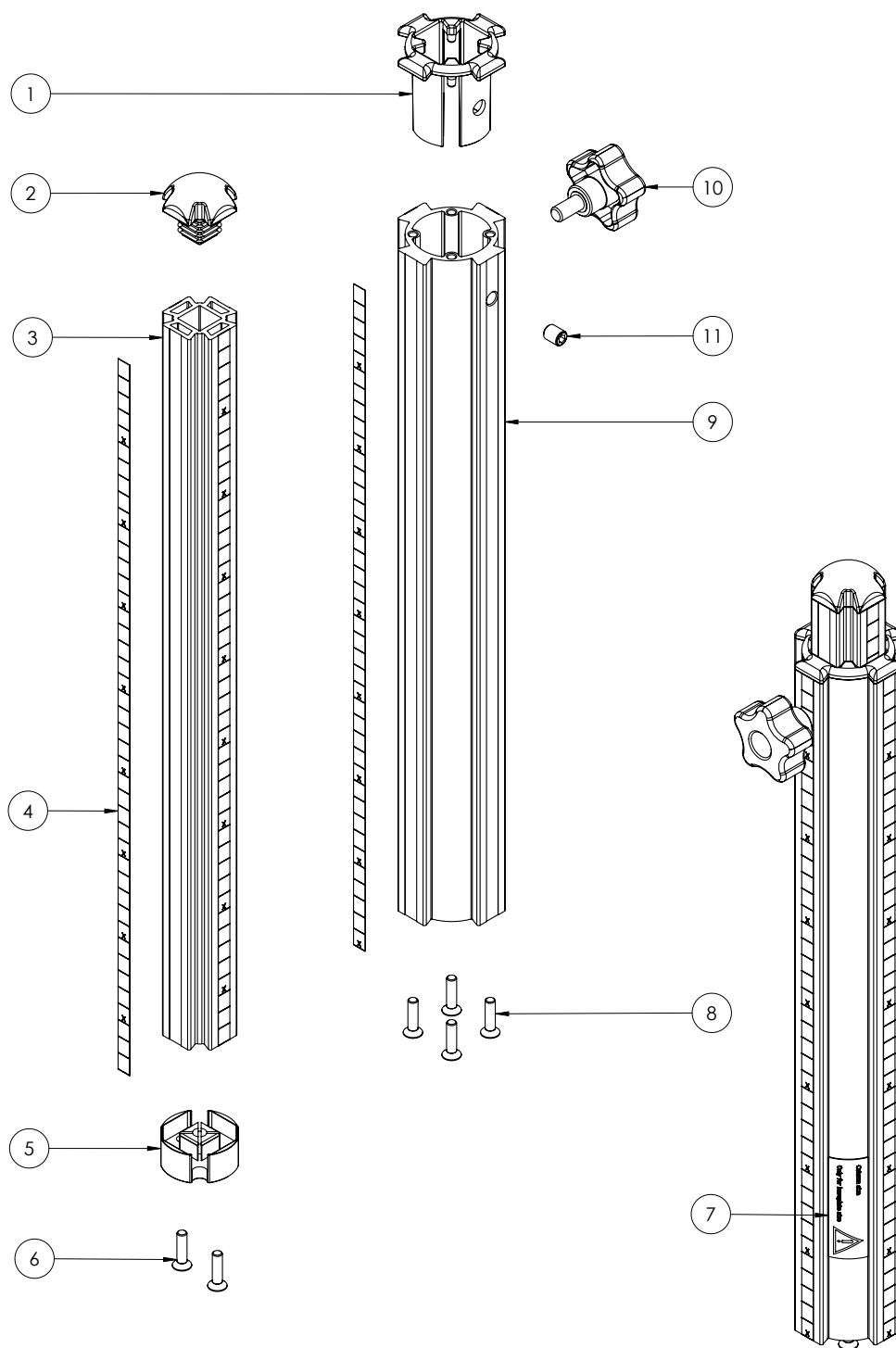


R82 <small>nothing compares to a smile</small> <small>nothing compares to a smile</small>	<small>Created by</small> Admin <small>Created date</small> 31-03-2011 <small>Project</small> 1039	<small>Revision</small> 3 <small>Revision by</small> sra <small>Revision date</small> 21/03/2018
		<small>Item no.</small> 117838


C:\PDM\Projects\1039\Columns\117838

Position	No.	Description	Quantity	Unit of Measure	Ean code
7	0494602	Etiket Column size 2 Meerkat	1	STK	5707292793706
7	0494603	Etiket Column size 3 Meerkat	1	STK	5707292793713
7	0494601	Etiket Column size 1 Meerkat	1	STK	5707292793690

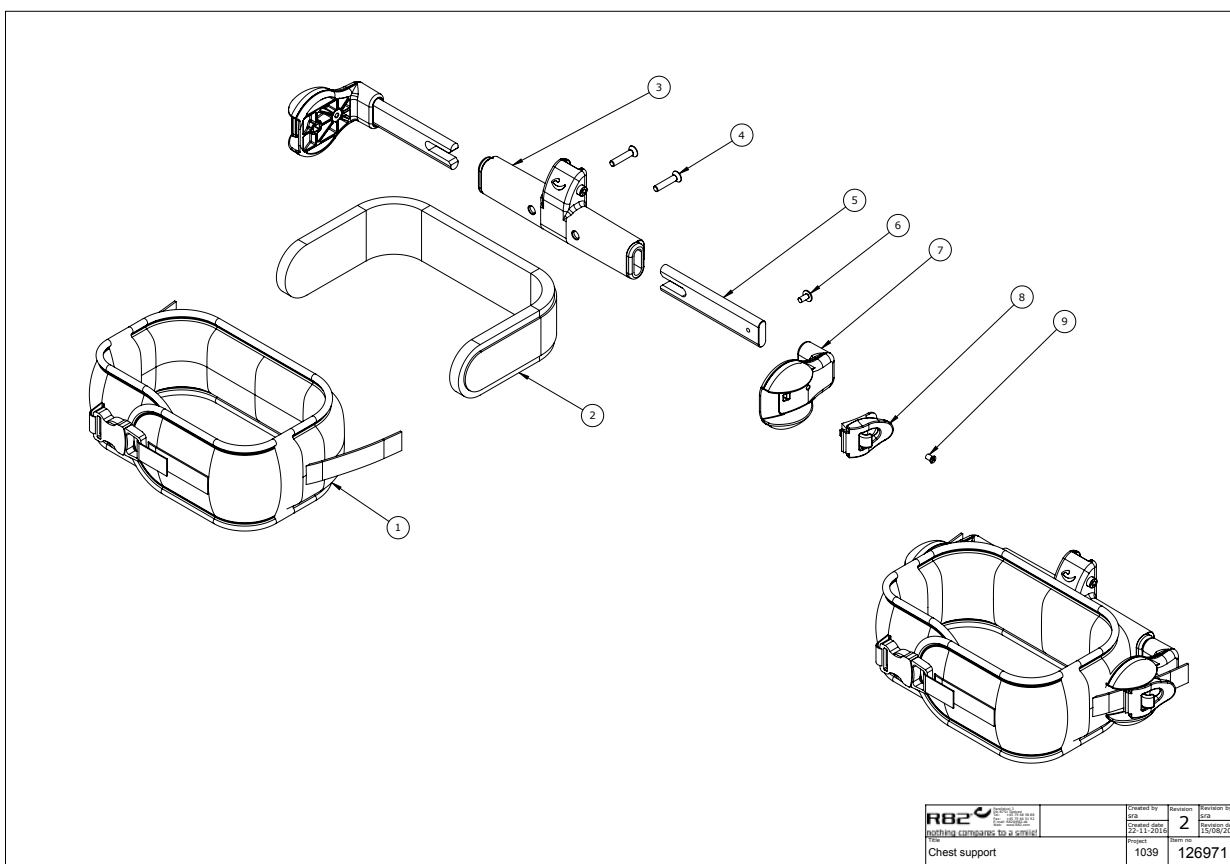


 nothing compares to a smile!	<small>Head Office 100, 101, 102, 103, 104, 105, 106, 107, 108, 109, 110, 111, 112, 113, 114, 115, 116, 117, 118, 119, 120, 121, 122, 123, 124, 125, 126, 127, 128, 129, 130, 131, 132, 133, 134, 135, 136, 137, 138, 139, 140, 141, 142, 143, 144, 145, 146, 147, 148, 149, 150, 151, 152, 153, 154, 155, 156, 157, 158, 159, 160, 161, 162, 163, 164, 165, 166, 167, 168, 169, 170, 171, 172, 173, 174, 175, 176, 177, 178, 179, 180, 181, 182, 183, 184, 185, 186, 187, 188, 189, 190, 191, 192, 193, 194, 195, 196, 197, 198, 199, 200, 201, 202, 203, 204, 205, 206, 207, 208, 209, 210, 211, 212, 213, 214, 215, 216, 217, 218, 219, 220, 221, 222, 223, 224, 225, 226, 227, 228, 229, 230, 231, 232, 233, 234, 235, 236, 237, 238, 239, 240, 241, 242, 243, 244, 245, 246, 247, 248, 249, 250, 251, 252, 253, 254, 255, 256, 257, 258, 259, 260, 261, 262, 263, 264, 265, 266, 267, 268, 269, 270, 271, 272, 273, 274, 275, 276, 277, 278, 279, 280, 281, 282, 283, 284, 285, 286, 287, 288, 289, 290, 291, 292, 293, 294, 295, 296, 297, 298, 299, 300, 301, 302, 303, 304, 305, 306, 307, 308, 309, 310, 311, 312, 313, 314, 315, 316, 317, 318, 319, 320, 321, 322, 323, 324, 325, 326, 327, 328, 329, 330, 331, 332, 333, 334, 335, 336, 337, 338, 339, 340, 341, 342, 343, 344, 345, 346, 347, 348, 349, 350, 351, 352, 353, 354, 355, 356, 357, 358, 359, 360, 361, 362, 363, 364, 365, 366, 367, 368, 369, 370, 371, 372, 373, 374, 375, 376, 377, 378, 379, 380, 381, 382, 383, 384, 385, 386, 387, 388, 389, 390, 391, 392, 393, 394, 395, 396, 397, 398, 399, 400, 401, 402, 403, 404, 405, 406, 407, 408, 409, 410, 411, 412, 413, 414, 415, 416, 417, 418, 419, 420, 421, 422, 423, 424, 425, 426, 427, 428, 429, 430, 431, 432, 433, 434, 435, 436, 437, 438, 439, 440, 441, 442, 443, 444, 445, 446, 447, 448, 449, 450, 451, 452, 453, 454, 455, 456, 457, 458, 459, 460, 461, 462, 463, 464, 465, 466, 467, 468, 469, 470, 471, 472, 473, 474, 475, 476, 477, 478, 479, 480, 481, 482, 483, 484, 485, 486, 487, 488, 489, 490, 491, 492, 493, 494, 495, 496, 497, 498, 499, 500, 501, 502, 503, 504, 505, 506, 507, 508, 509, 510, 511, 512, 513, 514, 515, 516, 517, 518, 519, 520, 521, 522, 523, 524, 525, 526, 527, 528, 529, 530, 531, 532, 533, 534, 535, 536, 537, 538, 539, 540, 541, 542, 543, 544, 545, 546, 547, 548, 549, 550, 551, 552, 553, 554, 555, 556, 557, 558, 559, 560, 561, 562, 563, 564, 565, 566, 567, 568, 569, 570, 571, 572, 573, 574, 575, 576, 577, 578, 579, 580, 581, 582, 583, 584, 585, 586, 587, 588, 589, 590, 591, 592, 593, 594, 595, 596, 597, 598, 599, 600, 601, 602, 603, 604, 605, 606, 607, 608, 609, 610, 611, 612, 613, 614, 615, 616, 617, 618, 619, 620, 621, 622, 623, 624, 625, 626, 627, 628, 629, 630, 631, 632, 633, 634, 635, 636, 637, 638, 639, 640, 641, 642, 643, 644, 645, 646, 647, 648, 649, 650, 651, 652, 653, 654, 655, 656, 657, 658, 659, 660, 661, 662, 663, 664, 665, 666, 667, 668, 669, 670, 671, 672, 673, 674, 675, 676, 677, 678, 679, 680, 681, 682, 683, 684, 685, 686, 687, 688, 689, 690, 691, 692, 693, 694, 695, 696, 697, 698, 699, 700, 701, 702, 703, 704, 705, 706, 707, 708, 709, 710, 711, 712, 713, 714, 715, 716, 717, 718, 719, 720, 721, 722, 723, 724, 725, 726, 727, 728, 729, 730, 731, 732, 733, 734, 735, 736, 737, 738, 739, 740, 741, 742, 743, 744, 745, 746, 747, 748, 749, 750, 751, 752, 753, 754, 755, 756, 757, 758, 759, 760, 761, 762, 763, 764, 765, 766, 767, 768, 769, 770, 771, 772, 773, 774, 775, 776, 777, 778, 779, 780, 781, 782, 783, 784, 785, 786, 787, 788, 789, 790, 791, 792, 793, 794, 795, 796, 797, 798, 799, 800, 801, 802, 803, 804, 805, 806, 807, 808, 809, 810, 811, 812, 813, 814, 815, 816, 817, 818, 819, 820, 821, 822, 823, 824, 825, 826, 827, 828, 829, 830, 831, 832, 833, 834, 835, 836, 837, 838, 839, 840, 841, 842, 843, 844, 845, 846, 847, 848, 849, 850, 851, 852, 853, 854, 855, 856, 857, 858, 859, 860, 861, 862, 863, 864, 865, 866, 867, 868, 869, 870, 871, 872, 873, 874, 875, 876, 877, 878, 879, 880, 881, 882, 883, 884, 885, 886, 887, 888, 889, 890, 891, 892, 893, 894, 895, 896, 897, 898, 899, 900, 901, 902, 903, 904, 905, 906, 907, 908, 909, 910, 911, 912, 913, 914, 915, 916, 917, 918, 919, 920, 921, 922, 923, 924, 925, 926, 927, 928, 929, 930, 931, 932, 933, 934, 935, 936, 937, 938, 939, 940, 941, 942, 943, 944, 945, 946, 947, 948, 949, 950, 951, 952, 953, 954, 955, 956, 957, 958, 959, 960, 961, 962, 963, 964, 965, 966, 967, 968, 969, 970, 971, 972, 973, 974, 975, 976, 977, 978, 979, 980, 981, 982, 983, 984, 985, 986, 987, 988, 989, 990, 991, 992, 993, 994, 995, 996, 997, 998, 999, 1000</small>	Created by sta	Revision 2	Revision by sta	Revision date 15/08/2016
Title	Project	Item no			
Chest support	1039	126971			


Position	No.	Description	Quantity	Unit of Measure	Ean code
1+2	869480-1	Belt, chest support sz. small	1	PCS	5707292796530
1+2	869481-1	Belt, chest support sz.medium	1	PCS	5707292797230
1+2	869482-1	Belt, chest support sz.large	1	PCS	5707292797247
6	0640913	Button head/flange M5*10 FZB	2	PCS	5707292558985
7	121395	Hip strap mounting bracket	2	PCS	5707292780140
8	0454005	Buckle 24mm, black	2	PCS	5707292786425
9	1102780	Phillips screw	2	PCS	5707292800183

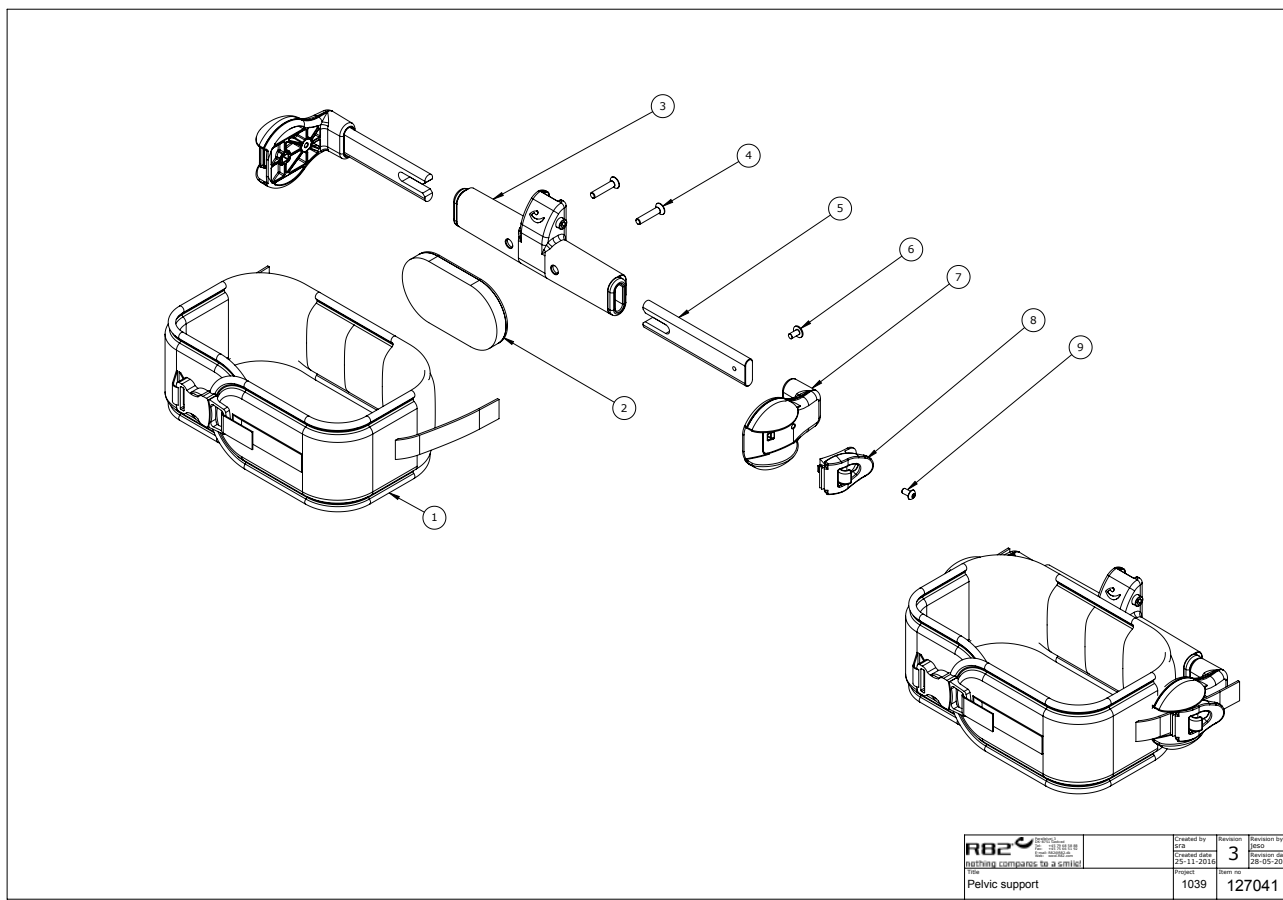


Position	No.	Description	Quantity	Unit of Measure	Ean code
1+2	869240-1	Belt, hip support sz. small	1	PCS	5707292797254
1+2	869241-1	Belt, hipsupport sz. medium	1	PCS	5707292797261
1+2	869242-1	Belt, hipsupport sz. large	1	PCS	5707292797278
6	0640913	Button head/flange M5*10 FZB	2	PCS	5707292558985
7	121395	Hip strap mounting bracket	2	PCS	5707292780140
8	0454005	Buckle 24mm, black	2	PCS	5707292786425
9	1102780	Phillips screw	2	PCS	5707292800183

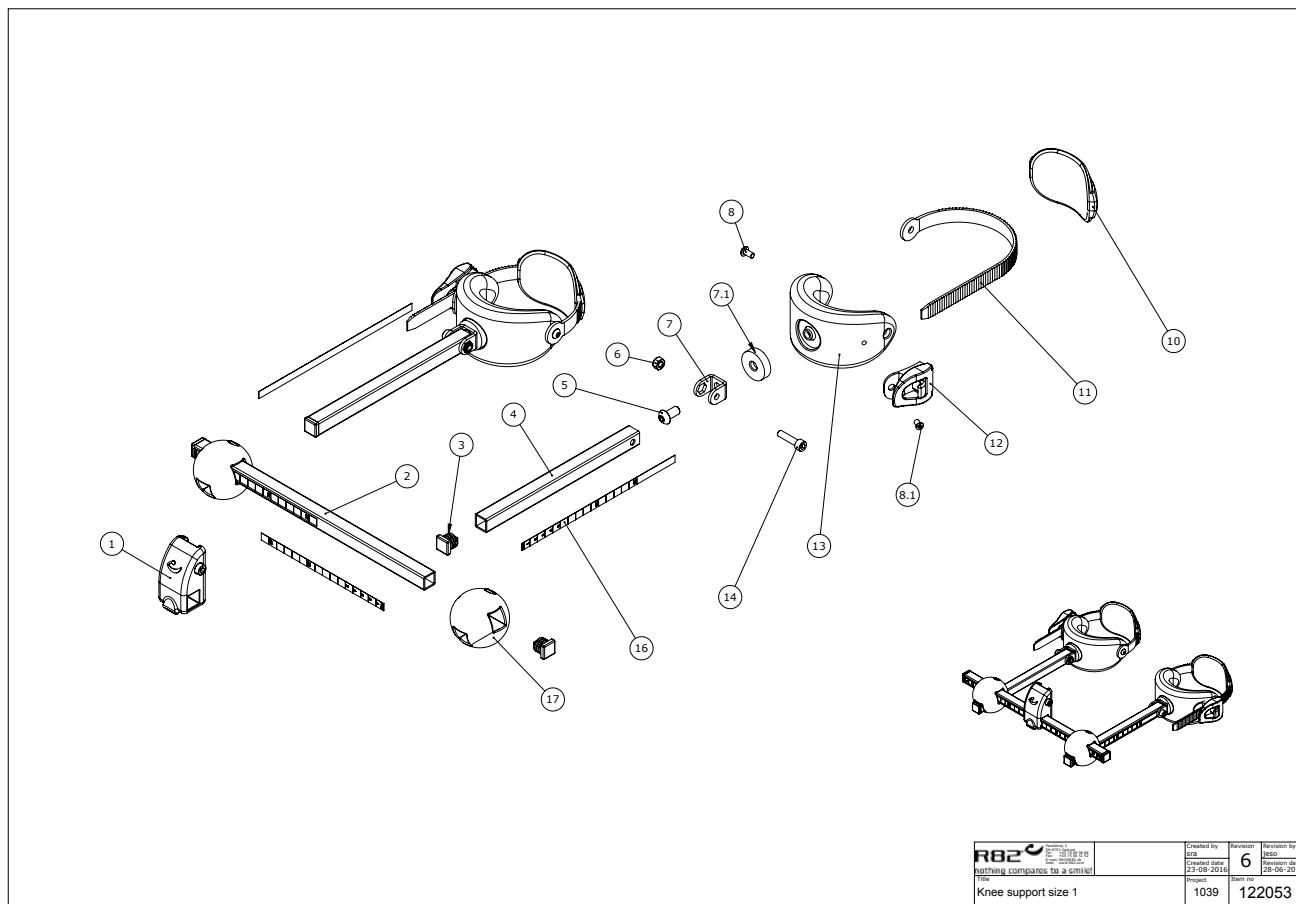


Position	No.	Description	Quantity	Unit of Measure	Ean code
1	117832	Column mount complete	1	PCS	5707292779885
8	0658010	Button head M5 x 10, stainless	2	PCS	5707292139375
8.1	1102780	Phillips screw	2	PCS	5707292800183
9	121133	Distance bush	2	PCS	5707292780096
10	127068	Knee pad size 1	2	PCS	5707292783431
11	0454010	Strap/ratchet 19*350mm, black	2	PCS	5707292786456
12	0454000	Buckle 19mm, black	2	PCS	5707292786418
13	128254	Knee support size 1	2	PCS	5707292795557

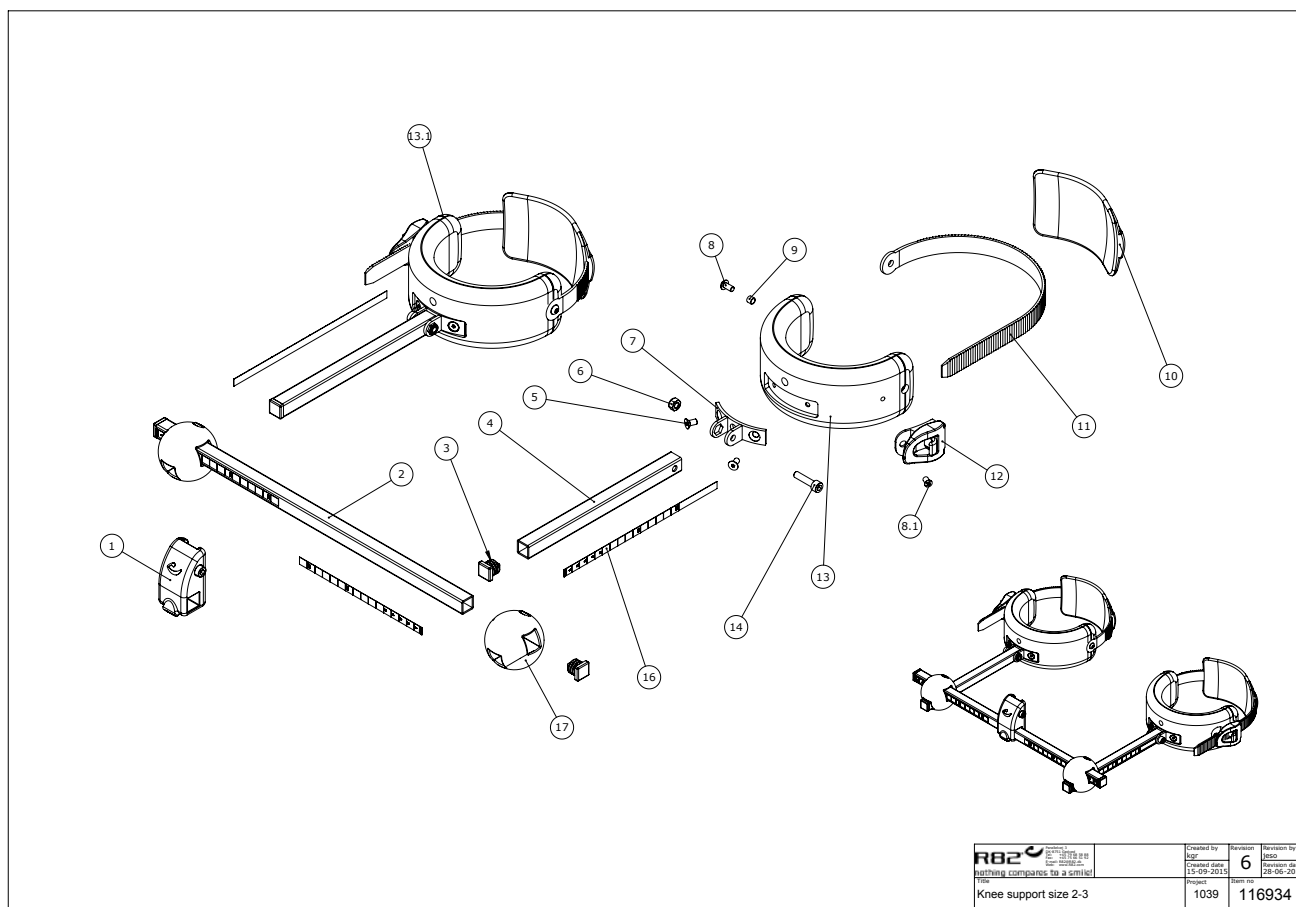


Position	No.	Description	Quantity	Unit of Measure	Ean code
1	117832	Column mount complete	1	PCS	5707292779885
8	0658010	Button head M5 x 10, stainless	2	PCS	5707292139375
8.1	1102780	Phillips screw	2	PCS	5707292800183
9	121133	Distance bush	2	PCS	5707292780096
10	127069	Knee pad size 2	2	PCS	5707292783448
10	127070	Knee pad size 3	2	PCS	5707292783455
11	0454010	Strap/ratchet 19*350mm, black	2	PCS	5707292786456
12	0454000	Buckle 19mm, black	2	PCS	5707292786418
13	128255	Knee support size 2	2	PCS	5707292795564
13	128256	Knee support size 3	2	PCS	5707292795571

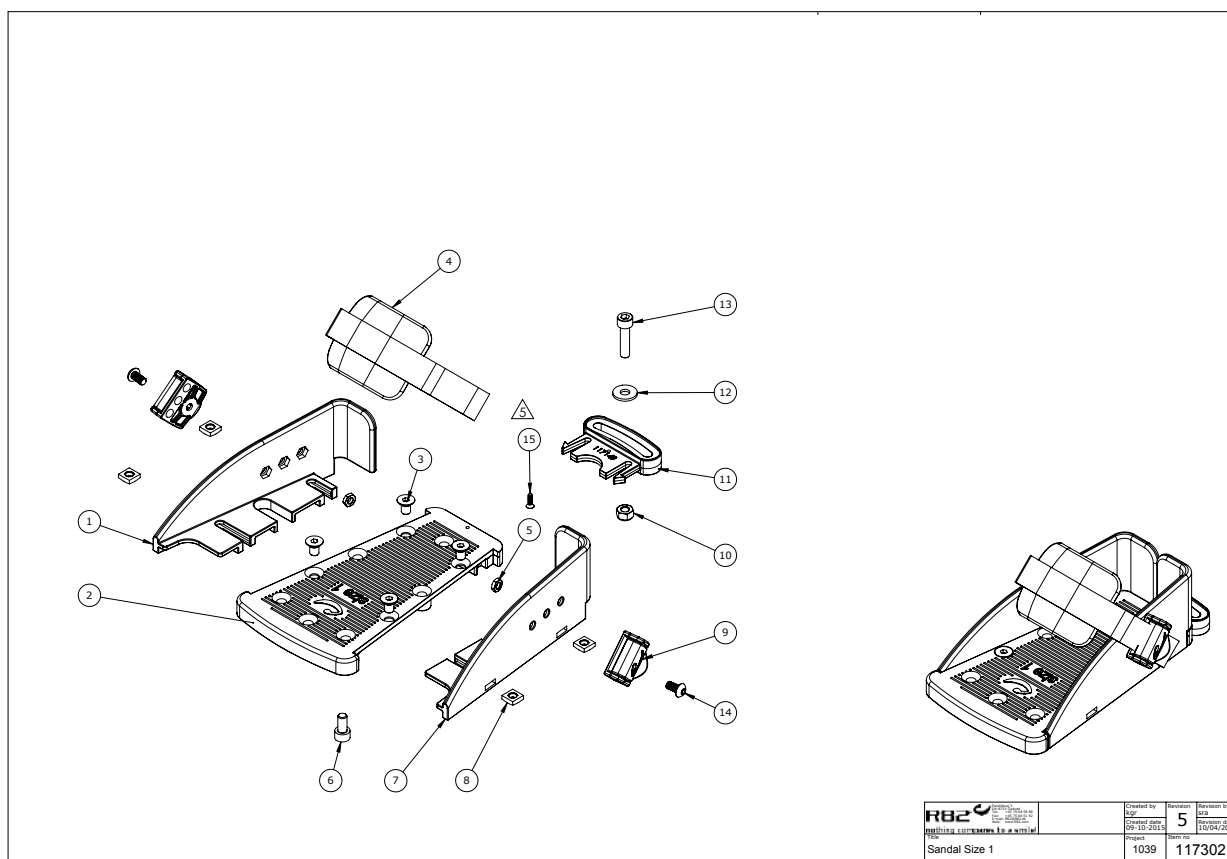

Position	No.	Description	Quantity	Unit of Measure	Ean code
4	127085	Foot strap size 1+2	2	PCS	5707292783585
4	127086	Strap size 3	2	PCS	5707292783424

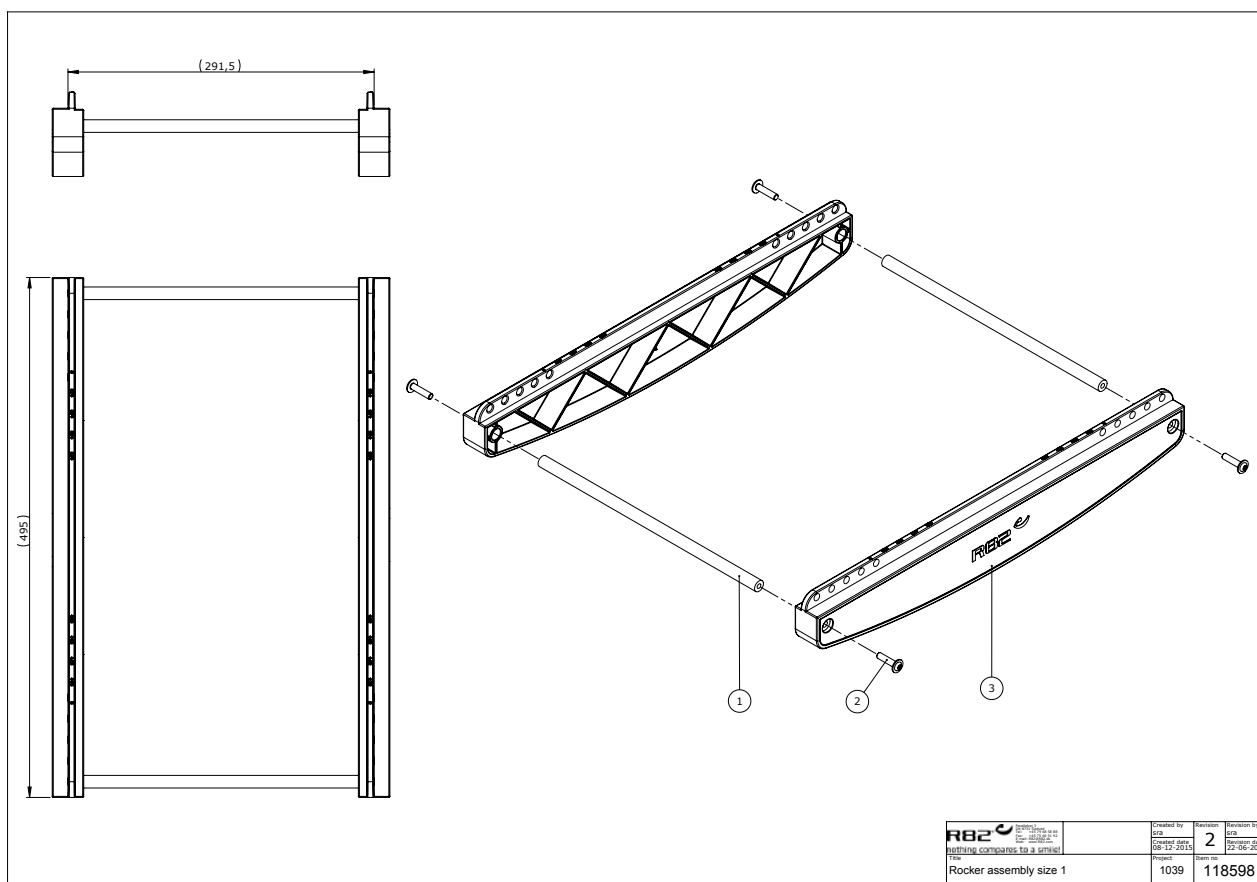

Position	No.	Description	Quantity	Unit of Measure	Ean code
1	117049	Rocker frame cross bar size 2	2	PCS	5707292780003
1	118599	Rocker frame cross bar size 1	2	PCS	5707292779991
2	0641621	Button head w/flange M6*20 FZB	4	PCS	5707292360960
3	117048	Rocker size 2	2	PCS	5707292779984
3	118717	Rocker Size 1	2	PCS	5707292779977

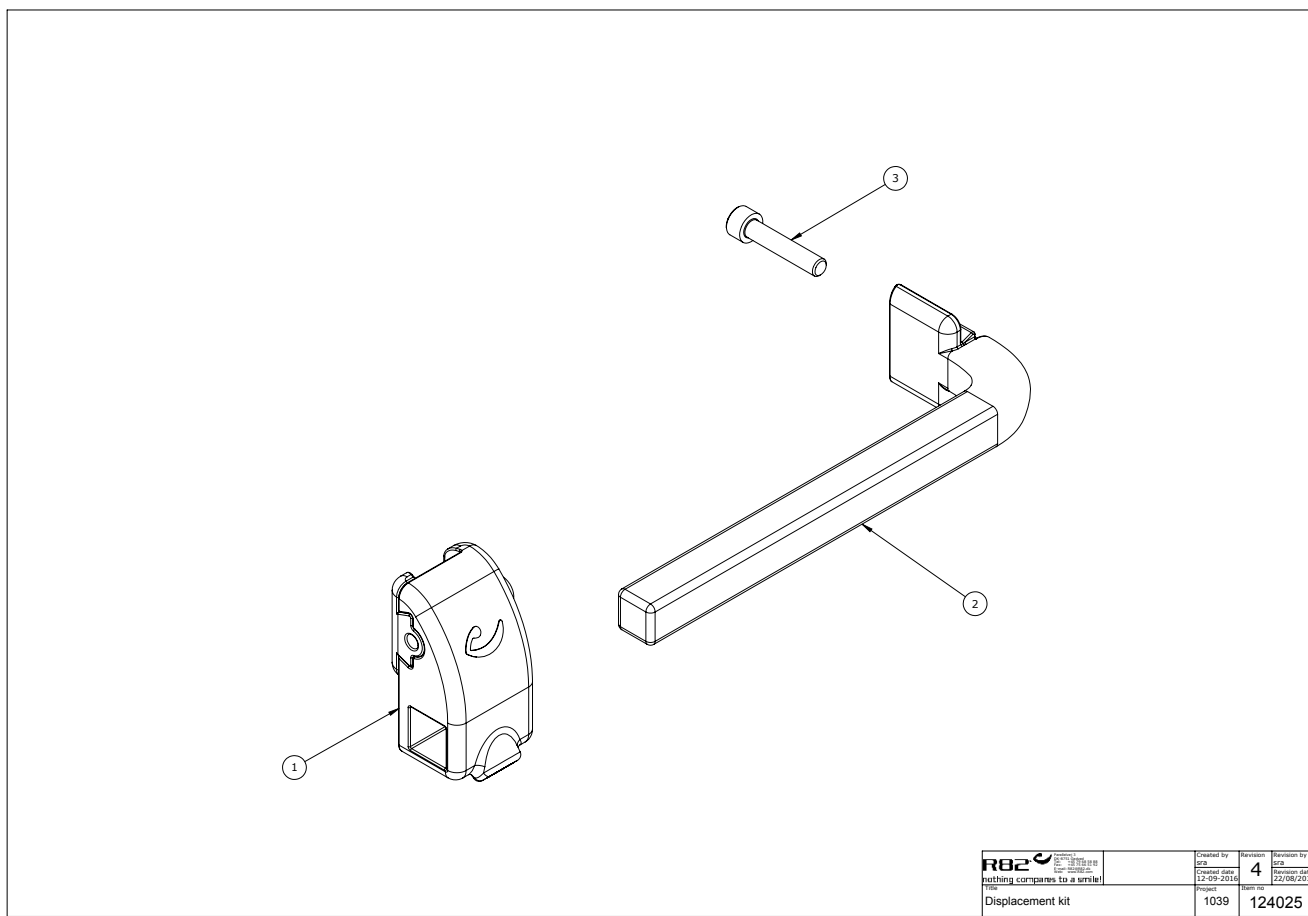

Position	No.	Description	Quantity	Unit of Measure	Ean code
1	117832	Column mount complete	1	PCS	5707292779885