

# multi frame

## Exploded views / Part lists


Rev. date (10.2010)

*On the following pages you will find the parts used to assemble this product. Please note that some spare parts cannot be purchased individually but are part of a complete kit. Please also note that for some spare parts extra delivery time may occur.*



## Contents

Multi Frame with 12½" wheels.....	3
Multi Frame with 22" wheels .....	7
Accessories .....	11

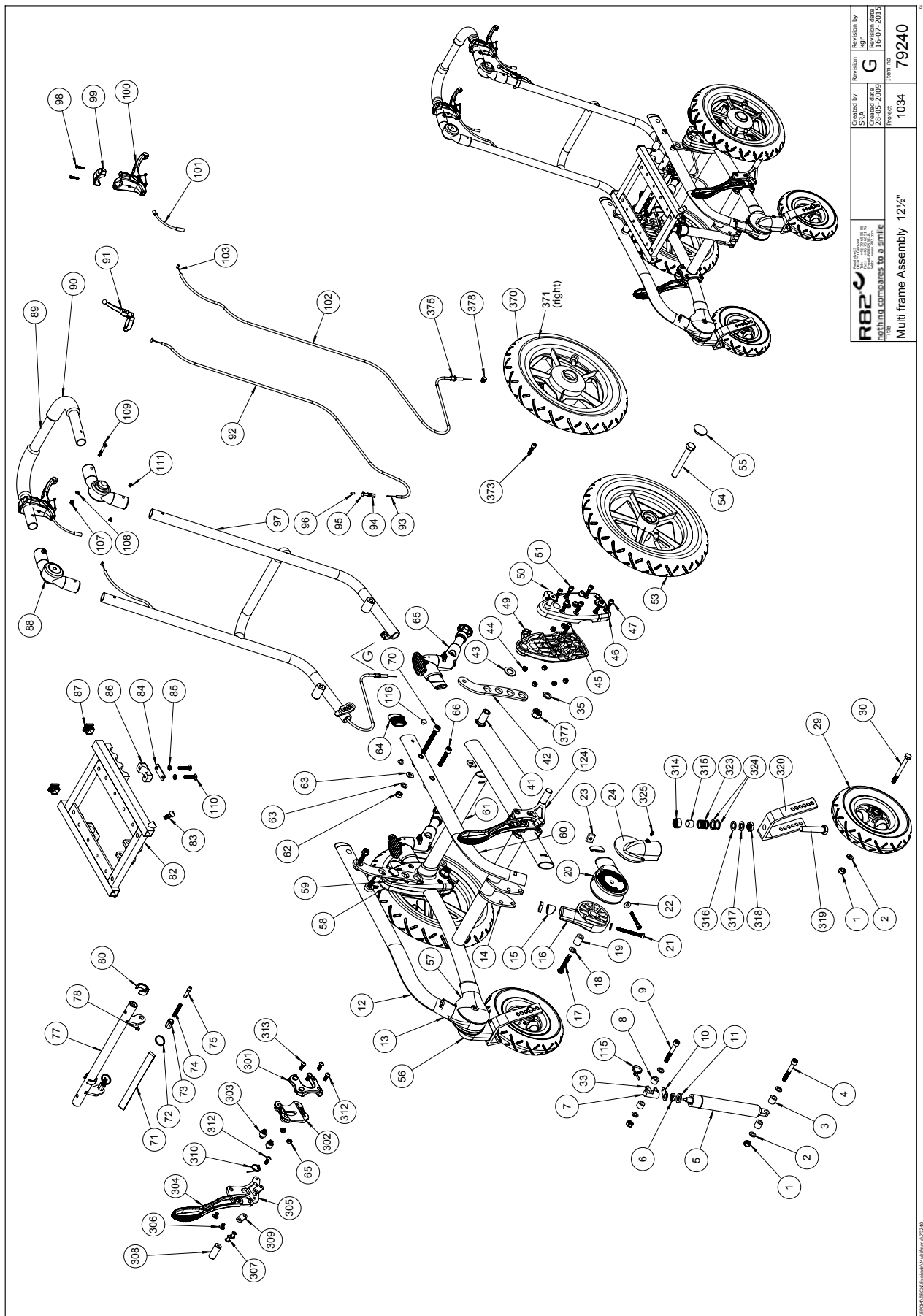


R82	Created by	Revision by
	28-05-2009	G
Nothing compares to a smile	Created date	Revision date
	28-05-2009	16-07-2015
Multi frame Assembly 12 1/2"	Project	Item no
	1034	79240

Position	No.	Description	Quantity	Unit of Measure	Ean code
1	1631080	Stainless lock nut 8MM	PIECES	4	5707292141453
2	1103200	Fazet washer 8MM	PIECES	6	5707292140296
3	1350024	Distance piece 14*15*8,2MM	PIECES	2	5707292140920
4	0603660	Insex screw 8*60	PIECES	1	5707292138279
5*	2205800	Block-o-lift 200N	PIECES	1	5707292143396
	2205830	Block-o-lift 250N	PIECES	1	5707292523457
6	1761000	Counter nut 10*1,0 fome thread	PIECES	1	5707292141538
7	2200010	Release house	PIECES	1	5707292142962
8	1350021	Distance piece 11*15*8,2MM	PIECES	2	5707292140890
9	0603600	Insex screw 8*55	PIECES	1	5707292138262
10	82268	Wire mounting by pedal	PIECES	1	5707292123039
11	1310712	Nylon washer, nature	PIECES	1	5707292356345
12*	75692-xx	Top frame, welded, right	PIECES	1	5707292504777
13	74704	Holder front wheel upper, right	PIECES	1	5707292411693
14*	75682-xx	Lower frame, welded, left, size 1	PIECES	1	5707292504661
	75683-xx	Lower frame, welded, left, size 2	PIECES	1	5707292504685
15	74683	Lock, moving part, short	PIECES	4	5707292425904
16	74705	Holder front wheel upper, left	PIECES	1	5707292411648
17	0603545	Insex head M8*45, stainless	PIECES	2	5707292138248
18	1105080	Stainless washer 8MM	PIECES	2	5707292140364
19	74654	Bush Ø16/Ø8, 1*21	PIECES	2	5707292411662
20	74703	Front wheel lower, left	PIECES	1	5707292411631
21	0602460	Insex M6*60, stainless	PIECES	4	5707292400727
22	1106025	Washer M6 O16/O6,4X1,6	PIECES	4	5707292404664
23	74090	Mounting for front fork	PIECES	6	5707292374066
24	5097016-1	Front wheel, left, compete	PIECES	1	5707292513854
29*	3009501	190*50 Solid wheel	PIECES	2	5707292483362
	3009500	190*50 Air wheel	PIECES	2	5707292483355
30	0602495	Insex M8*80, staines	PIECES	2	5707292423986
33	2200050	Small release arm	PIECES	1	5707292143075
35	1103500	Fazet washer 12MM	PIECES	4	5707292140326
41	75696	Bush for wheel support	PIECES	4	5707292503862
42	75706-1	Main tube (size 1)	PIECES	2	5707292532688
43	1102750	Washer 16MM	PIECES	4	5707292140258
44	1631060	Stainless lock nut 6MM	PIECES	14	5707292141446
45	0659580	Button head screw 6*30 MM	PIECES	2	5707292139481
46	74710-1	HJUL OPHÆNG YDRE VENSTRE	PIECES	1	5707292509567
47	0609827	STAINLESS INSEX M6*20	PIECES	2	5707292375049
48	1604500	STEEL NUT 12MM, LARGE PACK	PIECES	2	5707292141309
49	74708-1	Wheel suspension, left	PIECES	1	5707292509543
50	0612700	Insex M10*20,ZN/black	PIECES	4	5707292138767
51	0602425	Insex M6*25, stainless	PIECES	12	5707292404596
53*	3009515	12½*2 1/4" Air wheel	PIECES	2	5707292507020
	3009516	12½*2 1/4" Solid wheel	PIECES	2	5707292507037
54*	1920100	Screw M12*100, FZB	PIECES	2	5707292518118
	1902400	Steel set screw 12*110MM	PIECES	2	5707292142061
55	2704300	Black hub cap for wheel	PIECES	2	5707292144515
56	5097005-1	Front wheel, right, complete	PIECES	1	5707292513847
57	74702	Front wheel lower, right	PIECES	1	5707292411686

58	74711-1	Wheel suspension, right	PIECES	1	5707292509574
59	74709-1	Wheel suspension, right	PIECES	1	5707292509550
60*	75691-xx	Top frame, welded, left	PIECES	1	5707292504746
61*	74758	Cross tube 40X20, size 1	PIECES	1	5707292412317
	74607	Cross tube 40X20, size 2	PIECES	1	5707292403094
	74614	Cross tube 40X20, size 3	PIECES	1	5707292403155
62	1607001	Self locking nut 8MM	PIECES	2	5707292401281
63	1310717	Nylon washer O17-8,4*2MM	PIECES	4	5707292140821
64	74767	Plug for ellipse 20 * 40	PIECES	2	5707292412669
65	970953	Antitip device, long	SET	1	5707292411600
65	1631060	Stainless lock nut 6MM	PIECES	2	5707292141446
66	0603500	Insex screw 8*45	PIECES	2	5707292138231
70	0603750	Insex M8*75, FZB	PIECES	2	5707292138309
71*	5550116	PP adjustable strap, black	PIECES	1	5707292374714
	5550117	PP adjustable strap, black	PIECES	1	5707292374721
72	0216000	ring f.connectinglink 20*1,5MM	PIECES	2	5707292135230
73	6108222	Springhouse	PIECES	2	5707292185747
74	0310122	Compression spring 0,9*8U	PIECES	2	5707292135544
75	75746	Pin for spring lock	PIECES	2	5707292508928
76	75716	U-lock bracket (right)	PIECES	1	5707292474780
77*	75700-xx	Cross tube, welded, size 1	PIECES	1	5707292504821
	75701-xx	Cross tube, welded, size 2	PIECES	1	5707292504845
	75702-xx	Cross tube, welded, size 3	PIECES	1	5707292504890
78	0506519	Stainless blind rivet 4*12	PIECES	2	5707292137388
79	0659501	Button head w/flange M6*16 FZB	PIECES	2	5707292356000
80	75714	Fix ring	PIECES	2	5707292504562
81	75715	U-lock bracket (left)	PIECES	1	5707292504579
82*	75720-xx	Seat frame	PIECES	1	5707292504906
83	0603540	Insex low head M8x16 stainless	PIECES	2	5707292412485
84	75752	Pipeholder support plate	PIECES	2	5707292513779
85	1100490	Spring washer M6, stainless	PIECES	4	5707292411297
86	75713	Pipe holder	PIECES	2	5707292504555
87	2504500	Ferrule E20*1,0-2,0 INT.black	PIECES	2	5707292144126
88	0859090	Locking joint Ø22MM	PIECES	2	5707292440242
89*	75731	Hand brace (size 1)	PIECES	1	5707292474797
	75732	Hand brace (size 2)	PIECES	1	5707292482907
	75733	Hand brace (size 3)	PIECES	1	5707292484338
90	2500530	TPE ø22*30*200	PIECES	2	5707292381781
91	3503531	Gas pump lever with clamp	PIECES	1	5707292413529
92	5002987	Outer casing 103cm	PIECES	1	5707292520104
93	0371853	Stained wires 1,5MM 19T.	PIECES	1	5707292135889
94	0380000	Cable tighter M5*23	PIECES	1	5707292135933
95	0388006	Screw nipple 5*7,5MM	PIECES	1	5707292136114
96	0387000	End nipple for outer casing	PIECES	1	5707292136091
97*	75727-xx	Back frame, welded, size 1	PIECES	1	5707292504968
	75728-xx	Back frame, welded, size 2	PIECES	1	5707292506191
	75729-xx	Back frame, welded, size 3	PIECES	1	5707292506238
98+99	3503536	Clamp w/screws	PIECES	2	5707292427762
100	3503535	Brake parking lever wo/clamp	PIECES	2	5707292413536
101	0380060	Flexible V-Brake pipe.	PIECES	2	5707292425195

102	5002747	Outer casing for break	PIECES	2	5707292415899
103	0371853	Stained wires 1,5MM 19T.	PIECES	2	5707292135889
107	0609723	Stainless insex 5*40 UH.	PIECES	2	5707292435002
108	1102902	Washer 5MM, A2	PIECES	2	5707292411280
109	1631050	Stainless locking nut M5	PIECES	2	5707292510662
110	0659570	Button head screw 6*25 stainless	PIECES	4	5707292139474
111	0506508	Rive 5*14, stainless	PIECES	4	5707292359476
112	0235075	Cable tape, black	PIECES	2	5707292135308
113	75758	Main tube (size 3)	PIECES	2	5707292485120
114	1940010	Plastite 60, 2,9*6,5	PIECES	2	5707292142573
124	970939-L	Parking brake on frame, left	PIECES	1	5707292412386
301	74731	Cover inner, brake	PIECES	1	5707292412409
302	74730	Cover outer, brake	PIECES	1	5707292412416
303	6109200	M6 Thread bush, braking system	PIECES	2	5707292185952
304	74013	Braking arm	PIECES	1	5707292378033
305	74644	Brake bracket	PIECES	1	5707292407955
306	0641100	B.H. w/flange M6*10	PIECES	2	5707292139245
307	0503175	Black case, Ø6,35*17,5MM	PIECES	2	5707292137258
308	74764	Braking stick	PIECES	1	5707292435958
309	72338	Mounting 12X4X34,5	PIECES	2	5707292107794
310	74138	Spring for brake, right	PIECES	1	5707292377500
312	0659500	Button head screw	PIECES	3	5707292139450
313	0659550	Button head M6*20, stainless	PIECES	2	5707292139467
314	1631120	Stainless lock nut M12	PIECES	2	5707292502049
315	74804	Wheel bush	PIECES	2	5707292465177
316	0683900	Tight-washer 12*18*1	PIECES	2	5707292139726
317	1101600	Center spring washer 12MM	PIECES	2	5707292140197
318	1760012	Nut M12,FZB	PIECES	2	5707292519078
319	1902373	Steel set screw M12*60 FZB	PIECES	2	5707292519801
320	2965760	Front fork, 60mm	PIECES	2	5707292415226
323	0310105	Pressure spring 1,5*16,2	PIECES	2	5707292465191
324	0683950	Tight-washer 16*22*0,5	PIECES	4	5707292139733
325	1310542	Hub cap, black Ø8*Ø5MM	PIECES	2	5707292472014
370*	3009517	12½*2 1/4" air wheel, right	PIECES	1	5707292518071
	3009519	12½*2 1/4" solid wheel, right	PIECES	1	5707292518095
371*	3009518	12½*2 1/4" air wheel, left	PIECES	1	5707292518088
	3009520	12½*2 1/4" solid wheel, left	PIECES	1	5707292518101
373	0609827	Stainless insex M6*20	PIECES	2	5707292375049
375	0380000	Cable tighter M5*23	PIECES	2	5707292135933
377	1608000	Lock nut 12MM, large pack	PIECES	2	5707292141392
378	0380040	Cable bench screw M5	PIECES	2	5707292135988
*		Choose size and colour			
		Colour code: 23: blue, 27: red, 54: silver			

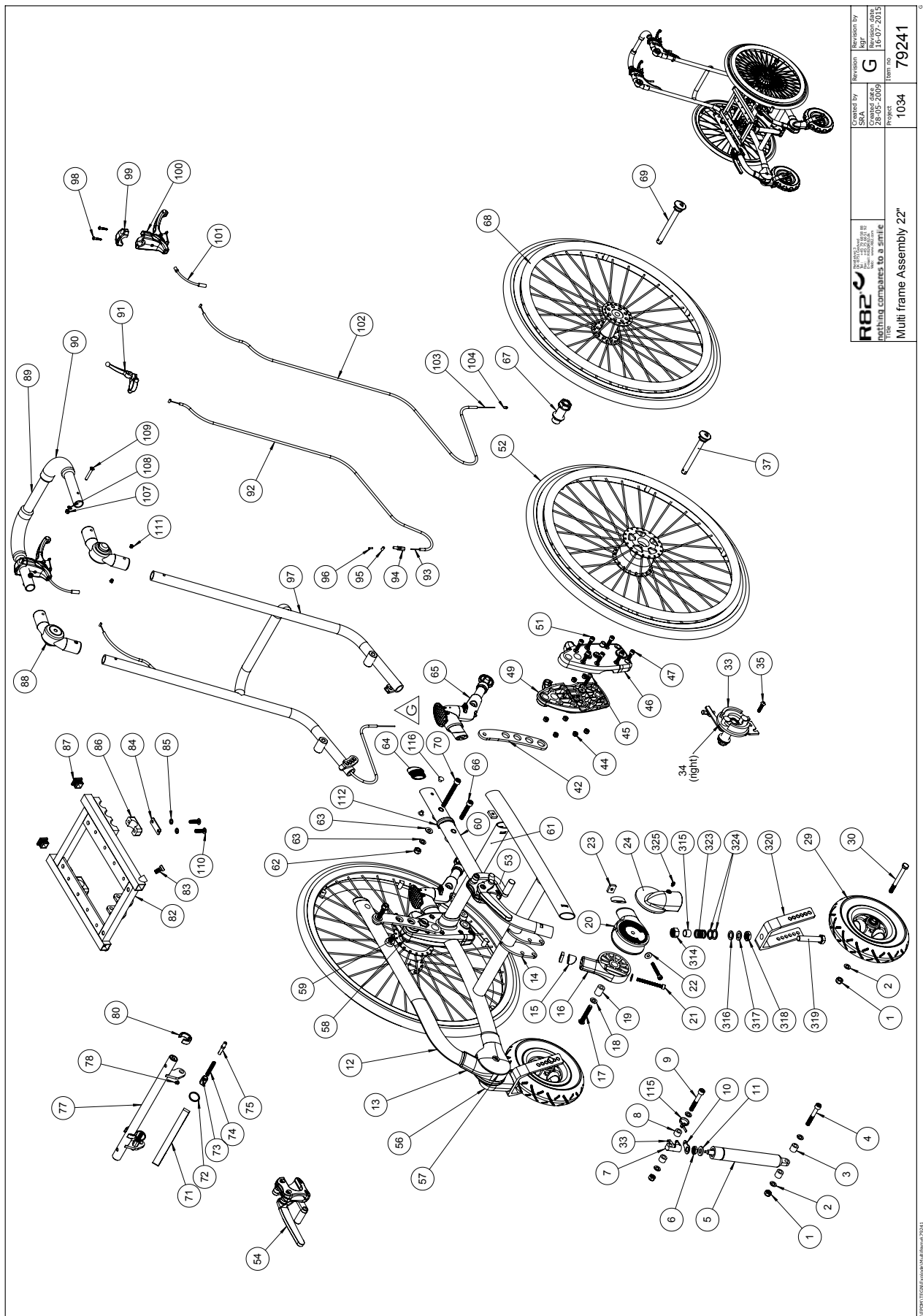

<b>R82</b> <small>nothing compares to a smile</small>	Created by	Revision by
	28-05-2009	G
<b>Multi frame Assembly 22"</b>	Created date	Revision date
	28-05-2009	16-07-2015
<b>79241</b>	Project	Item no
	1034	79241





Position	No.	Description	Quantity	Unit of Measure	Ean code
1	1631080	Stainless lock nut 8MM	PIECES	4	5707292141453
2	1103200	Fazet washer 8MM	PIECES	6	5707292140296
3	1350024	Distance piece 14*15*8,2MM	PIECES	2	5707292140920
4	0603660	Insex screw 8*60	PIECES	1	5707292138279
5*	2205800	Block-o-lift 200N	PIECES	1	5707292143396
	2205830	Block-o-lift 250N	PIECES	1	5707292523457
6	1761000	Counter nut 10*1,0 fome thread	PIECES	1	5707292141538
7	2200010	Release house	PIECES	1	5707292142962
8	1350021	Distance piece 11*15*8,2MM	PIECES	2	5707292140890
9	0603600	Insex screw 8*55	PIECES	1	5707292138262
10	82268	Wire mounting by pedal	PIECES	1	5707292123039
11	1310712	Nylon washer, nature	PIECES	1	5707292356345
12*	75692-xx	Top frame, welded, right	PIECES	1	5707292504777
13	74704	Holder front wheel upper, right	PIECES	1	5707292411693
14*	75682-xx	Lower frame, welded, left, size 1	PIECES	1	5707292504661
	75683-xx	Lower frame, welded, left, size 2	PIECES	1	5707292504685
15	74683	Lock, moving part, short	PIECES	4	5707292425904
16	74705	Holder front wheel upper, left	PIECES	1	5707292411648
17	0603545	Insex head M8*45, stainless	PIECES	2	5707292138248
18	1105080	Stainless washer 8MM	PIECES	2	5707292140364
19	74654	Bush Ø16/Ø8, 1*21	PIECES	2	5707292411662
20	74703	Front wheel lower, left	PIECES	1	5707292411631
21	0602460	Insex M6*60, stainless	PIECES	4	5707292400727
22	1106025	Washer M6 O16/O6,4X1,6	PIECES	4	5707292404664
23	74090	Mounting for front fork	PIECES	6	5707292374066
24	5097016-1	Front wheel, left, compete	PIECES	1	5707292513854
29*	3009501	190*50 Solid wheel	PIECES	2	5707292483362
	3009500	190*50 Air wheel	PIECES	2	5707292483355
30	0602495	Insex M8*80, stainles	PIECES	2	5707292423986
33	2200050	Small release arm	PIECES	1	5707292143075
33	2983062	Brake plate, left	PIECES	1	5707292505040
34	2983062	Brake plate, right	PIECES	1	5707292505057
35	1103500	Fazet washer 12MM	PIECES	4	5707292140326
42	75756	Main tube (size 1)	PIECES	2	5707292485144
44	1631060	Stainless lock nut 6MM	PIECES	14	5707292141446
45	0659580	Button head screw 6*30 MM	PIECES	2	5707292139481
46	74710-1	Wheel suspension, left	PIECES	1	5707292509567
47	0609827	Stainless insex M6*20	PIECES	2	5707292375049
49	74708-1	Wheel suspension, left	PIECES	1	5707292509543
51	0602425	Insex M6*25, stainless	PIECES	12	5707292404596
52*	3302310	22" Wheel with drumbrake, solid	PIECES	2	5707292414755
	3302510	22" Wheel with drumbrake, air	PIECES	2	5707292414823
53	75736	Wheel brake, left	PIECES	1	5707292474803
54	75737	Wheel brake, right	PIECES	1	5707292484765
56	5097005-1	Front wheel, right, complete	PIECES	1	5707292513847
57	74702	Front wheel lower, right	PIECES	1	5707292411686
58	74711-1	Wheel suspension, right	PIECES	1	5707292509574
59	74709-1	Wheel suspension, right	PIECES	1	5707292509550
60*	75691-xx	Top frame, welded, left	PIECES	1	5707292504746



61*	74758	Cross tube 40X20, size 1	PIECES	1	5707292412317
	74607	Cross tube 40X20, size 2	PIECES	1	5707292403094
	74614	Cross tube 40X20, size 3	PIECES	1	5707292403155
62	1607001	Self locking nut 8MM	PIECES	2	5707292401281
63	1310717	Nylon washer O17-8,4*2MM	PIECES	4	5707292140821
64	74767	Plug for ellipse 20 * 40	PIECES	2	5707292412669
65	970953	Antitip device, long	SET	1	5707292411600
66	0603500	Insex screw 8*45	PIECES	2	5707292138231
67	2983063	Axle set 1x QR 012*98mm Cougar	PIECES	2	5707292505064
68	3302410	22" WHEEL, RADIAL, AIR	PIECES	2	5707292414793
69	2983064	Quickrelease til pediatristol	PIECES	2	5707292509499
70	0603750	Insex M8*75, FZB	PIECES	2	5707292138309
71*	5550116	PP adjustable strap, black	PIECES	1	5707292374714
	5550117	PP adjustable strap, black	PIECES	1	5707292374721
72	0216000	ring f.connectinglink 20*1,5MM	PIECES	2	5707292135230
73	6108222	Springhouse	PIECES	2	5707292185747
74	0310122	Compression spring 0,9*8U	PIECES	2	5707292135544
75	75746	Pin for spring lock	PIECES	2	5707292508928
76	75716	U-lock bracket (right)	PIECES	1	5707292474780
77*	75700-xx	Cross tube, welded, size 1	PIECES	1	5707292504821
	75701-xx	Cross tube, welded, size 2	PIECES	1	5707292504845
	75702-xx	Cross tube, welded, size 3	PIECES	1	5707292504890
78	0506519	Stainless blind rivet 4*12	PIECES	2	5707292137388
79	0659501	Button head w/flange M6*16 FZB	PIECES	2	5707292356000
80	75714	Fix ring	PIECES	2	5707292504562
81	75715	U-lock bracket (left)	PIECES	1	5707292504579
82*	75720-xx	Seat frame	PIECES	1	5707292504906
83	0603540	Insex low head M8x16 stainless	PIECES	2	5707292412485
84	75752	Pipeholder support plate	PIECES	2	5707292513779
85	1100490	Spring washer M6, stainless	PIECES	4	5707292411297
86	75713	Pipe holder	PIECES	2	5707292504555
87	2504500	Ferrule E20*1,0-2,0 INT.black	PIECES	2	5707292144126
88	0859090	Locking joint Ø22MM	PIECES	2	5707292440242
89*	75731	Hand brace (size 1)	PIECES	1	5707292474797
	75732	Hand brace (size 2)	PIECES	1	5707292482907
	75733	Hand brace (size 3)	PIECES	1	5707292484338
90	2500530	TPE ø22*30*200	PIECES	2	5707292381781
91	3503531	Gas pump lever with clamp	PIECES	1	5707292413529
92	5002987	Outer casing 103cm	PIECES	1	5707292520104
93	0371853	Stained wires 1,5MM 19T.	PIECES	1	5707292135889
94	0380000	Cable tighter M5*23	PIECES	1	5707292135933
95	0388006	Screw nipple 5*7,5MM	PIECES	1	5707292136114
96	0387000	End nipple for outer casing	PIECES	1	5707292136091
97*	75727-xx	Back frame, welded, size 1	PIECES	1	5707292504968
	75728-xx	Back frame, welded, size 2	PIECES	1	5707292506191
	75729-xx	Back frame, welded, size 3	PIECES	1	5707292506238
98+99	3503536	Clamp w/screws	PIECES	2	5707292427762
100	3503535	Brake parking lever wo/clamp	PIECES	2	5707292413536
101	0380060	Flexible V-Brake pipe.	PIECES	2	5707292425195
102	5002747	Outer casing for break	PIECES	2	5707292415899

104	0380100	Clip for cable	PIECES	2	5707292136015
103	0371853	Stained wires 1,5MM 19T.	PIECES	2	5707292135889
107	0609723	Stainless inox 5*40 UH.	PIECES	2	5707292435002
108	1102902	Washer 5MM, A2	PIECES	2	5707292411280
109	1631050	Stainless locking nut M5	PIECES	2	5707292510662
110	0659570	Button head screw 6*25 stainless	PIECES	4	5707292139474
111	0506508	Rive 5*14, stainless	PIECES	4	5707292359476
112	0235075	Cable tape, black	PIECES	2	5707292135308
113	75758	Main tube (size 3)	PIECES	2	5707292485120
114	1940010	Plastite 60, 2,9*6,5	PIECES	2	5707292142573
124	970939-L	Parking brake on frame, left	PIECES	1	5707292412386
314	1631120	Stainless lock nut M12	PIECES	2	5707292502049
315	74804	Wheel bush	PIECES	2	5707292465177
316	0683900	Tight-washer 12*18*1	PIECES	2	5707292139726
317	1101600	Center spring washer 12MM	PIECES	2	5707292140197
318	1760012	Nut M12,FZB	PIECES	2	5707292519078
319	1902373	Steel set screw M12*60 FZB	PIECES	2	5707292519801
320	2965760	Front fork, 60mm	PIECES	2	5707292415226
323	0310105	Pressure spring 1,5*16,2	PIECES	2	5707292465191
324	0683950	Tight-washer 16*22*0,5	PIECES	4	5707292139733
325	1310542	Hub cap, black Ø8*Ø5MM	PIECES	2	5707292472014
*		Choose size and colour			
		Colour code: 23: blue, 27: red, 54: silver			

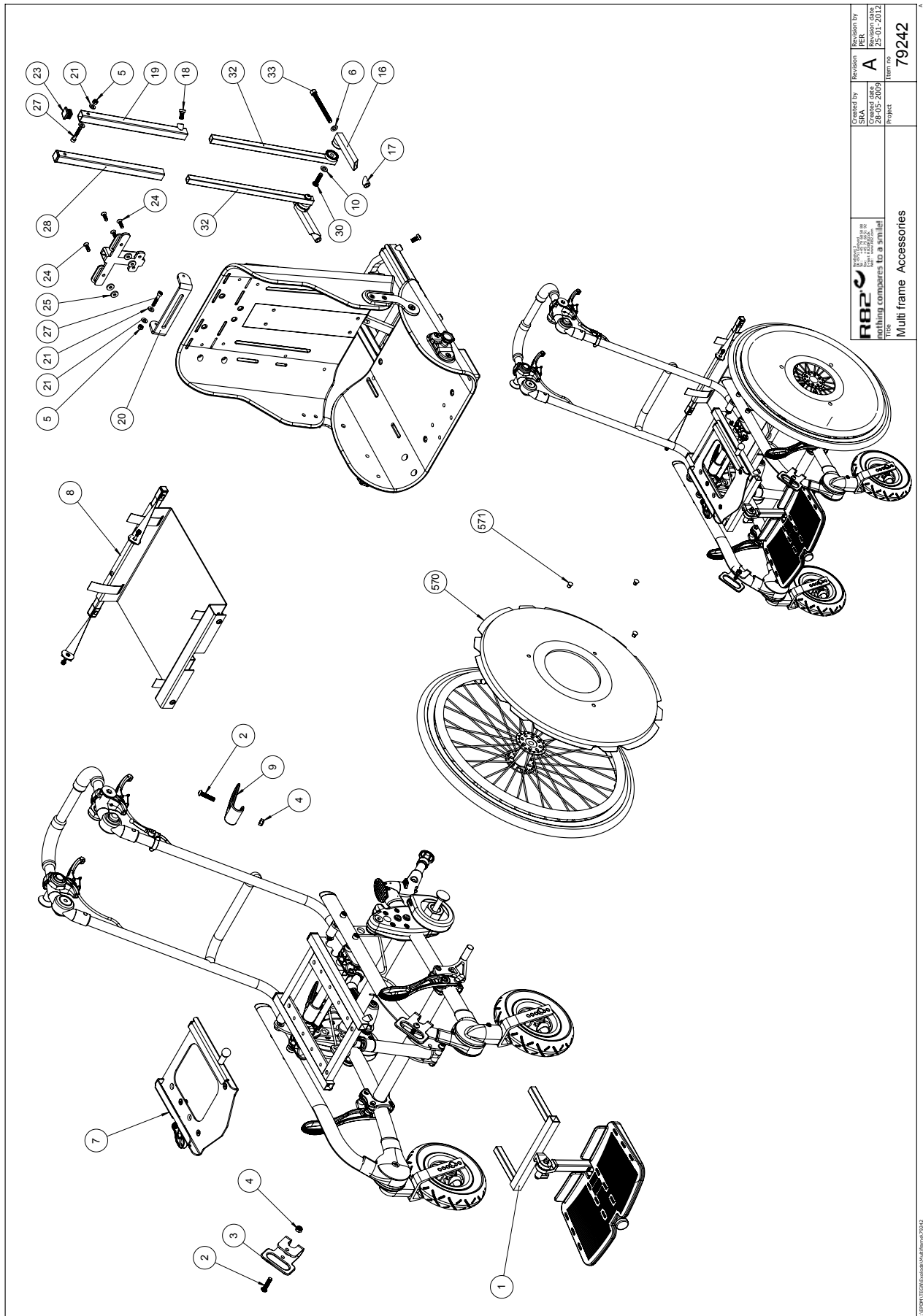

 nothing compares to a smile!	<div>Revised by SRA Created date 28-05-2009</div>	<div>Revision A</div>	<div>Revision date 25-07-2012</div>	Item no	
				79242	
Title Multi frame Accessories					

Position	No.	Description	Quantity	Unit of Measure	Ean code
1*	8910821-1	Rod for footrest, size 1, 15cm	PIECES	1	5707292460240
	8910821-2	Rod for footrest. size 2, 29cm	PIECES	1	5707292460257
	8910821-3	Rod for footrest size 3, 35cm	PIECES	1	5707292460264
	95145-1	Footplate size 1	PIECES	1	5707292331113
	95145-2	Footplate size 2	PIECES	1	5707292331120
	95145-3	Footplate size 3	PIECES	1	5707292398833
2	0663035	Button head screw 8*35 stainl.	PIECES	4	5707292139627
3	74639	Clamping bracket, front	PIECES	2	5707292406088
4	1631080	Stainless lock nut 8MM	PIECES	4	5707292141453
7	8911610-98	Interface for seat mounting	PIECES	1	5707292466334
8	970815	Vent tray, silver	PIECES	1	5707292418418
9	75735	Transportbeslag	PIECES	2	5707292484352
570	2982200	22" SPOKEPROTECTORS TRANSP.PVC	PIECES	1	5707292145031
571	2982401	CLIPS FOR SPOKE GUARDS, BLACK	PIECES	3	5707292427670