

Caribou

User information appropriate for use by people with visual, reading or cognitive disability

The latest version of all instructions are always available on the R82 website support.R82.org and can be printed in larger sizes

Information available in pictogram

The instructions are made as pictograms

Intended use and intended environment

The Caribou is a prone and supine stander providing the user very important physical and psychological benefits. It is developed for users suffering from muscular atrophy e.g. children with CP, GMFCS level 3-5. It is built with an optimum of security in mind and is only to be used on a level, non-tilting surface. It comes in 4 sizes, with the possibility for either manual or power operated tilt. This product is only for indoor use but can in an emergency be taken outside.

Maintenance instructions

Maintenance of a medical device is wholly the responsibility of the owner of that device. Failure to maintain a device in accordance with the instructions may invalidate the device's warranty. Furthermore, failure to maintain a device may compromise the clinical condition or safety of users and/or their caregivers.

Frequency Description

- | | |
|---------------------|---|
| 1 st Day | <ul style="list-style-type: none"> - Read the instructions for use (IFU) carefully before use and store them in a safe place for future reference - Store the enclosed tools on/at the product |
| Daily | <ul style="list-style-type: none"> - Use a dry cloth to clean the product - Visual inspection. Repair or replace any damaged or worn-out parts - Ensure that all fasteners are in place and fitted correctly - Check all fittings and buckles for signs of worn out parts - Be aware of any MAX mark indications before adjustment - Disinfect the upholstery. please check the disinfection section in this User Guide for more information - Inspect the detachable parts. If required, disinfect the parts with a 70% disinfectant solution. It is recommended to wipe of any residue and dirt using a cloth wrung in warm water and a mild detergent/soap without chlorine. Let the detachable parts dry before disinfection. - Check that all cables connecting the power system are in good shape (see IFU page 16) |
| Weekly | <ul style="list-style-type: none"> - Check that all castors are moving freely and that all wheel locks/brakes are easily activated |

Caribou	
	<ul style="list-style-type: none"> - Wipe off any residue and dirt from the product using a cloth with warm water and a mild detergent/soap and dry before use - All nuts and bolt on this product must be checked and tightened to avoid unnecessary failures - Oil the swing-away parts. We recommend you use a professional lubricant system - The product has to be checked and tightened by a qualified technician at least every 6 months to avoid failures.
Monthly	
Annually	<ul style="list-style-type: none"> - Inspect the frame for cracks or signs for worn-out parts and carry out annual service. Never use a product which appears to be sub-standard or faulty
Cleaning description	<p>This product can be washed at 60° using a mild detergent in 15-20 minutes in a washing machine de-signed for medical devices. Use the machine's drying function to dry the product. Clean the accessories separately.</p> <p>Before washing:</p> <ol style="list-style-type: none"> 1. Avoid degreasing of the piston rods, the actuators should be retracted to minimum stroke and without load before washing. 2. Remove all accessories and remove the power column <p>After washing:</p> <ol style="list-style-type: none"> 3. Replace the power column going backwards 4. Check that the handset functionalities work correct
Disinfection description	<p>The upholstery can be disinfected with a 70% disinfectant solution. It is recommended to wipe off any residue and dirt from the upholstery, using a cloth with warm water and a mild detergent/soap without chlorine and let it dry before disinfection.</p>
Dimensions	http://www.R82.org - Select product or click here
Mass	http://www.R82.org - Select product or click here
Used in combination with other products	N/A
Warning about dangerous combinations	<p>The CE mark must be removed when rebuilding the product, when used in combination with a product from another manufacturer or when using other than original R82 spare parts and fittings.</p>
List of accessories, detachable parts and materials	http://www.R82.org - Select product or click here
Programmable controller information	N/A
Operator control adjustments	N/A

Caribou	
Folding and dismantling	N/A
Transport instructions	-10°C to +50°C
Sound power level	Noise level: ≤ 50 db