

strong base

Exploded views / Part lists


Rev. date (11.2017)

On the following pages you will find the parts used to assemble this product. Please note that some spare parts cannot be purchased individually but are part of a complete kit. Please also note that for some spare parts extra delivery time may occur.



Contents


Strong base, manuel.....	3
Strong Base, spindle	5
Strong Base, interface.....	8

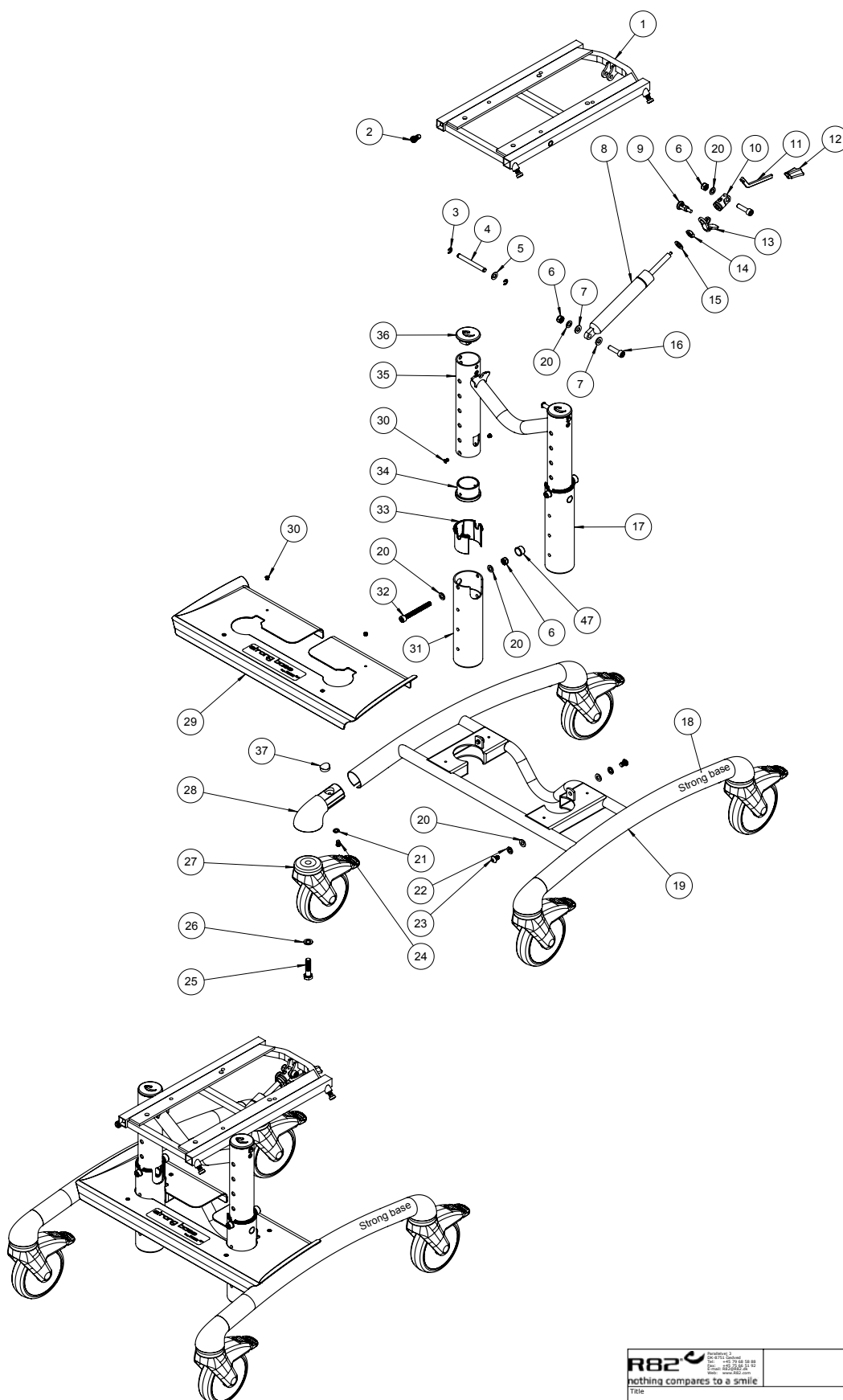

C:\EPDM\Projects\1038\79451

R82 <small>nothing compares to a smile</small>	<small> R82 A/S Parallelsvej 3 8751 Gedved Denmark Tel: +45 79 68 58 88 Fax: +45 75 66 51 92 Email: R82@R82.dk Web: www.R82.org </small>	Created by SRA	Revision D	Revision by PER
	Title Strong base mechanical	Created date 15.12.2009	Project 1038	Revision date 21-04-2015 Item no 79451


Position	No.	Description	Quantity	Unit of Measure	Ean code
1	75776-35	Seat frame, painted	1	PCS	5707292632265
2	0603530	Insex low head M8x16 stainless	4	PCS	5707292412492
3	1145060	Locking washer, axle stainless	4	PCS	5707292375056
4	75784	Axle for seatframe	2	PCS	5707292545923
5	1310645	Washer, nature Ø16*8*0,5mm	2	PCS	5707292446015
6	1607000	Lock nut 8mm, large pack	4	PCS	5707292141378
7	1310717	Nylon washer, nat o17-8,4*2mm	2	PCS	5707292140821
8	5040065	Gas spring 800N, mount Strongb	1	PCS	5707292558688
16	0615010	Insex M8*30 fzb, shortened	2	PCS	5707292367990
17	102200-35	Height Adj. column righ,painte	1	PCS	5707292632234
18	0493210	Strongbase label silver	2	PCS	5707292553775
19	75760-35	Frame, strongbase, painted	1	PCS	5707292632258
20	1105080	Stainless washer 8mm	10	PCS	5707292140364
21	1100490	Spring washer M6 stainless	4	PCS	5707292411297
22	1100500	Spring washer 8mm, stainless	4	PCS	5707292140142
23	0661200	Button head screw 8*12 stainl.	4	PCS	5707292139542
24	0659200	Button head screw 6*12 stainl.	4	PCS	5707292139436
25	0607990	Insex screw 12*30	4	PCS	5707292138446
26	1101600	Center spring washer 12mm	4	PCS	5707292140197
27	3006975	125mm directional/totallock	4	PCS	5707292543387
29	75761	Cover for frame	1	PCS	5707292485113
30	0506504	Rivet tas d 56 bs 4,1x8,5	8	PCS	5707292395818
31	102192-35	Height Adj. column left,painte	1	PCS	5707292632227
32	0603750	Insex M8*75, FZB	2	PCS	5707292138309
33	75788	Guide bush, frame	2	PCS	5707292545930
34	75764	Top bush	2	PCS	5707292545749
35	102186	H-Frame, For Strongbase	1	PCS	5707292558466
36	101098	Gearbox top, strongbase	2	PCS	5707292553522
47	2501406	Nut protection, black, f/M8	2	PCS	5707292375919
28+37+38	103234	Wheel mount	4	PCS	5707292565877

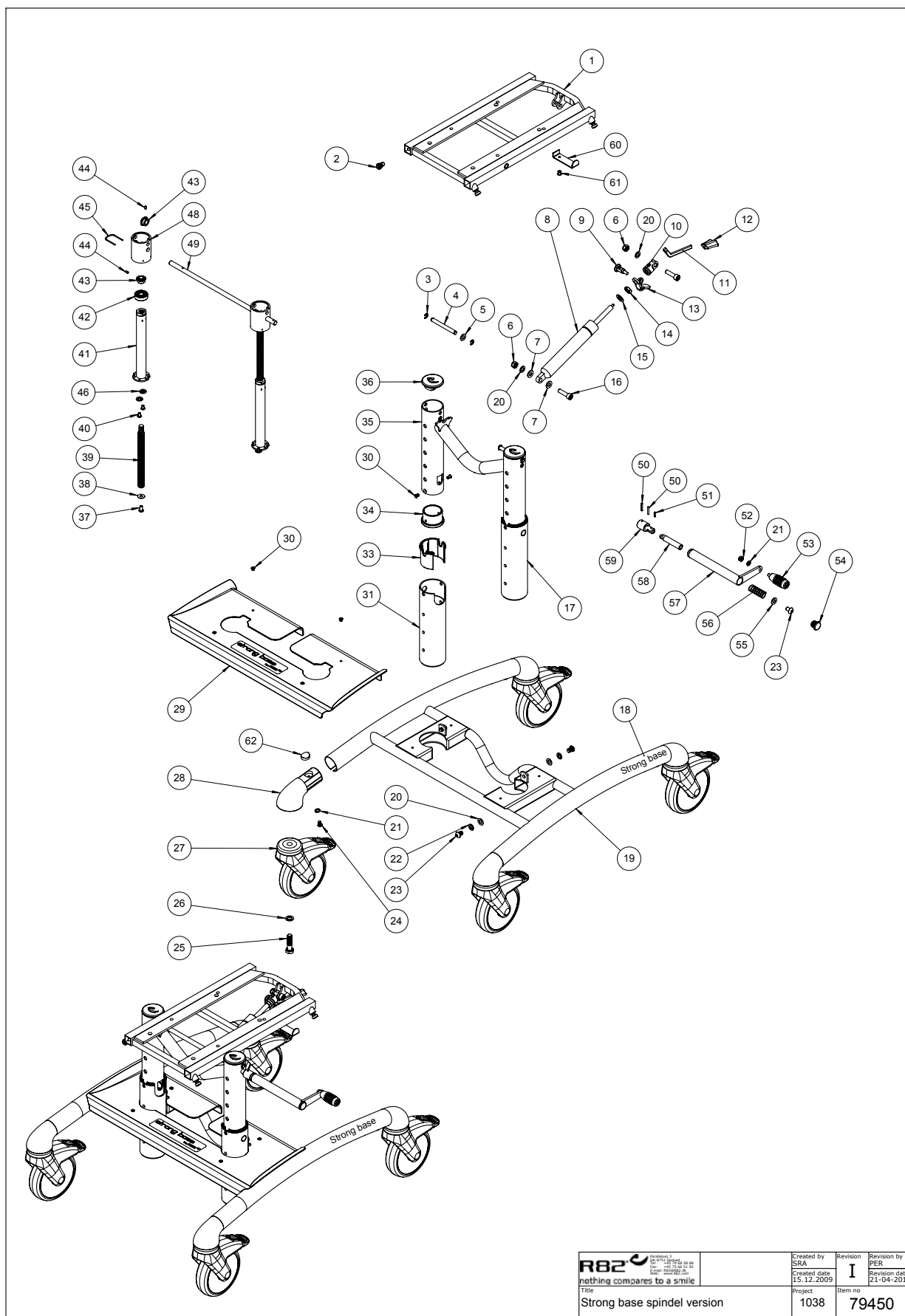

R82 <small>nothing compares to a smile</small>	<small>Revision 1</small> <small>Created by SRA</small> <small>Created date 15.12.2009</small>	<small>Revision</small> I	<small>Revision by PER</small> <small>Revision date 21-04-2015</small>
	<small>Title</small> Strong base spindel version	<small>Project</small> 1038	<small>Item no</small> 79450

Position	No.	Description	Quantity	Unit of Measure	Ean code
1	75776-35	Seat frame, painted	1	STK	5707292632265
2	0603530	Insex low head M8x16 stainless	4	STK	5707292412492
3	1145060	Locking washer, axle stainless	4	STK	5707292375056
4	75784	Axle for seatframe	2	STK	5707292545923
5	1310645	Washer, nature Ø16*8*0,5mm	2	STK	5707292446015
6	1607000	Lock nut 8mm, large pack	2	STK	5707292141378
7	1310717	Nylon washer, nat o17-8,4*2mm	2	STK	5707292140821
16	0615010	Insex M8*30 fzb, shortened	2	STK	5707292367990
17	102200-35	Height Adj. column righ,painte	1	STK	5707292632234
18	0493210	Strongbase label silver	2	STK	5707292553775
19	75760-35	Frame, strongbase, painted	1	STK	5707292632258
20	1105080	STAINLESS WASHER 8MM	6	STK	5707292140364
21	1100490	Spring washer M6 stainless	5	STK	5707292411297
22	1100500	Spring washer 8mm, stainless	4	STK	5707292140142
23	0661200	Button head screw 8*12 stainl.	5	STK	5707292139542
24	0659200	Button head screw 6*12 stainl.	4	STK	5707292139436
25	0607990	Insex screw 12*30	4	STK	5707292138446
26	1101600	Center spring washer 12mm	4	STK	5707292140197
27	3006975	125mm directional/totallock	4	STK	5707292543387
29	75761	Cover for frame	1	STK	5707292485113
30	0506504	Rivet tas d 56 bs 4,1x8,5	8	STK	5707292395818
31	102192-35	Height Adj. column left,painte	1	STK	5707292632227
33	75788	Guide bush, frame	2	STK	5707292545930
34	75764	Top bush	2	STK	5707292545749
35	102186	H-Frame, For Strongbase	1	STK	5707292558466
36	101098	Gearbox top, strongbase	2	STK	5707292553522
37	0641200	Button head screw 6*12	2	STK	5707292139252
38	1101950	Washer Ø6,1-16*1 mm, FZB	2	STK	5707292140210
39	100491	Spindel TR 14*4	2	STK	5707292553485
40	0640910	Button head M5*10 FZB	4	STK	5707292380142
41	101162	Spindle housing complete	2	STK	5707292553768
42	2985213	Ballbearing 6200-2RS	2	STK	5707292553461
43	101093	2020-1 Sprocket f. drive shaft	4	STK	5707292553645
44	0690805	Stopscrew w.peak M4*8,stainl.	4	STK	5707292554994
45	101152	Clips for bearing	2	STK	5707292553928
46	1100485	Spring washer M5, stainless	2	STK	5707292478603
48	101097	Gearbox, strongbase	2	STK	5707292553652
49	100470	Ø8 Drive axel	1	STK	5707292553669
50	0532220	Tighten wedge 3*20 stainless	2	STK	5707292555007
51	0532225	Tighten wedge 3*10 stainl.	1	STK	5707292556028
52	1606500	Lock nut 6 mm	1	STK	5707292141361
53	3515010	Handle foldable	1	STK	5707292554987
54	2502510	Plug 20*2,0-2,5 inside black	1	STK	5707292557759
55	1310610	Nylon washer din 125a, M8	1	STK	5707292374684
56	101108	Pressure spring for cranck	1	STK	5707292553751
57	101100	Krank body	1	STK	5707292553713
58	101101	Adapter joint for krank	1	STK	5707292553690
59	100474	Adapter joint for drive shaft	1	STK	5707292553676
60	102444-35	Handle Hook	1	STK	5707292632241

61	0506504	Rivet tas d 56 bs 4,1x8,5	1	STK	5707292395818
28+62+63	103234	Wheel mount	4	STK	5707292565877
MODULE	5040065	Gas spring 800N, mount Strongb	1	STK	5707292558688

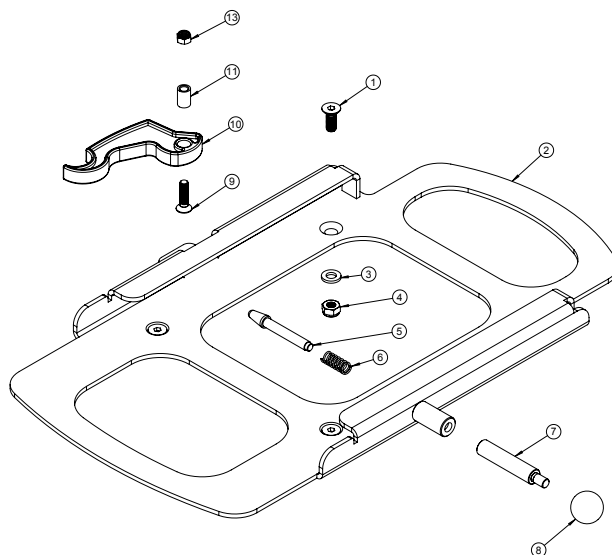

 nothing compares to a smile!	<small>Headoffice: 1 24-25 Fife Lane Unit 10 +44 774 64 64 66 Fax: +44 774 64 64 67 Email: info@r82.co.uk www.r82.co.uk</small>	Created by SRA	Revision B	Revision by sra
		Created date 24.01.2011		Revision date 19-05-20
Title		Project	Item no	
T-Frame Strongbase		1038	79453	

Position	No.	Description	Quantity	Unit of Measure	Ean code
1	0609728	Insex, stainless 6*16 uh	4	STK	5707292138491
2	75774-01	T-Frame	1	STK	5707292540249
3	1105060	Stainless washer 6mm	4	STK	5707292140357
4	1631060	Stainless lock nut 6mm	4	STK	5707292141446
5	6113000	Spindle, t-part/frame	1	STK	5707292186362
6	0310100	Pressure spring 0,9*8*23*10,5v	1	STK	5707292135513
7	6118750	Handle x:panda	1	STK	5707292530509
8	3502008	Knob handle Ø20 M6	1	STK	5707292146830
9	0615020	Insex screw counters. 5*20	1	STK	5707292138866
10	75905	Clamping grip	1	STK	5707292534590
11	75641	Bush 5-8*11,8	1	STK	5707292466501
13	1606300	Lock nut 5 mm	1	STK	5707292141354