

wombat living accessories

Exploded views / Part lists


Rev. date (09.2017)

On the following pages you will find the parts used to assemble this product. Please note that some spare parts cannot be purchased individually but are part of a complete kit. Please also note that for some spare parts extra delivery time may occur.




Contents

Pommel	3
Knee abduction pommel	4
Armrests size 1+2.....	5
Armrests size 3	7
Tray for Wombat.....	9
Tray	10
Hip supports	11
Handbrace seat mounted Size 1.....	12
Handbrace seat mounted Size 2.....	13
Handbrace back mounted.....	14
Back Extension Wombat Size 3.....	15
Lumba Support Size 1-2.....	16
Lumba Support Size 3	17
Lumba Support with sides Size 1	18
Lumba Support with sides Size 2	19
Lumba Support with sides Size 3	20
Central Brake	21
Anti-tip.....	22

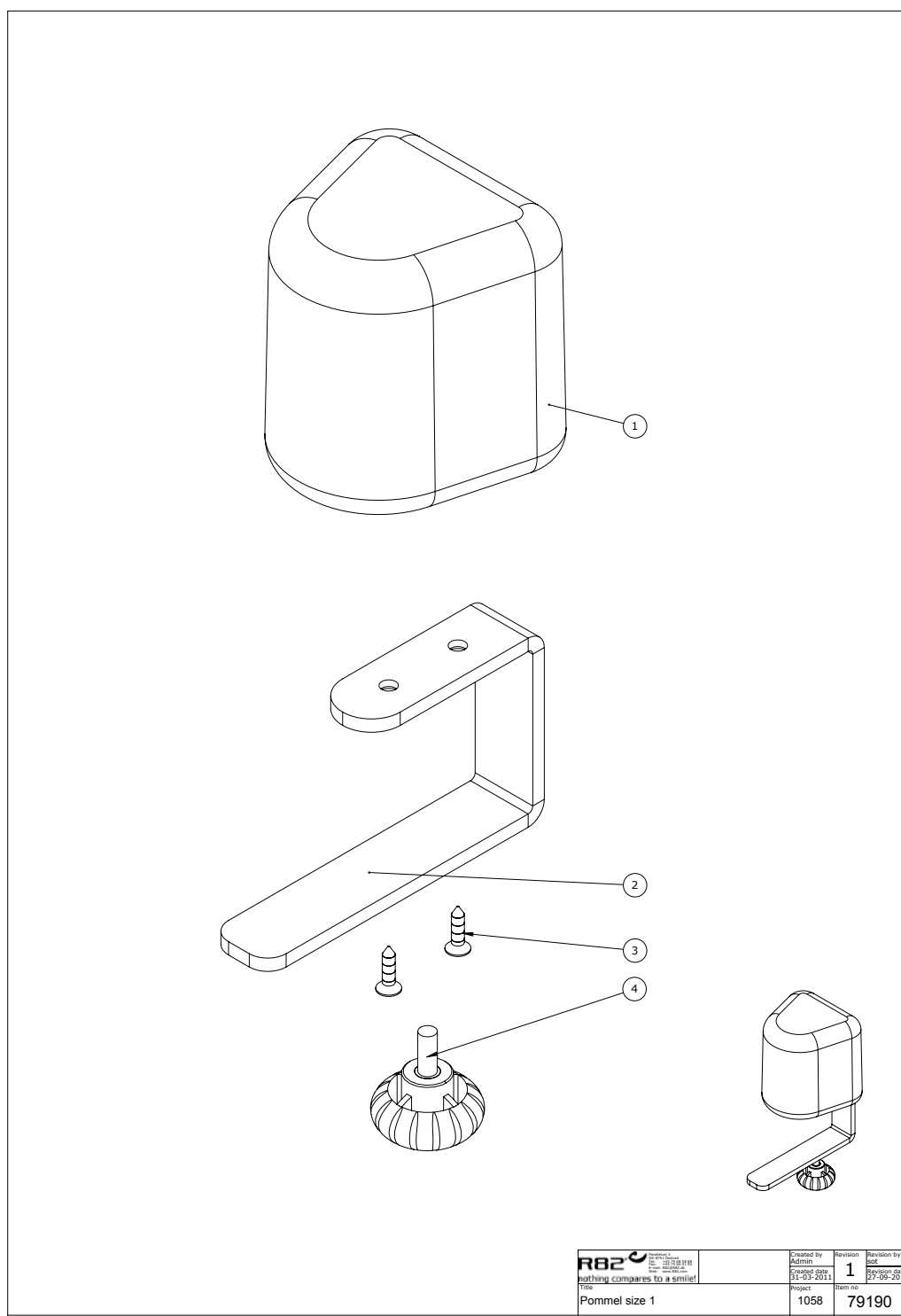


Position	No.	Description	Quantity	Unit of Measure	Ean code
1	6175111	Pommel size 1, black pur	PCS	1	5707292189103
1	6175112	Pommel size 2, black pur	PCS	1	5707292189110
1	6175114	Pommel size 4, black pur	PCS	1	5707292189134
1	6175113	Pommel sz. 3, black pur	PCS	1	5707292189127
2	82978	Pommel brace Wombat	PCS	1	5707292126221
3	1811275	Cutting screw 4*30mm	PCS	2	5707292141743
4	3652501	R82 handle M6*16mm screw	PCS	1	5707292413963

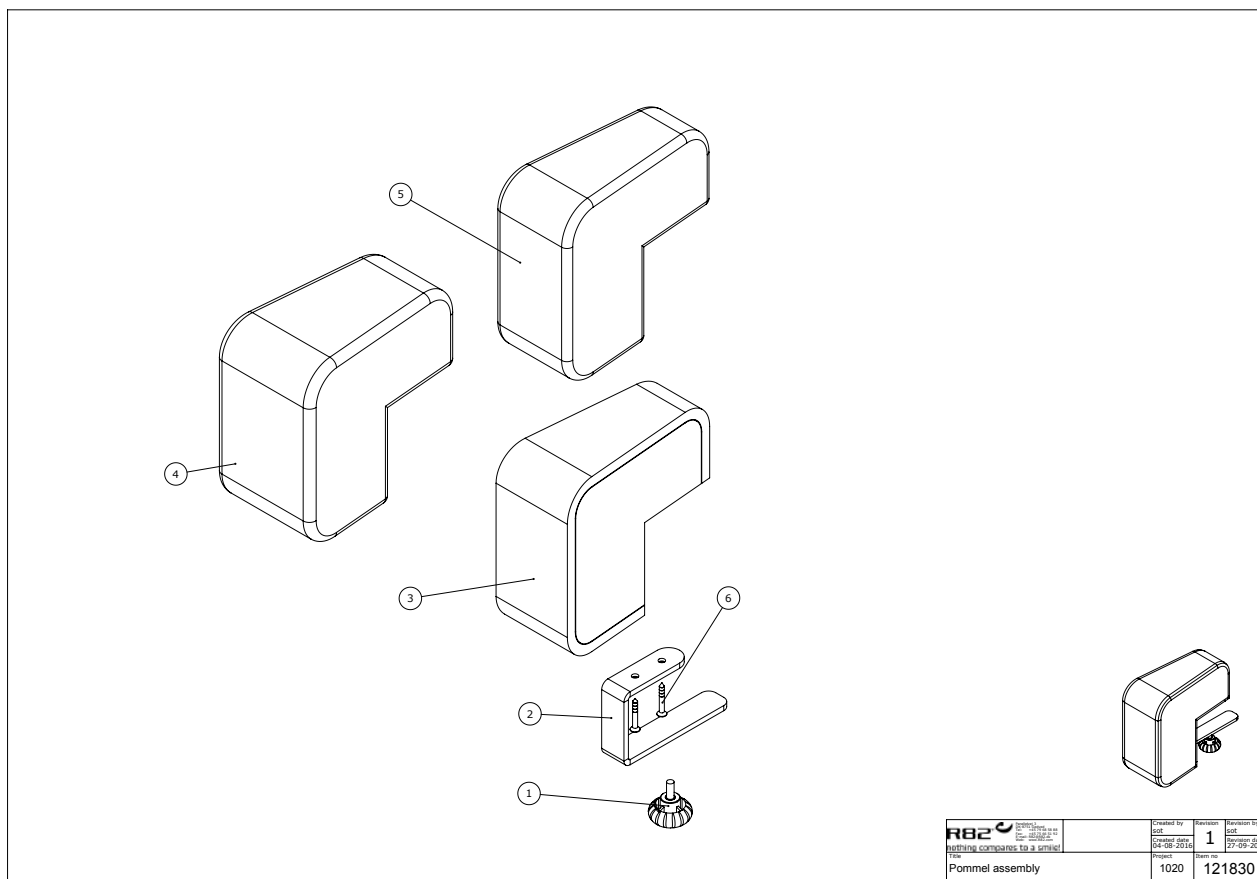


R82 <small> R82 A/S Parallelvej 3 8751 Gedved Denmark </small>	<small> R82 A/S R82 A/S R82 A/S </small>		<small> Created by JOD </small>	<small> Version 1 </small>	<small> Revision by JOD </small>
	<small> R82 A/S R82 A/S R82 A/S </small>		<small> Project 1020 </small>	<small> Date 27-09-2016 </small>	<small> Date 27-09-2016 </small>


Position	No.	Description	Quantity	Unit of Measure	Ean code
1	3652501	R82 handle M6*16mm screw	PCS	1	5707292413963
2	82978	Pommel brace Wombat	PCS	1	5707292126221
3	113633	Pommel with foam	PCS	1	5707292600882
4	95430-10160	Cover f. knee abduction pommel	PCS	1	5707292600646
6	1811275	Cutting screw 4*30mm	PCS	2	5707292141743

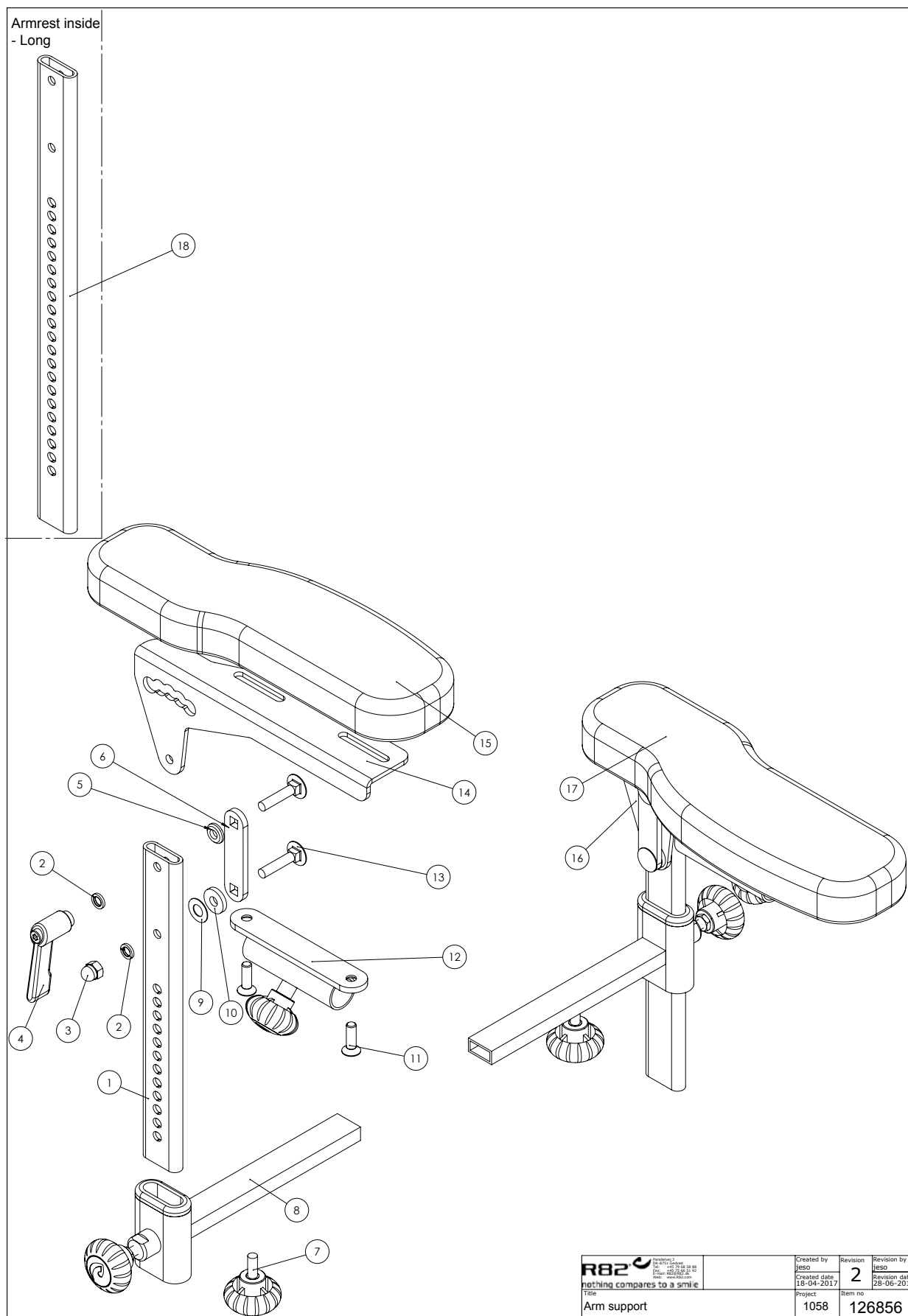


Position	No.	Description	Quantity	Unit of Measure	Ean code
1	82963-1	Innerprofil armrest Wombat Liv	PCS	2	5707292546890
2	1100900	Spring washer M6 fzb, 6.1/9.9	PCS	4	5707292459336
3	1780600	Top nut 6mm	PCS	2	5707292141552
4	3502200	Tightening handle 42mm M6 bush	PCS	2	5707292146861
5	128514	Lock washer Ø6	PCS	2	5707292795687
6	128508	Bracket for armrest	PCS	2	5707292795670
7	3652501	R82 handle M6*16mm screw	PCS	2	5707292413963
8	5082960	Hardware holder armrest	PCS	2	5707292511157
9	1310645	Washer, nature Ø16*8*0,5mm	PCS	2	5707292446015
10	1125618	Screenwasher fz6*18*3mm	PCS	2	5707292140494
11	0611000	Insex screw counters. 6*20	PCS	4	5707292138682
12	5082975	Hardware for table, mounted	PCS	2	5707292511164
13	1905300	Board bolt 6*25mm stainless	PCS	4	5707292142252
14	128509	Armrest right	PCS	1	5707292795663
15	6175154	Pur armrest size 2&3, right	PCS	1	5707292189240
16	128533	Armrest left	PCS	1	5707292795656
17	6175156	Pur armrest size 2&3, left	PCS	1	5707292189257
18	82666-1	Armrest, wombat, extended	PCS	2	5707292546906

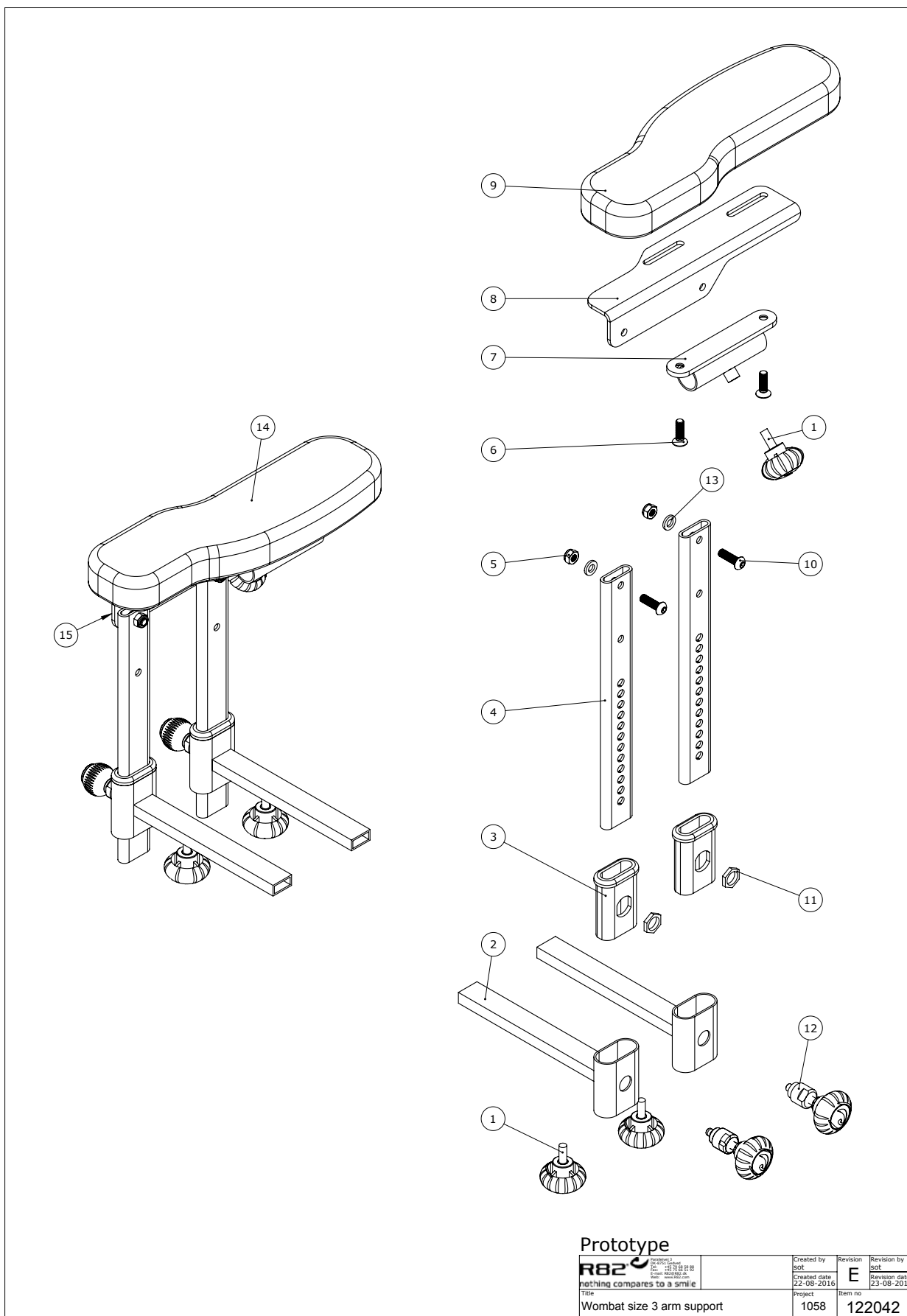


Position	No.	Description	Quantity	Unit of Measure	Ean code
1	3652501	R82 handle M6*16mm screw	PCS	4	5707292413963
2-3-11-12	5082960	Hardware holder armrest	PCS	4	5707292511157
4	82666-1	Armrest, wombat, extended	PCS	4	5707292546906
4	82963-1	Armlæn inv.wombat	PCS	4	5707292546890
5	1606500	Lock nut 6 mm	PCS	4	5707292141361
6	0611000	Insex screw counters. 6*20	PCS	4	5707292138682
1+7	5082975	Hardware for table, mounted	PCS	2	5707292511164
8	82718	Armrest bracket left size 3	PCS	1	5707292442727
8	82693	Bracket left	PCS	1	5707292401762
9	6175163	Armrest pu. Wombat sz 4. Right	PCS	1	5707292443151
9	6175156	Pur armrest size 2&3, left	PCS	1	5707292189257
10	0641620	Button head screw 6*20	PCS	4	5707292139276
13	1310640	Washer, nature Ø16*10*0,5mm	PCS	4	5707292140760
14	6175164	Armrest pu. Wombat sz 4. Left	PCS	1	5707292443168
14	6175154	Pur armrest size 2&3, right	PCS	1	5707292189240
15	82705	Armrest right size 3	PCS	1	5707292442734
15	82692	Bracket right	PCS	1	5707292401755

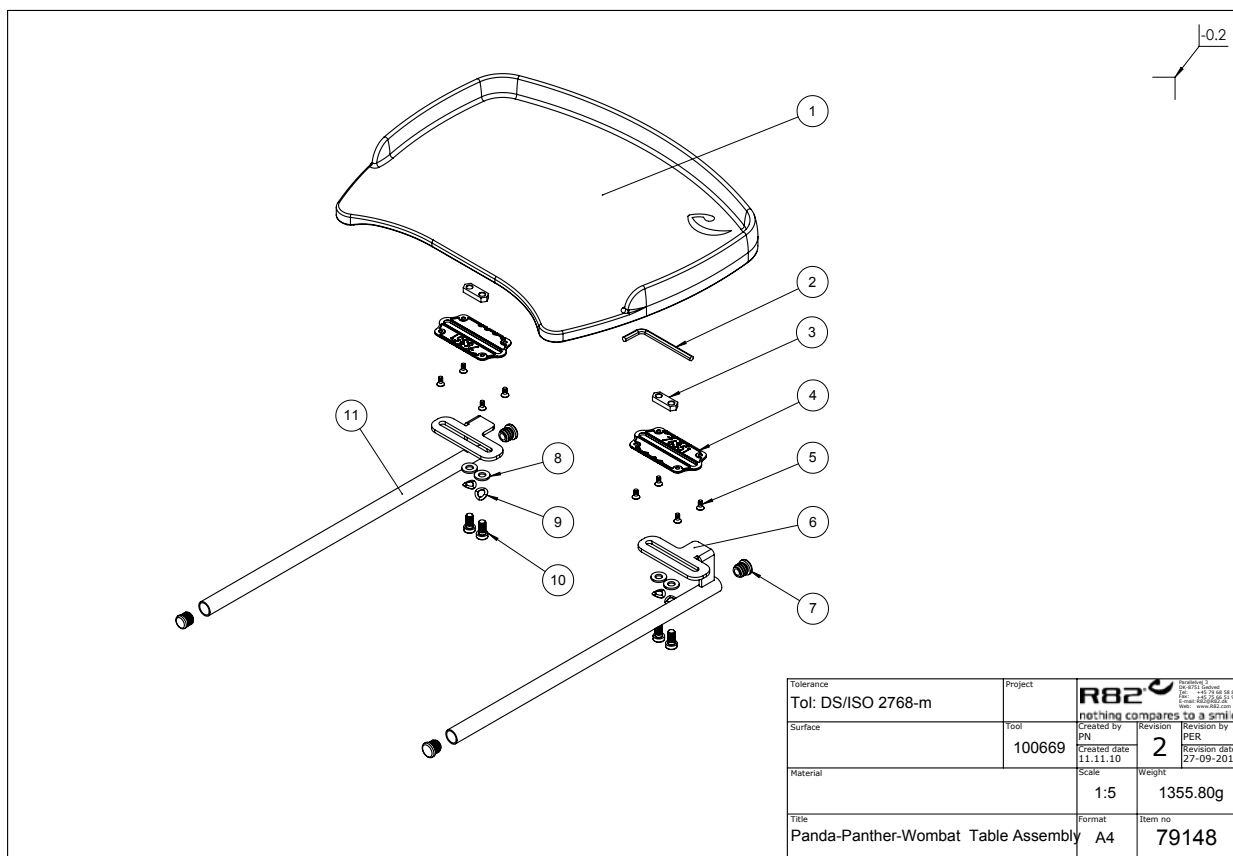


Position	No.	Description	Quantity	Unit of Measure	Ean code
1	76350	Table for Stingray, Grey	PCS	1	5707292541727
2	0589500	Insex keys n5	PCS	1	5707292137869
3	77557	Slide bar (M8x2)	PCS	2	5707292544995
4	76351	Mounting plate	PCS	2	5707292541741
5	1940011	Plastite 60, 4*8	PCS	8	5707292545978
6+7+11	5075364	Brackets for table sz 4	SÆT	1	5707292543608
8	1105080	Stainless washer 8mm	PCS	4	5707292140364
9	1100500	Spring washer 8mm, stainless	PCS	4	5707292140142
10	0661500	Button head screw 8*16 stainl.	PCS	4	5707292139559

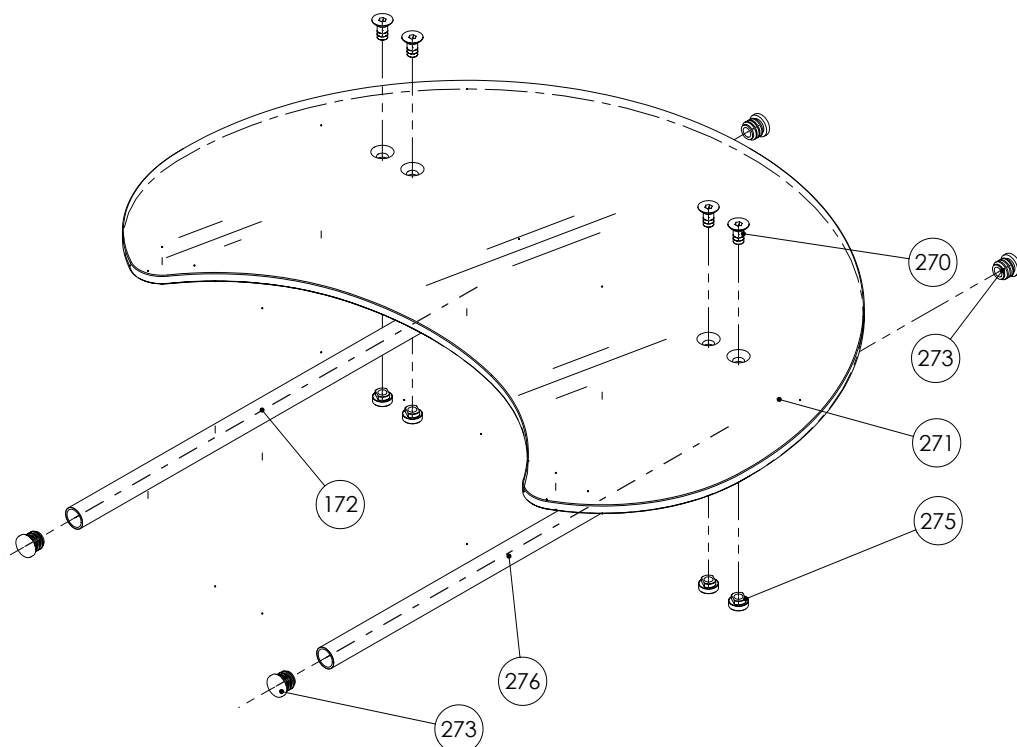


Position	No.	Description	Quantity	Unit of Measure	Ean code
172	82865	Mounting for tray, welded	PCS	1	5707292125590
270	0624800	Insex screw counters. 8*16	PCS	4	5707292139054
271	82998	Tray plate, 8 mm acryl (Size 1+2)	PCS	1	5707292126405
271	82695	Tray plate, 8 mm acryl (Size 3)	PCS	1	5707292402172
273	2502002	Ferrule d18*0,8-2,5 black	PCS	4	5707292433947
275	6118930	Nut M8	PCS	4	5707292353825
276	82864	Mounting for tray, welded	PCS	1	5707292125583

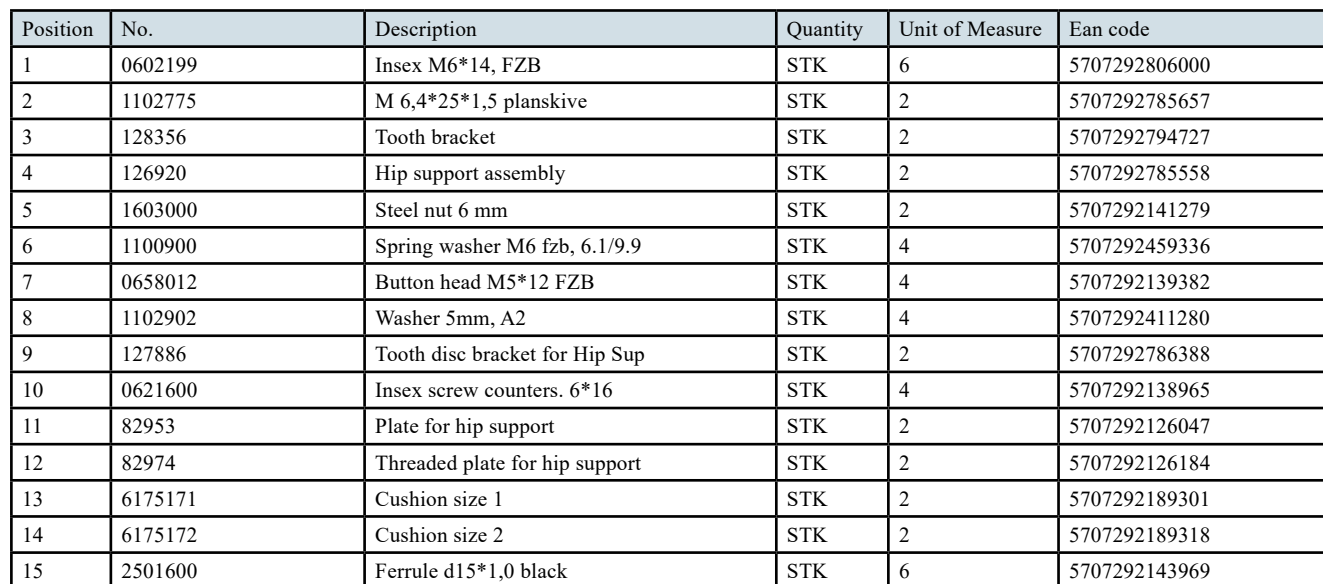


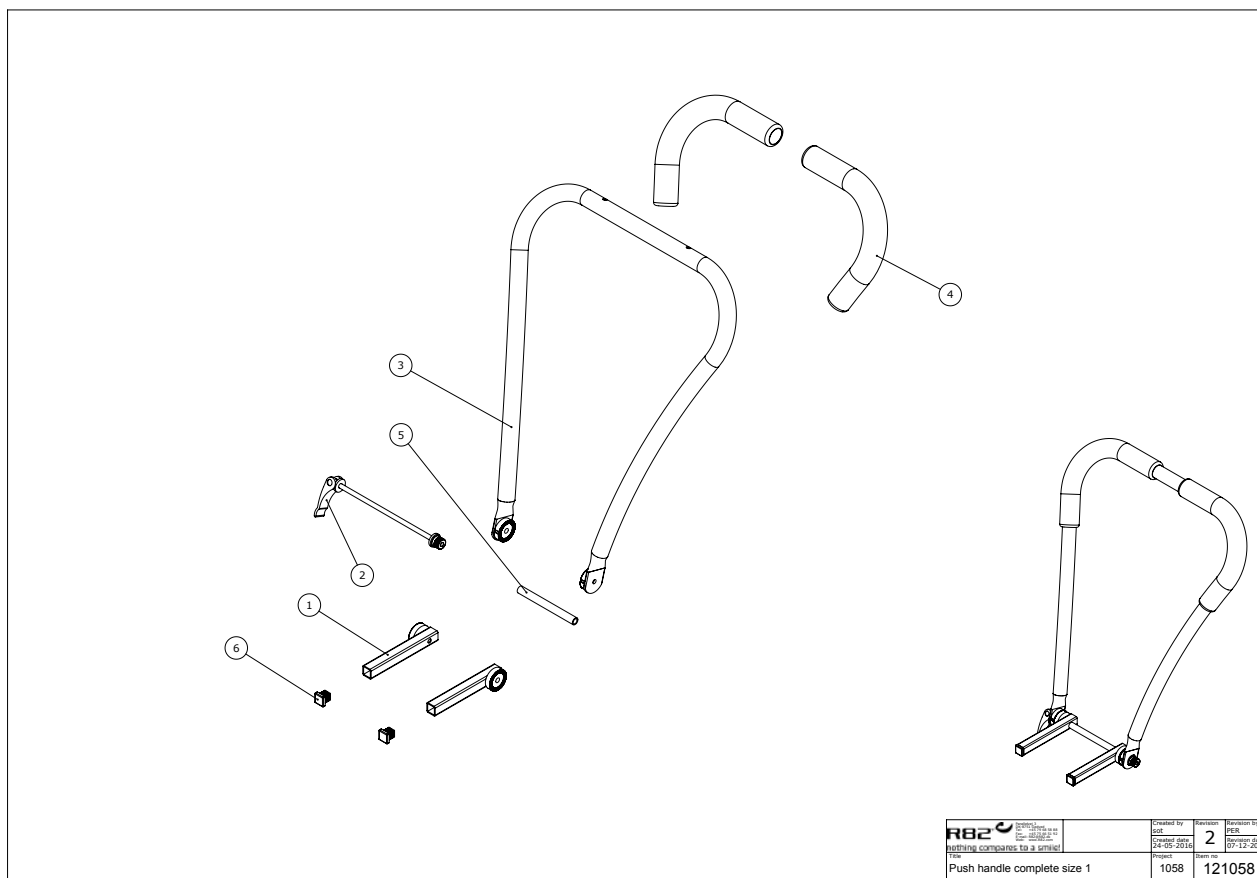


Position	No.	Description	Quantity	Unit of Measure	Ean code
1	120409	Bracket for handbrace size 1	PCS	2	5707292621641
2	121061	Handle size 1 angle adjustment	PCS	1	5707292622686
3	119439	Handbrace size 1 Wombat	PCS	1	5707292622112
4	2500520	Tpe cover Ø20mm	PCS	2	5707292401649
5	121080	Ø10 tube	PCS	1	5707292623317

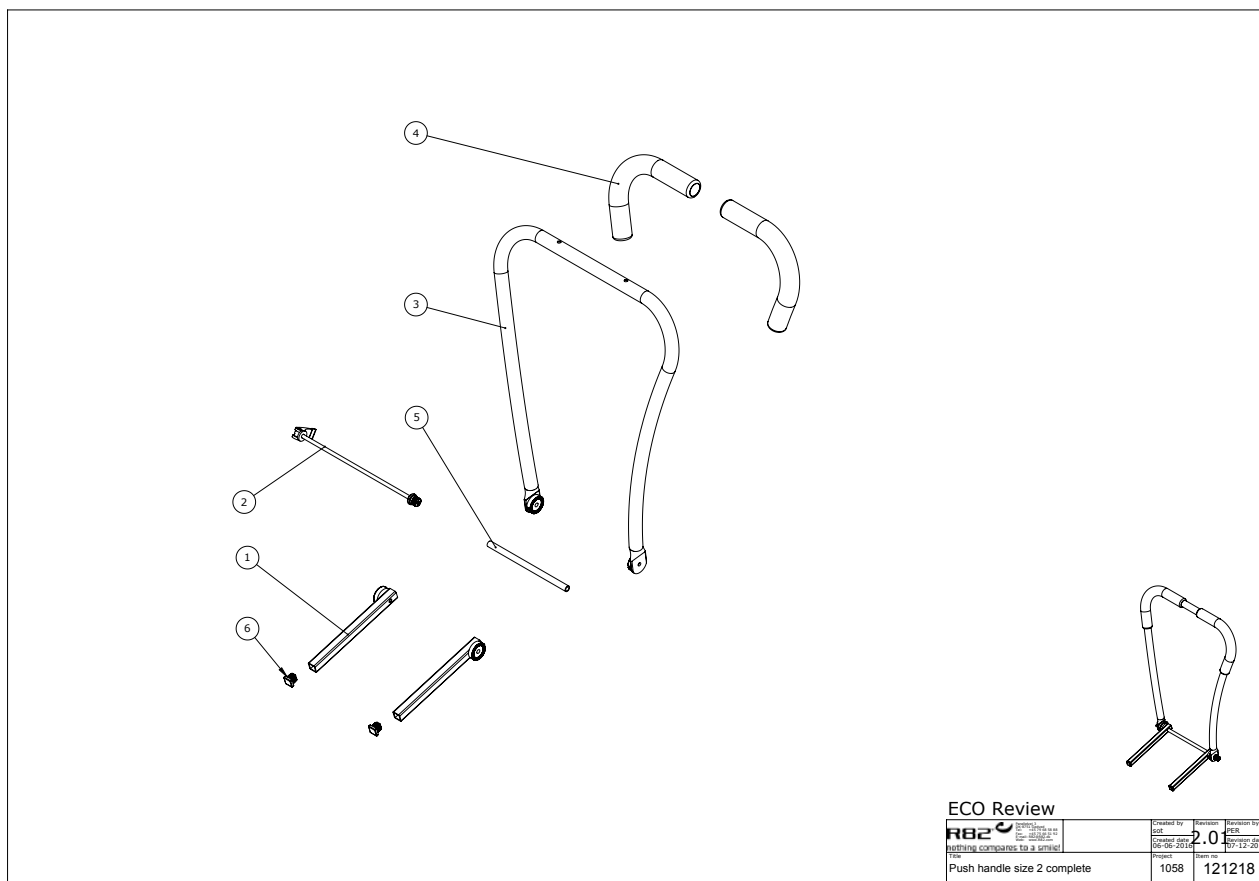


Position	No.	Description	Quantity	Unit of Measure	Ean code
1	120410	Bracket for handbrace size 2	PCS	2	5707292621931
2	121070	Handle size 2 angle adjustment	PCS	1	5707292622679
3	120635	Handbrace size 2 Wombat svejst	PCS	1	5707292622181
4	2500520	Tpe cover Ø20mm	PCS	2	5707292401649
5	121219	Ø10 tube 175mm	PCS	1	5707292623386
6	2504000	Ferrule e15*1,0-2,0 int.black	PCS	2	5707292144102

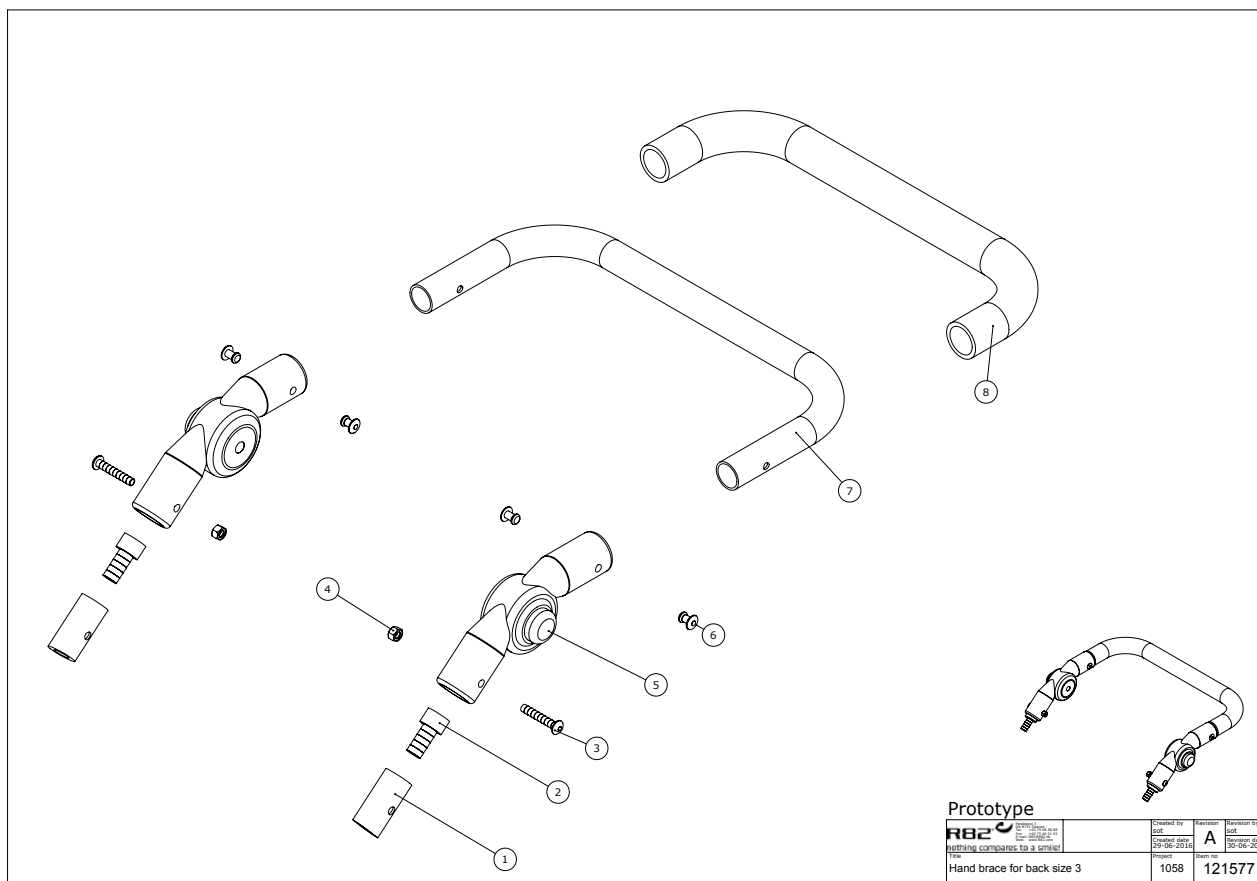


Position	No.	Description	Quantity	Unit of Measure	Ean code
1	80705	Bush for handbrace	PCS	2	5707292436900
2	0604000	Insex screw 10*20	PCS	2	5707292138361
3	0640935	Button head M5*35 rf	PCS	2	5707292485748
4	1606300	Lock nut 5 mm	PCS	2	5707292141354
5	0553090	LOCKING JOINT Ø20MM	PCS	2	5707292482235
6	0506508	Rivet 5*14, stainless	PCS	4	5707292359476
7	80706	Tube for handbrace	PCS	1	5707292436917
8	2500550	Grip 700mm length for Ø20mm	PCS	1	5707292388520

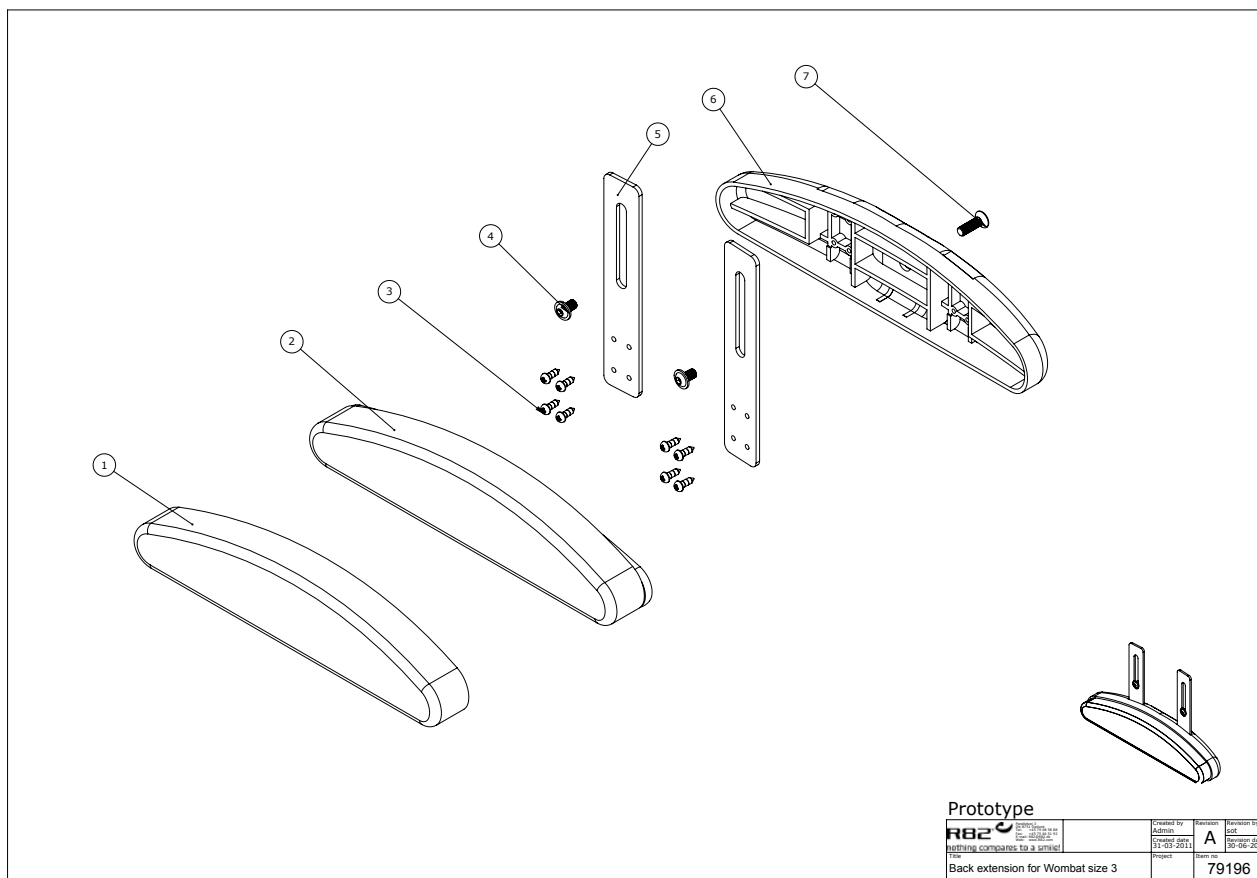

Position	No.	Description	Quantity	Unit of Measure	Ean code
1	121582	Upholstery Lumbar support Wombat3	PCS	1	5707292630032
2	5700483	Upholstered lower back supp	PCS	1	5707292399304
3	1940033	ECO-SYN Z M3,5*10 RUSTFRI	PCS	8	5707292422965
4	0659010	Button head with flange M6*10	PCS	2	5707292356819
5	82744	MOUNTINGPLATE	PCS	2	5707292398901
6	82748	LUMBAR SUPPORT	PCS	1	5707292400093
7	0612500	Insex screw counters. 6*25	PCS	1	5707292138712

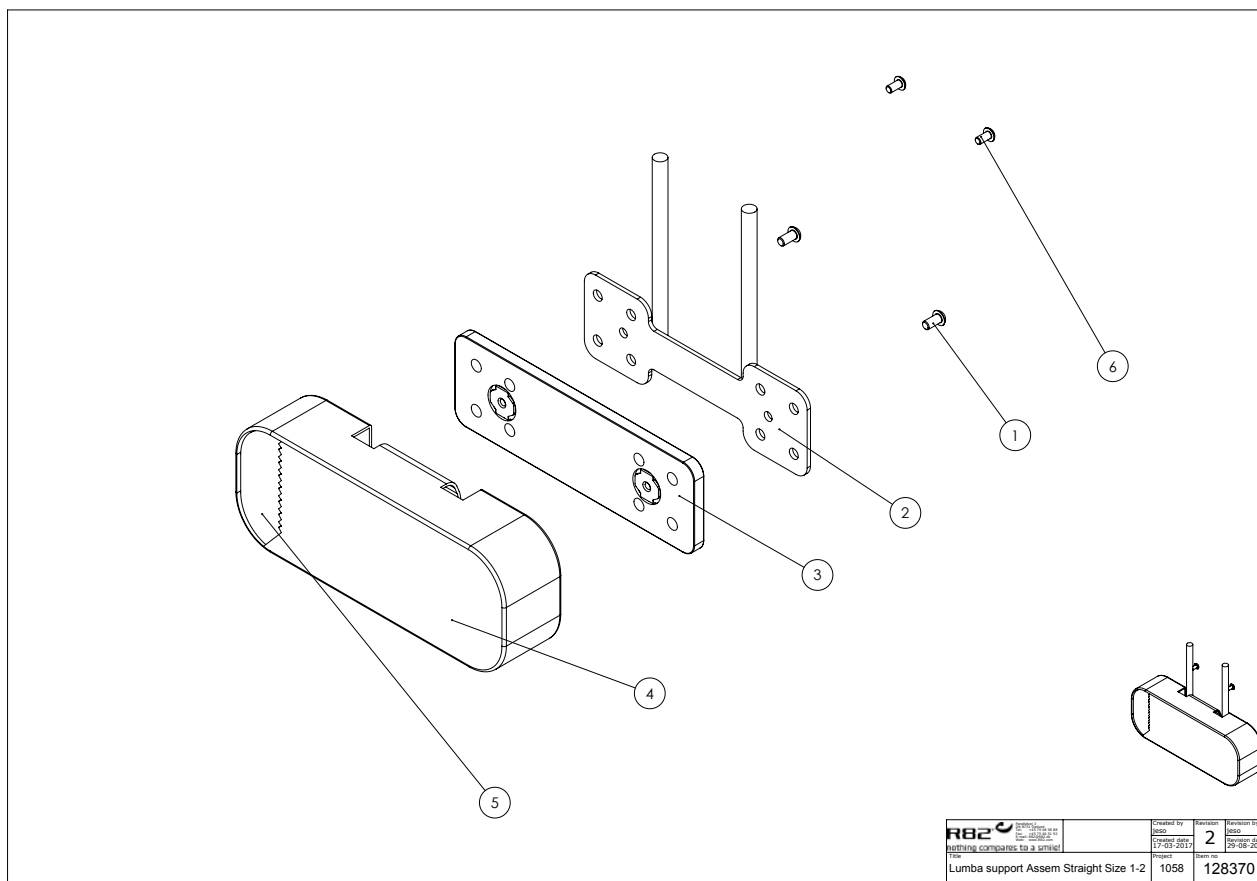


Position	No.	Description	Quantity	Unit of Measure	Ean code
1	0641200	Button head screw 6*12	PCS	2	5707292139252
2	126965	Lumbar support beslag	PCS	1	5707292785589
3	127681	Plate w yelo nut low back supp	PCS	1	5707292786975
4	128366	Cover for sacrum Support zs1+2	PCS	1	5707292806468
5	128372-1	Foam straight size 1-2	PCS	1	5707292807267
6	0640910	Button head M5*10 FZB	PCS	2	5707292380142

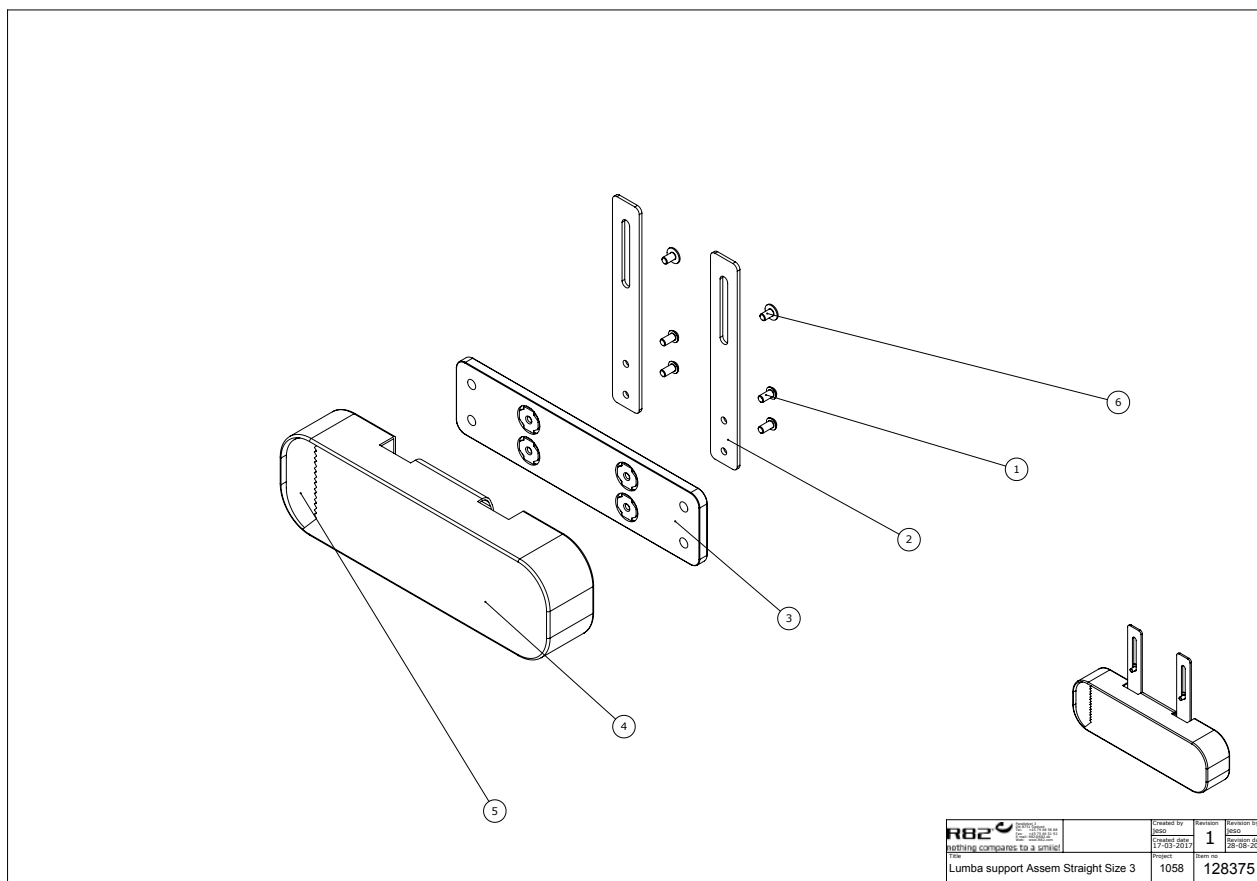


 nothing compares to a smile!	Revised by jeso	Revision 1	Revision date 28-08-2017
	Created by jeso	Created date 17-03-2017	
	File Lumba support Assem Straight Size 3	Project 1058	Item no 128375


Position	No.	Description	Quantity	Unit of Measure	Ean code
1	0641200	Button head screw 6*12	PCS	4	5707292139252
2	127608	Plate for lumbar support	PCS	2	5707292787019
3	127677	Plate w yelo not low back supp	PCS	1	5707292786968
4	128377	Cover for sacrum Support size3	PCS	1	5707292807311
5	128376-1	Foam straight size 3	PCS	1	5707292807274
6	0659010	Button head with flange M6*10	PCS	2	5707292356819

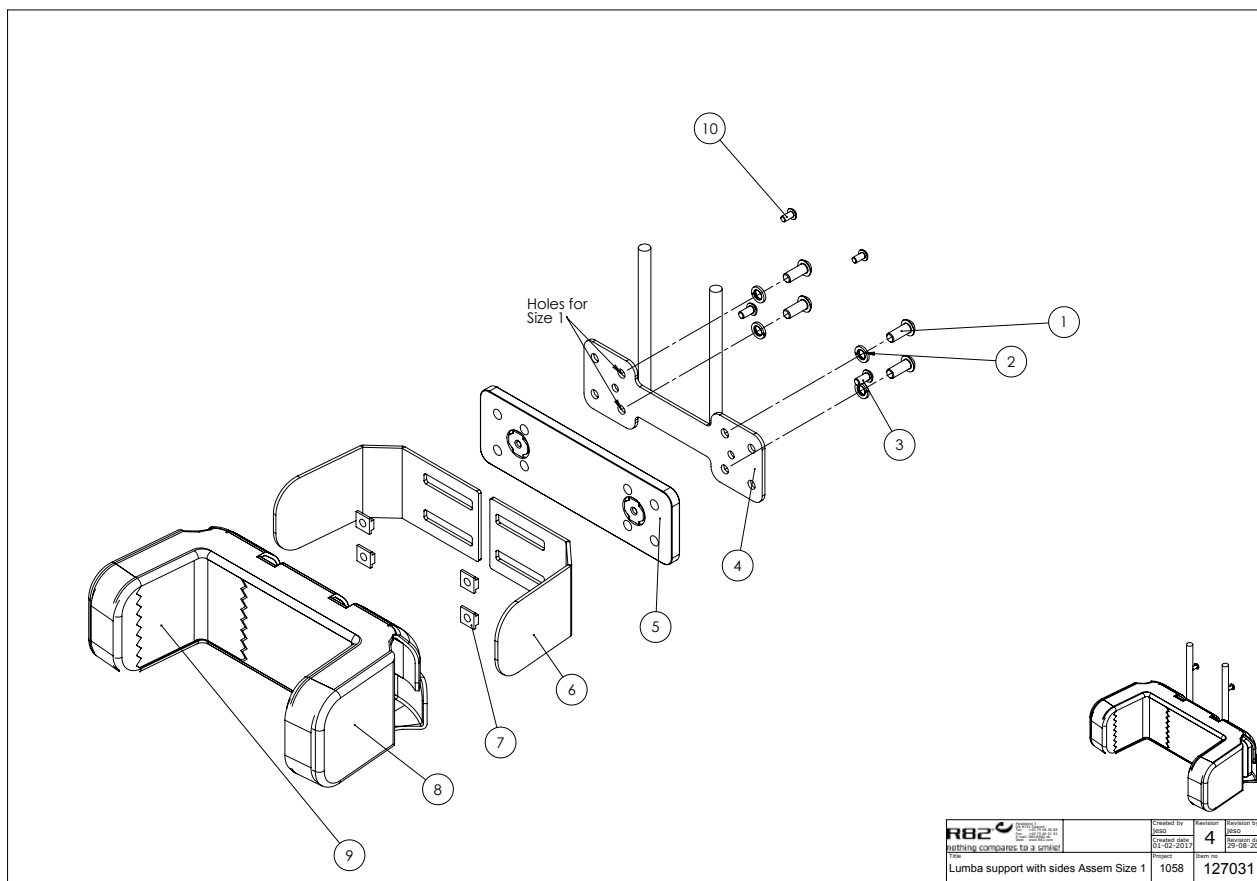


Position	No.	Description	Quantity	Unit of Measure	Ean code
1	0662000	Button head screw 8*20 stainl.	PCS	4	5707292139573
2	1100500	Spring washer 8mm, stainless	PCS	4	5707292140142
3	0641200	Button head screw 6*12	PCS	2	5707292139252
4	126965	Lumbar support beslag	PCS	1	5707292785589
5	127681	Plate w yelo nut low back supp	PCS	1	5707292786975
6	126967	lumbar back support left	PCS	2	5707292782502
7	75203-1	M8 slide with threads	PCS	4	5707292517593
8	128368	Cover for sacrum Support size 1	PCS	1	5707292807328
9	128224-1	Foam with sides size 1	PCS	1	5707292807281
10	0640910	Button head M5*10 FZB	PCS	2	5707292380142

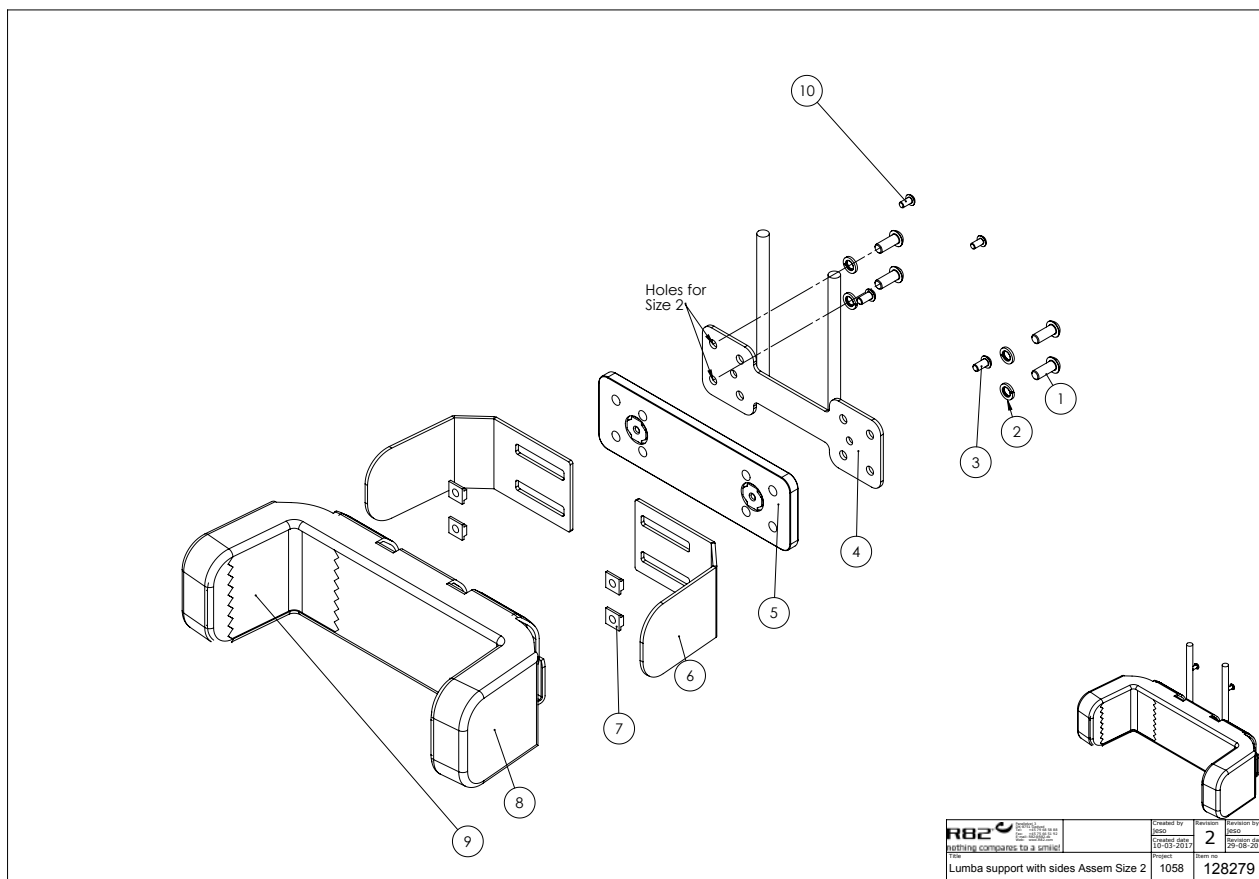


Position	No.	Description	Quantity	Unit of Measure	Ean code
1	0662000	Button head screw 8*20 stainl.	PCS	4	5707292139573
2	1100500	Spring washer 8mm, stainless	PCS	4	5707292140142
3	0641200	Button head screw 6*12	PCS	2	5707292139252
4	126965	Lumbar support beslag	PCS	1	5707292785589
5	127681	Plate w yelo nut low back supp	PCS	1	5707292786975
6	126967	lumbar back support left	PCS	2	5707292782502
7	75203-1	M8 slide with threads	PCS	4	5707292517593
8	128280	Cover for sacrum Support size2	PCS	1	5707292807335
9	128281-1	Foamwidth sides size 2	PCS	1	5707292807298
10	0640910	Button head M5*10 FZB	PCS	2	5707292380142

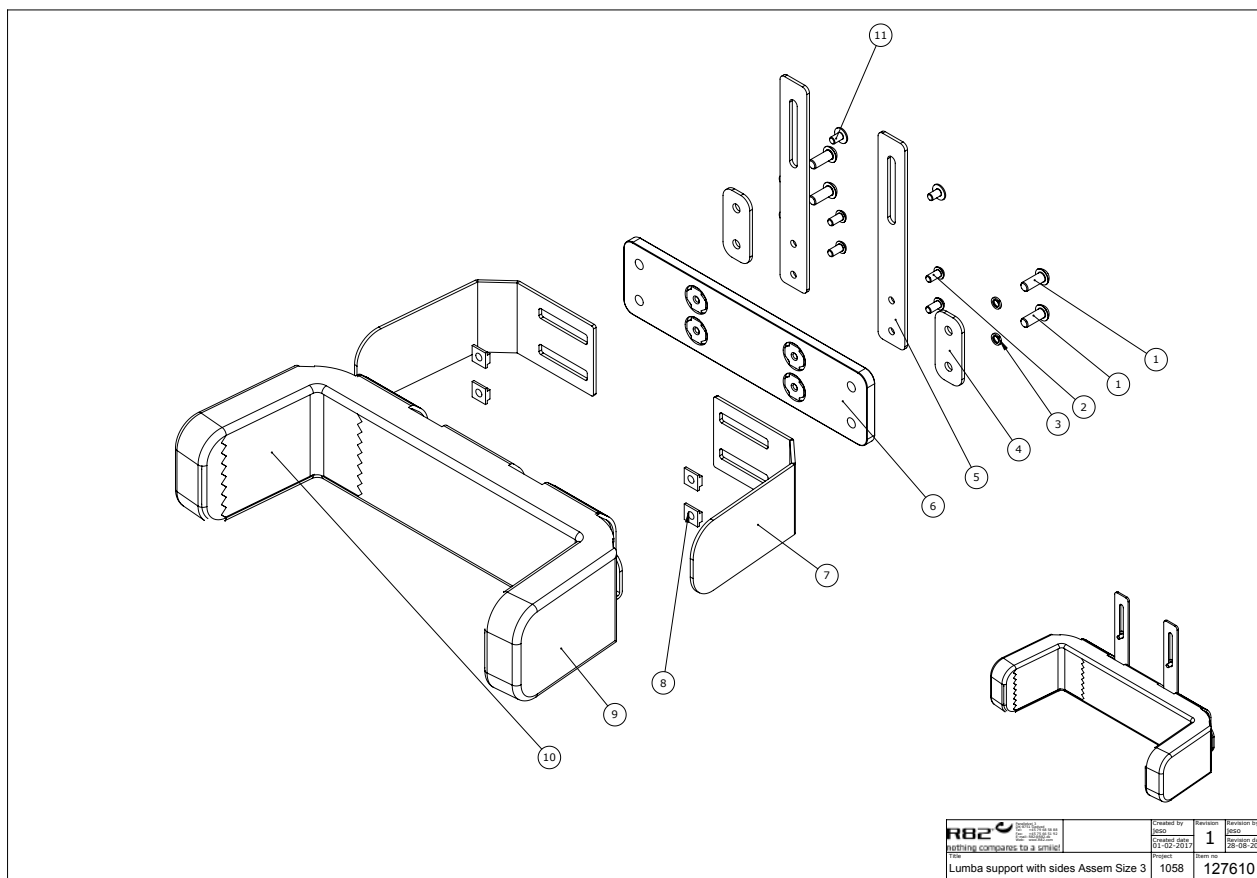

Position	No.	Description	Quantity	Unit of Measure	Ean code
1	0661500	Button head screw 8*16 stainl.	PCS	4	5707292139559
2	0641200	Button head screw 6*12	PCS	4	5707292139252
3	1100900	Spring washer M6 fzb, 6.1/9.9	PCS	4	5707292459336
4	127034	Plate for lumbar support	PCS	2	5707292782441
5	127608	Plate for lumbar support	PCS	2	5707292787019
6	127677	Plate w yelo not low back supp	PCS	1	5707292786968
7	127037	lower back support	PCS	2	5707292782489
8	75203-1	M8 slide with threads	PCS	4	5707292517593
9	128354	Cover for sacrum Support size3	PCS	1	5707292807342
10	128348-1	Foam with sides size 3	PCS	1	5707292807304
11	0659010	Button head with flange M6*10	PCS	2	5707292356819

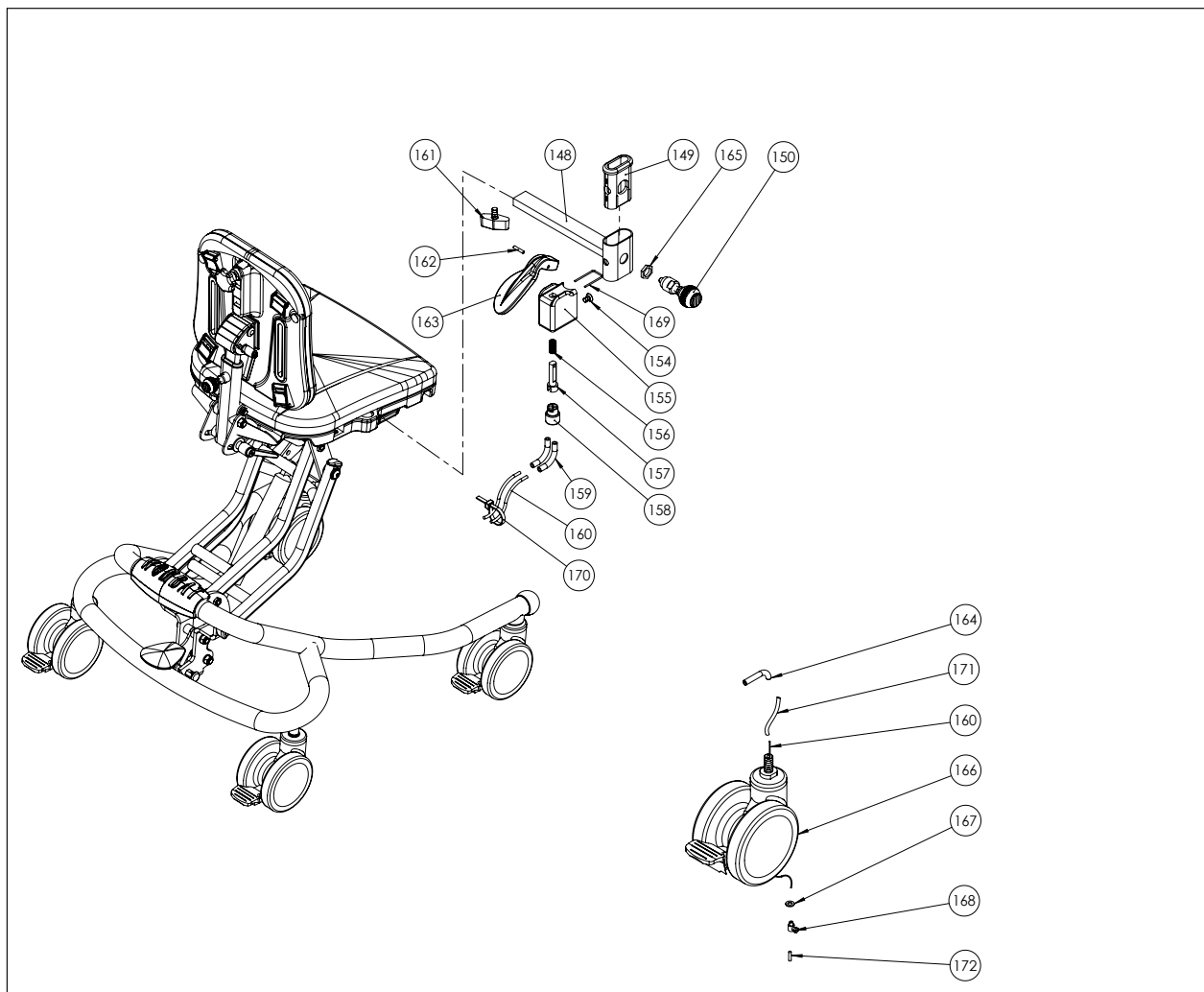

Position	No.	Description	Quantity	Unit of Measure	Ean code
154	0631809	Insex uh. M5*5	PCS	1	5707292353337
155	70140	Brakeholder, Rabbit	PCS	1	5707292442833
156	0310129	pressure spring 0,9*11*26mm*7v	PCS	1	5707292438782
157	72983	Pin for brake	PCS	1	5707292438836
158	70139	Plug	PCS	1	5707292101648
159	0389000	Angled cable tightener, 45°	PCS	2	5707292136152
160	0371850	Strained wire 1,5-19 tr.1	PCS	2	5707292135872
161	3652501	R82 handle M6*16mm screw	PCS	1	5707292413963
162	0529038	Cyl.stift Ø3,0x18 mm.	PCS	1	5707292439260
163	72964	Brake grip for Central brake	PCS	1	5707292499745
164	5082909	Cable guide	PCS	2	5707292503107
166	3006950	100 mm double wheel w/central	PCS	2	5707292145925
167	1102800	Fazet washer 4mm	PCS	2	5707292140265
168	0388010	Screwnippel 7*7mm	PCS	2	5707292136121
169	6108146	Safety brace Ø2	PCS	1	5707292185495
170	0235075	Cable tape, black	PCS	1	5707292135308
171	5002965	Cable for central brake size 3	PCS	2	5707292408846
171	5002964	Outer casing f. Centralbreak	PCS	2	5707292353689
172	0387000	End nipple for outer casing	PCS	2	5707292136091
Modul	5082870	Mounting bracket for armrest	PCS	1	5707292539618

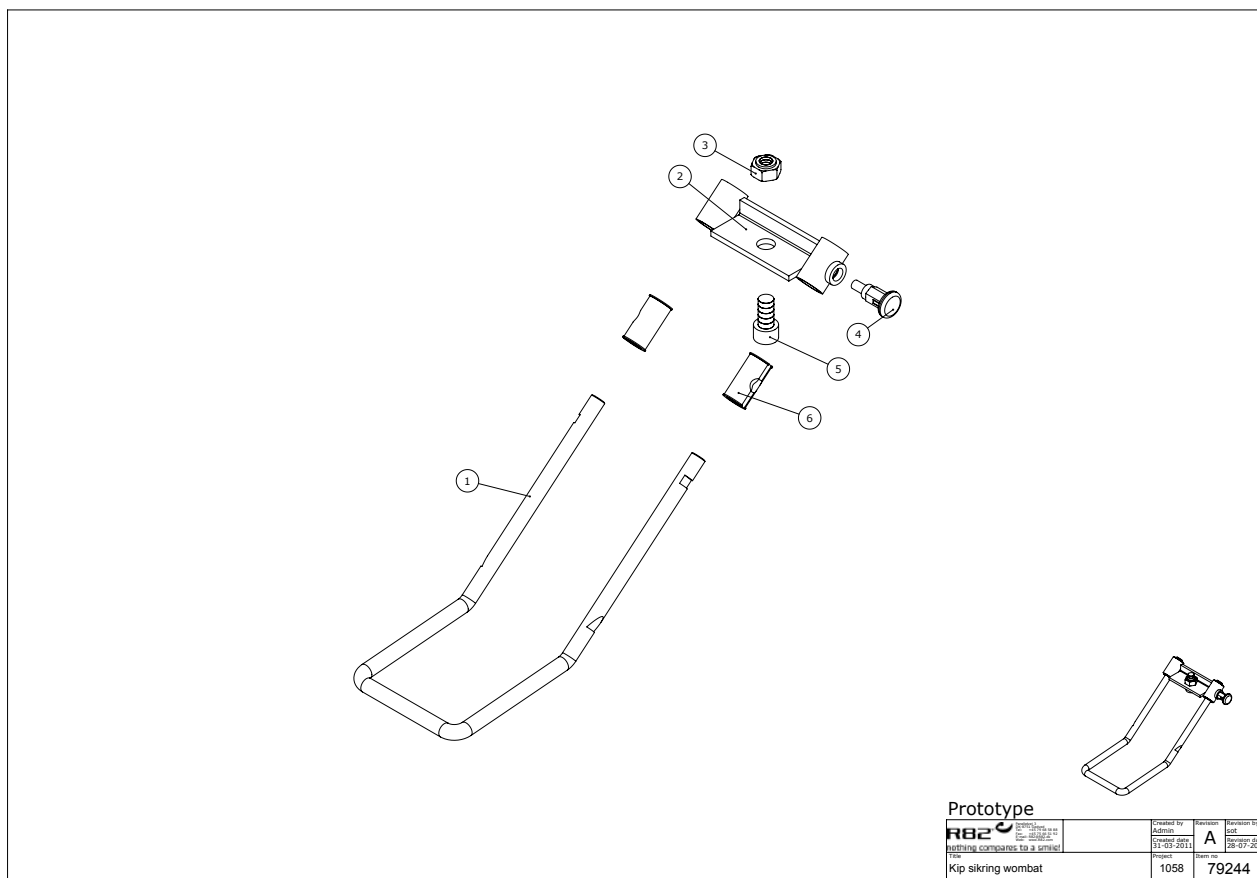

Position	No.	Description	Quantity	Unit of Measure	Ean code
1	121582	Upholstery Lumbar support Wombat3	PCS	1	5707292630032
2	5700483	Upholstered lower back supp	PCS	1	5707292399304
3	1940033	ECO-SYN Z M3,5*10 RUSTFRI	PCS	8	5707292422965
4	0659010	Button head with flange M6*10	PCS	2	5707292356819
5	82744	MOUNTINGPLATE	PCS	2	5707292398901
6	82748	LUMBAR SUPPORT	PCS	1	5707292400093
7	0612500	Insex screw counters. 6*25	PCS	1	5707292138712